

Attaching Air Scoop on the Perfecto₂[™]



Yes, you can.

Assembly, Installation and Operating Instructions

Model No. IRC5PW and IRC5PO2W

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage. If damage is found, DO NOT use. Contact Carrier/Invacare for further instructions.

Safety Summary

To ensure the safe and proper installation of the Air Scoop, these instructions and warnings **MUST** be followed:

⚠ WARNING

DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as owner's manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

⚠ ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

Attaching the Air Scoop

NOTE: For this procedure, refer to FIGURE 1.

Install the air scoop onto the filter access door by inserting the six tabs on the air scoop into the six slots on the sides of the filter access door.

NOTE: An audible click will be heard.

NOTE: The top of the air scoop should be flush with the top of the filter access door when properly installed.

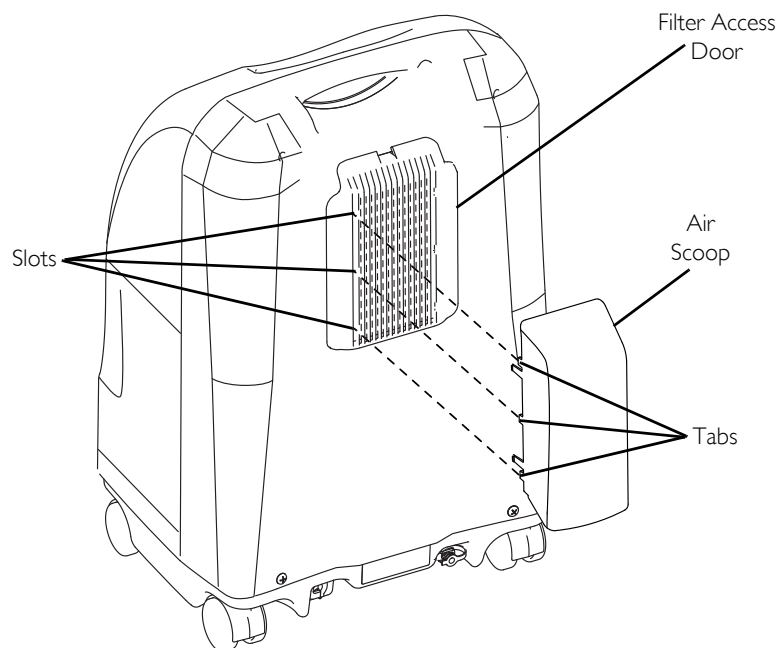


FIGURE 1 Attaching the Air Scoop

For more information regarding Invacare products, parts, and services, please visit www.invacare.com



Yes, you can.[®]

Invacare Corporation

USA
One Invacare Way
Elyria, Ohio USA
44036-2125
440-329-6000
800-333-6900
Technical Services
440-329-6593
800-832-4707

www.invacare.com

Invacare Corporation
2101 E. Lake Mary Blvd.
Sanford, FL 32773
407-321-5630



All rights reserved. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted.
© 2008 Invacare Corporation

Part No 1161718
Rev A - 10/08