
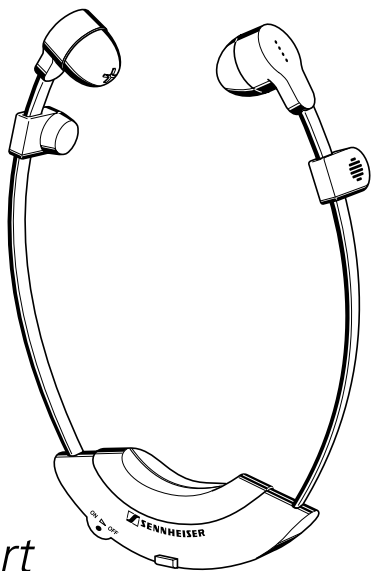
GEBRAUCHSANLEITUNG
INSTRUCTIONS FOR USE
NOTICE D'EMPLOI
ISTRUZIONI PER L'USO
INSTRUCCIONES PARA EL USO
GEBRUIKSAANWIJZING

A 200

audioport

Gebrauchsanleitung	3
Instructions for use	9
Notice d'emploi	15
Istruzioni per l'uso	21
Instrucciones para el uso	27
Gebruiksaanwijzing	33

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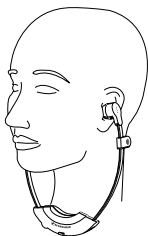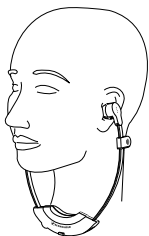
A 200

A 200 Stereo assistive listening device

The A 200 is a personal sound amplifier for people who may sometimes need to rely on artificial assistance for their hearing. It is especially suitable for people with slight to moderate hearing impairment. Due to the stereo amplification via two integrated electret microphones, the user can clearly determine the direction of the sound source. Dynamic headphone transducers ensure a natural and transparent sound reproduction with optimum speech intelligibility. The A 200 features an auxiliary audio input for connecting an external microphone (MKE 2013, available as an accessory) or infra-red receiver.

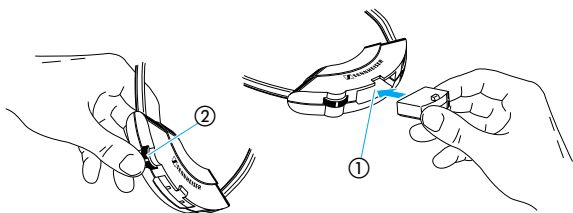
Thank you for choosing sennheiser!

We have designed this product to give you reliable operation over many years.

Please take a few moments to read these instructions carefully, as we want you to enjoy your new Sennheiser product quickly and to the full.

Sennheiser: Additional information

- ▶ Your local Sennheiser dealer will be happy to provide you with information on the extensive range of Sennheiser products. He has a variety of printed materials on Sennheiser products.
- ▶ For up-to-date information on Sennheiser products, you can also visit the Sennheiser WWW site at “<http://www.sennheiser.com>”.

Power supply

The A 200 is powered via the BA 151 rechargeable accuplug which is both cost-saving and environmentally friendly. The accuplug has an operating time of approx. 10–12 hours with the volume set to a medium value. The operating time can be extended by using a second accuplug (available as an accessory). With a second accuplug, one accuplug can be stored in the L 200 charging unit to ensure that it is fully charged when needed. Since the accuplug cannot be overcharged, it can remain in the charging unit even when it has been fully charged.

Putting the receiver into operation

Insert the BA 151 accuplug into compartment ① in the A 200.

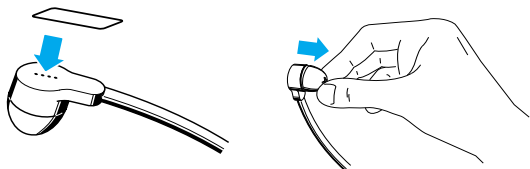
NB.: The BA 151 accuplug must be charged for 24 hours before first use! Take special note of the instructions on accuplug operation on page 12 of this manual!

Switching on/off the a 200, adjusting the volume

Use the rotary switch ② located on the bottom of the A 200 to switch on/off the device and adjust the volume.

The A 200 has a built-in limiter

If the user speaks or if loud and sudden noise occurs, the amplification is reduced immediately. Unpleasant volume peaks are thus avoided and the user perceives his own voice as sounding completely natural. The volume limiter is factory-adjusted to the optimum setting.

Adjusting the bass attenuation

You can use the two supplied sealing tapes to seal the small holes in the back of the headphone systems. The more holes you seal with the sealing tapes, the greater is the bass attenuation. This increases the intelligibility without increasing the volume - which is an advantage for people with a more severe hearing impairment.

Replacing the ear-cushions

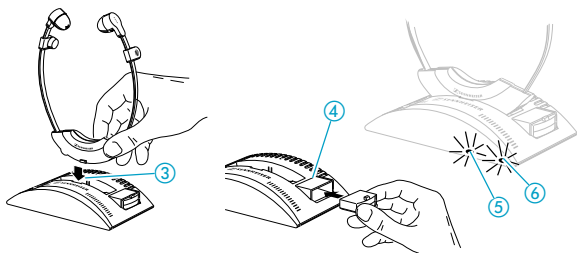
The ear-cushions can easily be removed from the headphones and cleaned with warm soapy water. From time to time, however, the ear-cushions should be replaced for reasons of hygiene. Replacement ear-cushions are available with different degrees of softness:

- ▶ Black ear-cushions, firmer version Cat. no. 37080
- ▶ Transparent ear-cushions, softer version Cat. no. 40949

Please order your ear-cushions from your Sennheiser dealer.

Volume up? – No!

When people use headphones, they tend to choose a higher volume than with loudspeakers. Listening at high volume levels for long periods can lead to permanent hearing defects. Please protect your hearing, Sennheiser headphones have an excellent sound quality even at low volumes.

Charging the accuplug / charge indication

To charge the accuplug, insert the personal sound amplifier containing the accuplug into the L 200 charging unit ③. A second BA 151 accuplug can be charged separately in the charging compartment ④. The automatic charging process is indicated by the LEDs ⑤ and ⑥. Since the accuplug cannot be overcharged, the personal sound amplifier and the accuplug can remain in the charging unit even when the accuplug has been fully charged.

Instructions on accuplug operation

Do not let rechargeable accuplugs become completely flat. If you do, they will be exhausted and cannot be recharged!

- ▶ If you notice that the accuplug is going flat (noise interference and sound distortions are sure signs of a flat accuplug), switch off the device and recharge the accuplug immediately. (If necessary, use a second accuplug to listen while the first accuplug is being recharged.)
- ▶ Immediately recharge the accuplug after each use.
- ▶ **Charge the accuplug for 24 hours before you use it for the first time!**

Environment and health

Devices powered by rechargeable accuplugs are kind to the environment. This energy source can be recharged many times. Conventional batteries, on the other hand, must be disposed of as special waste after they have been used just once.

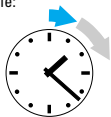
If an accuplug should be defective, it is then recycled - the dealer who sold it to you will take it back and will also supply you with a new, original Sennheiser accuplug. (Recycling methods differ in different countries).

Rechargeable accuplugs have a „charging cycle“

Depending on how long your receiver has been in use, you must also recharge the BA 151 accuplug for a certain amount of time. A simple rule of thumb will help you to decide how long is necessary:

Operating time x 1.5 = recharging time

Example:

1 hour listening
1.5 hours recharging

2 hours listening
3 hours recharging

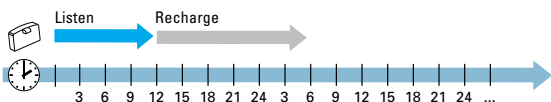
3 hours listening
4.5 hours recharging


Operation with a second accuplug

By using a second accuplug, you can increase the listening time with your A 200. The second accuplug should be kept in the charging compartment of the L 200 charging unit to ensure that it is ready and fully charged when needed.

- ▶ If you use your system with only one accuplug, you must stop after about 12 hours and recharge the accuplug for about 18 hours.

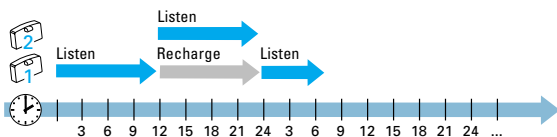- ▶ With a second accuplug, you can listen for more than twice the time. Simply place the discharged first accuplug back into the charging compartment. After the (partial) recharge, there is an additional (partial) amount of power available:


Important!

- ▶ Never open electronic devices! This must only be done by authorised personnel and is all the more important for current-carrying units.
- ▶ Keep the device away from central heating radiators and electric heaters. Never expose it to direct sunlight.
- ▶ Use the device in dry rooms only.
- Use a damp cloth for cleaning the device. Do not use any solvents.
- ▶ Do not wear your personal sound amplifier when eating or drinking in order to protect it from getting soiled.
- ▶ Many modern appliances (e.g. microwave, television set, lighting transformers) generate strong electromagnetic fields. In the proximity of these appliances, your A 200 personal sound amplifier can be subject to interference.

Technical data

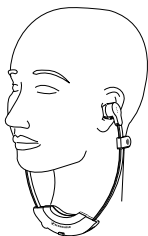Audio frequency response	200–12,000 Hz
Transducer principle:	
Headphones	dynamic
Microphones	2 electret mics, stereo
Acoustic amplification	max. 30 dB
Characteristic SPL at 1 kHz	max. 105 dB
Operating time	max. 12 h with one accuplug
Weight incl. accuplug	51 g
Supply schedule:	
	1 A 200 personal sound amplifier
	1 L 200 charging unit
	1 plug-in mains unit for L 200 (available for different operating voltages)
	1 BA 151 accuplug
	1 set of replacement ear-cushions
	1 set of sealing tapes

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Sennheiser electronic GmbH