


# Quick instructions Set 55 TV Set 50 TV

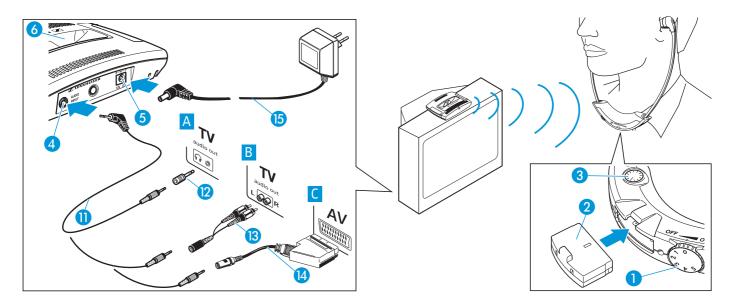

## Connecting the system components

- ▶ Plug in one end of the connecting cable ① to the socket on your sound source (e.g. television). If necessary, use one of the adapters ②, ③ or ② (not included). Plug in the other end of the connecting cable ① to the blue socket ④ on the transmitter.
- ▶ Connect the mains unit (5) to the transmitter (5) and to the mains.
- Place the receiver into the transmitter's charging compartment 6.
- ▶ Before you use the rechargeable battery 2 for the first time, charge it for at least 14 hours.
- Adjust the volume of your sound source to a sufficient (medium) level.

#### Switching on and using the stethoset receiver

- ► Turn the volume control 1 in the direction of "ON" until it clicks.
- First adjust the volume to a low level. This protects your hearing from damage. Then put the receiver on and adjust the volume to a comfortable level.
- ▶ Use the balance control ③ to adjust the volume between the left and right ear.
- If you cannot hear anything through the receiver, increase the volume of your sound source.

## Switching off the stethoset receiver

To switch the receiver off, turn the volume control 1 in the direction of "OFF" until it clicks.

# Charging the stethoset reciever

Switch the receiver off. Place the receiver into the transmitter's charging compartment 6 without exerting too much pressure. With a fully charged battery, the operating time is approx. 9 hours.

For details on the use of the Set 50 TV/Set 55 TV, please refer to the enclosed operating instructions.