

CARSON®

FREEHAND™ FH-25 Magnifier

TO USE HANDS-FREE

EXTEND BASE STAND BY GRABBING TABS AND PULLING OUT BOTTOM PORTION OF HANDLE. ADJUST MAGNIFIER HEAD TO DESIRED VIEWING ANGLE. SWITCH ON LIGHT FOR USE IN LOW-LIGHT CONDITIONS.



TO CHANGE BATTERIES

PULL OPEN BOTTOM PORTION OF HANDLE TO EXPOSE BATTERY COMPARTMENT. GRAB TOP OF BATTERY DOOR FROM BOTH SIDES AND PULL OPEN. REPLACE EXHAUSTED BATTERIES WITH 2 AA 1.5V BATTERIES (PLEASE FOLLOW POLARITY INDICATORS ON INSIDE OF BATTERY COMPARTMENT).

WARNING: DO NOT LEAVE BATTERIES INSTALLED DURING EXTENDED PERIODS OF DISUSE. DO NOT MIX NEW AND OLD BATTERIES. DO NOT MIX ALKALINE, STANDARD (CARBON-ZINC) OR RECHARGEABLE (NICKEL-CADMIUM) BATTERIES.