Mobile Shower Chair

Assembly, Installation and Operating Instructions

Model No. 6358E

Yes, you can.™

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. In case of shipping damage, DO NOT use. Contact Carrier/Dealer for further instruction.

SAFETY SUMMARY

To ensure the safe installation, operation and use of the Mobile Shower Chair, these instructions **MUST** be followed:

WARNING

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Always inspect the Mobile Shower Chair to ensure that it is properly positioned and stable before using.

Ensure that all hardware is tight at all times.

Mobile Shower Chair is NOT to be used as a transfer bench or transfer device.

Users with limited physical capabilities should be supervised or assisted when using the mobile shower chair.

Improper loading may result in Mobile Shower Chair tipping over. Ensure user weight is evenly distributed when seated.

Inspect and clean wheels periodically to maintain proper operation.

The mobile shower chair has a recommended weight limit of 250 lbs.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products

INSTALLATION WARNING

After ANY adjustments, repair or service and before use, make sure all attaching hardware is tightened securely - otherwise injury or damage may occur.

ASSEMBLING THE SHOWER CHAIR (FIGURE 1)

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. UNPACK CAREFULLY - Remove plastic from the end of the foot support tube (Item 1) and locate the package of screws and nuts.
2. Position the rear of the foot support tube (Item 1) so that it is aligned with the mounting holes in the rear of the side frame (Item 5). Bolt it to the rear side frame with the two screws (Item 16) and two locknuts (Item 15).

NOTE: Foot support (Item 1) should be positioned inside of the front legs and outside of the rear legs.

IMPORTANT: Tighten the screws (Item 17) and slotted rivnut (Item 18). They are assembled loosely to ease assembly of the foot support tube.

3. Align the holes in the back frame tube (Item 10) with the holes inside the frame (Item 5). Bolt together with the four screws (Item 14), and four nuts (Item 4).
4. Ensure that all nuts and screws are tight and that the chair is stable before use.

INSTALLING AND REMOVING COMMODE SEAT AND LID

NOTE: Refer to the WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Align toilet seat ring clamps onto back frame rail (Item 6). Push down on seat (Item 12) until it snaps firmly into place.

NOTE: Each clamp should set outside the pail holders.


2. When removing seat, simply grasp the back corner and lift up firmly.

NOTE: When you snap the seat on the commode frame, you will notice that the seat is very safe and secure. If you desire the seat to pivot easier, you may lubricate the rear seat rail area with a very small amount of Vaseline® or furniture polish.

3. Wheel locks are provided on two (2) casters. To operate, depress the “lock lever” to lock wheels; to unlock wheels, depress the release lever.

CLEANING THE COMMODE SEAT AND LID

Use a soft, damp cloth and mild soap for cleaning.

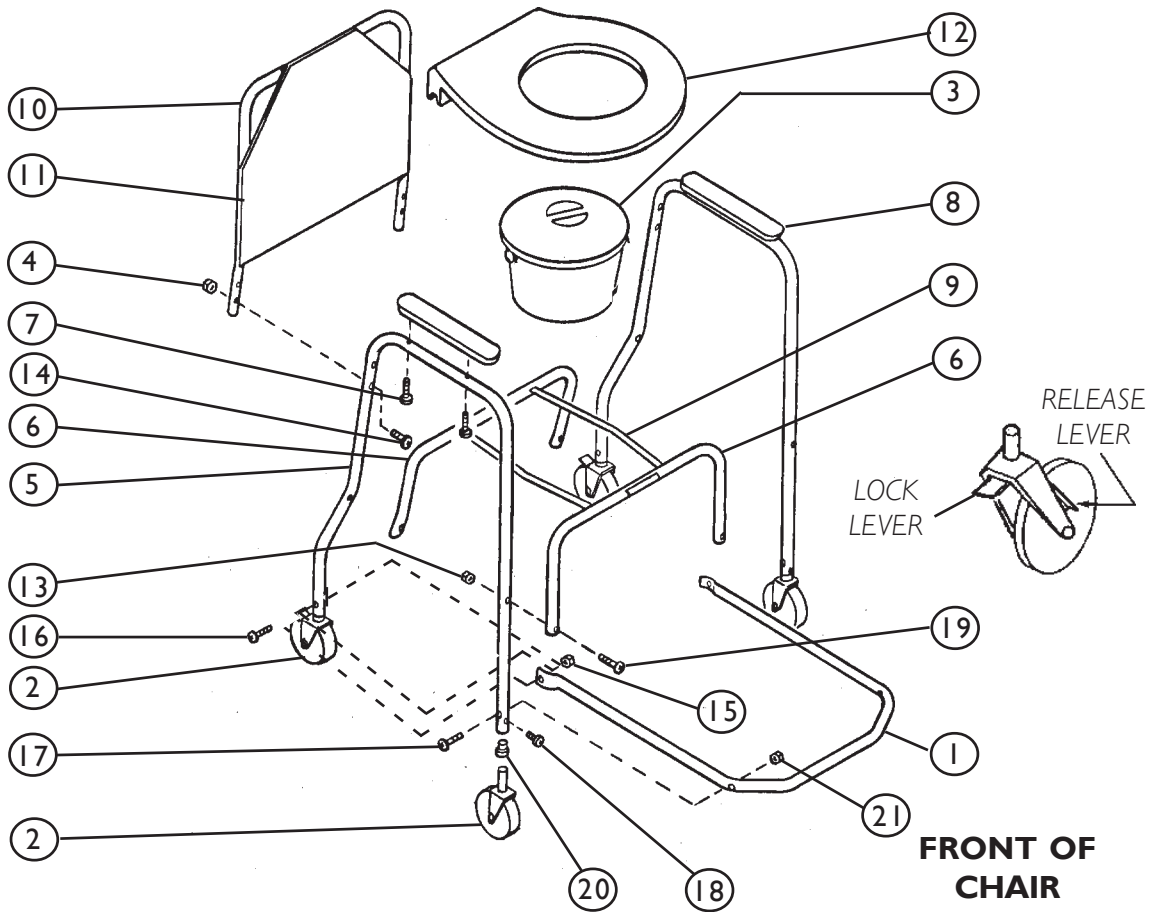PARTS LIST

ITEM	DESCRIPTION
1.	Foot Support Tube
2.	5" Caster Assembly w/o wheel Lock
3.	5" Caster Assembly w/ wheel lock
4.	Pail with Lid
5.	N.E. Nut with Nylon (1/4"-20)
6.	Side Frame
7.	Seat Rail - Front
8.	Seat Rail - Rear
9.	Screw Ph Hd Mach (#10-32X1/2")
10.	Armrest - 2 pc Desk Length
11.	Pail Holder-Aluminum
12.	Back Frame Tube
13.	Upholstery - Nylon Back
14.	Toilet Seat Ring
15.	Locknut (1/4"-20)
16.	Screw Ph Rd Hd (1/4"-20X2-1/4")
17.	Locknut (#10-32)
18.	Screw Ph Rd Hd (#10-32X1-3/8")
19.	Screw (#10-32x1-3/4")
20.	Ph. Pan Head Screw (#8-32x5/16")
21.	Screw Ph. Hd. (1/4"-20x1-1/4")
	Caster Bearing Sleeve
	Rivnut (#10-32)

FIGURE I - ASSEMBLING THE SHOWER CHAIR/ INSTALLING AND REMOVING COMMUNE SEAT AND LID

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© 2004 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare
Vaseline is a registered trademark of
Invacare Corporation www.invacare.com
Invacare Corporation.