Wheeled Walker Accessories

Assembly, Installation and Operating Instructions

Model Nos. 6270E (3 - Inch Single Fixed Wheel Attachment w/Glide Tips)

6267E (5 - Inch Swivel Wheels w/Glide Tips)

6265E (13 - Inch Rear Wheel Brakes) used with 3 or 5 - Inch Swivel Wheels

Yes, you can™

SAVE THESE INSTRUCTIONS

NOTE: Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/dealer for further instructions.

SAFETY SUMMARY

To ensure the safe installation of the Wheeled Walker Accessories, these instructions **MUST** be followed:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

STABILITY WARNINGS

Check to make sure that rubber tips and/or plastic glide tips are not ripped, worn or missing. Replace them **BEFORE** using the walker.

DO NOT use rubber tips on front leg extensions when using rear wheel/brake options.

Rear leg extensions with rubber tips or plastic glide tips **MUST** be in contact with floor surface at all times.

Glide tips are used on rear leg extensions **ONLY** to allow the walker to roll easily over indoor carpeted areas without the need to lift the walker off the ground.

Glide tips on rear leg extensions should only be used with front wheel attachments.

Glide tips are **NOT** recommended for use outdoors.

Glide tips are **NOT** recommended for use on all four (4) leg extensions at once.

DO NOT use glide tips and rubber tips at the same time.

Glide tips do not grip the floor the way rubber tips grip flooring, therefore, extra caution should be observed when in use.

When using glide tips on a walker, short steps should be taken to assure that weight is distributed evenly and directly over walker legs. These precautions will prevent uncontrollable movement of the walker.

INSTALLATION WARNINGS

Ensure all snap buttons fully protrude through adjustment holes of each leg extension. This ensures that the leg extensions are securely locked in position.

Wheel Accessories are **ONLY** to be used on the front walker legs with exception of Model No. 6265E - 13-Inch Rear Wheel Brakes.

The leg extensions should be adjusted so the walker is level. If an even height cannot be achieved, adjust the leg extensions so that the rear of the walker is no more than one (1) inch lower than the front.

DESCRIPTION

Wheeled Walker Accessories make a standard pick-up walker easier to use for people that find it difficult to lift or maneuver a standard walker and will allow the user to walk more naturally and with greater ease.

To use the walker with the wheel attachments, push the walker forward as you walk. The walker will move with you.

INSTALLING FIXED WHEEL ATTACHMENTS MODEL NO. 6270E (FIGURE 1)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.


1. Remove two (2) existing front leg extensions from walker by depressing the snap buttons.

NOTE: Front leg extensions will not be re-assembled to walker when using wheel attachments.

2. Depress snap button on walker legs and slide wheel attachment onto two (2) front legs of walker.

NOTE: When using fixed wheel attachments, wheels should be positioned on outside of walker.

3. Adjust each wheel attachment. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker is level.
4. Remove rear rubber tips and replace with glide tips. Glide tips will fit inside the leg extensions and may need to be tapped into place with mallet or hammer.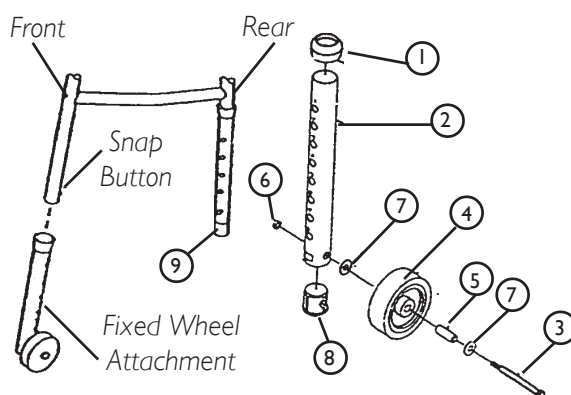

ITEM	DESCRIPTION	QUANTITY
1.	Anti-Rattle	1
2.	Adjustable Leg 10-inches	1
3.	Screw, Hex 1/4-20 x 2-1/2-inch	1
4.	Wheel-3-inch	1
5.	Bushing 3-inch Wheel	1
6.	Locknut (1/4-20)	1
7.	Washer	1
8.	Bushing	1
9.	Glide Tip	1

*Quantities are per component.

FIGURE 1 - INSTALLING FIXED WHEEL ATTACHMENTS MODEL NO. 6270E

INSTALLATION WARNINGS CONTINUED

Always test to see that the walker and attachments are properly and securely locked in place before using.

Swivel casters should be checked periodically for tightness and damage. If loose, tighten securely. If damaged, replace immediately.

INSTALLING SWIVEL WHEEL ATTACHMENTS MODEL NO. 6267E (FIGURE 2)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove the two (2) existing front leg extensions from the walker by depressing the snap buttons.


NOTE: The front leg extensions will not be reassembled to the walker when using the Swivel Wheel Attachments.

2. Depress the snap button on the walker legs and slide the swivel wheel attachments onto the front legs of the walker. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker is level.

INSTALLING REAR BRAKE ACCESSORIES MODEL NO. 6265E (FIGURE 2)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove the existing rear leg extensions from the walker by depressing the snap buttons and sliding the extensions off.
2. Install the 10-inch Rear Wheel Brakes or 10-inch SureGlide™ Brakes by depressing the snap buttons on the rear walker legs and sliding the accessories onto the rear legs.
3. Adjust to the desired adjustment hole on the rear accessories.
4. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker is level.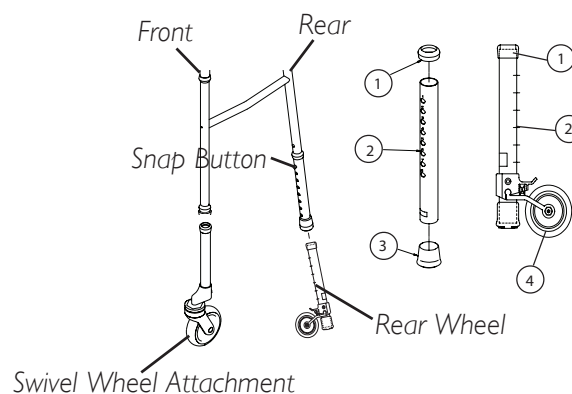

ITEM	DESCRIPTION	QUANTITY*
1.	Anti-Rattle	1
2.	Adjustable Leg w/Axle Hole	1
3.	Crutch Tip	1
4.	3-Inch Wheel	1

*Quantities are per component.

FIGURE 2 - INSTALLING SWIVEL WHEEL ATTACHMENTS MODEL NO. 6267E AND REAR BRAKE ACCESSORIES MODEL NO. 6265E

TIGHTENING SWIVEL CASTERS (FIGURE 3)

NOTE: Refer to the INSTALLATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

1. Remove swivel wheel attachments from front legs by depressing snap button.
2. Hold one (1) swivel wheel attachment with the wheel up as shown in FIGURE 3.
3. Turn hex nut **CLOCKWISE** with a wrench to tighten swivel caster.
4. Repeat STEPS 2-3 for remaining swivel wheel attachment.
5. Depress the snap buttons and slide the swivel wheel attachments onto the front legs of the walker. Ensure that the snap button of each leg fully protrudes through the desired adjustment hole and legs are adjusted so the walker sits level.

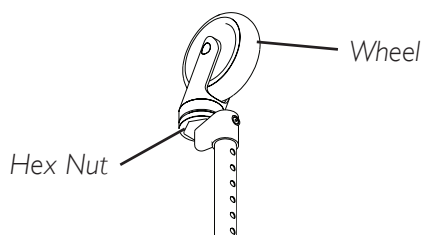
FIGURE 3 - TIGHTENING SWIVEL CASTERS

UNITED KINGDOM LIMITED WARRANTY

This warranty is extended only to the original purchaser of our products and does not affect statutory rights.

Invacare UK/Invacare Service Representative warrants its product to be free from defects for a period of three (3) years from the date of purchase. With regard to the original purchaser/user only, Invacare UK/Invacare Service Representative warrants its metal fabricated hardware components to be free from defects in materials and workmanship for the lifetime of the product except any wood, foam plastic or upholstered components. If within such warranty period the product shall be proven to Invacare UK's/Invacare Service Representative reasonable satisfaction to be defective, it shall be repaired or replaced, at Invacare UK's/Invacare Service Representative option; Invacare UK's/Invacare Service Representative sole obligation, and your exclusive remedy under this warranty being limited to such repair or replacement.

For warranty service, please contact Invacare UK's Service Department/Invacare Service Representative during normal working hours. Upon receiving notice of an alleged defect in a product, Invacare UK/Invacare Service Representative will issue a serialized return authorization. The product shall then be returned to either Invacare plant or service center as specified by Invacare UK/Invacare Service Representative. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY DOES NOT COVER NORMAL WEAR AND TEAR AND SHALL NOT APPLY TO PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS) OR TO PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE UK/INVACARE SERVICE REPRESENTATIVE.

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© 2003 Invacare Corporation
Invacare is a registered trademark of
trademarks of Invacare Corporation.
Invacare, S.A.
Invacare Corporation.