InTouch™ Propel™ Back

DEALER: This manual **MUST** be given to the user of this product.

USER: **BEFORE** using this product, read this manual and save for future reference.

For more information regarding
Invacare products, parts, and services,
please visit www.invacare.com

Yes, you can.

⚠ WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS CONTACT A QUALIFIED DEALER OR INVACARE TECHNICAL SUPPORT BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

NOTE: Updated versions of this manual are available on www.invacare.com.

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

MAINTENANCE

Maintenance MUST be performed ONLY by qualified personnel.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

TABLE OF CONTENTS

SPECIAL NOTES	3
SECTION 1—GENERAL GUIDELINES	7
Important Information	7
Seating Pressure Warning.....	7
Installation Warnings.....	7
Maximum Weight Rating.....	8
SECTION 2—INSTALLING	9
Installation Overview	9
Installing Mounting Hardware	9
Installing/Removing the Propel Back	11
Adjusting the Propel Back Width.....	12
Installing/Removing the Propel Back Cushion.....	13
Installing the Cushion	13
SECTION 3—OPERATION	14
Adjusting the Propel Back Depth and Angle	14
SECTION 4—MAINTENANCE	15
Replacing Upholstery	15
Replacing Cushion	15
Cleaning Instructions	16
Back Assembly	16
Cover	16
Foam.....	16
LIMITED WARRANTY	17

REGISTER YOUR PRODUCT

The benefits of registering:

1. Safeguard your investment.
2. Ensure long term maintenance and servicing of your purchase.
3. Receive updates with product information, maintenance tips, and industry news.
4. Invacare can contact you or your provider, if servicing is needed on your product.
5. It will enable Invacare to improve product designs based on your input and needs.

Register ONLINE at www.invacare.com

- or -

Complete and mail the form on the next page

PRODUCT REGISTRATION FORM

Register **ONLINE** at www.invacare.com - or -
Complete and mail this form

Name _____

Address _____

City _____ State/Province _____

Zip/Postal Code _____

Email _____ Phone No. _____

Fold
here

Invacare Model No. _____ Serial No. _____

Purchased From _____ Date of Purchase: _____

1. Method of purchase: (check all that apply)

Medicare Insurance Medicaid Other _____

2. This product was purchased for use by: (check one)

Self Parent Spouse Other

3. Product was purchased for use at:

Home Facility Other

4. I purchased an Invacare product because:

Price Features (list features) _____

5. Who referred you to Invacare products? (check all that apply)

Doctor Therapist Friend Relative Dealer/Provider Other _____

Advertisement (circle one): TV, Radio, Magazine, Newspaper No Referral _____

6. What additional features, if any, would you like to see on this product?

Fold
here

7. Would you like information sent to you about Invacare products that may be available for a particular medical condition? Yes No

If yes, please list any condition(s) here and we will send you information by email and/or mail about any available Invacare products that may help treat, care for or manage such condition(s):

8. Would you like to receive updated information via email or regular mail about the Invacare home medical products sold by Invacare's dealers? Yes No

9. What would you like to see on the Invacare website?

10. Would you like to be part of future online surveys for Invacare products? Yes No

11. User's Year of birth: _____

If at any time you wish not to receive future mailings from us, please contact us at Invacare Corporation, CRM Department, 39400 Taylor Parkway, Elyria, OH 44035, or fax to 877-619-7996 and we will remove you from our mailing list.

To find more information about our products, visit www.invacare.com.

Cut Along Line


Fold
here

BUSINESS REPLY MAIL
FIRST-CLASS MAIL PERMIT NO. 196 ELYRIA OH

POSTAGE WILL BE PAID BY ADDRESSEE

NO POSTAGE
NECESSARY
IF MAILED
IN THE
UNITED STATES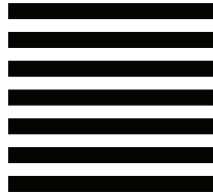

INVACARE CORPORATION
CRM DEPARTMENT
39400 TAYLOR PARKWAY
ELYRIA OH 44035-9836

Fold
here

Invacare Product Registration Form

Please Seal with
Tape Before Mailing

Cut Along Line

SECTION I—GENERAL GUIDELINES

⚠ WARNING

SECTION I - GENERAL GUIDELINES contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Important Information

The best way to avoid problems related to pressure sores is to understand their causes and your role in a skin management program.

Your therapist and physician should be consulted if you have any questions regarding weight relief, self-examination of skin, or individual limitations and needs.

All seating systems should be selected carefully. Working with your therapist, physician and equipment supplier is the best way to assure that a seating system choice matches your individual needs.

As the needs of the individual become more complex, the Propel Back evaluation becomes more important.

The Propel Back is not designed for use with tilt-in-space and/or recline model wheelchairs.

Seating Pressure Warning

Skin condition should be checked frequently after the installation and use of any new seating system.

Installation Warnings

After any adjustments, repair or service and before use, make sure that all attaching hardware is tightened securely.

DO NOT install the Propel Back assembly onto back canes with an outside diameter greater than 1-inch or less than .86-inch - otherwise injury or damage may occur.

Ensure the Propel Back is properly secured to the wheelchair before using.

The Propel Back **MUST** be resting flush against the tabs on the cane mounting brackets.

Both right and left back latches **MUST** be fully engaged.

⚠ WARNING

The position of the Propel Back is directly related to the wheelchair's stability. It may be necessary to reposition the front casters, rear wheels, back angle, and/or seat depth before use. Use **EXTREME** caution when using a new seating position.

Maximum Weight Rating

	16 - 21-INCH WIDTH
MAXIMUM WEIGHT RATING	300 lbs

SECTION 2—INSTALLING

⚠ WARNING

Check **ALL** fasteners periodically to ensure that mechanical connections and attaching hardware are tightened securely - otherwise injury or damage may occur. **DO NOT** continue to use this product if problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier. The Propel Back is not designed for use with tilt-in-space and/or recline model wheelchairs.

Installation Overview

NOTE: For this procedure, refer to FIGURE 2.1.

1. Remove any existing back upholstery or back system from the chair before installation. Refer to Owner's Manual shipped with wheelchair.
2. Install mounting hardware. Refer to Installing Mounting Hardware on page 9.
3. Install Propel Back onto the wheelchair. Refer to Installing/Removing the Propel Back on page 11.
4. Make any necessary depth and/or angle adjustments. Refer to Adjusting the Propel Back Depth and Angle on page 14 and Adjusting the Propel Back Width on page 12.

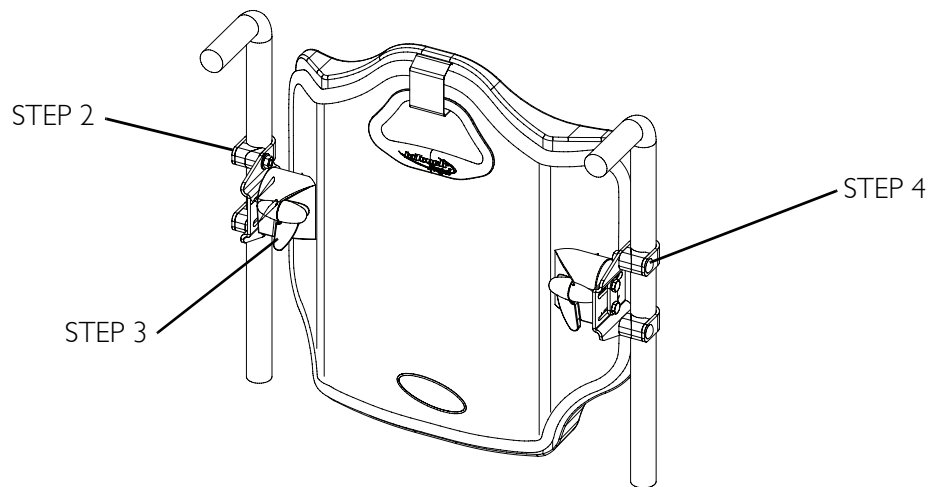
FIGURE 2.1 Installation Overview

Installing Mounting Hardware

NOTE: For this procedure, refer to FIGURE 2.2 on page 10.

NOTE: Cane brackets adjust to fit 7/8" to 1" back canes. No additional hardware is required.

1. Position the cane mounting bracket onto the back cane at the desired height.

NOTE: Ensure that the back tab is positioned towards the inside of the wheelchair and the round end is pointing upward.

2. Secure the cane mounting bracket and wedge blocks to the back cane with the two mounting bolts and hex nuts. Hand tighten securely.

NOTE: Some wheelchairs have adjustable back canes that are both 7/8" and 1". Ensure the wedge blocks are completely on either the 7/8" or 1" section and not straddling the two. Refer to Detail "A" in FIGURE 2.2.

NOTE: It may be necessary to raise or lower the back canes to achieve the desired bracket placement. Refer to Owner's Manual shipped with wheelchair.

3. Repeat STEPS 1 and 2 for the other cane mounting bracket.

NOTE: Cane mounting brackets MUST be parallel to each other.

4. Using the wrench supplied or 7/16 socket wrench, securely tighten the mounting bolts.

NOTE: DO NOT tighten the mounting bolts until the back height has been adjusted correctly.

NOTE: Check the alignment of the mounting brackets before tightening.

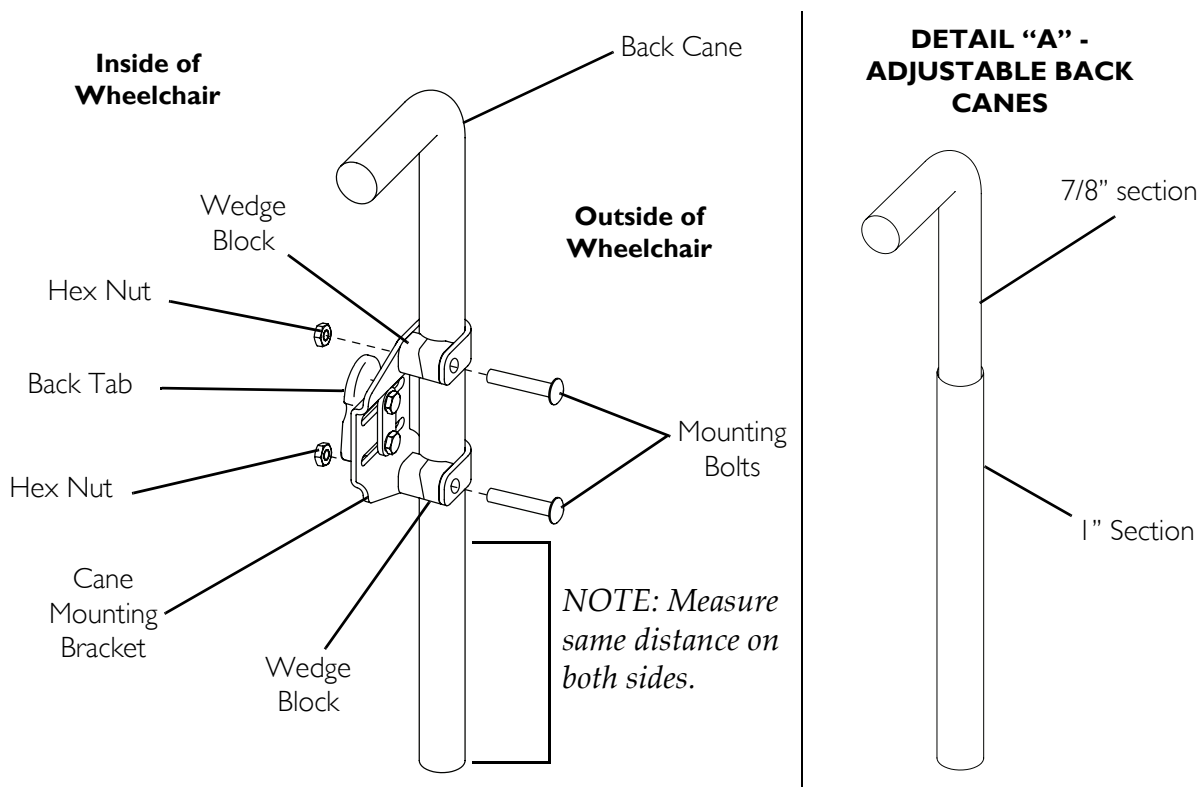
FIGURE 2.2 Installing Mounting Hardware

Installing/Removing the Propel Back

⚠ WARNING

The position of the Propel Back is directly related to the chair's stability. It may be necessary to reposition the front casters, rear wheels, back angle, and/or seat depth before use. Use **EXTREME** caution when using a new seating position.

NOTE: For this procedure, refer to FIGURE 2.3 on page 11.

NOTE: To remove the back, reverse this procedure.

NOTE: If the Propel Back does not fit properly on the back canes, the back width may need to be adjusted. Refer to Adjusting the Propel Back Width on page 12.

1. Align the back latch brackets with the tabs on the cane mounting brackets.
2. Push up the back latches into the unlocked position.

NOTE: The back latches on the back latch brackets must be in the up (unlocked) position to install or remove the back from the wheelchair.

3. Install the back latch brackets onto the tabs.

NOTE: Ensure the back latches rest flush on the tabs.

4. Push the back latches down (locked position) to secure the back to the tabs. Refer to Detail "A" of FIGURE 2.3.
5. After installation and before use, pull up on the back to ensure it is locked.

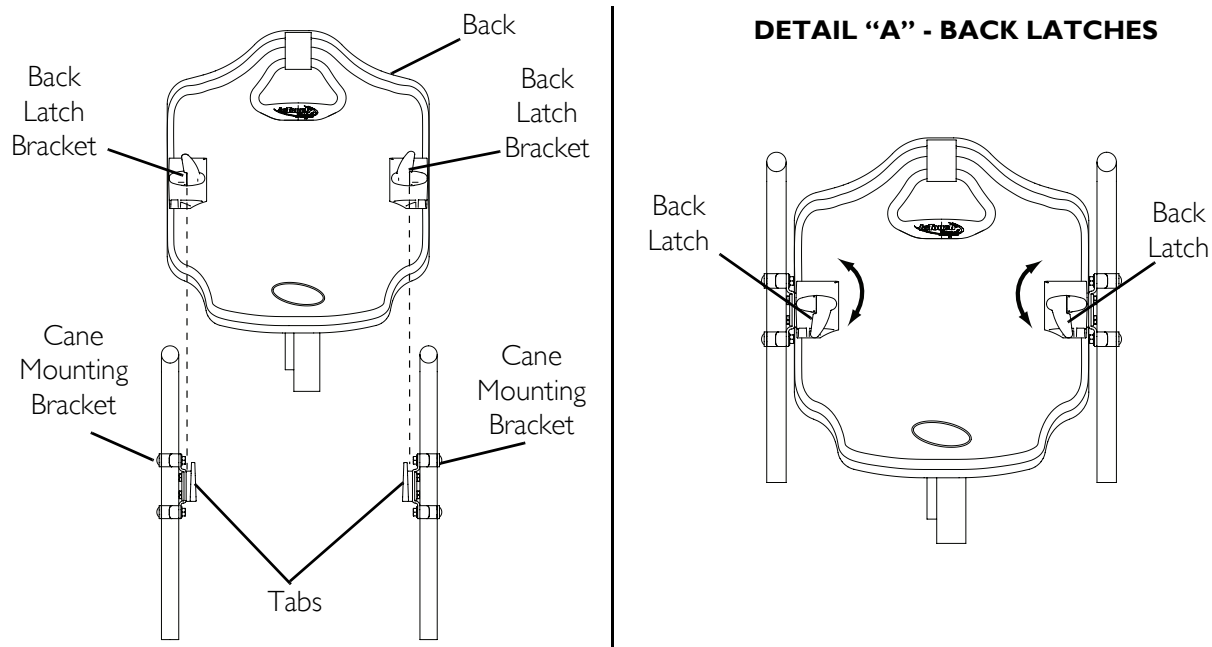
FIGURE 2.3 Installing/Removing the Propel Back

Adjusting the Propel Back Width

NOTE: For this procedure, refer to FIGURE 2.4.

1. Remove the cushion from the back frame. Refer to Installing/Removing the Propel Back Cushion on page 13.
2. Loosen, but DO NOT remove, the two mounting bolts and washers that secure the cane mounting bracket to the back frame.
3. Move the cane mounting brackets to the desired position.

NOTE: Keep the back frame centered for proper positioning.

4. Carefully set the back onto the back tabs.
5. Adjust the mounting brackets on both sides for desired fit keeping back centered.
6. Tighten the mounting bolts securely.
7. Replace the cushion. Refer to Installing/Removing the Propel Back Cushion on page 13.
8. Check the back height, adjust cane mounting bracket if necessary. Refer to Installing Mounting Hardware on page 9.

NOTE: Most wheelchairs' back canes are not fully vertical. Adjust the width of the brackets on the back frame to match. If the Propel Back seems too loose or too tight, the brackets can be manually adjusted to increase or decrease the amount of effort required to install or remove the back from the chair. This adjustment may also be used to reduce back noise caused by movement in the wheelchair.

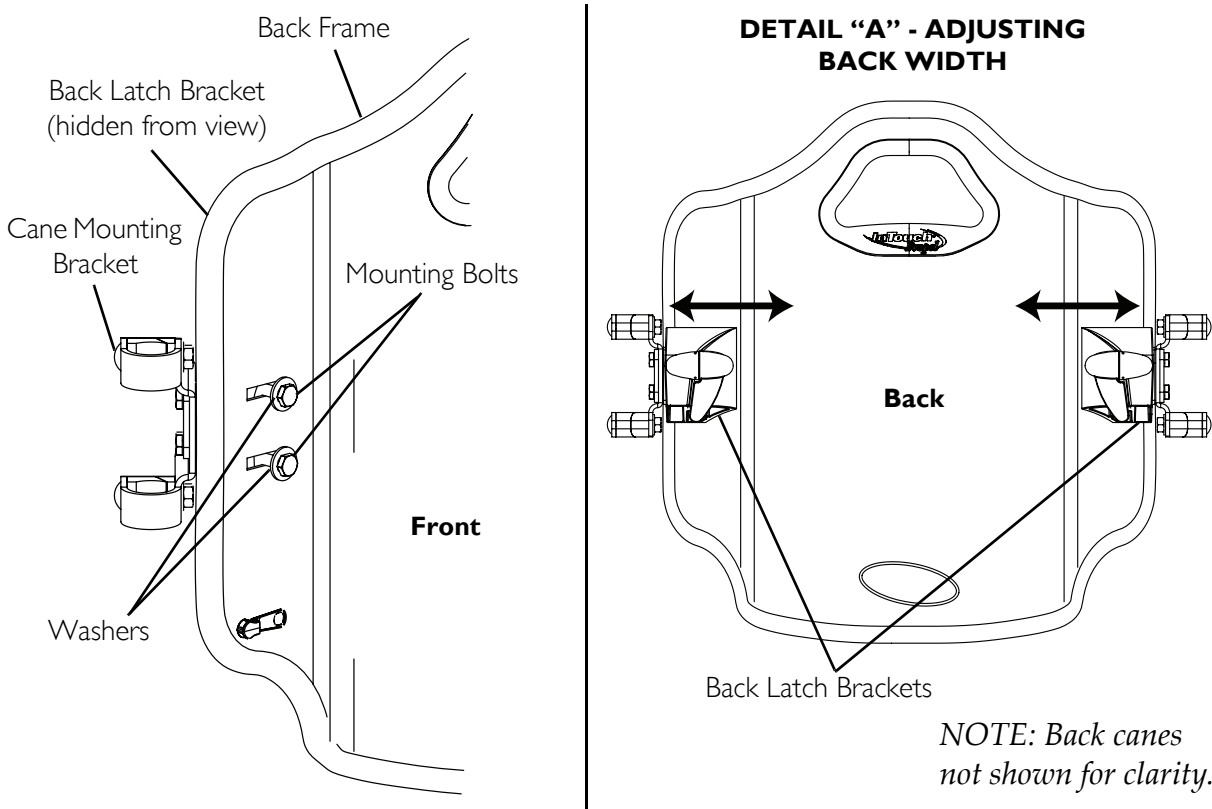
FIGURE 2.4 Adjusting the Propel Back Width

Installing/Removing the Propel Back Cushion

NOTE: For this procedure, refer to FIGURE 2.5.

NOTE: To remove the cushion, reverse this procedure.

Installing the Cushion

1. Secure the hook-and-loop strap at the top of the cushion to the hook-and-loop strip above the handgrip on the back frame.
2. Secure the cushion to the back frame with hook-and-loop strips on the rear of the cushion.

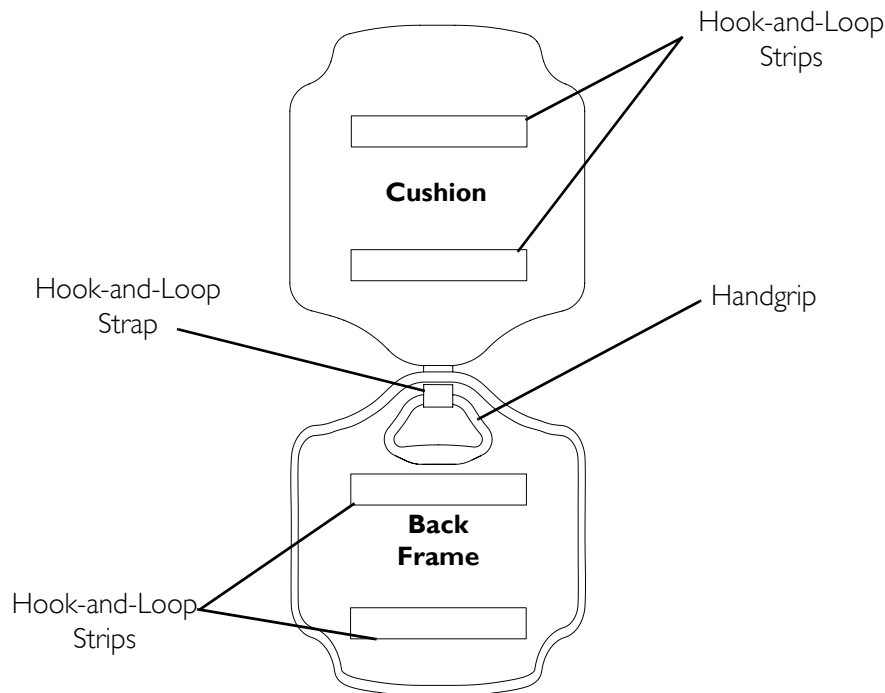
FIGURE 2.5 Installing/Removing the Propel Back

SECTION 3—OPERATION

⚠ WARNING

Check **ALL** fasteners periodically to ensure that mechanical connections and attaching hardware are tightened securely - otherwise injury or damage may occur. **DO NOT** continue to use this product if problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

Adjusting the Propel Back Depth and Angle

NOTE: For this procedure, refer to FIGURE 3.1.

1. Loosen, but **DO NOT** remove, the hex bolts securing the back tab to the cane mounting brackets on both sides.
2. Adjust the back to the desired angle and/or depth.

NOTE: Use the measurement marks for as a reference to align the left and right sides.

3. Tighten the hex bolts securing the back tabs to the cane mounting brackets.

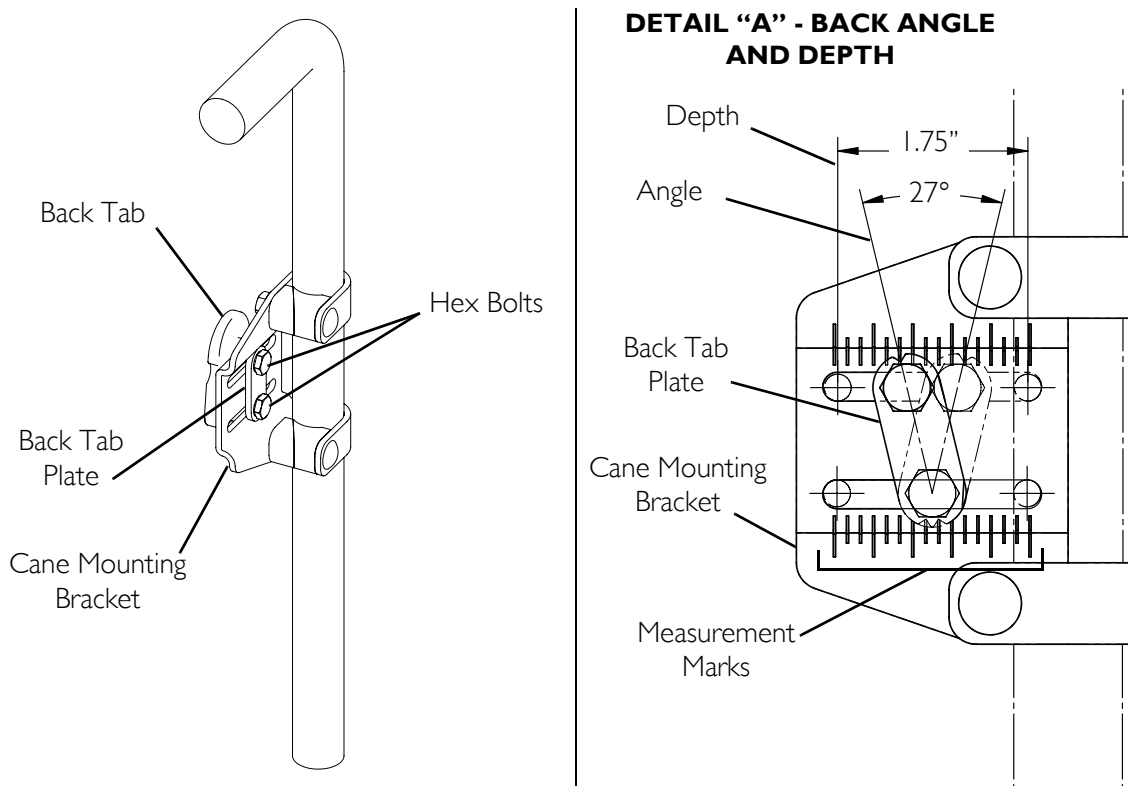
FIGURE 3.1 Adjusting the Propel Back Depth and Angle

SECTION 4—MAINTENANCE

⚠ WARNING

Check **ALL** fasteners periodically to ensure that mechanical connections and attaching hardware are tightened securely - otherwise injury or damage may occur.

DO NOT continue to use this product if problems are discovered. Corrective maintenance can be performed at or arranged through your equipment supplier.

Maintenance **MUST** be performed **ONLY** by qualified personnel.

Replacing Upholstery

NOTE: For this procedure, refer to FIGURE 4.1.

1. Remove the back cushion from the back frame. Refer to [Installing/Removing the Propel Back Cushion](#) on page 13.
2. Unzip the upholstery on the back of the cushion.
3. Remove the foam from the upholstery.
4. Install the foam into new upholstery.
5. Zip the new upholstery.
6. Install the cushion to the back frame. Refer to [Installing/Removing the Propel Back Cushion](#) on page 13.

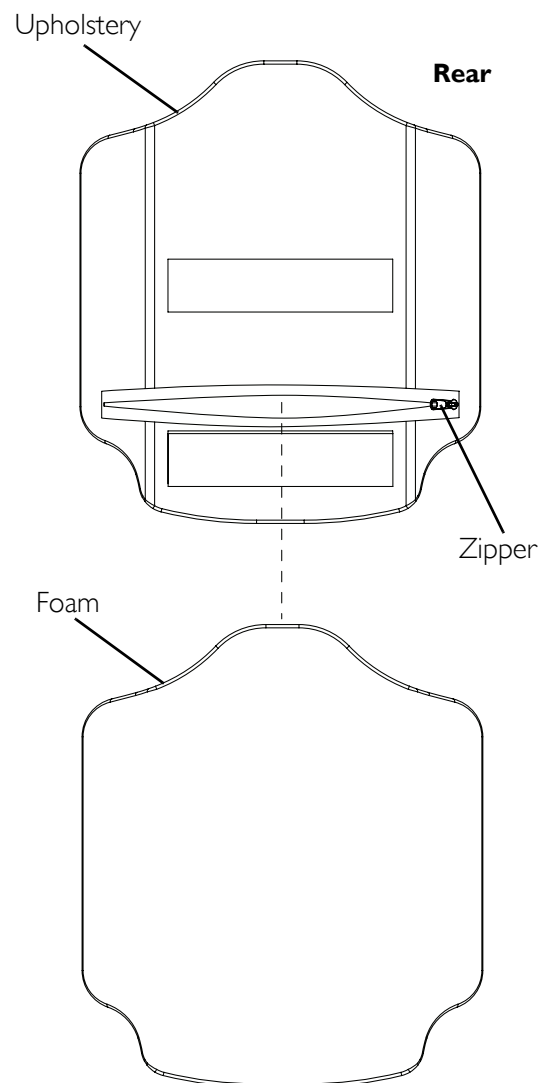
FIGURE 4.1 Replacing Upholstery - Replacing Cushion

Replacing Cushion

NOTE: For this procedure, refer to FIGURE 4.1.

1. Remove the back cushion from the back frame. Refer to [Installing/Removing the Propel Back Cushion](#) on page 13.
2. Unzip the upholstery on the back of the cushion.
3. Remove the foam from the upholstery.
4. Install the new cushion into the cover.
5. Zip the cover.
6. Install the back cushion to the back frame. Refer to [Installing/Removing the Propel Back Cushion](#) on page 13.

Cleaning Instructions

Back Assembly

Clean weekly with dampened cloth. Wipe entire surface.

Weekly visual inspection of parts including hardware, brackets, upholstery materials, foams (if accessible), and plastics, for deformation, corrosion, breakage, wear or compression.

Cover

CAUTION

Machine wash cold. Gentle cycle. Mild detergent. NO fabric softeners or bleach. DO NOT machine dry. Air dry ONLY out of sun.

Foam

Replace back foam if it becomes contaminated because of incontinence. The back foam is absorbent and cannot be cleaned.

⚠ WARNING

The cover is designed to protect the foam against a user's incontinence and to provide fire retardant so the cushion must not be used without its cover. If the cover is torn, it must be replaced.

Continued use with absent or damaged cover by an incontinent user can result in severe skin irritation, infection, and other skin care issues.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for a period of two years from date of purchase. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address below, provide dealer's name, address, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, COMMERCIAL OR INSTITUTIONAL USE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THESE INSTRUCTIONS.

THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESSED OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2006 Invacare Corporation
Invacare, the Medallion Design, and Yes, You Can, are registered trademarks of Invacare Corporation.
InTouch and Propel are trademarks of Invacare Corporation.
Invacare Corporation www.invacare.com