Owner's Manual  
for model: **SSN-414W**

# Automatic Wrist Blood Pressure Monitor with Heart Sense®

Made in Taiwan

## **Walgreens c/o SAI**

105 CHALLENGER ROAD, 6FL  
RIDGEFIELD PARK, NJ 07660

## **Toll Free Customer Service**

1.800.261.0130

- Large Three Row Digital Display
- 3 Zones of 40 Memory - 120 Memory Total
- Time and Date
- Memory Average of Last 3 Measurements
- Pulse Display

Distributor: Walgreen Co.  
Deerfield, IL 60015

Heart Sense® and Healthy Living® are registered trademarks used under license from Samsung America, Inc.

## In This Manual

Important Warnings/Medical Disclaimer	1
Healthy Living®	2-3
Commonly Asked Questions	4-8
Intended Use	8
Accuracy	9
Measurement Method	9-10
Special Conditions	10
Special Features	11
Main Unit and Accessories	12
Digital Display Symbols	13
Battery Installation	13
Selecting Time and Date	14
Important Reminders	15-16
Applying the Cuff	17
Taking a Blood Pressure Measurement	18-20
Storing Measurements	20
Recalling Stored Measurements	21
Reset Function	22
Taking Care of Your Monitor	23-24
Troubleshooting	25-26
Limited Lifetime Warranty	27-29
Specifications	30
Blood Pressure Diary	31-33

## Important Warnings/Medical Disclaimer

Before using your blood pressure monitor, you should read and understand all instructions and follow all warnings.

Information in this manual is provided for informational purposes only. This manual and product are not meant to be a substitute for the advice provided by your own physician or other medical professional. You should not use the information contained herein or this product for diagnosing or treating a health problem, disease, or prescribing any medication. If you have or suspect that you have a medical problem, promptly contact your healthcare provider.

## Healthy Living®

This blood pressure monitor is the first step to living a healthier life. But monitoring your blood pressure at home is just the beginning, It's also important to:

**EXERCISE REGULARLY** - Start out by taking a short walk each day. Before you know it, you'll feel energized. That's because along with being an easy way to shed a few pounds, walking also helps your bones and heart become stronger.

**EAT RIGHT** - Remember, you are what you eat! So when you reach for a snack, try grabbing a piece of fruit or a vegetable. When cooking meals at home try to use fats and oils sparingly. Sometimes a healthy diet is the best medicine of all! It's also a good idea to monitor your sodium intake because about one-third of individuals who have high blood pressure are also sodium sensitive.

## Healthy Living®

**BE HAPPY!** - A positive attitude is a big part of a healthy body. Try not to get too stressed out over things. Daily relaxation techniques such as gentle yoga or even quiet time by yourself can do a lot of good for the mind and body. Stay focused on the good things in life. Begin by congratulating yourself for that short walk you just took or the smart food choice you made. It's your first step to a healthier life!

**KEEP AN EYE ON YOUR WEIGHT!** - Surprisingly, losing even a small amount of weight can often times help control high blood pressure. Talk with your doctor to find a "healthy" weight for you and strive to come within a range of it.

**Remember** to consult your physician prior to beginning an exercise program, vitamin/mineral/dietary changes or programs and blood pressure monitoring.

## Commonly Asked Questions

### **Why Should I Monitor My Blood Pressure At Home?**


By monitoring at home, you may manage your health in between doctor visits. Home monitoring also provides you with a record of blood pressure measurements over time to give to your doctor - a valuable tool in monitoring hypertension.

Home monitoring may also help alleviate *White Coat Hypertension*. This term refers to the medical phenomena which causes many individuals' blood pressure to be higher when taken in a medical office setting. That's because many times the stress of being in the doctor's office may cause blood pressure to rise as much as 20-50 points higher than normal. When monitoring at home, you may get a truer indication of your blood pressure in a normal, relaxed state.

### **What Is Blood Pressure?**


The commonly used term "blood pressure" refers to the force or *pressure* that is exerted on the body's arteries as blood flows through them.

Each time the heart beats or *contracts*, it produces pressure in the arteries. This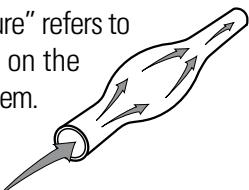

## Commonly Asked Questions



blood pressure is what moves blood through the body, supplying oxygen and nutrients to every organ.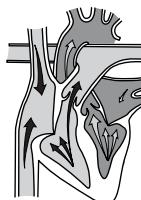

When the heart contracts it sends blood out into the body. This force of blood within the arteries is called **systolic** pressure.

The pressure that remains as the heart relaxes between beats is called **diastolic** pressure .


Blood pressure is recorded as systolic over diastolic, as shown here.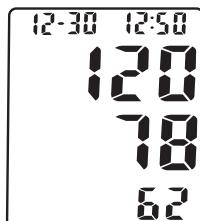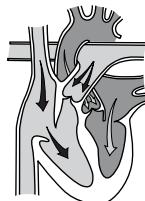

## Commonly Asked Questions

### ***What Is Considered Normal Blood Pressure?***

The well-respected World Health Organization (WHO) has set forth the following guidelines regarding high (commonly referred to as hypertension), low and borderline blood pressure.

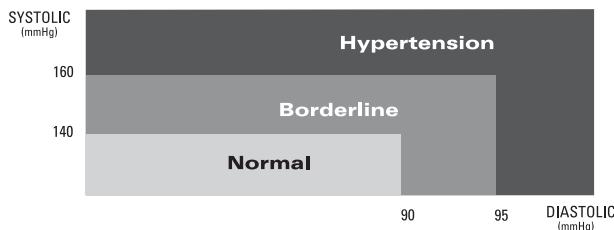### ***Why Does My Blood Pressure Fluctuate?***

Because blood pressure changes with every beat of the heart it is in constant fluctuation throughout any given day. In addition to these natural changes there are other factors that may affect blood pressure such as:

## Commonly Asked Questions

- Temperature (too hot or cold)
- Humidity
- Restlessness
- Lack of sleep
- Physical exercise
- Constipation
- Stress
- Fear
- Anger
- Anxiety
- Food and beverage consumption
- Individual physical conditions

In addition to these factors you should talk with your physician to see what may be affecting you.

### ***How Do I Know I Have High Blood Pressure?***

It's nearly impossible to know if your blood pressure is elevated without having it checked. People have the common misconception that one can "feel" their blood pressure rise. It is because of this mystery that hypertension has been named "the silent killer." Don't let it sneak up on you! Start monitoring your blood pressure every day with your new Walgreens monitor.

## Commonly Asked Questions

### **Why Is It Important To Control My Blood Pressure?**

Left uncontrolled, hypertension can place an incredible amount of stress on your heart and arteries, both of which become forced to work harder to keep blood flowing in a healthy manner throughout the body. Over time, this stress can result in health problems including over-enlargement of the heart, loss of elasticity of the arteries, heart disease, stroke and even death. Making some simple lifestyle changes now can help you to stay healthy in the future.

### Intended Use

Walgreens blood pressure monitors are intended for home use. They are noninvasive, meaning no part of the monitor enters your body, and provide systolic, diastolic and pulse measurements. Walgreens monitors are recommended for use by people over the age of 18 and are not suitable for clinical use.

## Accuracy

Walgreens monitors have been clinically tested against a scientific device called a *sphygmomanometer*, considered the gold standard in blood pressure measurement. All Walgreens monitors have performed equivalent to measurements taken with this scientific device and are within the accuracy limits prescribed by the American National Standard for Electronic or Automated Sphygmomanometers.

### Measurement Method

Walgreens monitors measure blood pressure and pulse by using what's called an *oscillometric* method, meaning they measure the fluctuations in pressure. This is done by first wrapping a fitted cuff around the upper arm (or wrist, depending on the model). The monitor then automatically fills the cuff with air creating pressure around the arteries inside the wrist. Mounted inside of the cuff is a special gauge that is capable of sensing the small *oscillations*.

## Measurement Method

(fluctuations) in pressure. These fluctuations are produced as a result of the pressure the cuff has placed around the arm and are, in a basic sense, the arteries pulsing (getting larger and smaller) with each beat. The monitor then measures how high and how quickly these pulses occur and converts that information into a digital value. A special valve, called an *electrode*, deflates the monitor automatically when the measurement is complete.

## Special Conditions

All Walgreens blood pressure monitors are safe and accurate to use with arrhythmias such as atrial or ventricular premature beats and atrial fibrillation.

## Special Features

This Walgreens monitor with Heart Sense® technology senses your blood pressure before the measurement is complete. The monitor then adjusts the air pressure amount according to your individual level. The results are quick, comfortable measurements each time you check your blood pressure.

### **Memory Feature**

Your Walgreens Automatic Wrist monitor features a 3 party memory capability or 3 "zones" - each zone stores 40 measurements for a total storage capacity of 120 measurements. This unique feature allows 3 users to use and store their blood pressure readings separately all on one monitor!

### **Time and Date**

This monitor automatically keeps track of the time and date remains on when the monitor is not in use for easy reference.

### **Large Three Row Display**


A large LCD displays the time, date and your blood pressure - systolic over diastolic - to make measuring your blood pressure easy to read.

## Main Unit and Accessories

### Main Unit


### Accessories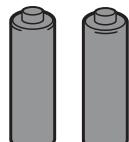

2 "AAA" Alkaline  
Batteries  
Storage Case

## Digital Display Symbols
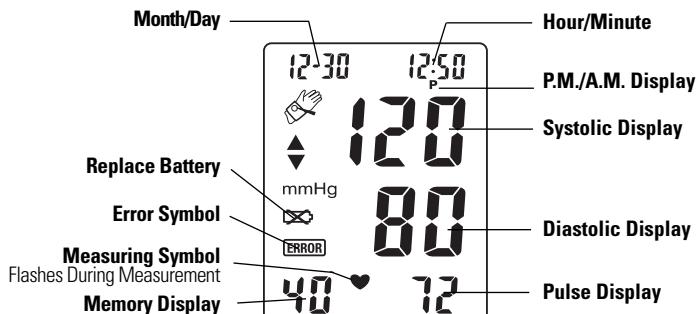
## Battery Installation

When the display shows ☒, it's time to replace the batteries. This monitor uses 2 AAA batteries.

Insert the batteries into the monitor making certain to align the polarities correctly.

Close the battery cover securely. It will snap into place when it is fully closed.

\* It's a good idea to remove the batteries if you know you will not be using your batteries for an extended amount of time.

## Selecting Time and Date

Your Walgreens monitor will keep track of the time and date unless the batteries have run down or are removed from the monitor.

- Press the MODE button on the unit. The month will begin to flash. Press the ▲ (Up) or ▼ (Down) button until the correct month appears on the display.
- Press the MODE button once again to store and now the day will begin to flash. Press the ▲ (Up) or ▼ (Down) button until the correct day appears on the display.
- Press the MODE button to store and the hour will begin to flash. Follow the same process as setting the month and day to set the hour and minute.
- Once you have selected the minute, press the MODE button. The unit will now display the time and date.

## Important Reminders

Here are some important reminders:

### ***Before Taking a Measurement***

- ✓ Sit quietly for about 10 minutes. This will allow your body to return to its normal, resting state.
- ✓ Take your blood pressure in a comfortable environment - not too cold, not too hot-as temperature may affect your measurement.
- ✓ If you've just consumed beverages containing caffeine such as coffee, non-herbal tea, or cola wait 30-45 minutes. Also, don't smoke just before taking a measurement.

## Important Reminders

### ***While Taking a Measurement***

- ✓ Sit with your legs uncrossed and your feet flat on the floor.
- ✓ For consistency, it's a good idea to measure you blood pressure using the same arm and at about the same time each day. If possible, we suggest using the left arm.
- ✓ Relax. Sit still and quietly while measuring. Talking or moving may elevate measurements. Do not touch the cuff or monitor during the measurement.
- ✓ When taking multiple measurement right after each other, make to wait at least 8-10 minutes in between. Waiting allows your blood vessels to return to their normal state.

## Applying the Cuff

1. Before taking a measurement, read the Important Reminders on pages 15-16.

2. If you are wearing a long sleeved shirt, roll up the sleeve, but make sure that it is not constricting circulation to your arm. Remove any jewelry or watches.

3. With your palm facing up, place the monitor on your wrist. Secure the cuff around your wrist using the velcro so it fits securely but is not too tight.

4. Place your arm on a table. The monitor should be facing you. It is important to make sure that the wrist cuff is at the same level as your heart.

## Taking a Blood Pressure Measurement

1. Once the cuff is properly applied and you are seated with your feet flat on the floor you are ready to take a measurement.

2. Press the ▲ (Up) or ▼ (Down) button and you will see a number appear: 1, 2, or 3. These are each of the 3 party memory zones explained on page 9 (Special Features).

\*If you select zone No. 1 your blood pressure will be recorded as one of the 40 readings available for that zone. The same is true for zones 2 and 3. When taking measurements, be sure and select the correct zone number so that the measurement is stored in the appropriate zone.

3. Press the START/STOP button. The monitor will begin to inflate to the level that's right for you. An upwards facing arrow on the digital display indicates the monitor is inflating. Remain still and do not talk during the measurement process.

## Taking a Blood Pressure Measurement

4. You will feel the cuff get tighter around your wrist - this is normal. It should not inflate to the point of discomfort. If this occurs, the cuff may be secured too tightly. To stop the inflation, press START/STOP. The cuff will deflate automatically. Rewrap the cuff allowing more room between your wrist and the cuff.

5. Inflation will stop automatically to begin the measurement process. You will see a downward facing arrow and the numbers decreasing as the cuff is deflating. You will also see a flashing heart to indicate the measurement is taking place.

6. When your measurement is complete the monitor will deflate automatically, releasing the air pressure in the cuff.

7. The entire measurement process should take less than a minute. Your blood pressure and pulse readings are displayed and automatically stored in the memory.

\* The monitor will automatically power off after about one minute.

## Taking a Blood Pressure Measurement

**NOTE:** Blood pressure constantly changes during the course of a day. Occasionally, high or low readings do not necessarily mean your monitor is malfunctioning or that you are ill. It is important to speak with your doctor to learn what your normal blood pressure is if you experience abnormal measurements.

## Storing Measurements

When you take a measurement the monitor automatically stores it in memory space number one. As you continue to take measurements the monitor will push the previous measurement back one space. In other words, the value in space one will always be the most recent blood pressure measurement taken.

Your monitor features a 3 party memory capability or 3 "zones" - each zone stores 40 measurements for a total memory capacity of 120 readings. This unique feature allows 3 users to use and store their blood pressure measurements separately all on one monitor. This feature is also useful for one user to track their blood pressure throughout the day: morning, afternoon, and evening.

## Recalling Stored Measurements

- 1.Press the ▲ (Up) or ▼ (Down) button to select a user zone. You will see a number appear: 1, 2, or 3. (Fig. 1)
- 2.Press the (M) MEMORY button and the average of the last 3 readings will be displayed for that zone. For example, if you select zone No. 1 and you press MEMORY, the average of the last 3 measurements stored in zone No. 1 will appear on the display.
- 3.Press ▼ (Down) to view the next stored measurement; press ▲ (Up) to go back to the previous measurement.
- 4.To go back to the main screen for "Recalling Stored Measurements" and to choose another zone, press ▲ (Up) to scroll through the measurements until you reach the most recent measurement and press once more. You can also press START/STOP to return to the main screen.
- 5.Follow steps 1-3 once again to review measurements in another zone.

\*Fig.1

\*Fig.2

## Reset Function

Your Walgreens monitor will not lose the data stored in the memory even when you change the batteries. To erase all of the data in memory zone, refer to the instructions below.

Note: Single measurements cannot be erased.

Warning: Resetting the memory will permanently erase all of the stored data in the memory zone that you select.

- Press ▲ (Up) ▼ (Down) to choose a zone: 1, 2, or 3.
- Press the (M) MEMORY button. (Fig. 3)
- Press the MODE and ▲ (Up) button at the same time to clear all of the stored data. (Fig. 4)

\*Fig.3

\*Fig.4

## Taking Care of Your Monitor

It's important to treat your monitor with care. Here are some basic tips:

- ✓ Keep the monitor out of direct sunlight, high humidity, and extreme temperature changes.

- ✓ Try not to drop it on the floor.
- ✓ Do not take the monitor or the cuff apart. Do not attempt to fix any of the monitor's parts at home. If your monitor needs repair, see details in warranty section.
- ✓ Keep the monitor free of dust and other materials. Avoid getting any part of the monitor or cuff extremely wet. If the monitor needs cleaning it's best to wipe it with a cloth slightly dampened with water. We recommend that you clean your monitor about once a month.

## Taking Care of Your Monitor

- ✓ For your safety, do not wrap the cuff around any other body part other than your wrist.
- ✓ Do not harshly fold or bend the cuff as this may damage it on the inside.
- ✓ Do not inflate the monitor's cuff if it is not wrapped around your wrist.

## Troubleshooting

Error/Symbol	What Does It Mean?	How To Fix It
<b>ERROR</b>	Monitor may have been inflated without the cuff around the wrist <i>OR</i> Cuff may not be positioned correctly on the wrist.	Read page 17 (Applying the Cuff).
	You may have moved while the cuff was inflating, it is now attempting to reinflate. Check the cuff to make sure it's positioned correctly.	Read pages 15 & 16 (Important Reminders).
	You may have moved or spoken during the measurement process.	Remember that it is important not to move and speak as it may elevate measurements.
	You may have been startled by a noise or vibration while measuring.	Make sure you are in a quiet and comfortable place when taking a measurement.

## Troubleshooting

Error/Symbol	What Does It Mean?	How To Fix It
	You have been ignoring the flashing replace battery symbol (  ).	Replace with 2 new "AAA" alkaline batteries. Make sure the polarities are positioned correctly in the battery compartment before continuing to measure your blood pressure.
	The batteries have run down.	Replace with 2 new "AAA" alkaline batteries so you may continue to measure your blood pressure without any interruptions.
Blood pressure value seems too high or too low	Cuff may be positioned too high or too low on the wrist.	Read page 17 (Applying the Cuff).
The monitor keeps reinflating	You may have moved during inflation <i>OR</i> If you have a very weak pulse the monitor may continue to reinflate until it senses your pulse.	Relax for a few minutes then take a measurement.
Unable to take a measurement after receiving an error symbol.	The cuff may not be positioned correctly.	Read page 17 (Applying the Cuff).

## Limited Lifetime Warranty

Manufacturer warrants to the original purchaser that this Auto Wrist Blood Pressure Monitor (the "Product") will be free from defects in material and workmanship for the life of the original owner under normal use. This Product is for home use only and is not intended for commercial or clinical use. If the Product does not work properly because of defects in material or workmanship, Manufacturer will, at its option, repair or replace the Product, including the cuff, with the same or a comparable model free of charge (excluding shipping & insurance charges). This warranty only extends to the original retail purchaser. Purchase receipt or other proof of date of original purchase is required before warranty performance.

This warranty extends solely to failures due to defects in materials or workmanship occurring during normal use. It does not cover normal wear of the Product. This warranty is void if: the Product housing has been removed, the Product's label, logo or serial number have been removed or if the Product fails to function properly as a result of accident, misuse, abuse, neglect, mishandling, misapplication, defective batteries, faulty installation, setup, adjustments, improper maintenance, alteration, maladjustment of controls, modification, power surges, commercial or clinical use of the product, service by anyone other than an authorized service center or acts beyond the manufacturer's or distributor's control.

## Limited Lifetime Warranty

The warranty and remedy provided herein are exclusive and in lieu of all other express and implied warranties and unless stated herein, any statements or representations made by any other person or entity are void. NEITHER THE MANUFACTURER, DISTRIBUTOR NOR THEIR AFFILIATES SHALL BE LIABLE FOR ANY LOSS, INCONVENIENCE, INJURY, OR DAMAGE INCLUDING DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES RESULTING FROM THE USE, IMPROPER USE, FAILURE TO HEED ANY WARNING, DISCLAIMER OR INSTRUCTION OR INABILITY TO USE THIS PRODUCT, WHETHER ARISING OUT OF ANY BREACH OF THIS WARRANTY OR ANY OTHER LEGAL THEORY. ALL EXPRESS AND IMPLIED WARRANTIES, INCLUDING THE WARRANTIES OF MERCHANTABILITY, ARE LIMITED TO THE APPLICABLE WARRANTY PERIOD SET FORTH ABOVE. Manufacturer and Distributor assume no responsibility for injuries, damages or penalties incurred or suffered resulting from the use of this product in an improper manner or manner or location other than for that for which it is intended or from failure to heed any warning, disclaimer or instruction.

## Limited Lifetime Warranty

Some states do not allow the exclusion or limitation of incidental or consequential damages, or limitations on how long an implied warranty lasts, so the above exclusions or limitations may not apply to you. This warranty gives your specific legal rights and you may also have other rights which vary from state to state.

Should your Product prove defective under this warranty, please return the Product in the original carton prepaid to:

**Walgreens c/o SAI  
105 Challenger Road, 6th Floor  
Ridgefield Park, NJ 07660**

**Your dated register receipt or other proof of purchase is required along with your returned unit.** Also include your name, address, daytime phone and detailed description of the problem. Sorry we cannot deliver to PO Boxes.

Include US\$8.95 check or money order (NON-REFUNDABLE) for return shipping & insurance within the continental U.S. This warranty applies only to products purchased in the U.S. For products purchased in, but used outside, the U.S., this warranty covers only warranty services within the U.S. (and does not include shipping outside the U.S.).

# Specifications

<b>Model</b>	<b>SSN-414W</b>	
<b>Monitor Type</b>	Wrist cuff	
<b>Measurement Method</b>	Oscillometric	
<b>Display</b>	Liquid crystal digital display	
<b>Measurement Range</b>	Pressure:	0-300 mm Hg
	Pulse:	40-199 beats/Minute
<b>Accuracy</b>	Pressure:	within ± 3mm Hg
	Pulse:	within ± 5%
<b>Memory</b>	Stores up to 40 measurements for each zone (120 memory total)	
<b>Automatic Power Off</b>	About 2 minutes after measurement is complete	
<b>Battery Type</b>	2 AAA batteries (Alkaline)	
<b>Battery Life</b>	About 6 months when used once a day	
<b>Storage Temperature</b>	Temperature:	-4°F to 140°F
	Relative Humidity:	less than 95%
<b>Operating Temperature</b>	Temperature:	50°F to 104°F
	Relative Humidity:	less than 95%
<b>Dimensions (approx.)</b>	Monitor: $3\frac{1}{4}$ " (w) x $2\frac{1}{2}$ " (d) x $1\frac{3}{8}$ "(h) Cuff: fits wrists 5"- 8" around	
<b>Weight (approx.)</b>	5 oz. (including batteries)	

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