User Manual

DEALER: This manual MUST be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

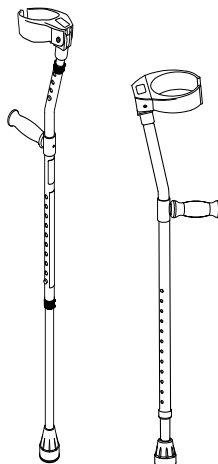
Yes, you can.

Elbow Crutches

Model	Size	Type
8153-J	Junior	Double Adjust
8153-A	Adult	
8153-T	Tall	
124-J	Junior	Single Adjust
124-A	Adult	
124-T	Tall	

FR	Manuel d'utilisation.....	page 9
NL	Gebruikershandleiding.....	pagina 15
ES	Manual de usuario.....	página 23
PT	Manual do utilizador.....	pagina 29
IT	Manuale per l'utente.....	pagina 37

I General

I.1 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING

Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

Symbol used on the product to indicate weight limitation when used as a pair.

This product complies with Directive 93/42/EEC concerning medical devices.

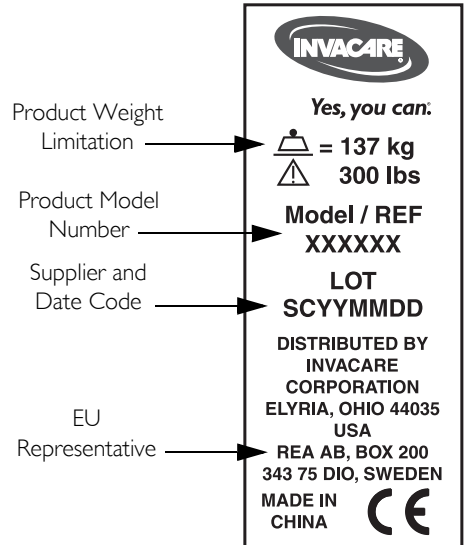
Gives useful tips, recommendations and information for efficient, trouble-free use.

I.2 Life

The expected life of the product listed in this manual when used in accordance with the safety instructions, maintenance intervals and care instructions is two years. Maintenance should be done at least every six months.

I.3 Product Labeling

2 Safety

Check ALL parts for shipping damage. If shipping damage is noted, DO NOT use. Contact carrier/ dealer for further instruction.

2.1 Intended Use

The crutches are designed for persons with limited walking ability and serve to increase their ability to walk. The crutches can be used inside and outside the house.

2.2 General Warnings

WARNING

DO NOT USE THIS PRODUCT OR ANY AVAILABLE OPTIONAL EQUIPMENT WITHOUT FIRST COMPLETELY READING AND UNDERSTANDING THESE INSTRUCTIONS AND ANY ADDITIONAL INSTRUCTIONAL MATERIAL SUCH AS OWNER'S MANUALS, SERVICE MANUALS OR INSTRUCTION SHEETS SUPPLIED WITH THIS PRODUCT OR OPTIONAL EQUIPMENT. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS OR INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, INJURY OR DAMAGE MAY OCCUR.

ACCESSORIES WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

WARNING

Each individual should ALWAYS consult with their physician or therapist to determine proper adjustment and usage. A physical/occupational therapist should assist in the height adjustment for maximum support.

The cuffs are designed to add stability to the elbow crutch and are not intended to support the users weight.

Care should be taken to ensure that all hand and height adjustments are secure before using this or any mobility aid.

If hand grips are exposed to extreme temperature [above 100°F (37°C) or below 32°F (0°C)], high humidity and/or becomes wet, prior to use, ensure hand grips DO NOT twist on the crutch handle - otherwise damage or injury may occur.

Before use, ensure that all hardware is tightened securely.

If exposed to extreme temperature [above 106°F (41°C)], the crutch will be hot and first degree burns may occur. Allow crutch to cool before using.

Exercise extra care when walking on a slippery or wet surface.

3 Technical Specifications

WARNING

ALWAYS observe the weight limit on the labeling of your product. Check that all labels are present and legible. Replace if necessary.

Model	Patient Height (Min - Max)	Support Angle	Leg and Arm Height Adjustment Range (Min - Max)	Weight Limitation
8153-J	4' 5" - 5' 2" (135 cm - 157.5 cm)	10°	Leg 23" - 32" (58.4 cm - 81.3 cm) Arm 10.5" - 13.5" (26.7 cm - 34.3 cm)	300 lb (137 kg)*
8153-A	5' 2" - 5' 11" (157.5 cm - 180 cm)	10°	Leg 29" - 38" (73.7 cm - 96.5 cm) Arm 10.5" - 13.5" (26.7 cm - 34.3 cm)	300 lb (137 kg)*
8153-T	5' 11" - 6' 8" (180 cm - 202.5 cm)	10°	Leg 32" - 41" (81.3 cm - 104.1 cm) Arm 11" - 14" (27.9 cm - 35.6 cm)	300 lb (137 kg)*
124-J	4' 5" - 5' 2" (135 cm - 157.5 cm)	20°	Leg 21" - 31" (54.0 cm - 79.5 cm)	220 lb (100 kg)
124-A	5' 2" - 5' 11" (157.5 cm - 180 cm)	20°	Leg 25" - 36" (63.0 cm - 91.0 cm)	350 lb (160 kg)
124-T	5' 11" - 6' 8" (180 cm - 202.5 cm)	20°	Leg 31" - 43" (80.0 cm - 108.0 cm)	350 lb (160 kg)

*Weight limitation is for crutches used as a pair.
Not all models are available in every region.

4 Usage

4.1 Leg Height Adjustment

1. Loosen the leg collar (8153 models only).
2. Depress the snap button.
3. Slide the leg up or down until the desired height adjustment is achieved. The handgrip should normally be positioned at wrist height. Consult a therapist for height adjustment information.
4. Tighten the leg collar (8153 models only).

WARNING

Ensure that the snap buttons fully protrude through the height adjustment hole, otherwise injury or damage may occur. An audible “click” will be heard when the snap buttons engage in the height adjustment holes.

4.2 Cuff Height Adjustment

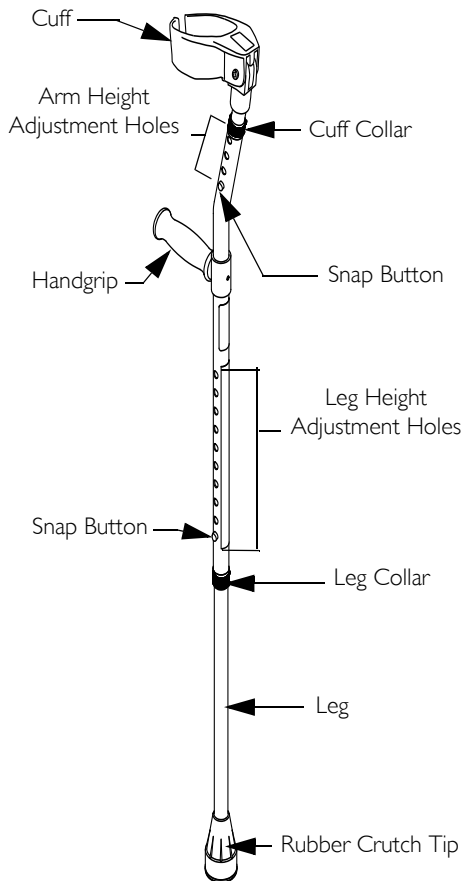
This procedure applies to models 8153-J, 8153-A and 8153-T only.

1. Loosen the cuff collar.
2. Depress the snap button.
3. Slide the cuff up or down until the desired height adjustment is achieved. The handgrip should normally be positioned at wrist height. Consult a therapist for height adjustment information.
4. Tighten the cuff collar.

WARNING

Ensure that the snap button fully protrudes through the height adjustment hole, otherwise injury or damage may occur. An audible “click” will be heard when the snap button engages in the height adjustment hole.

Models 8153-J, 8153-A and 8153-T shown.

5 After Use

5.1 Care and Maintenance

DO NOT use an abrasive detergent, cleaner or cloth to clean or dry the elbow crutches.

1. Regular cleaning of the elbow crutches is recommended. Wipe all surfaces with a damp cloth and allow all surfaces to dry before use.
2. Regular inspection of parts including hardware, brackets, and plastics, for deformation, corrosion, breakage, wear or compression is recommended. Replace the crutches if any of these conditions exist.
3. If hand grips are loose, DO NOT use the crutches. If rubber crutch tips are worn or cracked, replace the crutch tips IMMEDIATELY.

5.2 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly.

Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

5.3 Reuse

WARNING: Risk of Injury

Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, DO NOT use the product. Otherwise injury may occur.

6 Limited Warranty - North America

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product to be free from defects in materials and workmanship for three years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back cover. Provide dealer's name, address, model number, the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. DO NOT return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE. THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER. THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

7 Customer Service and Warranty Information - Europe

United Kingdom	Invacare Limited, Pencoed Technology Park, Pencoed, Bridgend CF35 5HZ Tel: (44) (0) 1656 776 222 Fax: (44) (0) 1656 776 220 www.invacare.co.uk, UK@invacare.com
Ireland	Invacare Ireland Ltd, Unit 5 Seatown Business Campus, Seatown Road, Swords, County Dublin - Ireland Tel: (353) 1 810 7084 Fax: (353) 1 810 7085 www.invacare.co.uk, ireland@invacare.com
France	Invacare France Opération, Route de Saint Roch, 37230 Fondettes Tel: (33) (0) 2 47 62 64 66 Fax: (33) (0) 2 47 42 12 24 www.invacare.fr, contactfr@invacare.com
Belgium & Luxembourg	Invacare nv, Autobaan 22, B-8210 Loppem, Brügge Tel: (32) (0) 50 83 10 10 Fax: (32) (0) 50 83 10 11 www.invacare.be, belgium@invacare.com
Switzerland	Invacare AG, Benkenstrasse 260, CH-4108 Witterswil Tel: (41) (0) 61 487 70 80 Fax: (41) (0) 61 487 70 81 switzerland@invacare.com
Deutschland	Invacare Aquatec GmbH, Alemannenstraße 10, D-88316 Isny Tel: (49) (0) 75 62 7 00 0 Fax: (49) (0) 75 62 7 00 66 www.invacare-aquatec.de, info@invacare-aquatec.com
Portugal	Invacare Lda, Rua Estrada Velha, 949, P-4465-784, Leça do Balio Tel.: (351) (0)225 1059 46/47 Fax: (351) (0)225 1057 39 portugal@invacare.com
Italia	Invacare Mecc San s.r.l., Via dei Pini 62, I-3016 Thiene (VI) Tel: (39) 0445 38 00 59 Fax: (39) 0445 38 00 34 www.invacare.it, italia@invacare.com
España	Invacare SA, c/Areny s/n, Polígon Industrial de Celrà, E-17460 Celrà (Girona) Tel: (34) (0) 972 49 32 00 Fax: (34) (0) 972 49 32 20 www.invacare.es, contactsp@invacare.com
Nederland	Invacare BV, Celsiusstraat 46, NL-6716 BZ Ede Tel: (31) (0)318 695 757 Fax: (31) (0)318 695 758 www.invacare.nl, nederland@invacare.com, cseede@invacare.com

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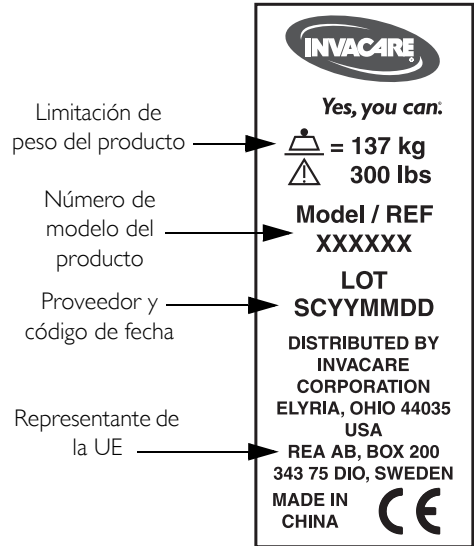
The diagram shows a rectangular product label with the following text and symbols:

- At the top: **INVACARE** logo.
- Below the logo: **Yes, you can:**
- Weight information:  = 137 kg and  300 lbs.
- Model information: **Model / REF XXXXXX**
- Lot information: **LOT SCYYMMDD**
- Distributor information: **DISTRIBUTED BY INVACARE CORPORATION ELYRIA, OHIO 44035 USA**
- Manufacturer information: **REA AB, BOX 200 343 75 DIO, SWEDEN**
- Origin: **MADE IN CHINA**
- CE mark: 

Arrows from the left point to the following labels:

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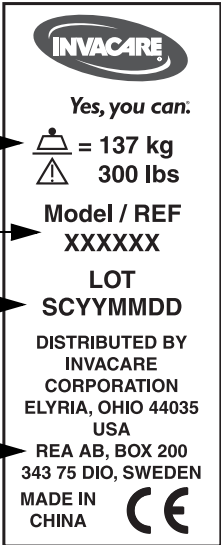
The image shows a rectangular product label for INVACARE. At the top left is the INVACARE logo. Below it, the text reads "Yes, you can:". To the left of the label, four lines of text with arrows point to specific parts of the label: "Limite di peso del prodotto" points to the weight limit, "Numero di modello del prodotto" points to the model number, "Fornitore e data di produzione" points to the lot number, and "Distributore per la UE" points to the distributor information. The label itself contains the following text: "Model / REF XXXXXX", "LOT SCYYMMDD", "DISTRIBUTED BY INVACARE CORPORATION ELYRIA, OHIO 44035 USA", "REA AB, BOX 200 343 75 DIO, SWEDEN", "MADE IN CHINA", and the CE mark.

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Distributore per la UE → **DISTRIBUTED BY
INVACARE
CORPORATION
ELYRIA, OHIO 44035
USA
REA AB, BOX 200
343 75 DIO, SWEDEN
MADE IN CHINA** 

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© 2011 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by [™] and [®]. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Invacare Rea AB