TOPCOM[®]

BPM ARM 1900f

USER GUIDE
HANDLEIDING
MODE D'EMPLOI
BEDIENUNGSANLEITUNG
ANVÄNDARHANDBOK
BRUGSANVISNING
ANVÄNDARHANDBOK
KÄYTTÖOHJE
MANUAL DE USUARIO

V 2.1

0366

Exclusively produced for Topcom Europe NV,
Grauwmeer 17, 3001 Heverlee, Belgium
by Rossmax Int'l Ltd.
N° 6018 Huyi Rd., Waigang, Jiading District,
Shanghai China 201806

This product is in compliance with the essential requirements and other relevant provisions of the
directive 93/42/EEC.

The declaration of conformity can be found on:
<http://www.topcom.net/support/cedclarations.php>

- GB** The features described in this manual are published with reservation to modifications.
- NL** De in deze handleiding beschreven mogelijkheden worden gepubliceerd onder voorbehoud van wijzigingen.
- F** Les possibilités décrites dans ce manuel sont publiées sous réserve de modifications.
- D** Die in dieser Bedienungsanleitung umschriebenen Möglichkeiten, werden vorbehaltlich Änderungen publiziert.
- S** Funktionerna i denna bruksanvisning publiceras med reservation för ändringar.
- DK** Vi forbeholder os retten til ændringer af de specifikationer, der er beskrevet i denne brugsanvisning.
- N** Funksjoner beskrevet i denne manualen kan endres uten nærmere informasjon.
- SF** Tässä ohjekirjassa julkaissut tiedot voivat muuttua ilman ennakoilmoitusta.
- ES** Las características descritas en este manual pueden ser objeto de futuras modificaciones.

| | | |
|-----------|--|-----------|
| 1 | Introduction | 4 |
| 2 | Safety instructions | 4 |
| 3 | Technical specifications | 5 |
| 4 | Helpfull tips | 5 |
| 5 | Blood pressure | 6 |
| 5.1 | What is Blood Pressure? | 6 |
| 5.2 | Why Measure Your Blood Pressure? | 6 |
| 5.3 | Blood Pressure Standard | 6 |
| 5.4 | Blood Pressure Fluctuation | 7 |
| 6 | Battery installation | 7 |
| 7 | Parts & buttons description | 8 |
| 8 | Display | 8 |
| 9 | Applying the cuff | 9 |
| 10 | Measurement | 10 |
| 10.1 | Procedure | 10 |
| 11 | Recalling memorized results | 11 |
| 12 | Power adapter (optional) | 11 |
| 13 | Topcom Warranty | 12 |
| 13.1 | Warranty period | 12 |
| 13.2 | Warranty handling | 12 |
| 13.3 | Warranty exclusions | 12 |

The CE symbol indicates that the unit complies with the essential requirements of the directive 93/42/EEC and 89/336/EEC.

1 Introduction

Congratulations with your purchase of the Topcom BPM 1900f. This fully automatic, wrist mounted blood pressure monitor is both easy to use and ideally suited for daily measurements. The large display includes systolic, diastolic and pulse, all clearly shown at the completion of each reading.

In addition, you can store up to 90 measurements, ideal for users who want to monitor and track their blood pressure on a regular basis. The BPM 1900 f is compact and portable, making it ideal for home use and travel.

Please read this manual carefully before use. For specific information on your own blood pressure, contact your physician. Put the user guide away in a safe place for future reference.

2 Safety instructions

1. This product is for household use only. It is not intended as a substitute for consultation with your doctor.
2. For users diagnosed with common arrhythmia (atrial or ventricular premature beats or atrial fibrillation), diabetes, poor circulation of blood, kidney problems, or for users suffered from stroke, or for unconscious users, the device might not be suitable for use. Consult your doctor in case of any doubt.
3. The unit should not be operated by children so to avoid hazardous situations.
4. The unit contains high-precision assemblies. Therefore, avoid extreme temperatures, humidity, and direct sunlight. Avoid dropping or strongly shocking the main unit, and protect it from dust.
5. Leaky batteries can damage the unit. Remove the batteries when the unit is not used for a long time.
6. Do not press the START key when the cuff is not placed around the arm.
7. Do not disassemble the main unit or cuff.
8. If the unit is stored in a cold place, allow it to acclimate at room temperature before use.
9. Clean the blood pressure monitor body and the cuff carefully with a slightly damp, soft cloth. Do not press. Do not bend the pre-formed cuff inside out. Do not wash the cuff or use chemical cleaner on it. Never use thinner, alcohol or petrol (gasoline) as cleaner.
10. To ensure accurate measurements, it is recommended that calibration be completed every two years.

3 Technical specifications

| | |
|-----------------------|--|
| Power source | 6V DC: 4 x LR06 (AA) batteries |
| Measurement method | Oscillometric |
| Measurement zone | Pressure: 20~280 mmHg Pulse: 40~180 beats/minute |
| Accuracy | Pressure: ± 3 mmHg Pulse: within $\pm 5\%$ of reading |
| Pressure sensor | Semi-conductor |
| Inflation | Pump driven |
| Deflation | Automatic Pressure release valve |
| Memory capacity | 90 memories |
| Auto-shut-off | 60 seconds after last key operation |
| Operation environment | Temperature : 5°C~45°C(41°F~113°F) Humidity : 40~85% RH max. |
| Storage environment | Temperature : -10°C~60°C (14°F~140°F) Humidity : 10~95% RH max. |
| Weight | 400g (without batteries) |
| Arm circumference | 24~36 cm |
| Limited patient | Over 18 years old |
| Dimensions | 150(L) x 100(W) x 68(H) mm |
| K.B. | Short time operation 2 minutes. |
| Classification | Classification of type BF |

*Specifications are subject to change without notice.

4 Helpful tips

Here are a few helpful tips to help you obtain more accurate readings:

- Do not measure your blood pressure immediately after consuming a large meal. To obtain more accurate readings, please wait one hour before measuring.
- Do not smoke or drink alcohol before measuring your blood pressure.
- You should not do be physically tired or exhausted while taking a measurement.
- It is important that you relax during measurement. Try to take 15-minutes rest before a reading.
- Do not take measurements if you are under stress or under tension.
- Take your blood pressure at normal body temperature. If you are feeling cold or hot, wait a while before taking a measurement.
- If the monitor is stored at very low temperature (near freezing), have it placed at a warm location for at least one hour before using it.
- Wait about 5 minutes before taking the next pressure measurement.

5 Blood pressure

5.1 What is Blood Pressure?


Blood pressure is the pressure exerted on the artery tube while blood flows through the arteries. The pressure measured when the heart contracts and sends blood out of the heart is systolic (highest). The pressure measured when the heart dilates with blood flowing back into the heart is called diastolic (lowest) blood pressure.

5.2 Why Measure Your Blood Pressure?

Among the various health problems afflicting modern people, problems associated with high blood pressure are by far the most common. High blood pressure's dangerously strong correlation with cardiovascular diseases and high morbidity has made measuring blood pressures a necessity of identifying those at risk.

5.3 Blood Pressure Standard

The World Health Organization (WHO) and National High Blood Pressure Education Program has developed a Blood pressure standard, according to which areas of low and high-risk blood pressure are identified. This standard however, is a general guideline as individual's blood pressure varies among different people and different groups...etc.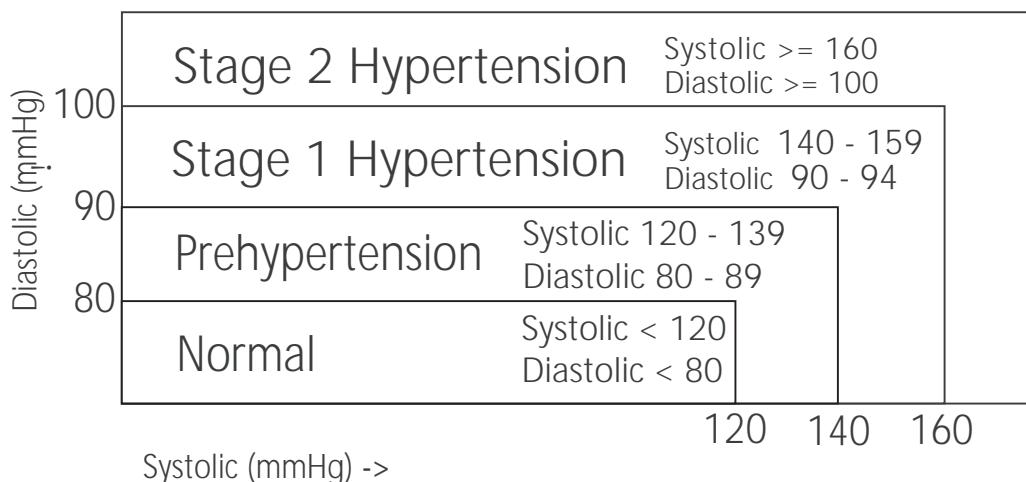


It is important that you consult with your physician regularly. Your physician will tell you your normal blood pressure range as well as the point at which you will be considered at risk.

5.4 Blood Pressure Fluctuation


Blood pressure fluctuates all the time!

You should not be worried if you encountered two or three measurements at high levels. Blood pressure changes over the month and even throughout the day depending on circumstances (state of mind, temperature, ...)

Ex. Fluctuation within the day for a Male, 35 years old.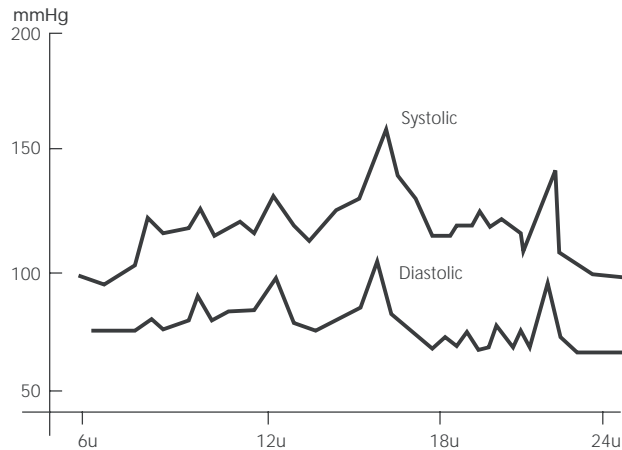

6 Battery installation

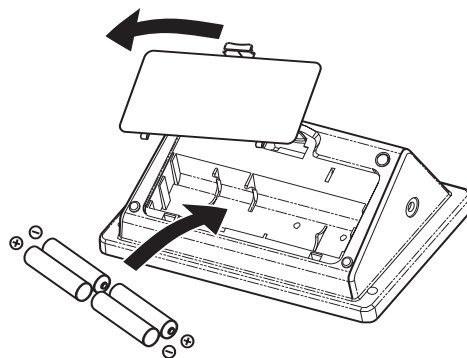1. Press down and lift the battery cover in the direction of the arrow to open the battery compartment.
2. Install or replace 4 "AA" sized batteries in the battery compartment according to the indications inside the compartment.
3. Close the battery compartment.


Dispose of the batteries at a designated battery disposal unit and not in the household waste. Do not shortcircuit & dispose of in fire. Remove the battery if this device is not going to be used for a long period.


NiMH

7 Parts & buttons description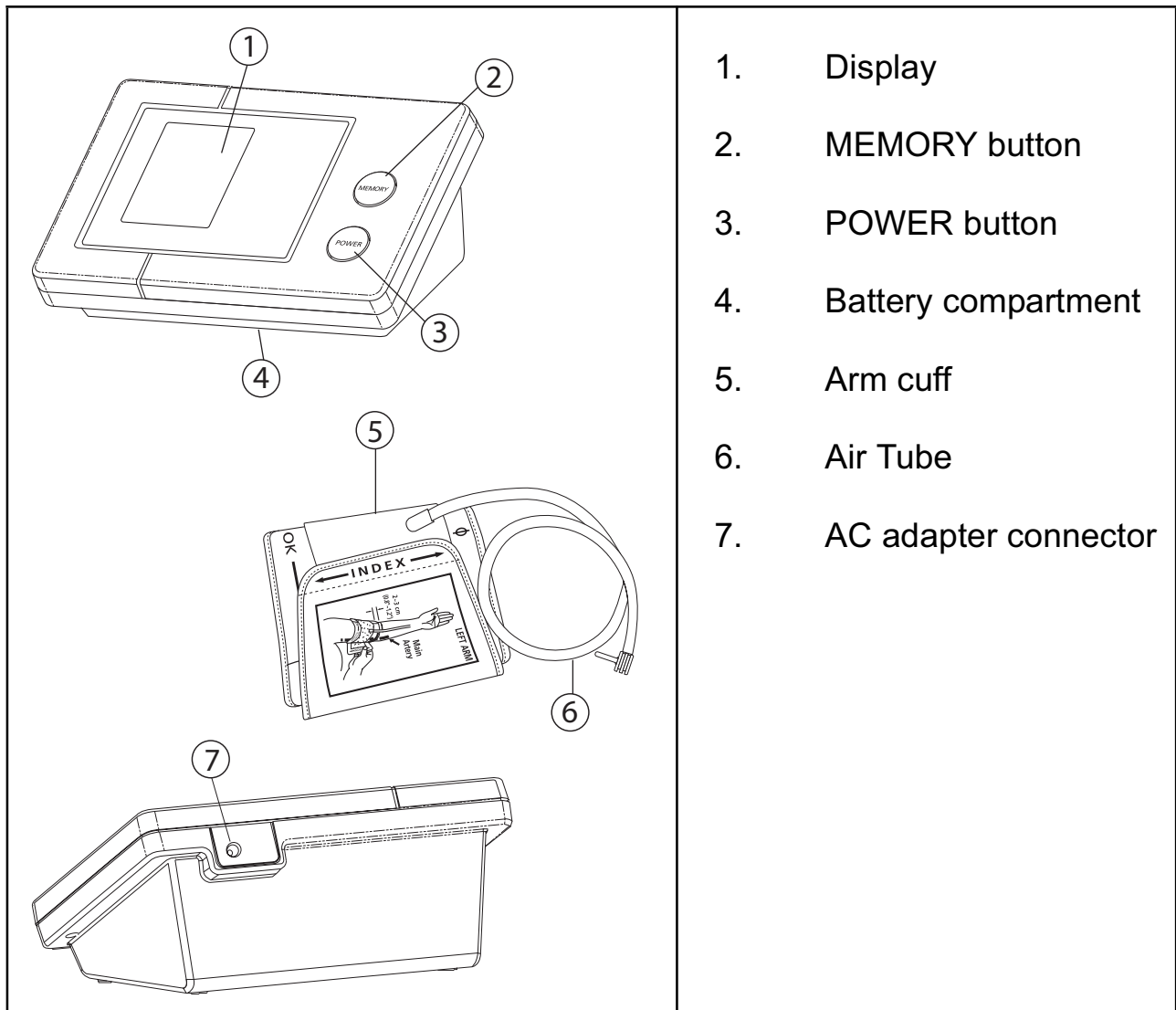


1. Display
2. MEMORY button
3. POWER button
4. Battery compartment
5. Arm cuff
6. Air Tube
7. AC adapter connector

8 Display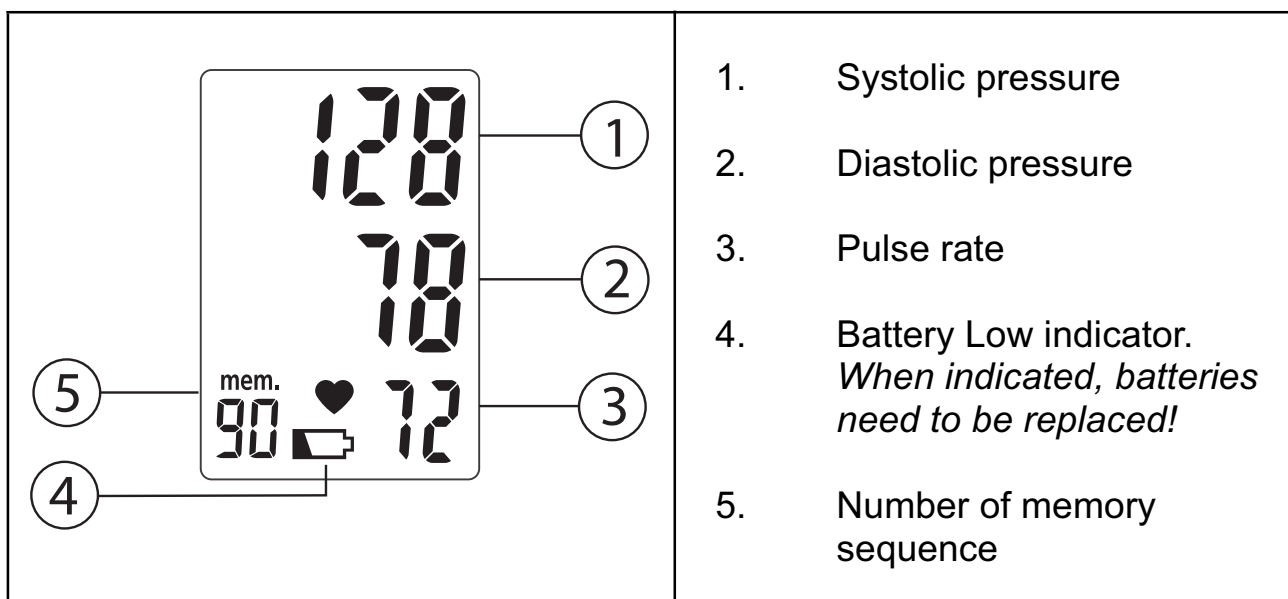

1. Systolic pressure
2. Diastolic pressure
3. Pulse rate
4. Battery Low indicator.
When indicated, batteries need to be replaced!
5. Number of memory sequence

9 Applying the cuff

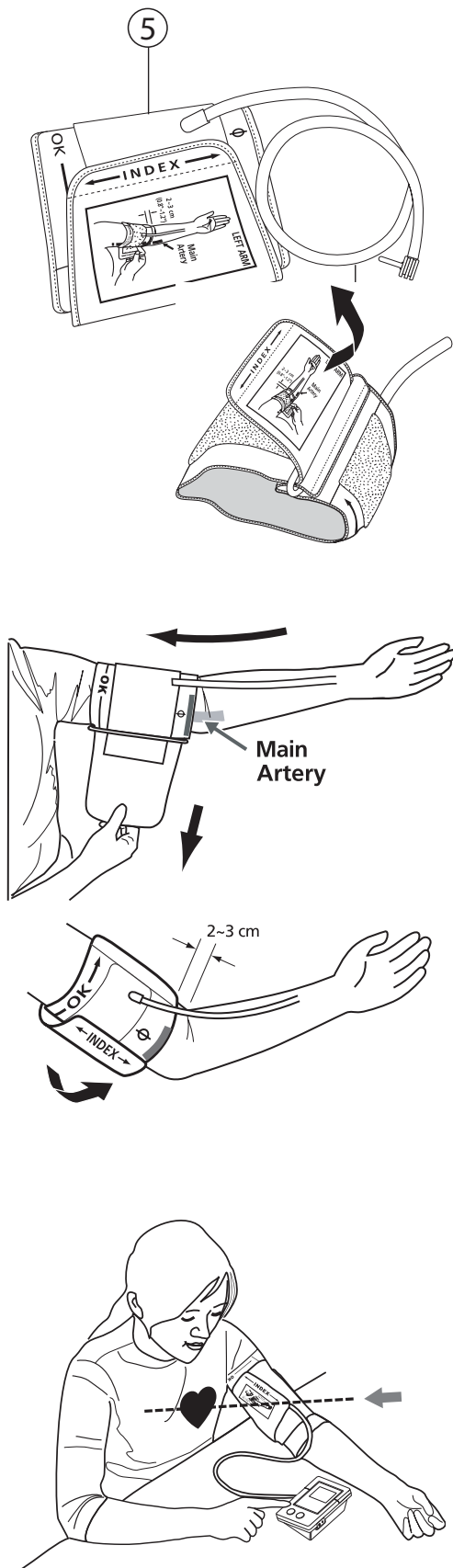




1. Plug in the cuff connecting tube into the unit.
2. Unwrap the arm cuff, leaving the "Index" end of the cuff through the D-ring of the cuff.
3. Put your left arm through the cuff loop. The "OK" range indication should be positioned closer to yourself with the rubber tube pointing in the direction of your arm. Position the artery mark (ϕ) over the main artery (on the inside of your arm) in the upper arm.
4. Turn your left palm upward and place the edge of the arm cuff at approximately 2 to 3 cm above the inner side of the elbow joint. Tighten the cuff by pulling the "Index" end of the cuff. If the Index line falls within the OK range, indicated at the edge of the cuff, this cuff is suitable for your use. If the Index line falls out of the OK range, you may need a cuff for use with other circumferences for measurement.
5. Relax your entire body, especially the area between your elbow and fingers. Place your elbow on a table so that the cuff is at the same level as your heart. Avoid leaning back while the measurement is being taken.

Your heart is located slightly below your arm pit.

10 Measurement

10.1 Procedure

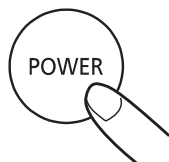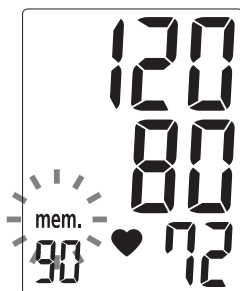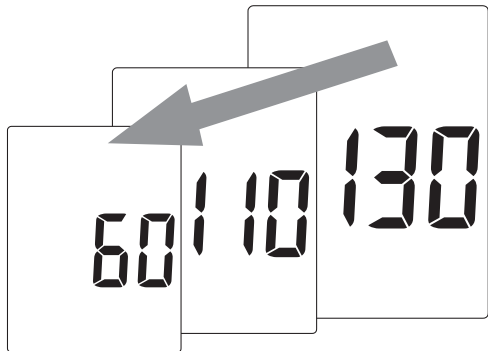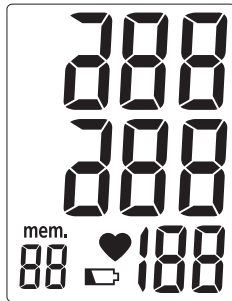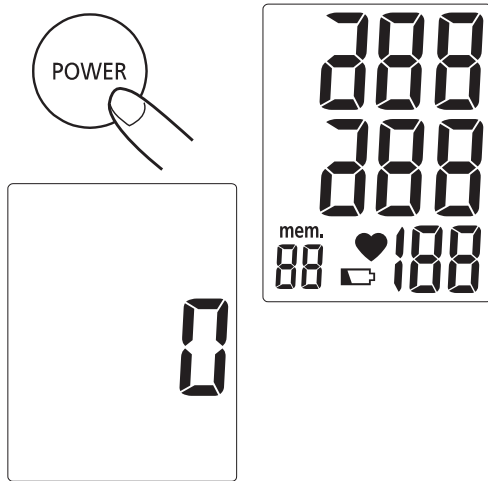1. Press the POWER button. All display segments will appear for approximately one second before returning to "0"
2. The unit will automatically inflate to the appropriate inflation level based on the user's pulse oscillations. Measurement will then begin.


To interrupt the measurement, simply press any key. The cuff will deflate immediately after a key is pressed. During the measurement, do not talk or move your arm or hand muscles.

3. When the measurement is completed, systolic, diastolic and pulse will be shown simultaneously and be saved automatically in the memory system. Up to 90 results can be saved.
4. Press the POWER button to turn off the power. If no button is pressed, the unit will shut off automatically in 1 minute.

When a mistake was detected during measurement the Error mark appears on the display.

11 Recalling memorized results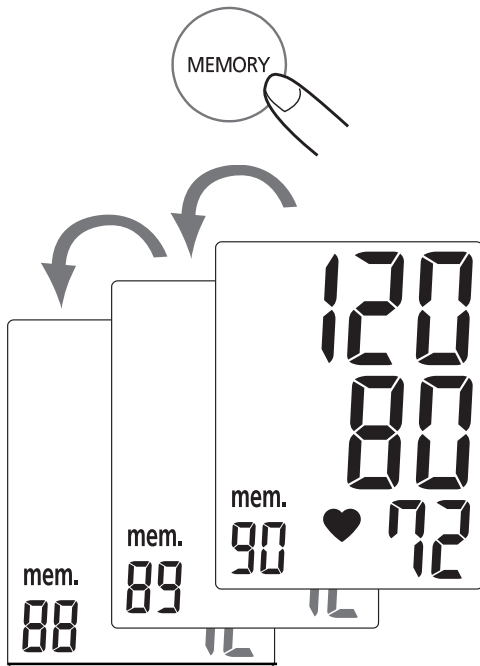



To recall stored blood pressure measurement results from the memory:

- Press the MEMORY button. The last memorized result will be displayed.
- Press the MEMORY button again to recall the previous result...

When the batteries are installed, the data in the memories will not be lost even when the Blood Pressure Monitor is switched off. Remove the batteries for more than 10 seconds to erase the memories.

12 Power adapter (optional)

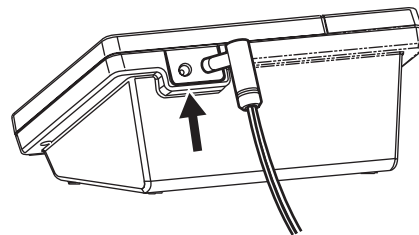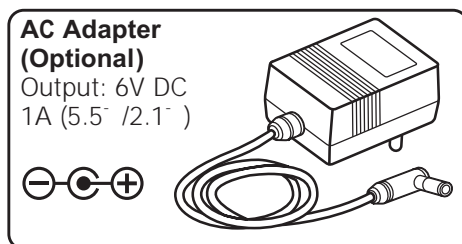- Connect the AC adapter jack with the AC adapter connector on the back of the unit.
- Plug the AC adapter into the wall socket.

Only use the optional Topcom BPM Arm 1900f power adapter.

13 Topcom Warranty

13.1 Warranty period

The Topcom units have a 24-month warranty period. The warranty period starts on the day the new unit is purchased. Consumables or defects causing a negligible effect on operation or value of the equipment are not covered. The warranty has to be proven by presentation of the original purchase receipt, on which the date of purchase and the unit-model are indicated.

13.2 Warranty handling

A faulty unit needs to be returned to a Topcom service centre including a valid purchase note.

If the unit develops a fault during the warranty period, Topcom or its officially appointed service centre will repair any defects caused by material or manufacturing faults free of charge.

Topcom will at its discretion fulfil its warranty obligations by either repairing or exchanging the faulty units or parts of the faulty units. In case of replacement, colour and model can be different from the original purchased unit.

The initial purchase date shall determine the start of the warranty period. The warranty period is not extended if the unit is exchanged or repaired by Topcom or its appointed service centres.

13.3 Warranty exclusions

Damage or defects caused by incorrect treatment or operation and damage resulting from use of non-original parts or accessories not recommended by Topcom are not covered by the warranty.

The warranty does not cover damage caused by outside factors, such as lightning, water and fire, nor any damage caused during transportation.

No warranty can be claimed if the serial number on the units has been changed, removed or rendered illegible.

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