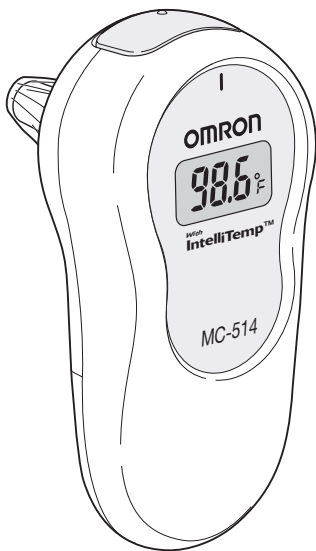
OMRON®

INSTRUCTION MANUAL

EAR THERMOMETER

Model **MC-514** with IntelliTemp™

ESPAÑOL

ENGLISH

TABLE OF CONTENTS

Before Using the Thermometer

| | |
|-----------------------------------|---|
| Introduction | 3 |
| Important Safety Notes | 4 |
| Before Taking a Measurement | 5 |

Operating Instructions

| | |
|------------------------------------|---|
| Know Your Unit | 6 |
| Conversion °F/ °C | 6 |
| Battery Installation | 7 |
| Operating Mode | |
| One Second Mode | 8 |
| IntelliTemp™ Technology Mode | 9 |

Care and Maintenance

| | |
|---|----|
| Care and Maintenance | 10 |
| Error Indicators and Troubleshooting Tips | 11 |
| Quick Reference Guide | 12 |
| FCC Statement | 13 |
| Warranty Information | 14 |
| Specifications | 15 |

INTRODUCTION

Thank you for purchasing the Omron® IntelliTemp™ MC-514 Ear Thermometer.

Fill-in for future reference

DATE PURCHASED: _____

SERIAL NUMBER: _____

Staple your purchase receipt here.

Your new IntelliTemp™ thermometer offers a safe, accurate and quick temperature reading from the ear canal. The thermometer detects the infrared heat given off by the eardrum (tympanic membrane), and surrounding tissues, and it convert this heat into an equivalent oral temperature.

No need to worry about broken glass or mercury hazards.

Talking, eating or drinking does not affect an ear temperature. These factors do affect an oral temperature.

The unit is easy to use on children and adults. Temperatures can be taken even while sleeping.

We recommend that you practice taking a temperature with the Omron® IntelliTemp™ Ear Thermometer on each member of your family when they are healthy. This way you will be confident in your technique when using the ear thermometer on a sick family member.

Contact your doctor if you have any questions regarding specific temperatures.

The MC-514 comes with the following components:

- Thermometer
- Storage Case
- 1 CR2032 Battery
- 21 Lens Covers
- Instruction Manual in English and Spanish

The MC-514 IntelliTemp™ Ear Thermometer is intended for home use.

SAVE THESE INSTRUCTIONS

IMPORTANT SAFETY NOTES

To assure the correct use of the product basic safety measures should always be followed including the precautions listed below:

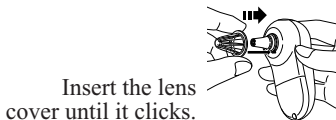
- Read all information in the instruction book and any other literature included in the box before using the unit.
- Contact your physician for specific information about your temperature. Follow the instructions of your healthcare provider.
- Operate the unit only as intended. Do not use for any other purpose.
- Do not use a cellular phone near the unit. It may result in operational failure.
- Do not use if ear infection is present.
- Changes or modifications not approved by Omron Healthcare will void the user warranty. Do not disassemble or attempt to repair the unit or components.

BEFORE TAKING A MEASUREMENT

To ensure a reliable reading follow these recommendations.


The thermometer will not work unless a lens cover is applied correctly.

Attach a new lens cover for every reading for hygienic reasons.


Replacing Lens Cover

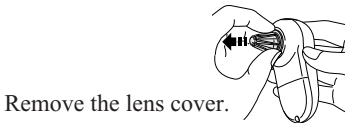Use Omron® lens cover Model MC-505LC.

KNOW YOUR UNIT

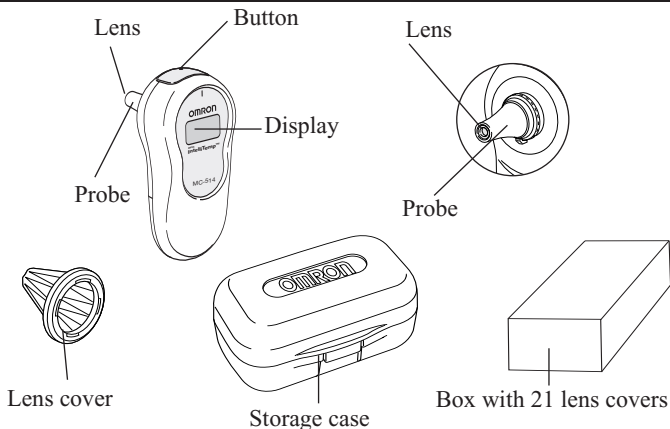CONVERSION FAHRENHEIT TO CENTIGRADE (°F/°C)

The thermometer is initially set at Fahrenheit °F.

1. In the off position with lens cover applied, press and hold the button for three (3) seconds until the “°F” flashes you will hear one beep.
2. Press button again, the “°C” flashes.
3. After five (5) seconds the unit will turn off. The desired mode has been set. Repeat the process to reverse measurement scale if desired.

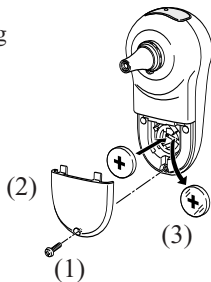
NOTE: Changing the measurement scale deletes the previous measured value.

BATTERY INSTALLATION

When the battery is removed from the unit the stored reading will be deleted.

1. Remove the screw from the battery cover using a small screwdriver.
2. Open the battery cover.
3. Place the battery in the space provided.

Place the positive (+) side of the battery upwards. Insert the battery in the direction of the arrow.

4. Replace the battery cover and tighten the screw.
When the Battery Indicator appears on the display screen remove the old battery.

Remove the battery with a thin stick that is not easily breakable.

- DO NOT use any metal device.

NOTE: Dispose of any used battery in accordance with local regulations.

ONE SECOND MODE

When unit is used for the first time the “M” symbol will not be displayed.

One Second Mode

1. Press button to turn unit on.
2. When the “°F” or “°C” symbol flashes-gently insert the probe while pulling back the ear lobe this provides a direct line to the eardrum to maximize the accuracy of results.
3. Listen for ready beep (1 beep); an “0” symbol will appear indicating unit is functioning, angle is correct and thermometer is ready for measurement.
4. Press button again while unit is placed in the ear; you will hear 4 beeps which means reading is complete.

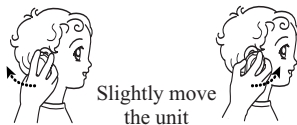
INTELLITEMP™ TECHNOLOGY MODE

Omron® IntelliTemp™ technology makes it easy and safe to use with infants and adults. IntelliTemp™ mode is recommended when the direction of the eardrum may be difficult to find such as measuring the temperature of an infant. The unit will scan the eardrum 250 per second and displays the highest temperature detected during measurement.


IntelliTemp™ Directions

Follow directions 1 through 3 for One Second Mode.

4. Slightly move the unit left and right. The unit automatically detects the temperature of the eardrum for a maximum of 10 seconds.

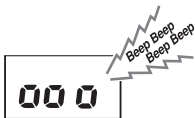5. When the unit beeps twice, press the blue button-leaving probe in the ear.


6. Measurement is complete when unit beeps 4 times.
7. Remove probe from ear. Discard used lens cover, and replace with a new one, so that the unit is ready for use again.
8. To repeat measurements, wait until the “°F” or “°C” flashes.

CARE AND MAINTENANCE

- Use of lens covers is recommended to help avoid spreading germs and to keep the thermometer lens clean.
- Clean the thermometer with 70% isopropyl alcohol or clean water.
- Clean the lens by using only a soft dry cloth or cotton swab.
- Store the unit in the protective case while not in use.
- Avoid the thermometer being exposed to direct sunlight, high temperatures, high humidity and dust.
- Do not subject the unit to strong shock, drop, or step on the unit.
- Keep battery away from children.
- Do not open the thermometer except to replace the battery.

Storing your thermometer

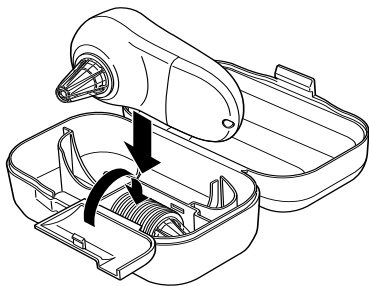- Keep your thermometer in the storage case after attaching a lens cover to protect the lens.

ERROR INDICATORS

| SYMBOL | CAUSE | CORRECTION |
|--|--|--|
| | Lens cover is not on probe. | Attach the lens cover, and then take a measurement. |
| | Temperature of the unit is unstable. The temperature of the unit is 50° F or below, which is outside the operating range. The temperature of the unit is 104°F or above, which is outside the operating range. High room temperature. | Be sure unit remains at room temperature where measurement will take place. Allow 30 minutes to stabilize thermometer before using. Normal range: (50°F to 104°F) (10°C to 40°C). |
| | The measured temperature was above the range of the thermometer. The measured temperature was below the range of the thermometer. | The probe may not have been properly positioned in ear. The probe must be snug against opening of ear. Be sure lens cover is clean and ear canal is not blocked with earwax. |
| “°F” or “°C” flashes, but “0” symbol is NOT displayed. | The probe or lens cover may be dirty. Eardrum may be blocked with earwax. | Clean probe with cotton swab, replace lens cover. Remove earwax |
| | Low Battery. | Replace the battery. Refer to page 7 for battery installation. |

TROUBLESHOOTING TIPS

| PROBLEM | CAUSES AND SOLUTIONS |
|--|---|
| No power. No display appears on the unit. | Replace worn batteries. Check the battery installation for proper placement of the battery polarities. |

QUICK REFERENCE GUIDE

- Press Button. Display will illuminate for 2 seconds.
- Insert into ear.
- Listen for 1 beep.
- Press button again – wait for 4 beeps.
- Remove from ear. Read results.
- Wait 10 minutes before the 4th measurement.

FCC STATEMENT

NOTE:**POTENTIAL FOR RADIO/TELEVISION INTERFERENCE (for U.S.A. only)**

This product has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. The product generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If the product does cause harmful interference to radio or television reception, which can be determined by turning the product on and off, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna
- Increase the separation between the product and the receiver.
- Connect the product into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

POTENTIAL FOR RADIO/TELEVISION INTERFERENCE

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Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

WARRANTY

Limited 5 Year Warranty

Your MC-514 Omron® IntelliTemp™ Ear Thermometer, excluding the battery, is warranted to be free from defects in materials and workmanship appearing within 5 years from the date of purchase when used in accordance with the instructions provided with the unit. The above warranties extend only to the original retail purchaser.

Repair or replacement is our only responsibility and your only remedy under the above warranty.

To obtain warranty service contact Omron Healthcare's Customer Service by calling **1-800-634-4350** for the address of the repair location and the return shipping and handling fee. Information for warranty service is available on our website at www.omronhealthcare.com.

Enclose the Proof of Purchase. Include a letter, with your name, address, phone number, and description of the specific problem. Pack the product carefully to prevent damage in transit. Because of possible loss in transit, we recommend insuring the product with return receipt requested.

ALL IMPLIED WARRANTIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE APPLICABLE WRITTEN WARRANTY ABOVE. Some provinces do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

OMRON SHALL NOT BE LIABLE FOR LOSS OF USE OR ANY OTHER INCIDENTAL, CONSEQUENTIAL OR INDIRECT COSTS, EXPENSES OR DAMAGES. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above exclusions may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which may vary from state to state.

FOR CUSTOMER SERVICE

Visit our web site at:

www.omronhealthcare.com

Call toll free:

1-800-634-4350

SPECIFICATIONS

| | |
|-----------------------------------|--|
| Name: | Omron® IntelliTemp™ Ear Thermometer |
| Model: | MC-514 |
| Power source: | DC 3 Volt (lithium battery CR2032) |
| Battery life: | Approximately 2500 measurements |
| Sensor: | Thermopile |
| Temperature indication: | In 4 digits, every 0.1°F or °C |
| Accuracy: | 93.2 ~ 96.7°F (34.0 ~ 35.9°C) : ±0.5°F (±0.3°C) 96.8 ~ 102.2°F (36.0 ~ 39.0°C) : ±0.4°F (±0.2°C) 102.3 ~ 108.0°F (39.1 ~ 42.2°C) : ±0.5°F (±0.3°C) |
| Measurement range: | 93.2 to 108°F (34 to 42.2°C) |
| Weight: | Approx. 1.8 oz (50 g) (including the battery) |
| Operating temperature / humidity: | Ambient temperature: 50 to 104°F (10 to 40°C) Relative humidity: 30 to 85% |
| Storage temperature / humidity: | Ambient temperature: -4 to 140°F (-20 to 60°C) Relative humidity: 10 to 95% |
| Memory: | Memory function displays the previous reading |
| *ASTM Standards: | E 1965-98 |
| Contents: | Thermometer, Storage Case, Instruction Manual, and 21 Lens Covers |
| Accessories: | Omron MC-505LC Lens Covers |
| UPC: | 0 73796 33514 4 |

The specifications may be changed without prior notice.

* ASTM laboratory accuracy requirements for the thermometer only (not including a lens filter) in the display range of 96.8 °F to 102.2 °F (36 °C to 39 °C) for infrared thermometers is ± 0.4 °F (± 0.2 °C), whereas for mercury-in-glass and electronic thermometers, the requirements per ASTM Standards E 667-86 and E 1112-86 is ± 0.2 °F (± 0.1 °C).

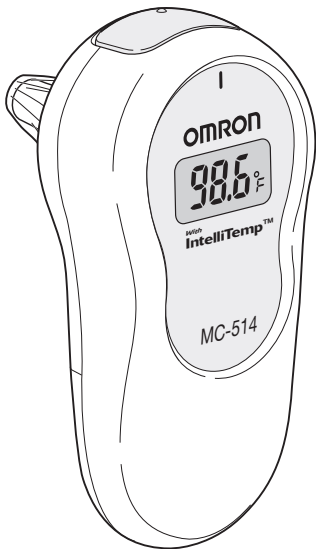
Distributed by:
OMRON HEALTHCARE, INC.
1200 Lakeside Drive
Bannockburn, Illinois 60015
www.omronhealthcare.com

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© 2005 Omron Healthcare, Inc.