Dealer: This manual **MUST** be given to the end user.

User: **BEFORE** using this product, read this manual and save for future reference.

**For more information regarding
Invacare products, parts, and
services, please visit
www.invacare.com**

Yes, you can.

 **WARNING**

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT INVACARE TECHNICAL SERVICES BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE, SERIOUS INJURY OR PROPERTY DAMAGE MAY RESULT.

TABLE OF CONTENTS

SPECIAL NOTES	5
INTRODUCTION	7
TYPICAL PRODUCT PARAMETERS	8
SUPPLEMENTAL INFORMATION	9
Compressor.....	9
Extension Cords.....	9
HANDLING	10
Shipping Instructions.....	10
Unpacking.....	10
Inspection.....	10
SECTION 1—IMPORTANT SAFEGUARDS	11
To Reduce the Risk of Electrocution:	12
To reduce the risk of burns, electrocution, fire and injury to Persons:	13
SECTION 2—OPERATING INSTRUCTIONS	16
Setup.....	16
Treatment Instructions.....	20

TABLE OF CONTENTS

TABLE OF CONTENTS

SECTION 3—BATTERIES 22

- Battery Specifications 22
- Conditioning the Battery 24
- Using the Portable Nebulizer Battery 25
- Charging the Battery 25

SECTION 4—CLEANING AND DISINFECTING ... 27

- Cleaning the Outer Case..... 28
- Cleaning the Nebulizer 29
- Cleaning the Compressor 31
- Cleaning the Mouthpiece..... 32

SECTION 5—MAINTENANCE 33

- Routine Care..... 33
- Storage 33
- Filter Care..... 34

SECTION 6—TROUBLESHOOTING 35

LIMITED WARRANTY 37

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

NOTICE

The information contained in this document is subject to change without notice.

 **WARNING**

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

INTRODUCTION

Your Invacare Stratos Portable Nebulizer is the latest in portable compressor nebulizer technology. The purpose of the nebulizer is to convert your liquid medication into a fine mist that can be drawn into your lungs to relieve your respiratory condition. The nebulizer is very portable and lightweight. For even greater convenience, ask your dealer about the optional battery pack that is sold separately.

To maximize your benefit from the nebulizer, we recommend you read and understand the information in the operating manual. By following your doctor's advice and these simple instructions, the nebulizer system will become a valuable part of your daily treatment routine.

TYPICAL PRODUCT PARAMETERS

TYPICAL PRODUCT PARAMETERS

Sound Level:	55 dBA
Operating Temperature Range:	50°F - 104°F (10°C to 40°C); 10 to 90% RH
Foam Intake Filter:	Rayon Fiber Filter
Safety Systems:	High temperature compressor shutdown.
Regulatory Listing:	In compliance with EN 60601-1
Dimensions without battery: with battery:	3.6 x 1.6 x 4.1 inches 3.6 x 2.6 x 4.1 inches
*Weight without battery: with battery:	0.70 lbs (318 g) 1.2 lbs (613 g)
Power Supply:	100-240V ~, 47-63 Hz, 12VDC, 1.25A
Maximum Pressure:	35 psi
Maximum Flow Rate:	5.2 LPM
Storage Conditions:	-13°F to 158°F (-25° to 70°C); 10 to 90% RH
Operating Flow Rate:	2.5 L/min. @ 10 PSI operating pressure
Mode of Operation:	Intermittent Use 20 min. ON/40 min. OFF

*NOTE: *Weight is without nebulizer and tubing.*

SUPPLEMENTAL INFORMATION

Compressor

If the unit shuts down frequently or fails to start, you may have a low voltage situation.

Low voltage can also be suspected when:

- The compressor fails to start or motor does not get up to full power or speed.
- Other motor operated appliances fail to operate properly or too many motor operated appliances are on same circuit.

Extension Cords

Make certain extension cord is in good condition. Make certain extension cord wire is NOT LESS than 18 gauge for 25 feet.

HANDLING

Shipping Instructions

The shipping container has been designed to assure protection of this device.

If this device is to be reshipped by common carrier, it should be packed in the same carton.

Unpacking

1. Check for any obvious damage to the carton or its contents. If damage is evident, DO NOT use. Notify Carrier/Invacare or your local Invacare Dealer.
2. Remove all the loose packing from the carton.
3. Carefully remove all the components from the carton.

NOTE: Retain carton and packing materials for use in shipping or storing the unit until use is required.

Inspection

1. Examine exterior for nicks, dents, scratches or other damage. Inspect all components.

SECTION I—IMPORTANT SAFEGUARDS

WARNING

SECTION I - IMPORTANT SAFEGUARDS contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Dangers, Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Federal law restricts this device to sale by or on the order of a physician.

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following.

SECTION I—IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING

DANGER

Risk of electric shock. DO NOT disassemble. Refer servicing to qualified service personnel.

FIRE RISK - DO NOT use this product in an oxygen enriched environment (i.e – oxygen tent).

To Reduce the Risk of Electrocution:

ALWAYS unplug this product IMMEDIATELY after using.

DO NOT use while bathing.

DO NOT place or store product where it can fall or be pulled into a tub or sink.

DO NOT reach for product that has fallen into water. Unplug IMMEDIATELY.

DO NOT place in or drop into water or other liquid.

DO NOT reach for a product that has fallen into water. Unplug IMMEDIATELY.

⚠ WARNING

**To reduce the risk of burns,
electrocution, fire and injury to Persons:**

This product should NEVER be left unattended when plugged in.

Close supervision is necessary when this product is used by, on, or near children and physically challenged individuals.

Use this product only for its intended use as described in this manual and only with the prescribed medication. Use this product only under physician's direction.

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

NEVER operate this product with a damaged cord or plug, if it is not working properly, if it was dropped or damaged, or if it was dropped into water. Call qualified service personnel for examination and repair.

SECTION I —IMPORTANT SAFEGUARDS

WARNING

Keep the nebulizer tubing, cord, and unit away from heated or hot surfaces, including space heaters, electric blankets, stoves and similar electrical appliances.

NEVER drop or insert objects into any opening or hose.

NEVER use while sleeping or drowsy.

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free from lint, hair and the like. Blocked openings may cause the unit to shut down. Contact dealer immediately.

The Stratos compressor nebulizer system uses the surrounding air to effectively administer the prescribed medication through the nebulizer. The user and administrator should be aware of surroundings so as not to have contaminated air filtered through the compressor nebulizer system. (Examples can be air contaminated with paint or cleaning fumes, weed killer, car exhaust, etc.)

DO NOT use the Stratos compressor nebulizer system in these environments. Otherwise, serious bodily injury may occur.

This unit is oil-less. DO NOT lubricate.

⚠ WARNING

This equipment is NOT suitable for use in the presence of a flammable anesthetic mixture with air or with nitrous oxide.

DO NOT operate the compressor while it is still in the carrying case.

When not administering treatment, keep out of the reach of small children. Some of the parts of the nebulizer are small enough for children to swallow. The tubing may present a strangulation hazard

NEVER touch the unit with wet hands.

DO NOT pull on the power cord to unplug the unit from an electrical outlet.

DO NOT overload wall outlets.

Pump maintenance MUST be performed by a qualified service technician.

SAVE THESE INSTRUCTIONS

SECTION 2—OPERATING INSTRUCTIONS

⚠ WARNING

Remove the compressor from the carrying case before use.

DO NOT operate unit without filter.

Avoid operating this product in a dusty environment. Otherwise, premature compressor wear could occur.

Setup

NOTE: For this procedure, refer to FIGURE 2.1 on page 17.

1. Wash your hands before you begin the treatment procedure.
2. Place the device on a table so that the controls can be reached when seated.
3. Make sure the nebulizer is firmly seated in holder.
4. Remove the nebulizer from the compressor by pulling straight up.

SECTION 2—OPERATING INSTRUCTIONS

5. Open the nebulizer by pulling up on the top while securely holding the bottom.
6. Make sure the cone is correctly positioned in the cup.

⚠ WARNING

DO NOT exceed the maximum allowed quantity of six ml of medication. The maximum capacity of the nebulizer medication cup is 6 ml.

CAUTION

NEVER pour medication into the nebulizer seat area for the compressor. Damage to the compressor will occur.

7. Add the prescribed amount of medication to the cup of the nebulizer using an eyedropper, pre-measured dose or ampoule (FIGURE 2.1).
8. Close the nebulizer by firmly pressing the top down onto the cup until a click is heard.

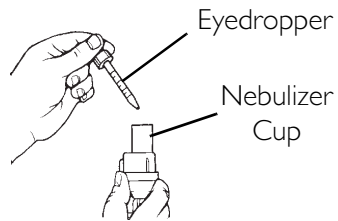
FIGURE 2.1 Adding Medication

SECTION 2—OPERATING INSTRUCTIONS

9. Place the tubing adapter into the compressor outlet and rotate it counter clockwise to lock the adapter in place. The tubing connector is then firmly inserted into the bottom of the nebulizer.

CAUTION

Use only the original tubing adapter. DO NOT squeeze or kink the tubing.

10. In high humidity conditions, moisture or condensation can occur in the tubing adapter. If this happens, simply turn on the compressor with the tubing adapter attached and run for 2 - 3 minutes or hang the tubing adapter so the moisture can drain out.
11. Connect the mouthpiece to the nebulizer.

⚠ WARNING

DO NOT use the nebulizer mouthpiece, compressor or tubing with other compressors, nebulizer mouthpieces or tubing. Otherwise, this may produce incorrect flow, resulting in improper treatment.

12. To use with battery, charge for 16 hours. Refer to Conditioning the Battery on page 24.
 13. To use the nebulizer with the power cord, connect the power cord plug into the power outlet on the side of the nebulizer compressor. Connect the power cord to the adapter and insert the plug into the electrical outlet.
 14. To use the nebulizer with the DC auto adapter, connect the DC auto adapter to the power outlet on the side of the nebulizer compressor. Connect the auto adapter plug of the DC cord to the auto adapter receptacle.
-

⚠ DANGER

To avoid the risk of possible electrical shock, the nebulizer must be operated on the specified power source.

Treatment Instructions

NOTE: For this procedure, refer to FIGURE 2.2.

NOTE: The Invacare Stratos Portable Nebulizer has been designed for intermittent use. Normal treatment time should be five to ten minutes. Turn off the nebulizer after 20 minutes of use and allow the device to cool down 40 minutes before turning it on again.

1. Sit in a relaxed and upright position.
2. Turn on the device by pushing the ON/OFF button.
3. Begin treatment by placing the nebulizer mouthpiece between your teeth and close your mouth. Inhale deeply and slowly as the aerosol begins to flow and then exhale slowly.
4. When treatment is completed, turn off the nebulizer and unplug the power adapter.
5. Remove the mouthpiece from the nebulizer.

FIGURE 2.2 Using the Nebulizer

SECTION 2—OPERATING INSTRUCTIONS

6. Remove the nebulizer and follow the cleaning instructions. Refer to Cleaning and Disinfecting on page 27.

SECTION 3—BATTERIES

Battery Specifications

Battery type	NiMH
Supply voltage	12 VDC (use only original adapter)
Complete charge	16 hours
Capacity	750 mAh
Autonomy	Minimum 35 minutes at full charge

The optional battery used with the Invacare Stratos Portable Nebulizer requires a few simple precautions to ensure proper operation and to avoid damage to the nebulizer or to the battery. When used correctly, the battery can operate the nebulizer for approximately 35 minutes. Battery operation may vary due to charge conditions and care.

WARNING

Use only the original batteries supplied by Invacare. Carefully follow the instructions for operating the unit. Any other kind of use constitutes an improper use and is therefore dangerous. Invacare cannot be held liable for any damage caused by improper, incorrect or unreasonable use.

CAUTION

In case of failure and/or malfunction, turn the unit off and read the operating manual. **DO NOT** attempt to open the plastic case or to replace any components.

DO NOT short circuit battery contacts. Make sure battery contacts do not come in contact with keys, coins or other metal objects.

The battery has protection from overheating and will not operate if temperature is 104°F (40°C) or higher. Allow the battery to cool before restarting.

⚠ WARNING

To charge battery, use only the original power cable (AC) or auto adapter cable (DC). Using non-original equipment may result in damage to the battery.

Charging of the battery requires at least 16 hours. Failure to condition the battery prior to its first use may result in a reduced battery life expectancy.

DO NOT dispose of the battery pack in the household waste. Proper disposal at an appropriate collection site provided in your local area is strongly recommended.

DO NOT dispose batteries in fire, as they could explode.

Conditioning the Battery

NOTE: The definition of conditioning the battery is simply charging the battery.

1. Disconnect the battery from the compressor.
2. Connect the power cable (AC) or auto adapter cable (DC) to the battery.

The orange light will indicate if the battery is properly charging. Battery charging to full capacity is dependent upon many factors. Charging of the battery requires at least 16 hours. Failure to condition the battery prior to its first use may result in a reduced battery life expectancy. To assure maximum life performance of the battery after conditioning, leave the battery connected to a power source when not in use.

The battery operation time depends upon the charge condition of the battery, the temperature, the storage time of the battery prior to use, the amount of medication used for treatment, and the number of charge/discharge cycles. **The manufacturer of the battery recommends the battery be conditioned every four months.**

Using the Portable Nebulizer Battery

1. Align the rubber feet of the compressor with the indentations on the battery pack.
2. While holding the battery pack and compressor, push in the two tabs on the side of the battery pack.
3. Gently pull the compressor and battery pack in opposite directions to ensure the tabs have seated properly.

NOTE: The GREEN light on the battery will not illuminate while the nebulizer is being powered. The RED light will turn on when the battery power level is low. When the RED light is on, immediately find an alternative power source.

4. To remove the battery pack, press in the two tabs on the side of the battery pack and pull the two pieces apart.

Charging the Battery

The battery can be charged with either the power cable (AC) or the auto adapter cable (DC). Plug the cable into the appropriate power source. The ORANGE battery light will turn on indicating that the charging cycle has begun. The battery will reach full charge after at least 16 hours.

SECTION 3—BATTERIES

Completion of the charge cycle is indicated by a green battery light. Full charge will provide approximately 35 minutes of operation.

NOTE: In order to ensure maximum performance of the battery, keep the battery plugged in and charging when it is not in use. This will not overcharge or damage the battery.

NOTE: The battery must be plugged into a power source for the battery to charge. The battery will not charge when it is attached to the compressor nebulizer system.

NOTE: NEVER leave the battery connected to either power cable when the cable is disconnected from the power source. This will cause the battery to discharge.

SECTION 4—CLEANING AND DISINFECTING

Following the cleaning instructions after each use will prevent any remaining medication in the bottle from drying and causing the device to nebulize ineffectively.

It is recommended that the disposable nebulizer and the mouthpiece be thoroughly rinsed with hot tap water after each use and cleaned with a mild detergent after the last treatment of the day. The same pieces must also be disinfected daily by soaking in a vinegar/water solution or a commercially available disinfectant. If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions

Cleaning the Outer Case

DANGER

DO NOT remove outer case of this unit. Removing the case creates a serious electrical shock hazard. All disassembly and maintenance of this unit **MUST** be done by a qualified technician.

WARNING

Unplug unit before cleaning.

DO NOT submerge in water to clean. Clean outer case with damp cloth.

Cleaning the Nebulizer

⚠ WARNING

To prevent possible risk of infection due to contamination, follow these instructions:

- Clean the nebulizer before initial operations.
 - Clean the nebulizer after each aerosol treatment.
 - **ALWAYS** use fresh cleaning solution.
 - **DO NOT** towel dry the nebulizer components after cleaning.
-

NOTE: Your physician and/or an Invacare dealer may specify certain cleaning procedures. If so, follow their recommendations. If not, clean according to the following instructions:

1. Remove the nebulizer and mouthpiece from the tubing.
 2. Disassemble nebulizer chamber by turning counterclockwise and separating.
 3. Fill two plastic containers or bowls:
 - A. Washing: Hot water/detergent solution.
 - B. Soaking: Hot water/vinegar solution (one part vinegar to three parts water).
-

SECTION 4—CLEANING AND DISINFECTING

4. Thoroughly clean the nebulizer and mouthpiece in hot water/detergent solution.
5. Remove from the water/detergent solution and rinse with clear hot tap water.
6. Soak in hot water/vinegar solution for 30 minutes.
7. Rinse with hot tap water again and air dry thoroughly.

NOTE: If using medical disinfectant cleaners, follow manufacturer's instructions carefully.

8. Keep the outer surface of the tubing dust-free by wiping regularly.

NOTE: The nebulizer air tubing does not have to be washed internally because only filtered air passes through the tubing

Changing the Nebulizer

The nebulizer is designed for single patient use.

The nebulizer must be replaced after a long period of inactivity, or when it is obstructed by dry medication, dust, etc. The reusable nebulizer should last from six months to one year with general use. The disposable nebulizer should last 30 days.

Cleaning the Compressor

CAUTION

Unplug unit before cleaning.

NEVER submerge the nebulizer compressor in water or the AC/DC adapter or auto adapter, hold under running water or spray with water or other liquids. This will result in compressor damage.

If fluid gets into the area around the compressor outlet, wipe the area off with a dry cloth and let the nebulizer sit on its front end for five minutes. Then run the compressor for two minutes with no nebulizer or tubing adapter in place.

⚠ WARNING

DO NOT clean using abrasive cleaners.

1. Make sure the nebulizer is off and is not plugged into any power source.
2. Clean the casing of the main unit daily by using a soft cloth moistened with water or a mild detergent.

Cleaning the Mouthpiece

1. Boil the mouthpiece in water for a maximum of five minutes or autoclave at a maximum of 121° C for ten minutes.
2. Allow the mouthpiece to air dry.
3. Once dry, the mouthpiece may be stored inside the nebulizer top.

SECTION 5—MAINTENANCE

DANGER

DO NOT remove the compressor case. Failure to comply could result in electrical shock. All disassembly must be performed by an authorized Invacare service center.

WARNING

All maintenance on your Invacare Stratos Portable Nebulizer system must be performed by an authorized Invacare service center. Maintenance manuals and parts lists are available through Invacare upon request.

Routine Care

To keep your device in the best condition and protect the unit from damage, follow these directions:

- DO NOT subject the device or the parts to any strong shocks such as dropping the device on the floor.
- DO NOT store the device in extreme hot or cold temperatures, high humidity or under direct sunlight.

Storage

1. Put the nebulizer and mouthpiece in the carrying case.
-

SECTION 5—MAINTENANCE

2. Coil the air tubing.
3. Wrap the AC cord in a coil.
4. Zip the bag closed.
5. Store the device in a safe area.

NOTE: Store the repackaged device in a dry area. DO NOT place other objects on top of the repackaged device.

Filter Care

The accessible filter **MUST** be replaced after one month of use or when it turns GREY. The filter should be replaced if the nebulizer has been in storage for three months or longer.

CAUTION

Use only the filters provided with the Invacare Stratos Portable Nebulizer device.

DO NOT use cotton or any other material.

DO NOT wash or clean the filter.

Use only Invacare filters.

DO NOT operate without a air filter.

SECTION 6—TROUBLESHOOTING

PROBLEM	SOLUTION
Nebulizer overheats. The nebulizer has an internal thermal protector to prevent the device from overheating. If the internal temperature reaches the preset level, the unit will automatically shut down, and the power LED will turn RED.	Turn the unit OFF for a minimum of 40 minutes to allow the nebulizer to cool.
The nebulizer does not operate.	Check all power sources to the nebulizer. Push the ON/OFF button again.
Medication does not nebulize properly.	<p>Check for proper connection of the tubing adapter.</p> <p>Make sure the nebulizer is in an upright position and not tilted beyond 30°.</p> <p>Make sure the nebulizer is not empty.</p> <p>Clean the nebulizer as described in the manual.</p> <p>Make sure the cone has been properly placed in the nebulizer cup.</p> <p>Replace the nebulizer.</p> <p>Check the intake filter and replace it if it is dirty.</p>
Nebulizer delivers no mist.	Make sure the cone is in the cup and properly seated.

SECTION 6—TROUBLESHOOTING

PROBLEM	SOLUTION
Nebulizer does not work, and the light indicator is red.	The thermal protector has been activated due to: a. The device operating outside the conditions outlined in this manual. b. The device has been operating near heat sources or in hot environments. Allow the nebulizer to cool down for a minimum of 40 minutes. Wait until the RED light turns off before turning the nebulizer on again.
Optional battery is not working optimally.	Battery was not properly conditioned (charged for at least 16 hours before first use). If it has been in use for approximately four months, it may be time to recharge the battery.
Optional battery is not charging.	Battery will not charge if connected to the nebulizer. Check all power cord connections and cord conditions. Check power outlet.

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser who purchases this product when new and unused from Invacare Corporation or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person. This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants this product when purchased new and unused to be free from defects in materials and workmanship for a period of three years from date of purchase from Invacare or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty period any such product shall be proven to Invacare's satisfaction to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not apply to normal wear and tear or failure to adhere to the product instructions, nor does it include shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the Invacare service department during normal business hours at the toll-free telephone number identified on the back cover of this manual.

LIMITED WARRANTY

Upon receiving notice of an alleged defect in a product, Invacare will issue a serialized return authorization. It shall be the responsibility of the purchaser to pack the product(s) or part(s) in an appropriate manner to avoid shipping damage and return the product(s) or part(s), at the purchaser's expense, to either Invacare's plant or service center as specified by Invacare in advance. The defective product(s) or component part(s) must be returned for warranty inspection using the serial number as identification within 30 days of the return authorization date. **DO NOT** return any product(s) or part(s) to our factory or service center without prior consent. **C.O.D.** shipments will be refused - please pre-pay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE, OR STORAGE; PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT (INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS); PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; OR PRODUCTS REPAIRED BY ANYONE OTHER THAN INVACARE OR AN AUTHORIZED INVACARE SERVICE CENTER. SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

LIMITED WARRANTY

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESSED OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

SOME STATES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGE, OR LIMITATION OF HOW LONG AN IMPLIED WARRANTY LASTS, SO THE ABOVE EXCLUSION AND LIMITATION MAY NOT BE APPLICABLE.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2006 Invacare Corporation
trademarks of Invacare Corporation.
Stratos is a trademark of Invacare