
OWNER'S MANUAL

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Envoy™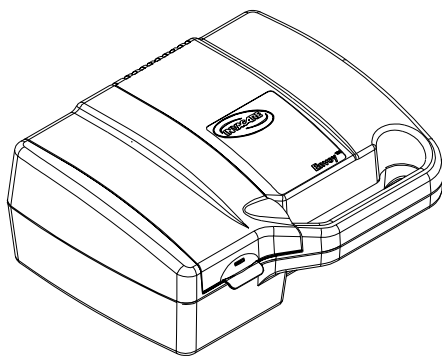

**AEROSOL COMPRESSOR
COMPRESSEUR AÉROSOL
COMPRESOR PARA AEROSOL**

DEALER: THIS MANUAL MUST BE GIVEN TO THE END USER.

USER: BEFORE USING THIS AEROSOL COMPRESSOR READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE.

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**For more information regarding
Invacare products, parts, and services,
please visit: www.invacare.com**

Yes, you can.™

SPECIAL NOTES

DANGER/WARNING/CAUTION notices as used in this manual apply to hazards or unsafe practices which could result in personal injury or property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

THIS AEROSOL COMPRESSOR IS INTENDED FOR HOUSEHOLD USE.

CHECK ALL PARTS FOR SHIPPING DAMAGE. IF SHIPPING DAMAGE IS NOTED, DO NOT USE. CONTACT DEALER/ CARRIER FOR FURTHER INSTRUCTIONS.

WARNING

DO NOT INSTALL, MAINTAIN OR OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE DANGERS / WARNINGS / CAUTIONS AND INSTRUCTIONS, CONTACT YOUR HOME HEALTHCARE PROVIDER BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

SAVE THESE INSTRUCTIONS

TABLE OF CONTENTS

SPECIAL NOTES	2
IMPORTANT SAFEGUARDS	4
SPECIFICATIONS AND SUPPLEMENTAL INFORMATION	6
FEATURES	8
HANDLING PROCEDURES	9
Shipping and Transportation Instructions	9
Unpacking	9
Inspection	9
Storage	9
OPERATION	9
MAINTENANCE	12
Cleaning the Outer Case	12
Cleaning the Nebulizer	12
Cleaning or Replacing the Foam Intake Filter	13
ACCESSORIES	13
LIMITED WARRANTY	14
GUIDE DE L'UTILISATEUR	16
MANUAL DE INSTRUCCIONES	30

IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING

“Caution: Federal law restricts this device to sale by or on the order of a physician.”

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

DANGER

TO REDUCE THE RISK OF ELECTROCUTION

ALWAYS unplug this product **IMMEDIATELY** after using.

DO NOT use while bathing.

DO NOT place or store product where it can fall or be pulled into a tub or sink.

DO NOT reach for product that has fallen into water.
UNPLUG IMMEDIATELY.

DO NOT place or drop into water or other liquid.

WARNING

TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURY TO PERSONS

This product should **NEVER** be left unattended when plugged in.

Close supervision is necessary when this product is used by, on, or near **CHILDREN AND PHYSICALLY CHALLENGED INDIVIDUALS.**

Use this product only for its intended use as described in this manual. Use this product only under physician's direction. **DO NOT** use attachments not recommended by the manufacturer.

WARNING

NEVER operate this product with a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or dropped into water; call **Qualified Service Personnel** for examination and repair.

Keep the cord away from **HEATED** or **HOT** surfaces.

NEVER drop or insert any object into any opening.

NEVER use while sleeping or drowsy.

NEVER block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free from lint, hair and the like. Blocked openings may cause the unit to shut down. Contact dealer immediately.

DO NOT use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered in a closed environment such as an oxygen tent.

This unit is oil-less. **DO NOT** lubricate.

CAUTION

Risk of electric shock. Do not disassemble. Refer servicing to qualified service personnel.

SPECIFICATIONS AND SUPPLEMENTAL INFORMATION

STANDARD SPECIFICATIONS

Electrical Requirements:	115 VAC, 60Hz, Single Phase
Running Current:	1.5 A @ 10 psig @ 115 volts (approx.)
Sound Level:	55 dBA
Ambient Temperature Range:	50°F - 104°F (10°C - 40°C)
Power Wattage:	80 watts @ 10 psig
Thermal Protector:	UL 547 Class B
Foam Intake Filter:	Scot Industrial Foam, open cell 100 ppi, polyurethane foam.
Safety Systems:	High temperature compressor shutdown.
Width:	11-inches
Height:	4.4-inches
Depth:	8.5-inches
Weight:	5lbs.
Shipping Weight:	6 lbs.
Regulatory Listing:	In compliance with UL 1431 Personal Hygiene and Healthcare Appliances and CSA 22.2 Electromedical Equipment.

ELECTRICAL

This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.

Compressor:

If the unit shuts down frequently, you may have a low voltage situation.

Low voltage can also be suspected when:

1. The motor does not get up to full power or speed.
2. Home fuses or circuit breakers activate when starting compressor.
3. Lights dim or remain dim when compressor is started.
4. Other motor operated appliances fail to operate properly or too many motor operated appliances are on same circuit.

The motor of this compressor has a thermal overload protector. If the motor should overheat, the overload protector will shut off the motor. If the unit shuts down and cannot be restarted, the motor must be replaced. Call your dealer immediately.

Extension Cords:

Make certain extension cord is in good condition. Make certain extension cord wire is NOT LESS than 18 gauge for 25 feet, 16 gauge for 50 feet and 14 gauge for 100 feet.

MAINTENANCE

Only authorized or factory-trained personnel should perform preventative maintenance.

FEATURES

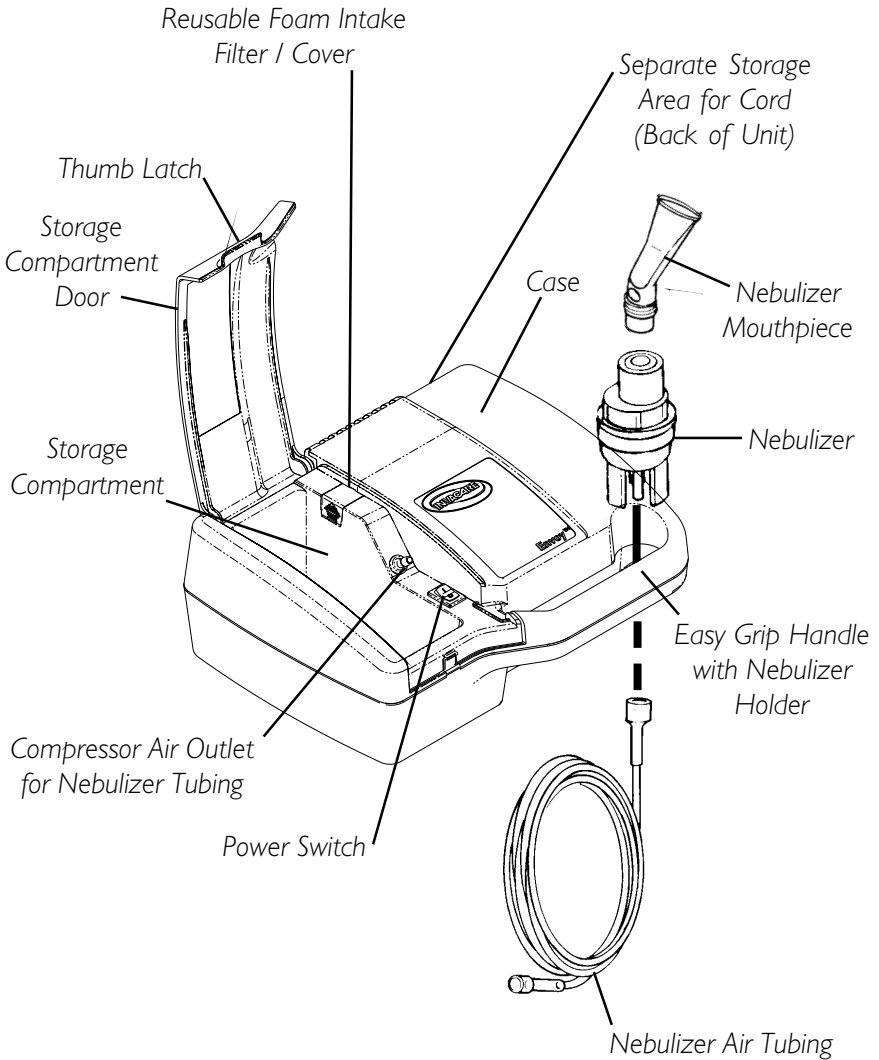
FIGURE I - FEATURES

HANDLING PROCEDURES

SHIPPING AND TRANSPORTATION INSTRUCTIONS

The shipping container has been designed to assure protection of this device.

If this device is to be reshipped by common carrier, it should be packed in the same carton. Extra cartons are available from Invacare.

UNPACKING

1. Check for any obvious damage to the carton or its contents. If damage is evident, do NOT use. Notify Carrier/Invacare or your local Invacare Dealer.
2. Remove all the loose packing from the carton.
3. Carefully remove all the components from the carton.

NOTE: Retain carton and packing materials for use in shipping or storing the unit until use is required.

INSPECTION

1. Examine exterior for nicks, dents, scratches or other damage. Inspect all components.

STORAGE

1. Store the repackaged device in a dry area.
2. DO NOT place other objects on top of the repackaged device.

OPERATION

Your nebulizer package includes a nebulizer mouthpiece, nebulizer, and air tubing. These items are sold as a package only.

NOTE: Before initial operation, nebulizer should be cleaned. Follow the recommended Cleaning Instructions in this Owner's Manual.

OPERATION

WARNING

DO NOT run unit without filter.

Avoid operating this product in a dusty environment which could cause premature compressor wear.

SETUP

1. Place the device on a table so that the controls can be reached when seated.
2. Open the storage compartment door by lifting up the latch with your thumb and pulling out with your fingers.
3. Remove nebulizer, nebulizer mouthpiece, and air tubing from the storage compartment.
4. Secure one end of the tubing to the compressor air outlet (**FIGURE 1**).
5. Connect the other end to the nebulizer.

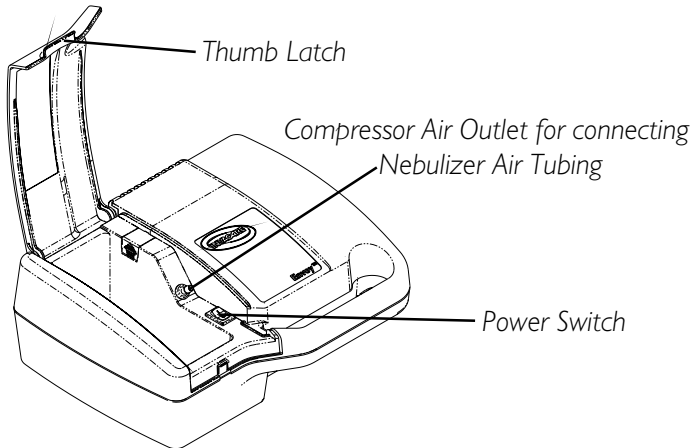
FIGURE 1 - LOCATION OF COMPRESSOR AIR OUTLET

WARNING

In the case of high humidity, condensation may form inside the internal tubing. Run the unit for two (2) minutes before connecting the nebulizer to the tubing.

6. Add liquid medication to the nebulizer according to your physician's prescription (**FIGURE 2**).

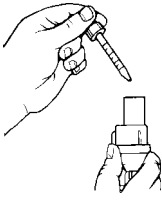
FIGURE 2 - ADDING LIQUID MEDICATION TO THE NEBULIZER

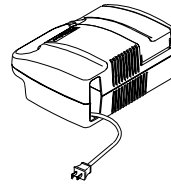
FIGURE 3 - LINE CORD LOCATION

WARNING

Before plugging in the line cord, make sure that the power switch is in the OFF (O) position.

7. Remove the line cord from the storage bin in the rear of the unit (**FIGURE 3**).
8. Plug into a 115V/60Hz wall outlet.
9. Insert the nebulizer mouthpiece into the top of the nebulizer.

NOTE: Before attempting to start a cold unit, let the unit reach room temperature in a heated area.

10. Press the power switch ON (I) to start the compressor.
11. Place mouthpiece between your teeth and close your mouth (**FIGURE 4**).

FIGURE 4 - USING THE NEBULIZER

12. As aerosol begins to flow, inhale deeply, slowly, and hold breath for 10 seconds. Then exhale slowly through the mouthpiece.

NOTE: If you need to rest during a treatment, place the nebulizer in the round holder located within the handle opening. (Refer to Features Section in this manual.)

NOTE: Unit may be operated with the storage compartment door closed. Route nebulizer air tubing through side opening in storage compartment door.

NOTE: Before storing supplies between treatments, clean and dry nebulizer. (Refer to Maintenance Section in this manual.) Tighten caps on medication jars.

MAINTENANCE

WARNING

Electrical shock hazard. DO NOT remove outer case of this unit. All disassembly and maintenance of this unit MUST BE done by a qualified Invacare Dealer.

CLEANING THE OUTER CASE

WARNING

UNPLUG unit before cleaning. DO NOT submerge in water to clean. Clean outer case with damp cloth.

CLEANING THE NEBULIZER

NOTE - IRC1192 ONLY: With proper maintenance your nebulizer should last 30 days. Since the nebulizer is disposable, it is recommended that an extra nebulizer be kept at all times.

NOTE - IRC1192D ONLY: With proper maintenance your Durable Sidestream nebulizer should last for one (1) year.

NOTE: Clean the Nebulizer after every use and at least once a day.

NOTE - IRC1192D ONLY: Durable Sidestream nebulizer is dishwasher safe.

NOTE: Your physician and/or a Invacare Dealer may specify certain cleaning procedures. If so, follow their recommendations. If not, clean according to the following instructions:

1. Remove the nebulizer and mouthpiece from the tubing.
 2. Disassemble nebulizer chamber by turning counterclockwise and separating.
 3. **Fill two (2) plastic containers or bowls:**
 - A. **Washing:** Hot water/detergent solution.
 - B. **Soaking:** Hot water/vinegar solution (one [1] part vinegar to three [3] parts water).
 4. Thoroughly clean the nebulizer and mouthpiece in hot water/detergent solution.
 5. Remove from the water/detergent solution and rinse with clear hot tap water.
 6. Soak in hot water/vinegar solution for 30 minutes.
 7. Rinse with hot tap water again and air dry thoroughly.
-

MAINTENANCE/ACCESSORIES

NOTE: If using medical disinfectant cleaners, follow manufacturer's instructions carefully.

8. Keep the outer surface of the tubing dust-free by wiping regularly.

NOTE: The nebulizer air tubing does not have to be washed internally because only filtered air passes through the tubing.

CLEANING OR REPLACING THE FOAM INTAKE FILTER

NOTE: Clean the filter once a month or sooner if filter discolors.

1. Unplug unit.
2. Open cover of storage compartment to gain access to filter.
3. Remove filter and clean with soap and water. Let dry thoroughly before use.
4. Replace filter if it becomes clogged, torn or has a worn appearance.

NOTE: Regular filter cleaning/replacing is necessary to help ensure proper compressor performance.

WARNING

DO NOT run unit without filter.

ACCESSORIES

Intake Filter

Filter Cover

Owner's Manual

MS2100 Disposable Mobil-Mist™ Nebulizer (5 ml Capacity), w/Mouthpiece, T-Piece, and 7 ft. tubing.

MS2200 Disposable Sidestream® Nebulizer (10 ml Capacity), w/Mouthpiece and 7 ft. Tubing.

MS2400 Durable Sidestream® Nebulizer (10 ml Capacity), w/Mouth-piece and 7 ft. Tubing. One (1) Year Limited Warranty.

IRCI 198 TruZone® PFM Peak Flow Meter.

NOTE: See Invacare Dealer for additional Nebulizers.

WARRANTY

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants the compressor of the Invacare Envoy Aerosol Compressor to be free from defects in materials and workmanship for a period of five (5) years from date of purchase. The nebulizer chamber, nebulizer air tubing, and any other accessories are excluded in this warranty. If within such warranty period any such product shall be proven to Invacare's satisfaction to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include normal wear and tear or shipping charges incurred in replacement part installation or repair of such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement.

For warranty service, please contact Invacare's Service Department during normal working hours at the Toll Free Number listed.

Upon receiving notice of an alleged defect in a product, Invacare will issue a serialized return authorization. It shall then be the responsibility of the purchaser to return, at purchaser's cost, to either Invacare's plant or service center as specified by Invacare in advance. Defective component part(s) must be returned for warranty inspection using the serial number as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without prior consent. **C.O.D** shipments will be refused - please prepay shipping charges.

Limitations and Exclusions: The foregoing warranty shall not apply to products subjected to negligence, accident, improper operation, maintenance or storage, commercial or institutional use, products modified without Invacare's express written consent (including, but not limited to, modification through the use of unauthorized parts or attachments) or to products damaged by reason of repairs made to any component without the specific consent of Invacare, or to a product damaged by circumstances beyond Invacare's control.

The foregoing warranty is exclusive and In lieu of all other express warranties. Implied warranties, if any, including the implied warranties of merchantability and fitness for a particular purpose, shall not extend beyond the duration of the expressed warranty provided herein and the remedy for violations of any implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein. Invacare shall not be liable for any consequential or incidental damages whatsoever.

Some states do not allow the exclusion or limitation of incidental or consequential damage, or limitation on how long an implied warranty lasts, so the above exclusion and limitation may not apply to you.

This warranty shall be extended to comply with state/**PROVINCIAL** laws and requirements.

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© 2003 Invacare Corporation
Invacare is a registered trademark of Invacare Corporation.
Yes, you can. and Envoy are trademarks of Invacare Corporation.
Sidestream is a registered trademark of Medic-Aid Ltd.
TruZone is a registered trademark of Monaghan Medical Corporation
Invacare Corporation