

# All in one peel

## User manual



## **The Theory of this unit**

### **Hydro peeling:**

The hydro peeling can remove the dead skin and the dirt in the skin effectively. At the same time, it can lead in all kinds of nutrient, make the skin health.

### **Crystal dermabrasion**

The Microdermabrasion System works by vacuuming ultra-fine aluminum oxide micro-crystals across the surface of the skin, exfoliating gently and precisely, removing dead skin cells and revealing the new softer, fresher skin below.

The finely abrasive action also stimulates the production of new skin cells, collagen and elastin, further improving the skin's appearance.

### **Diamond dermabrasion**

The Diamond Peel treatment erases epidermal layers at varying depths in a controlled manner, minimizing skin trauma. It mechanically alters the epidermis and invigorates cell renewal to create a healthy, balanced skin surface. It is then possible to improve or eliminate many skin conditions and skin imperfections.


### **Jet peeling**

It makes use of pressurized Oxygen to accelerate a liquid agent (saline) to supersonic velocities, creating an exfoliative effect on the skin. The liquid agent (saline) is compounded with different vitamins and supplements that will address the problem of the patient. The high velocity, at the speed of 200m/s, peels the superficial layers of the skin allowing better penetration of the compounded substances, and prevents skin inflammation and infection by saline solution which is put out in spray. The anaerobic bacteria in the sebaceous glands can not exist when oxygen is spray into the skin. At the mean time this system supply sufficient nutrition to the skin, accelerate the cell regrowth, subsequent reepithelialization takes place resulting in younger, healthier skin.

## Advantages:

1. Remove the epidermis cutin layer,refreshing the skin
2. Improve the cells organization,remove the defects.
3. Promote the collagen and elastic collagen regeneration,improve the mechanism,strengthen the elasticity,Reconstruct the healthy skin
4. Seperately control for the crystal fow and air pressure adjustment,can master the exfoliating depth,lightening the red and uncomfort.
5. Improve the following skin problems:Thick Epidermis cutin layer,dark skin,Ueven pigment,Wrinkle,Lack of elastic skin,Sun burn skin,.all kinds of splash,and uneven skin

## Panel illustration

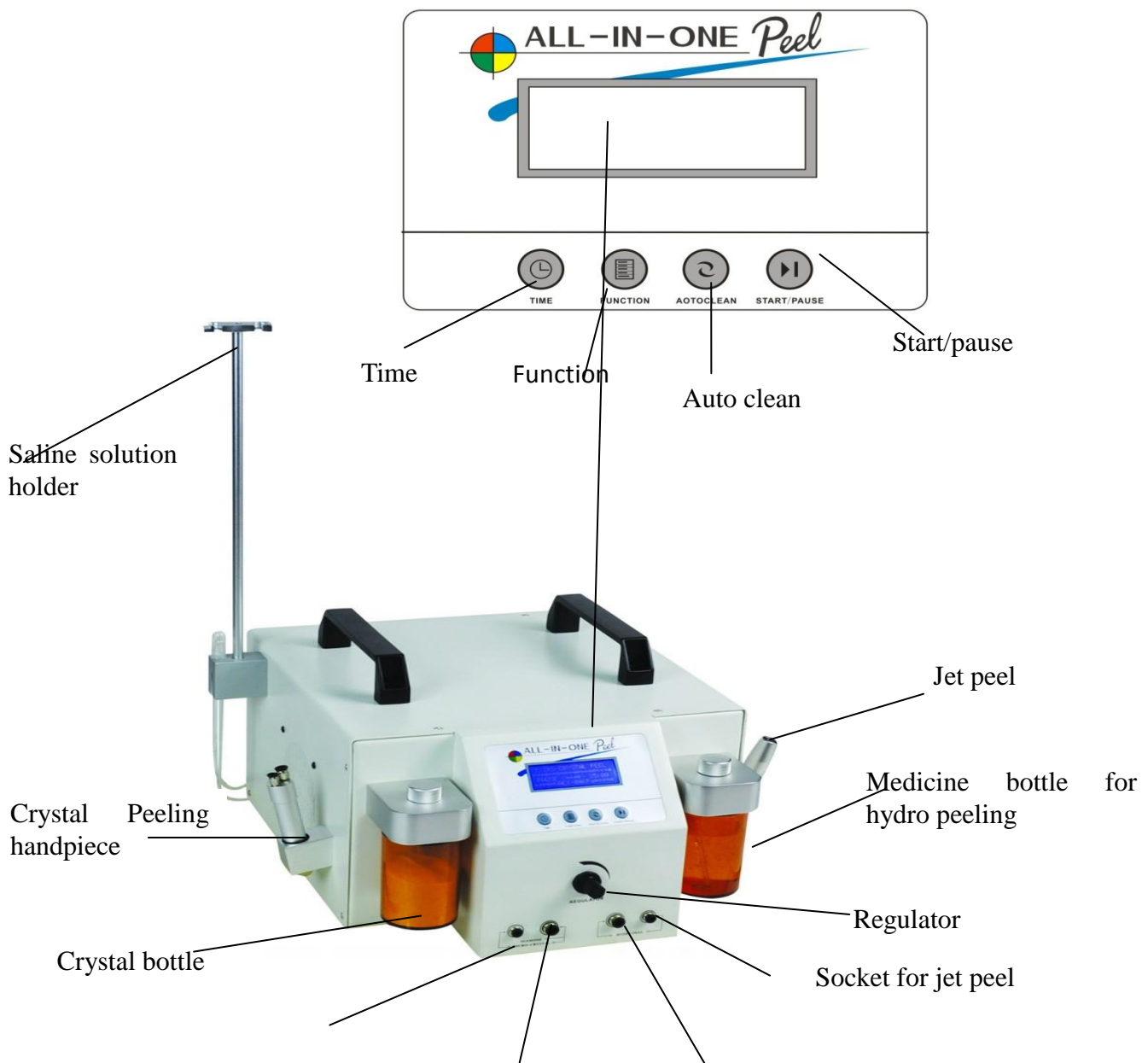

Socket for crystal  
peeling


## Operation:



### 1. Start

Plug in, connect the spare parts, press the power on button, then the machine is ready now

### 2. Jet peel



1. Connect the machine with oxygen tank or air compressor first

2. Press  to choose the function, choose JET PEEL

3. Start: Press  button, step on the foot switch at the same time, then the jet peel can work. Press  again, jet peel stop working



### 3. Hydro peeling

1. Press  to choose the function, choose Hydro peel

2. Start: Press  button, the hydro peeling start working, Press  again, Hydro peeling stop working



### 4. Crystal peeling

1. Press  to choose the function, choose Micro-crystal peel


2. Start: Press  button, the crystal peeling start working, Press  again, crystal peeling stop working

### 5. Diamond peel

1. Press  to choose the function, choose Diamond peel


2. Start: Press  button, the Diamond peeling start working, Press  again, diamond peeling stop working

### Auto clean function

1. When using micro-crystal peeling, if the crystal get stuck in the handpiece, you can press the tips of the handpieces, and press  (auto clean) to solve the problem

1. There is a bottle in the machine to prevent the machine when using the hydro peel, if the water comes into the machine, it will come to that bottle, and once the bottle is full of

water,the machine will alarm,

Then you can just press  button to drain off the water in the bottle,thus can protect the machine

### **Caution after treatment:**

1. After treatment,cool the skin or use collagen products to maintain the skin
2. Do not exposure in the sun
3. Use cold water for cleaning the face
4. If feel itchy after teatment,do not catch the face
5. According to your skin,you can adopt the second treatment every one to three weeks

### **Safety Warning**

Never operate this equipment on a person has or suffers from the following medical conditionsdescribed below. (The salon or operator may consider having a client sign a waiver to the effect that he/she is free of any poor health conditions.)

1. Heart Disease
2. Heart Murmur or Irregular Heart Rate
3. Has a Pacemaker or other type of heart rate balancer.
4. Pregnancy
5. Epilepsy
6. Alcoholism
7. Ulcers
8. Disease of Stomach, Intestine, Liver, or Pancreas.
9. Breast Cancer.
10. Undergoing treatment for any form of Cancer.
11. Had major surgery within the past year.
12. Asthma
13. Cold, Flu, or Upset Stomach
14. Any serious illness or disease not listed above.

### **Do not use on these areas:**

1. Sensitive Skin
2. Skin Rash

3. Damaged, Inflamed, or Infected Skin.
4. Surgical scar less than 12 months old.
5. Skin Swollen.
6. The probes can not point to Eye ball.

### **Product services and Maintenance**

1. The warranty date is one year from the date of purchasing. We are responsible for the repairing and the maintenance the device when the warranty date is expired, but we will charge the service fee or the cost of the component. Please contact the distributor to purchase the extra accessory or the components.

2. We do not responsible for the free warranty as the situation as following:

A. The failure caused by open the machine without any autyORIZATION by the distributor or the manufacturer. Or modification of the product in private.

B. The failure caused by dropping sown the machine.

C. The failure caused by unreasonable maintenance.

D. The failure caused by incorrect operation, not following the direction of the instruction.

3. We keep the rights to modify the system without notice.

## **Warranty Card**

<b>Maintaining sheet</b>		
Product description : _____	Purchasing date : _____	
Outlet: _____	Add: _____	Tel.: _____
Client name : _____	Add: _____	Tel: _____

Maintaining date	Fault case	Maintaining content

