

POSTURITE2

User Manual



POSTURITE2

Thank you for purchasing a Posturite 2 - an integrated postural and pressure management seating support you can enjoy all day long comfort - working or resting.

Your Posturite2 is made up from a selection of components which are featured below.




BACK OPTIONS:

WIRE BACK
WINGED BACK
TRUNK BACK

CUSHION OPTIONS:

LOWZONE GEL2
AIRZONE CONTOUR

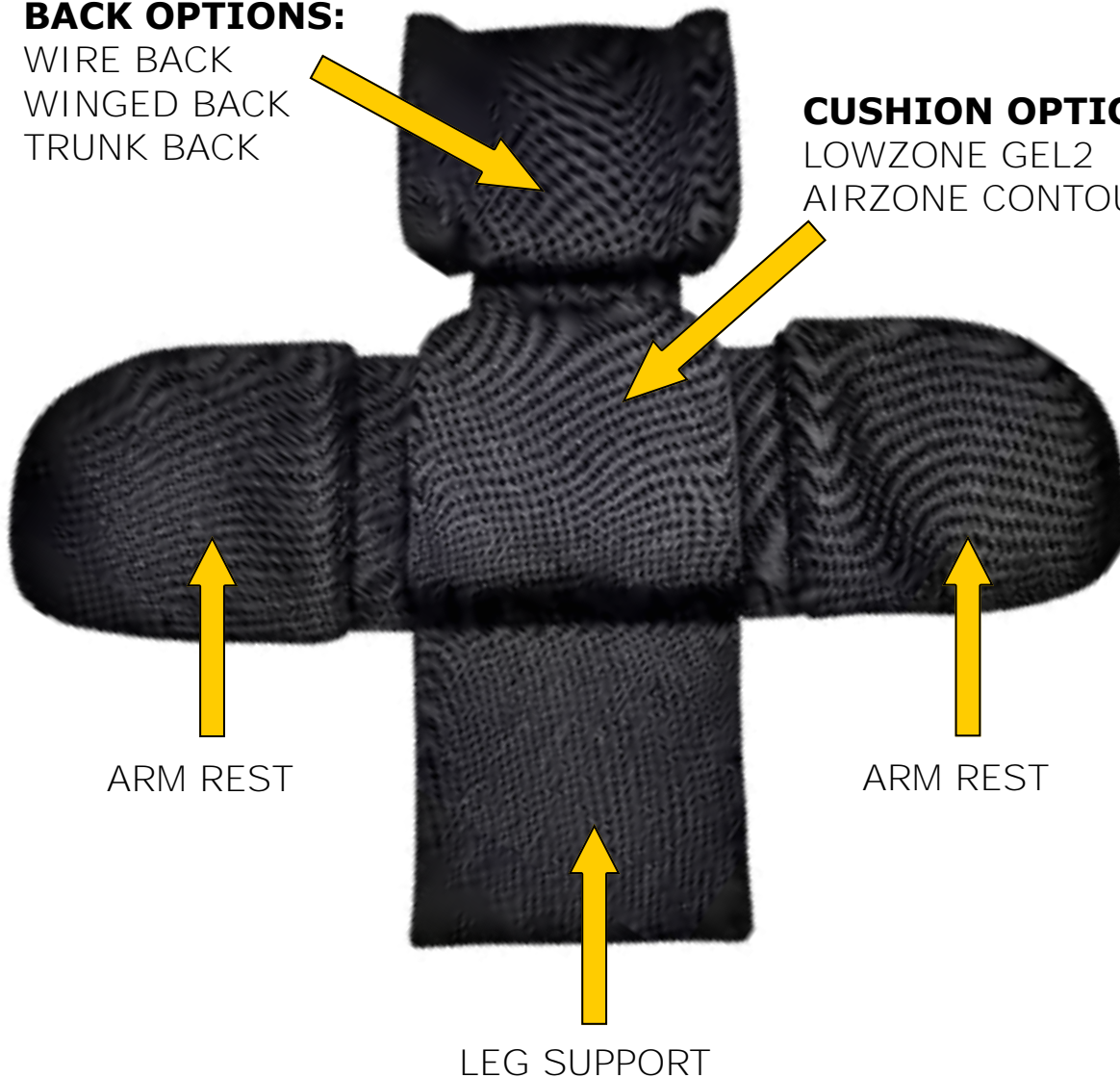

ARM REST

ARM REST

LEG SUPPORT

Component Details

Fitting & Adjustments

LEG SUPPORT

The addition of the leg support provides extra support when extended.

ARM RESTS

The arm rests can be used to improve the fit, comfort and support of your chair by either positioning over the chair arms or tucked in.



The arm rests sit over the chair and are secured in place with the adjustable strap.



To tuck in the arms, simply fold over and press the Velcro strips together.

We hope you find these instructions useful, if you have any queries please contact us on 01531 635678.



Component Details

Fitting & Adjustments

THE WIRE BACK

The Wire back offers a range of adjustment options as it can be tightened or opened out to provide trunk support, positioned higher or lower on the chair for stability and create offset lumbar support.



The back can be adjusted to provide optimum comfort and support.



The wire back is secured around the chair/wheelchair back using an adjustable strap as shown.



The wire back can be opened out to accommodate the **clients' trunk and seating arrangement**.

The width setting is held in place using the mid point strap with D rings.

Introduction & Guarantee

The Posturite2 system has been designed for those who need to adapt their seating to improve its performance. The range of components are selected to enhance your comfort and posture, providing greater pressure distribution and create a functional seating position.

Great care has been taken ensuring the Posturite II is simple to use and easy to maintain.

If you have any questions please contact your therapist

The Posturite2 is very versatile with many features:

- **Simple, lightweight, portable, supportive and comfortable to fit in any chair**
- **Adaptable back designs with active laterals that naturally adjust to fit your shape**
- **Ergonomical cushions to allow you to carry weight on the fleshy areas and away from high risk, such as coccyx and I.T.**

The system includes cushion foams, covers and straps and is guaranteed against faulty materials and workmanship for a period of **ONE YEAR** from the date of manufacture.

This guarantee is valid for reasonable wear and tear only. It is not valid in cases of misuse, or uses other than those specified in the manufacturer's instruction manual. We do not guarantee fabrics against shrinkage or staining so please pay careful attention to cleaning instructions.

Should spare covers be ordered subsequent to the purchase of this product, we cannot guarantee an exact colour match as dye strengths vary. We reserve the right to withdraw patterns or colours from the range, but will always endeavour to find the closest match available.


Certain movement patterns and behaviours may accelerate fabric wear. This cannot be covered by guarantee.

System components



When the system is first delivered it is fully assembled, folded up and clipped in place, with its own carry bag.

Simply unclip the Posturite2 and lay it out flat. Please familiarise yourself with each component using the diagram below.

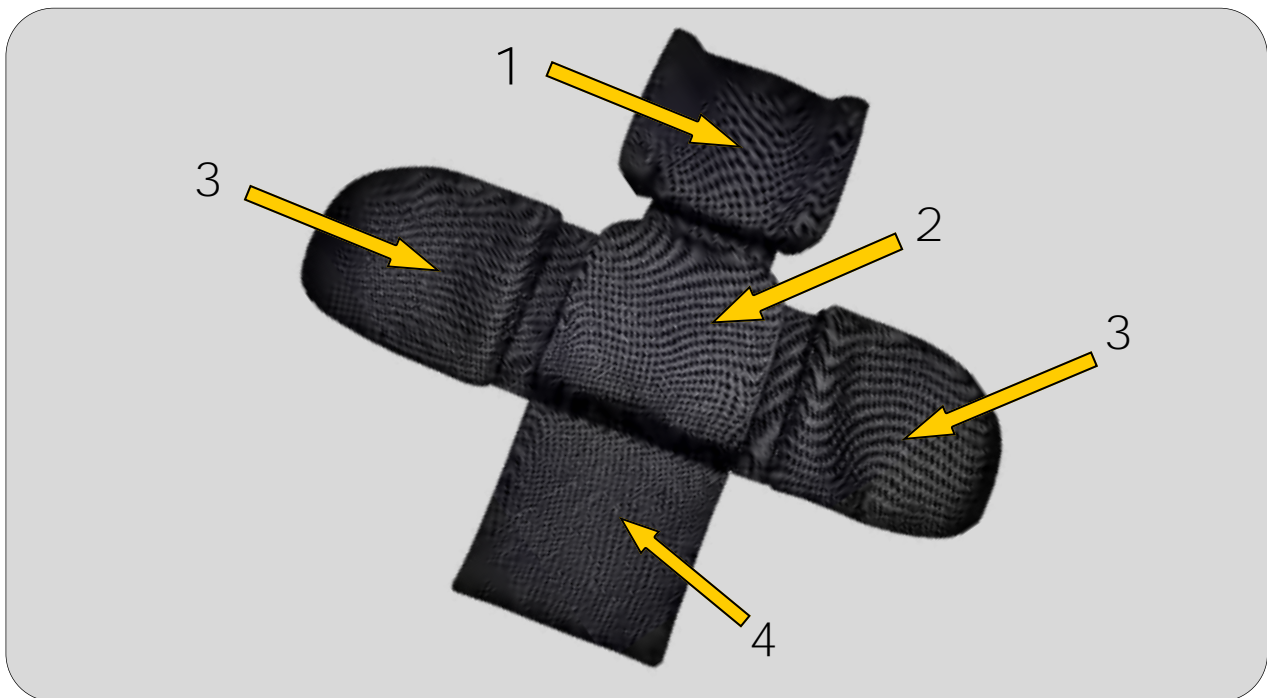

- 1. BACKCUSHION - TRUNK SUPPORT **SHOWN** (OPTIONAL)
WIRE BACK (OPTIONAL)
WING BACK (OPTIONAL)
- 2. SEAT CUSHION - LOWZONEGEL2 (OPTIONAL)
AIRZONE (OPTIONAL)
- 3. ARM RESTS - FLEXI-POSITION (OPTIONAL)
- 4. LEG SUPPORT - COMFORT LEG SUPPORT (OPTIONAL)

Once you recognise each component, please fit and adjust the unit as shown on the following pages to ensure correct adjustment and comfort.

Component Details

Fitting & Adjustments

To place the seat cushion correctly, turn the unit so the straight edge is towards the back of the chair and the rolled front edge lines up with the front of the chair.

Once the unit is placed on the chair, next lift up the back section against the chair and fit accordingly. The seat and back are connected and can be positioned to suit you.


THE WINGED BACK

Locate the base of the winged back either on top or behind the seat cushion as shown. Once in desired position, clip and tighten the webbing strap.




THE TRUNK BACK

Place the trunk back either on top or behind the seat cushion as shown. Once in place, clip and tighten the webbing strap as indicated.

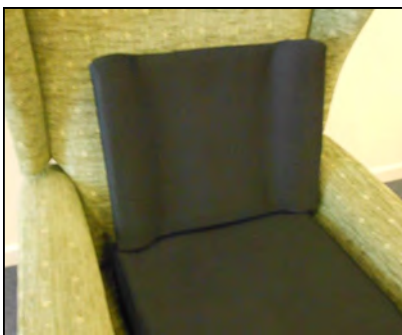

Maintenance

- The Posturite2 is suitable for long term use and designed for easy laundering both in domestic and commercial settings
- The components can still be used while the covers are being washed
- Spare covers are available to purchase

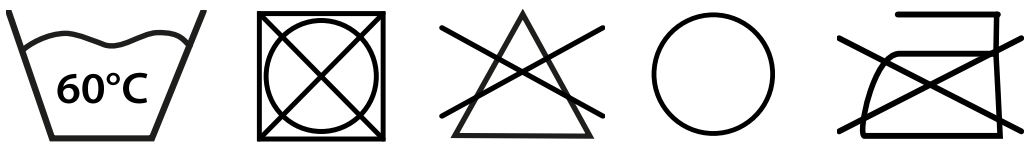

MACHINE WASHING - OUTER MESH COVER AND PU LINING

Machine wash at 60°C or less with mild detergents. Do **NOT** tumble dry. Wash the cover inside out with the positioning straps secured inside the cover.

WIPE DOWN - PU CUSHION COVERS

Wipe clean with a soft cloth moistened with water and a neutral detergent. Do **NOT** use bleach or any biological cleaners. Machine wash* or dry clean with detergent up to 60°C - some surface wrinkling may occur, however this has no adverse effect on the fabric properties. Do **NOT** tumble dry.

*ALWAYS REMOVE THE FOAM INSERTS PRIOR TO WASHING

| The Helping Hand Company offer an extensive range of products to help manage your changing needs. | | |
|---|---|---|
| Pressure Management | Postural Support | Temperature Regulation |
| Proven to prevent and relieve the symptoms of pressure and bed sores | Combining stability, postural support and pressure management | Ingenious temperature regulating products to keep you comfortable |
| For more information please contact your therapist, call us direct or take a look at our website www.lowzone.co.uk | | |

Bromyard Road,
Ledbury,
Herefordshire
HR8 1NS
Tel: 01531 635678
www.posturite2.co.uk

