




The Intel[®] Reader

Care Innovations Assistive Technologies

Kris Killough Southeast Marketing Development Manager

The Intel® Reader

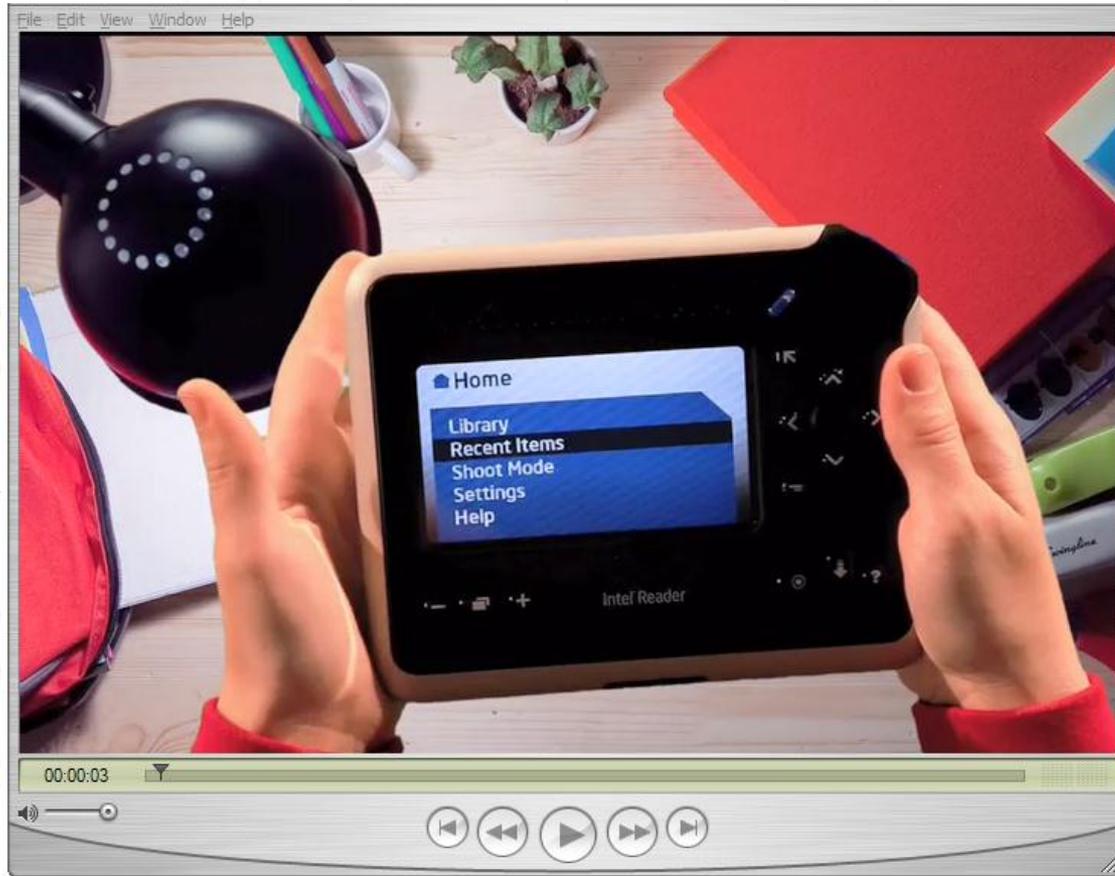

Example Users

Student



Professional



Senior



Freedom... Flexibility... Independence... Productivity

Basic Requirements

- PL-94-142, ADA, IDEA, FAPE, NCLB, etc...
- Every child deserves a free and appropriate public education
- Educated in the least restrictive environment
- Ability to learn, the ability to work, and the ability for self-care

Institution Goals

- Appropriate IEPs for student achievement
- Keep students in general education classroom
- Reduce material prep time required by teachers
- Reduce AT training cost/time for teachers, students, parents
- Reduce AT equipment and maintenance complexity and costs

How the Intel Reader Helps in Education

Institution

- Meets the IEP AT needs of students with a learning disability or visual impairment
- Goes with student, eliminates duplicate units in multiple settings
- Little training needed for teachers and parents
- Reduces teacher material prep time
- Less expensive than many other accommodations

Student

- Self-empowered
- Convert text to speech as needed
- Use in class, library, work, home
- Participate real-time in class with peers
- Minimal fine motor skills needed to operate
- Easy to learn in as little as 15 min

Student focus is on “reading to learn” rather than struggling to read

Hand-held Capture



Mobile to use when and where you need

Playback



**Words are highlighted as they are read aloud.
Use buttons to navigate the audio book, change the words per minute
and other settings, or see the original photo of the document.**

Example Uses

Users can create their own accessible audio books

Multi-modal reading

Reduce material prep time

Export Intel Reader created files to other AT devices

Import and playback other content



Export Intel Reader Files

Intel Reader file types

- JPG images
- DAISY* books
- txt file
- MP3 audio



Use on
computer




ClassMate
Reader



VictorReader
Stream



BrailleNote

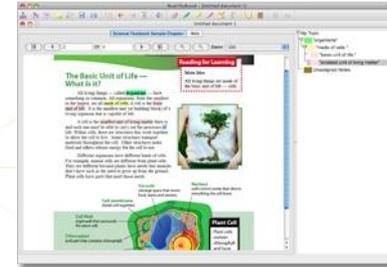

MP3 players

Import Files

**DAISY*/NIMAS*
audio books
(RFBD)**



TXT files



MP3 and WAV



Bulk Capture



The Intel Portable Capture Station makes it easy to capture multiple pages

Box Contents

Intel Reader box:

- Reader
- Battery
- USB cable
- Earphones
- Carrying case
- Quick Start Guide
- User Manual
- Warranty
- Audio CD
 - Braille label: “Reader Audio Guide”
 - WAV and audio book of User Manual and the Quick Start Guide
- Power cord and power adapter
 - Braille label: “Intel Reader”
- Practice Page
 - Braille label: “Practice Page”



Intel Portable Capture Station box:

- Capture Station
- Quick Start Guide
- User Manual
- Warranty
- Audio CD
 - Braille label: “Capture Station Audio Guide”
- Power cord and power adapter
 - Braille label: “Intel Reader”



Comparison to eReaders


General	Intel Reader	Typical eReaders
Use	<i>Captures and reads aloud most everyday printed materials</i>	<i>Displays books purchased from an online store</i>
Captures printed text	Yes	No
Read text aloud	Yes	Some
Camera	5 Megapixel	None
Content		
Read books	Yes	Yes
Read bills, menus, receipts	Yes	Some
Available content	Captures most any printed text	Limited to Content Marketplace
Accessibility		
Spoken menus	Yes	No
Audible tones for all navigation	Yes	No



Demonstration: The Intel® Reader

The Intel Reader should not be relied on as the sole means of reading when a reading inconsistency or misreading of written text might put a person at risk for personal injury or economic harm. The Intel Reader is not intended for reading primarily mathematical, scientific or financial data.

Intel Reader Controls

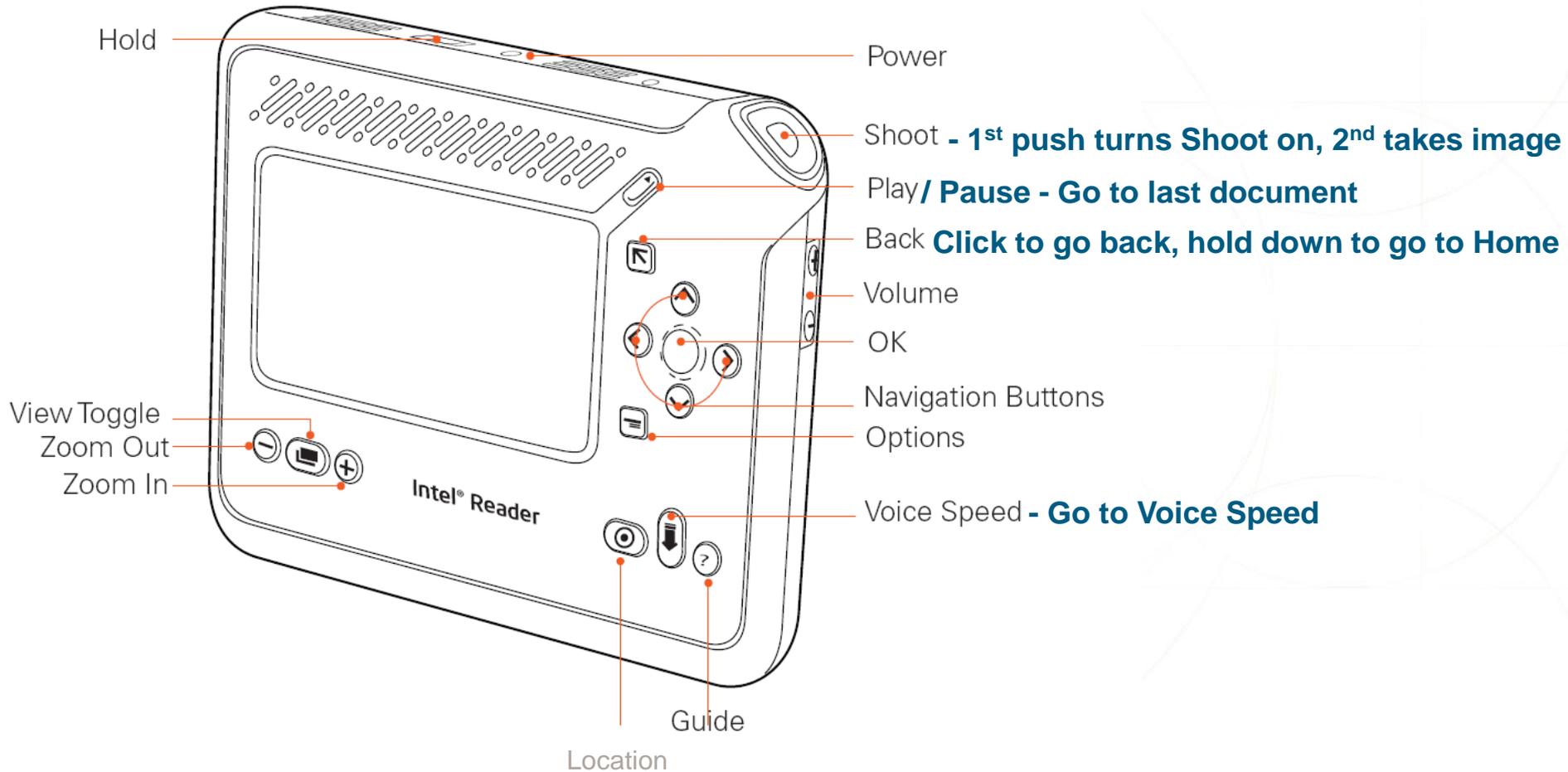

Capture Practice Page Point, Shoot, and...

1. Put page sideways on the desk in front of you
2. Press the **Shoot Button**, the Reader will announce "Shoot Mode"
3. Hold the Intel Reader above the page
4. Press the **Shoot Button** again, the device announces "Focusing"
5. The Intel Reader makes a clicking sound similar to the sound a camera makes when you take a picture
6. Turn page upside down and shoot another picture
7. Turn page sideways and shoot another




Play Practice Page ...Listen

1. Press the **Play/Pause** button to listen to text
2. Press the **Play/Pause** button to stop playback




Change the Voice Speed

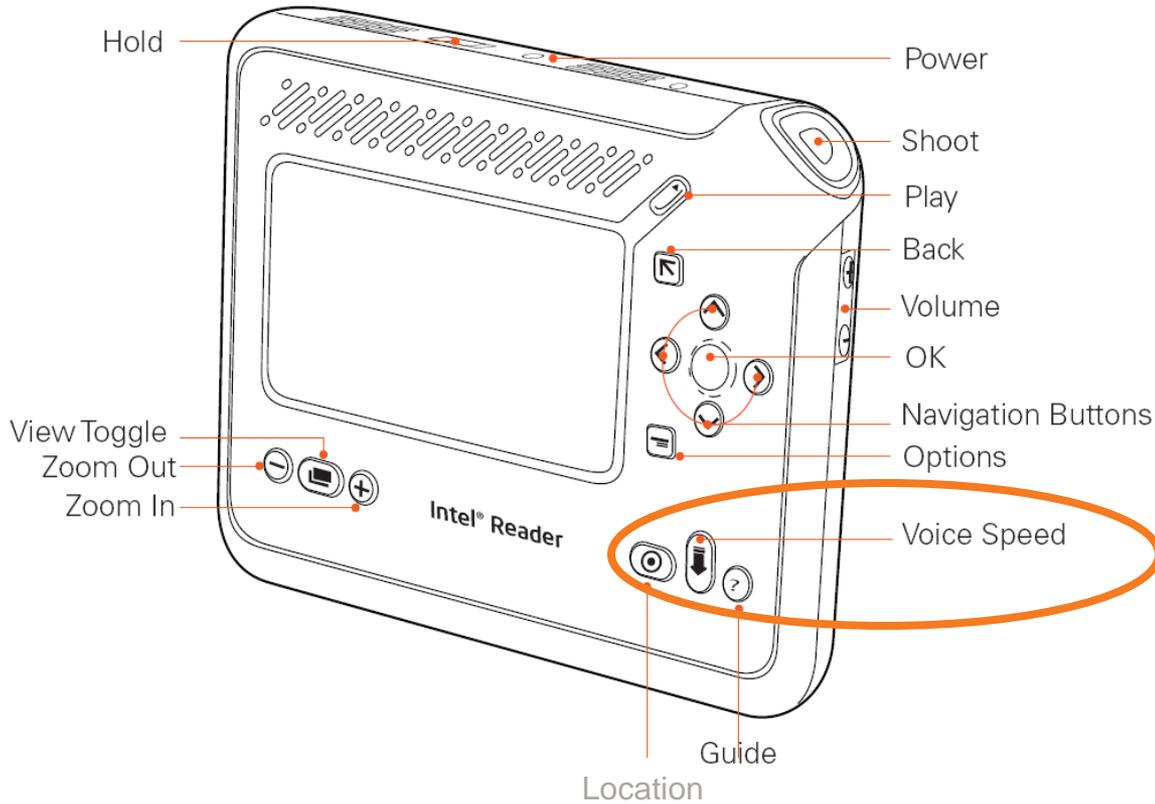


1. Press **Voice Speed** button
2. Use the **Navigation Buttons** to select a different speed
3. Press **OK**
4. Press **Play/Pause** to play the text at the new speed
5. Press **Play/Pause** to stop playback

Change the Text Size

1. Press **+** button to enlarge the text
2. Press **-** button to decrease the text
3. Press **Play/Pause** to play the text at the new size
4. Press **Play/Pause** to stop playback

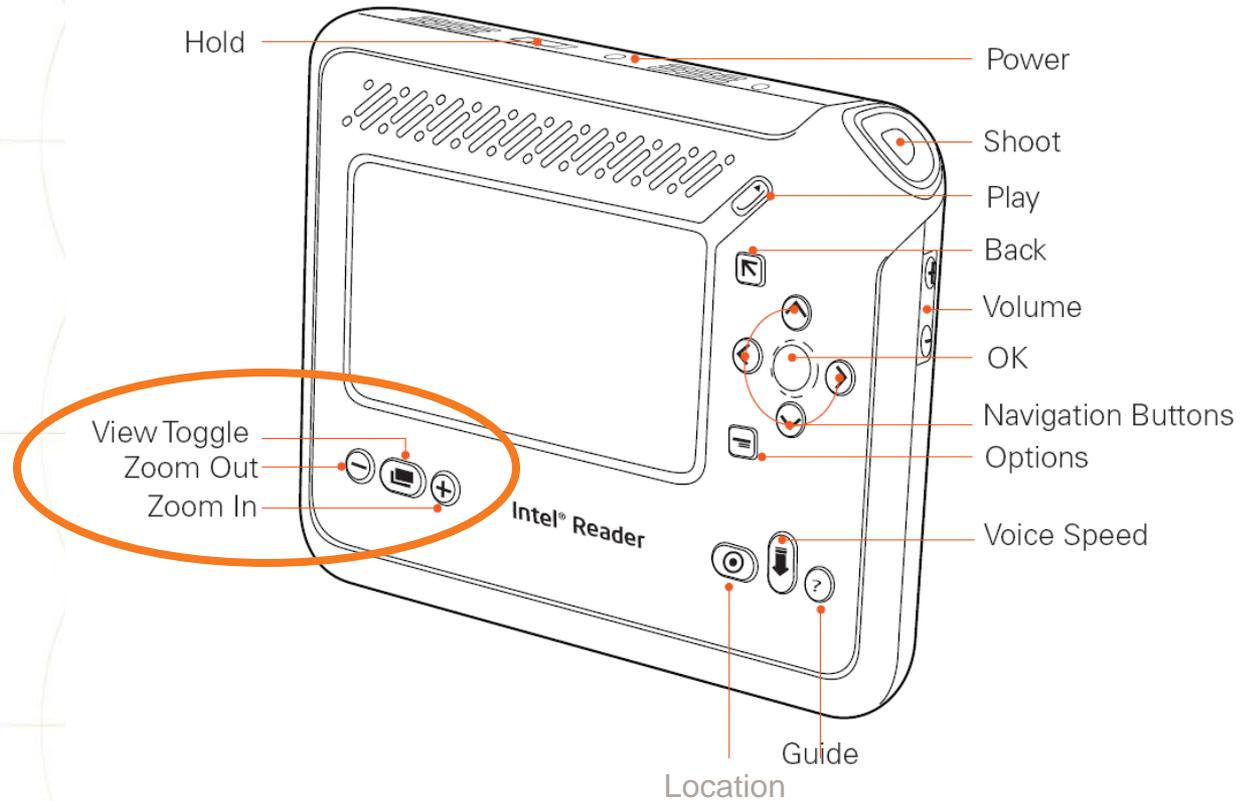


Play Mode Navigation

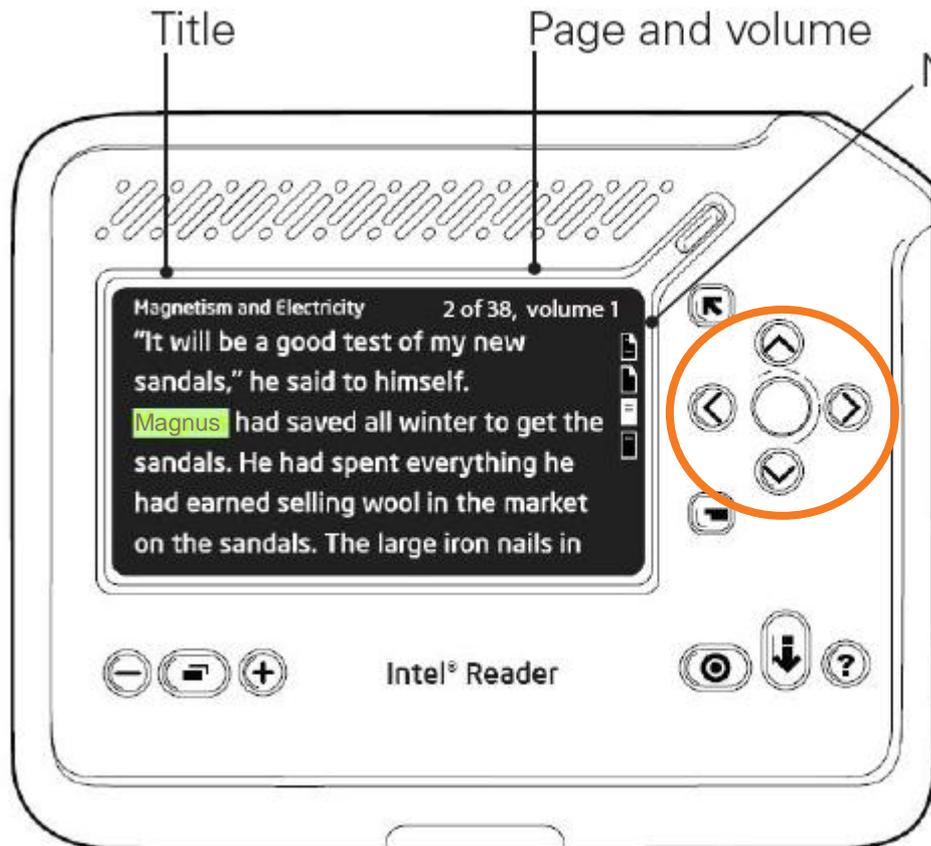

1. Up/down arrows select navigation level
 - **Heading, Page, sentence, or word.**
 - **Beginning/end** of document (hold down the arrow button)
2. Left/right arrows navigate forward/backward by the selected level




Other Features Include...

1. **Pause** playback and hold down **OK** button to **spell word**
2. **View Toggle** button toggles between **text** view and **image** view
3. **Options** button will allow you to do such things as:
 - Add pages to the end of a document file
 - Save file as an **MP3** audio file
 - Add file to a **Category**
 - **Rename** file
 - **Delete** file
 - Look at file **information**
 - Go to **Location** Menu

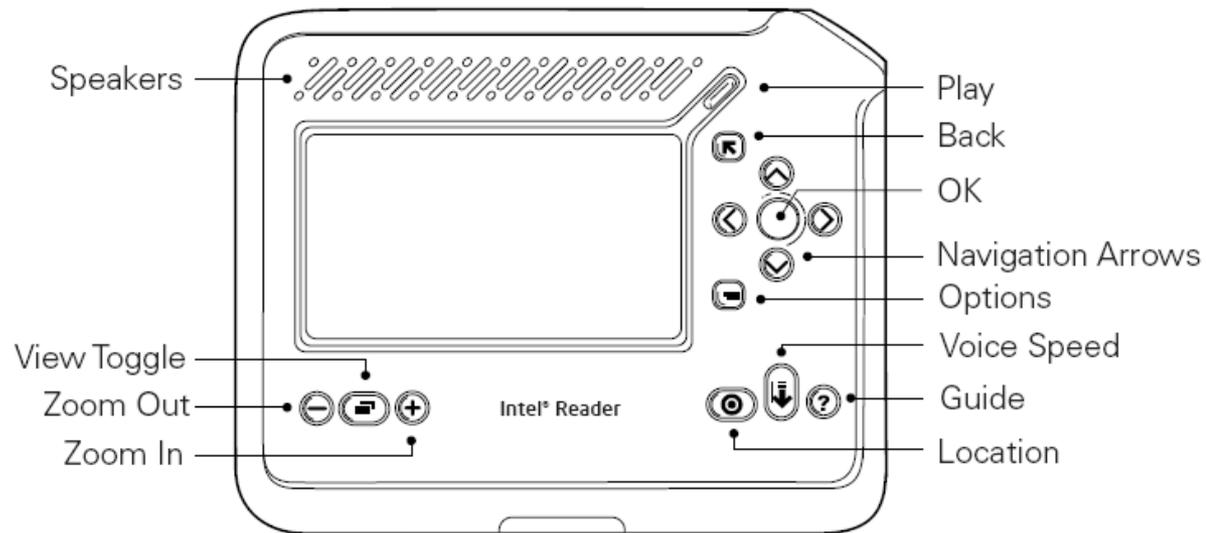

Intel Portable Capture Station

1. Put slanted part of handle away from you
2. Open case, lay flat, raise arm, swivel cradle
3. Put Intel Reader in cradle
4. Insert mini-USB plug
5. Lay book under hold-down strip
6. Align book to upper left corner
7. Capture area is 12.5" x 18"
8. Press the Blue Button on the front edge of the tray to turn on Shoot mode
9. Shoot at least 4 images
10. Press Play/Pause button

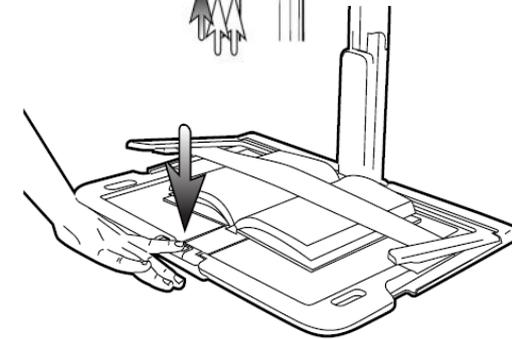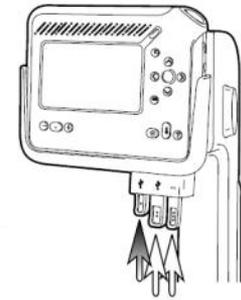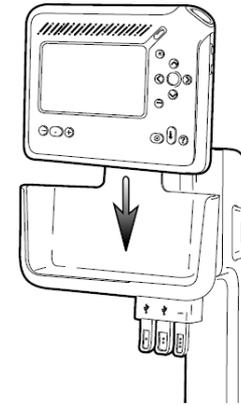



will

~~I want to~~ achieve
my full potential



Intel® Reader

Point... Shoot... Listen

Thank You