



SUPAFLEX CUSHION USER MANUAL SX32

The packaging of this mattress is sealed prior to despatch and the mattress itself will be clean prior to closure. If the mattress has become exposed or damaged during transit, please contact us at the address below

CMHR and VE layered foam cushion construction, with soft upper surface and supportive base. Two way stretch, vapour permeable, removable cover with sewn seams and zipped closure.



FOAM FOR CARE
Azalea Close, Clover Nook Industrial Park,
Alfreton, Derbyshire DE55 4QX
Tel 01773 838800 Fax 01773 838870

Flammability

The Supaflex Cushion meets the requirements of BS6807:2006 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

User Instructions

This Supaflex Cushion has been designed for use in wheelchairs and/or day care chairs, by patients who require additional comfort♣, with a weight guide of up to 95kg / 15 Stone.

Ensure that the surface of the chair seat is clear from debris before placing the cushion as debris can damage the coating of the fabric.

The label on this cushion has a unique batch number which should be used in the event of a warranty claim. NEVER remove the label.

The cushion should be placed in the seat of the chair with the castellated surface uppermost. Instructions for use are screen printed onto the cover of the product for each of reference.

♣ Indication of use based upon interface pressure readings– not a prescriptive criteria. Always defer to clinician’s judgement. Safe working load based on test methods devised at the Recticel International Development Centre, Wetteren, Belgium.

Warranty

Covered against faulty manufacture and materials as replacements goods.

Normal wear and tear expected would not be included.

All warranties subject to terms and conditions of trading.

Please refer to our warranty schedule for terms.

In order to claim under warranty refer to the batch number printed on the product label.

Removing the Cover

The cover of Supaflex Cushion is removable for cleaning purposes. Always ensure that, if removed, the cover is returned to the same inner. Using this cover on a different cushion core MAY render the warranty invalid.

The castellated foam is the upper surface should be placed into the “lid” of the cover with the flat surface on the base. The back of the cushion is indicated by a black mark on the foam itself

Cleaning

For general cleaning and superficial dirt, use a disposable wipe with a solution of warm water and neutral detergent. Do not use abrasive or phenolic-based cleaners.

To disinfect in situ, use a 0.1% sodium hypochloride solution (1,000 ppm chlorine).

Removable, contaminated covers should be laundered at 65°C for not less than 10 minutes or at 95°C for not less than 3 minutes.

Thoroughly rinse and dry the cover before re-use or storage. Store in a cool dry area.

Do not mangle or iron.

Spin and tumble-dry at not more than 60°C.

Storage & Care

- Store the cushion flat – do not fold
- Do not place heavy objects on the cushion surface when not in use
- Do not allow sharp objects to penetrate the cover coating
- Keep the cushion clean
- Regularly inspect the interior and exterior of the cushion
- Do not store in damp conditions