




The packaging of this mattress is sealed prior to despatch and the mattress itself will be clean prior to closure. If the mattress has become exposed or damaged during transit, please contact us at the address below

## TRI-FLEX CUSHION USER MANUAL

**Multi-layer CMHR foam construction with internal gel sac. Multistretch, wipe clean waterproof, vapour permeable, PU coated kitted nylon cover**

**FOAM FOR CARE**  
Azalea Close, Clover Nook Industrial Park,  
Alfreton, Derbyshire DE55 4QX  
Tel 01773 838800 Fax 01773 838870

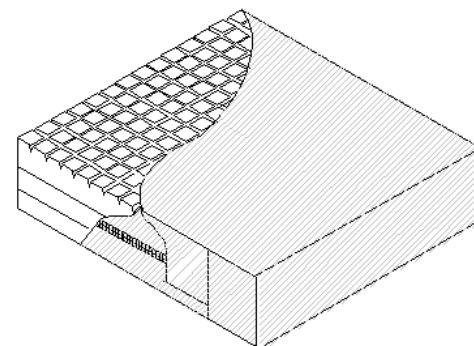

**The manufacturing process operates under Class 1 of the Medical Devices Directive by fully complying with the essential requirements for CE marking this product.**

### **Flammability**

The Tri-Flex Cushion meets the requirements of BS6807: 1996 – Section 2 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

### **User Instructions**

The Tri-flex Cushion has been designed for use in wheelchairs and/or day care chairs by patients in the high risk user category♣, with a weight guide of up to 121 Kg / 19 stone.

Ensure that the surface of the chair is clear from debris before placing the cushion – debris can damage the coating of the fabric.

Before positioning the patient on the cushion ensure the cushion is placed the correct side upwards. Ensure the Tri-Flex logo is visible at the front of the cushion.

The label on this cushion has a unique batch number which should be used in the event of a warranty claim. NEVER remove the label.

♣ *Indication of use based upon interface pressure readings– not a prescriptive criteria. Always defer to clinician’s judgement. Safe working load based on test methods devised at the Recticel International Development Centre, Wetteren, Belgium.*

### **Warranty**

Covered against faulty manufacture and materials as replacements goods.

Normal wear and tear expected would not be included.

All warranties subject to terms and conditions of trading.

Please refer to our warranty schedule for terms.

In order to claim under warranty refer to the batch number printed on the product label.

### **Removing the Cover.**

The cover of this cushion is removable for cleaning purposes. Always ensure that, if remove, the cover is returned to the same inner. Using this cover on a different cushion core MAY render the warranty invalid.

Screen printing on the cover of the cushion indicates the base and front. The castellated foam is the upper surface and should be placed into the “lid” of the cover with the flat surface on the base. The back of the cushion is indicated by a black mark on the foam itself.

The flap covers a water-resistant zip on three sides. This zip should be positioned to the side and read of the cushion

Following the above instructions will ensure that there is no front seam other than that at the lower edge of the cushion

Instructions for use are screen printed onto the base of the cushion for ease of reference

### **Cleaning**

For general cleaning and superficial dirt, use a disposable wipe with a solution of warm water and neutral detergent. Do not use abrasive or phenolic-based cleaners.

To disinfect in situ, use a 0.1% sodium hypochloride solution (1,000 ppm chlorine).

Removable, contaminated covers should be laundered at 65°C for not less than 10 minutes or at 95°C for not less than 3 minutes.

Do not mangle or iron.

Spin and tumble dry at no more than 60°C.

Thoroughly rinse and dry the cover before re-use or storage.

### **Storage & Care**

- Store the cushion flat – do not fold
- Do not place heavy objects on the cushion surface when not in use
- Do not allow sharp objects to penetrate the cover coating
- Keep the cushion clean
- Regularly inspect the interior and exterior of the cushion
- Do not store in damp conditions