

TwinSheet

Instruction for Use

immedia[®]
a part of Etac




Illustrations

Illustration 1



Illustration 2

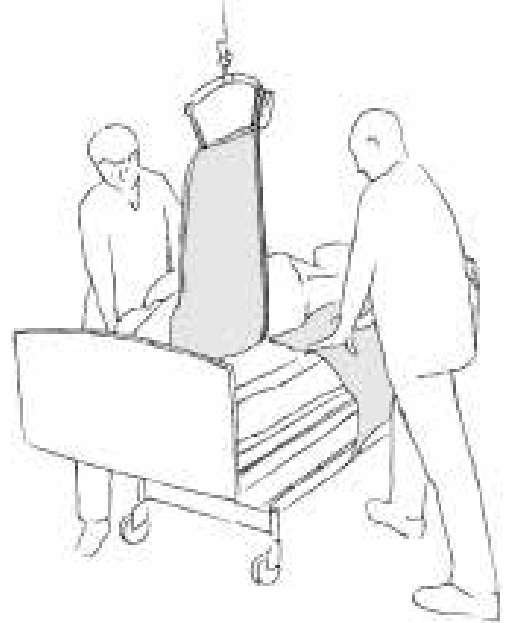

Contents

Heading	Page
General/Symbols/Intended use/Intended enviroment	4
Practical handling.....	5
Notice/Materials/Cleaning	6
Articles/Combinations.....	7
Service Information.....	8

General

Thank you for choosing an Etac product.

To avoid accidents and injury when moving and handling the products, read the manual carefully.

The person described as "the user" in this user manual is the person who is lying or sitting on the product. The assistant or assistants are the people who manoeuvre the product.



This symbol appears alongside the text in the manual. It draws the reader's attention to points at which there may be a risk to the health and safety of the user or assistant.

The products comply with the standards applicable for Class 1 products in the European Council Directive MDD 93/42/EEC on medical devices.

Lot no/Batch no:

x	x	x	x	x	x	x	x	x	x	x
Production year					Running number					

At Etac we strive to improve our products all the time and therefore we reserve the right to make changes to products without prior warning. All measurements given on illustrations and similar material are for guidance only and Etac cannot be held liable for errors and defects.

The information given in this manual, including recommendations, combinations and sizing, does not apply to special orders and modifications. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided. If in doubt, please contact Etac.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly.

For further information on the Etac transfer range, see www.etac.com.

The product can be scrapped in accordance with national regulations.

Symbols

	Washing		Iron		Never leave on the floor		User mass limit = maximum rated load
	Handwash		Do not dryclean		Warning		The product can be scrapped in accordance with national regulations
	Wipe off		Do not bleach		Read the User Manual		Manufacturer
	Tumbledry		Risk of sliding down		Batch no./Lot no.		Class 1 - European Council Directive MDD 93/42/EEC on medical devices
	Keep out of the sun						

Intended use

The TwinSheet is a 2-part absorbent sheet which makes it easy for assistants to turn a user into the side position in connection with hygiene and baths, wound care, etc.

The TwinSheet can be used on its own or with turning systems (see combinations).

Intended environment

Acute care, Long-term care, Home care

Practical handling

Positioning

Fit the TwinSheet down part first. Then fit the Twinsheet top part (Illustration 1). Close the Velcro fasteners.

Position the user so that the TwinSheet seam does not irritate him/her.

Use

If possible, bend the users legs.

Attach a lift to the functional handles on the top part of the TwinSheet. The user can be turned into the side position by raising the lift. The assistants can either push down on the TwinSheet or pull on the handles, turning the user in the middle of the bed (Illustration 2). Make sure that the users spine does not twist.

Positioning

If the user is to remain in the side position after bath/care, the users back can be supported using a positioning wedge or pillow.

After moving the user, it will often be beneficial to smooth the glide system under the user. To do this, pull diagonally on the corners, preferably one layer at a time.

Notice

Special features

Always check the product before use and after washing.

Never use a defective product. If the product shows signs of wear-and-tear, it must be scrapped.

Read these instructions carefully.

It is important that assistants receive instruction in manual handling. Etac offers advice and training for assistants. For further information, contact Etac.

Always use the correct manual handling techniques.

Encourage the client to assist where possible.

To ensure that the client feels safe and that every manual handling is smooth, always plan the manual handling in advance.



There is a risk that the user may slide off. Never leave the user alone on the edge of the bed.



Never leave the product on the floor.



Always make a risk assessment, and secure that the assistive product can be used at the individual user and in combination with other devices that it is safe for the user and caregivers.



It is recommended that the guard rail is in place when the user is left on the glide system, unless a risk assessment has found that the user can be safely left on the glide system without the need of guard rails.



Always lock the glide system when you have completed the transfer.



Do not use the product as a sling.

If in any doubt - please contact Etac.

Materials

Absorbtion: 2200 ml/m².

Top part: polyester. Green.

Middle layer: 80% polyester/20% viscose.

Barrier: polyurethane.

Backside: polyester. Blue.

Handles: polyester.

Cleaning

The product should be checked regularly, preferably each time it is used, and especially after wash.

Check that there is no damage to seams or fabric.



Never try to repair a product yourself.
Never use a defective product.

Washing instructions:

Do not use fabric softener.



Articles

Article no.	Description	Size (mm)
IM150/75	TwinSheet, top part, absorbent sheet	W1500 x L750
IM150/85	TwinSheet, down part, absorbent sheet	W1500 x L850
IM4306	TwinSheet Set (1 x IM150/75 + 1 x IM150/85)	
IM4310	TwinSheet Set (1 x IM150/75 + 2 x IM150/85)	

Combinations

Article no.	Description	Size (mm)
IM53NS	Positioning Wedge with non-slip cover	W300 x H200 x L530
IM53/80NS	Positioning Wedge with non-slip cover	W300 x H200 x L800
IM140/200LPL	Glide mattress without non-slip locking system, green	W1400 x L2000
IM140/200LPLG	Glide mattress without non-slip locking system, grey	W1400 x L2000
IM140/200PU	Waterproof cotton sheet	W1400 x L2100
IM85/2003LPL	Nylon sheet with 3 x non-slip locking system	B850 x L2000
IM85/200LPL	Nylon sheet with non-slip locking system	B850 x L2000
IM85/200LPLNS	Nylon sheet with non-slip along edge + non-slip locking system	W850 x L2000

Service Information

Thank you for choosing an Etac product. If the customer makes adjustments, repairs or combinations not predetermined by Etac, the Etac CE certification and Etac warranty will be voided.

Warranty: Two-year warranty on material and manufacturing defects, provided that the product is used correctly. Before taking a product in service always make a risk assessment according to national and/or local legislation.

First inspection:

- Is the packaging intact?
- Read the label on the packaging and check the article no. and product description
- Check that Short Instruction is enclosed – updated User Manual/Instruction for use can be downloaded from www.etac.com or contact your Etac Customer Service or the local distributor.
- Check the label on the product - does it include article no., product description, lot/batch no., cleaning instructions and supplier name?

Periodic Inspection:

- Make sure that your Manual transfer system always is in a perfect condition.
- After wash – always check materials, stitching, handles, buckles.
- If the product shows signs of wear and tear, it must be removed from service immediately.
- Lot no/Batch no:

x	x	x	x	x	x	x	x	x	x
Production year		Running number							



Visual inspection/Check the product:

Ensure that the materials, seams, stitching, handles, buckles are intact/faultless.



Stability test:

Try to bend the product verify that it feels firm, stable and solid.



Mechanical load or stress/Test handles:

Draw hard in the handles in opposite directions and control the material, seams and stitching.
Test buckles:
Lock the buckle and draw in opposite directions, control the buckle and the stitching.



Test low/high friction:

Place the product on a firm surface or a bed, place your hands on the material and test the friction by pushing your hands down into the product.
Low friction – slides effortlessly
High friction – no sliding/moving

Check always:

<p>Manual transfer aids with handles, straps and buckles:</p>	<p>Manual transfer aids for sitting and lying transfer:</p>
<p>Manual transfer aids with high or low friction:</p>	<p>The product can be scrapped in accordance with national regulations</p> <p>Read the User Manual</p>

If in doubt, please contact Etac Customer Service or local distributor for more information and guidance.
For more information about training and the use of these products, contact the Etac transfer experts – www.etac.com.

Manufacture:

Etac Immedia A/S
Egeskovvej 12
DK-8700 Horsens, Denmark
etac.immedia@etac.com

Export:

Etac Immedia A/S
Egeskovvej 12
DK-8700 Horsens
etac.immedia@etac.com
www.etac.com/exp

Etac Sverige AB
Box 203, 334 24 Anderstorp, S
Tel +46 371-58 73 00
Fax +46 371-58 73 90
info@etac.se www.etac.se

Etac AS
Pb 249, 1501 Moss, N
Tel +47 815 69 469
Fax +47 69 27 09 11
hovedkontor@etac.no www.etac.no

Etac A/S
Egeskovvej 12, 8700 Horsens, DK
Tel +45 79 68 58 33
Fax +45 75 68 58 40
info@etac.dk www.etac.dk

Etac GMBH
Bahnhofstraße 131, 45770 Marl, D
Tel +49 236 598 710
Fax +49 236 598 6115
info@etac.de www.etac.de

Etac Holland BV
Fluorietweg 16a, 1812RR Alkmaar, NL
Tel +31 72 547 04 39
Fax +31 72 547 13 05
info.holland@etac.com www.etac.com

R82 UK Ltd.
Unit D4A, Coombswood Business Park, East Coombswood Way,
Halesowen, West Midlands B62 8BH, UK
Tel 0121 561 2222
Fax 0121 559 5437
R82uk@r82.com www.r82.co.uk

Snug Seat Inc.
12801 E. Independence Blvd., P.O. Box 1739, Matthews, NC 28106, US
Tel 1 800 336 7684
Fax 704 882 0751
information@snugseat.com www.snugseat.com

The logo for Etac, featuring a stylized 'e' inside a circle followed by the word 'etac' in a bold, lowercase sans-serif font, with a registered trademark symbol (®) to the upper right.

Creating possibilities