


USER MANUAL FOR **CAMRY** KNEE CPM.




- 1) Whenever we switch ON the machine

It displays: -

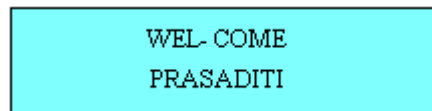

If the splint is not at zero position, it will take initialization first and become completely straight.

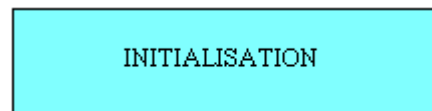

- 2) After initialization, put patient's leg on machine splint with knee exactly matching to machine hinge. If required adjust length of femoral splint by loosening bolt

below that splint. Tighten that bolt after length adjustment. Apply straps on leg tightly, it displays: -

FLECTION 000
MAX ANGLE 110

Press UP/ LOW keys to set desired flexion angle and press NEXT key.
UP key will increase flexion angle by 5 degrees, LOW key will decrease flexion angle by 5 degrees

3) It then displays: -

TRETEMENT 20 MIN
ADJ & PRESS NEXT

Press UP/ LOW keys to set timer and press NEXT key.

4) It will display: -

HOLD TIME 04 SEC
ADJ & PRESS NEXT

Press UP/ LOW keys to set desired Hold time and press NEXT key.

When you will press NEXT key on this screen, the treatment will be started.

USING CAMRY FOR FIX FLECTION DEFORMITY

1) On the very first screen of "FLEXION ANGLE 000", press and hold DEL key for 3 sec. It will display: -

ADJUST SPLINT &
PRESS NEXT KEY

2) Then it displays: -

CURRENT ANGLE xxx
ADJ & PRESS NEXT

Use UP/LOW keys to bring empty splint up /down to adjust splint as per the Patient's comfort / deformity angle and press NEXT key.

3) Then it displays: -

FLEXTION xxx:
MAX ANGLE 110

Press UP/ LOW keys to set desired flexion angle and press NEXT key.

4) It displays: -

EXTENSION XXX
MAX ANGLE 000

Press LOW key to set desired extension angle and press NEXT key.

5) It then displays: -

TRETEMENT 20 MIN
ADJ & PRESS NEXT

Press UP/ LOW keys to set timer and press NEXT key.

6) It will display: -

HOLD TIME XX SEC
ADJ & PRESS NEXT

Press UP/ LOW keys to set desired hold time and press NEXT key.
When you will press NEXT key on this screen, the treatment will be started

ONLINE MENU

You can use following functions when the process is ON.

1. Patient can hit **RED PANIC KEY** on remote in unbearable pains.
2. To increase or decrease set flexion angle online, press UP key **while the splint is coming up** that is flexing. It will stop at whatever position it is and display set flexion angle initially by you: -

FLEXTION xxx:
MAX ANGLE 110

Press UP/ LOW keys to do necessary modifications and press NEXT key. The process will be continued further and set angle will be **modified from the next cycle only**.

3. You can terminate the process anytime in between by pressing DEL.
4. Always handover remote with panic key to patient which will help him in unbearable pains. There should be always one attendant to monitor and use ON-OFF switch directly in any extreme conditions to switch OFF the supply.