



SPECIAL STROLLER

RACERevo™



CE

USER MANUAL AND MAINTENANCE GUIDELINES

AKCES[®]MED



CAUTION! Manufacturer takes responsibility for proper usage of the device only when the product was purchased from qualified Akces-Med employee or at specialized medical store.

QUALITY
MANAGEMENT
SYSTEM



ISO 13485
CERTIFIED COMPANY

Medical product first class according to the Regulation 93/42/EEG of European Parliament and Council regarding medical products. For this product the producer issued relevant declaration of conformity.

Producer's identification number in the Registration Department of Healing Products,
Medical Products and Biocide Products:

PL/CA01 04088/W

Akces-Med Sp. z o.o. Company introduced quality management system
ISO 13485 regarding production, sales and service.
The system is certified by Det Norske Veritas..

CONTENTS:

1. The purpose of user manual.....	6
2. Indications for use.....	6
3. General characteristics of RACER evo stroller.....	7
3.1. Standard equipment.....	7
3.2. Additional equipment.....	8
4. Weight and measurements.....	10
5. Size table.....	10
6. Description how to assemble racer evo stroller.....	11
7. Detailed description how to operate RACERevo stroller.....	11
7.1 How to unfold stroller.....	11
7.2 How to fold stroller.....	13
7.3 Adjustment of footrest's length.....	14
7.4 Adjustment of backrest's tilt angle.....	15
7.5 Adjustment of footrest position.....	15
7.6 Foot stabilizing belts.....	15
7.7 Head rest.....	15
7.8 Vest.....	16
7.9 Side pelottes.....	16
7.10 Abduction block.....	17
7.11 Wheels.....	17
7.12 Hand brake.....	18
7.13 Safety rail.....	18
7.14 Adjustable handle.....	18
7.15 Covers for folding mechanism.....	19
8. Assembly of additional equipment.....	20
8.1 RC_001 Thigh abduction belts.....	20
8.2 RC_002 Sun umbrella.....	21
8.3 RC_003 Table.....	21
8.4 RC_005 Canopy.....	22
8.5 RC_004 Canopy with cover.....	22
8.6 RC_006 Bag.....	23
8.7 RC_007 Pelvic belt.....	23
8.8 RC_008 PU wheels.....	24

8.9 RC_010 Headrest cotton cover	24
8.10 RC_011 Cup holder.....	24
8.11 RC_012 Elastico cushion seat.....	25
8.12 RC_017 Winter footmuff.....	26
8.13 RC_029 Hood for winter footmuff.....	27
8.14 RC_028 Bag DeLux.....	27
8.15 RC_069 Men's bag.....	28
8.16 RC_070 Lady,s bag.....	28
8.17 RC_018 Low head supporting pannels.....	28
8.18 RC_019 High head supporting pannels.....	28
8.19 RC_020 Canopy with side covers.....	29
8.20 RC_026 Trunk belt.....	30
8.21 RC_065 Narrowing pads (set) 5 cm.....	30
8.22 RC_034 Narrowing pads (set) 3 cm.....	31
8.23 RC_071 Tray Plexiglas®.....	31
8.24 RC_015 Calf belt with side supports.....	32
8.25 RC_014 H Harness.....	33
8.26 RC_025 6 points harness.....	34
8.27 RC_048 Oxygen tank holder.....	35
8.28 RC_037 Rain cover.....	37
8.29 RC_022 Side pannels.....	38
9. The use of Racer evo stroller.....	39
9.1. Preparation of the patient.....	39
9.2. How to use Racer evo stroller.....	40
10. Safety rules.....	41
11. Maintenance and cleaning.....	44
12. Stroller's marking.....	46

INTRODUCTION

RACERevo stroller was created with the aim to help patients with illnesses which make independent sitting and standing difficult or impossible. (It refers to triplegia, paresis and serious paraplegia)

Taking sitting position by a child who is disabled from birth is highly important stage of complex rehabilitation process. Sitting position precedes taking an upright position. What is more, it has a significant influence on psychomotor development of the child. The fact that the patient can be safely transported is beneficial not only for patients but also for his/her care person. Proper use of safety wedges, belts and pelottes makes it possible to have correct upright standing, even in case of patients with flaccid paralysis. It prevents contractures as well as further deformations. Moreover, stroller is additionally equipped with swivel front wheels 360°. Regarding stroller, it is designed to be used indoors and outdoors.

Remember to read this user manual carefully. You will find not only useful information but also guidelines regarding safe and effective use of the product.

If you have any questions or remarks, do not hesitate to contact our medical consultant (ph. 508 382 509) or directly AKCES-MED Sp. Z o.o. (ph. 17 864 04 70).

Thank you for purchasing our product.



CAUTION! Before using the product, user is obliged to familiarize himself/herself with user manual. Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.

1. THE PURPOSE OF USER MANUAL

The present user manual contains basic information which is essential for proper usage, preparation, maintenance, cleaning and guarantee conditions regarding RACERevo stroller.

User manual should be kept in easily accessible place. It is designed for people who look after disabled children, and for both doctors as well as physiotherapists who operate the device.



CAUTION! Before using the product, user is obliged to familiarize himself/herself with user manual.

Remember that complying with guidelines is highly important. It assures safety and extends durability as well as aesthetics of the device.

2. INDICATIONS FOR USE


The product is recommended for patients with :

- Cerebral palsy – different types.
- Myelomeningocele.
- Brain and cord injuries – different etymology.
- Muscle disorders connected with paresis and paralysis.
- Genetic diseases connected with paresis.
- After brain and spine injuries (spine injuries: spinal cord and lower limbs injuries)
- Other diseases connected with paresis, paralysis and damages of the locomotor system which make it impossible to move single-handedly.

3. GENERAL CHARACTERISTICS OF RACERevo STROLLER

3.1 Standard equipment

1. Adjustable and ergonomically shaped handle
2. Backrest
3. Seat pillow
4. Head rest
5. 4 points vest
6. Side pelottes
7. Safety rail
8. Thigh abduction belt
9. Footrest with stabilizing belts
10. Hand brake's lever
11. Clip for backrest angle adjustment
12. Folding mechanism
13. Frame
14. Cover for folding mechanism
15. Calf side supporting pannels
16. Swivel wheel lock
17. Wheel nut
18. Pumped wheels
19. Pumped wheels

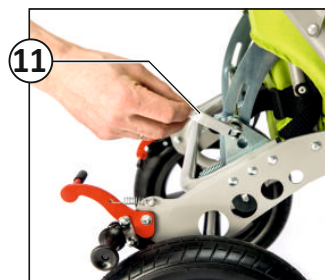

3.2 Additional equipment





4. WEIGHT AND MEASUREMENTS

Description	Weight [kg]		Stroller measurements [cm]			Measurements of folded stroller [cm]		
	User weight	Stroller weight	Stroller height	Stroller width	Stroller length	Stroller height	Stroller width	Stroller length
Pictogram								
Size 1	35	21	107	65	123	58	65	110
Size 2	50	23	106	64	135	58	64	110
Size 3	60	26	103	69	165	58	69	125

5. SIZE TABLE

Description	Pictogram	Seat width [cm]		Seat depth [cm]		Backrest width [cm]		Backrest height [cm]		Distance from seat to footplate [cm]		Backrest tilt angle [°]		Footer length [cm]		Distance from backrest to abduction block [cm]		Footplate tilt angle [°]		Weight [kg]		Stroller measurements [cm]					Measurements of folded stroller [cm]		
		Min	Max	Min	Max	Min	Max	Min	Max	Min	Max	Min	Max	Min	Max	Min	Max	Min	Max	User weight	Stroller weight	Stroller height	Stroller width	Stroller length	Stroller height	Stroller width	Stroller length		
Size 1		-	32	-	33	-	30	-	65	-	28	-	120	-	24	-	19	-	16	110	35	21	107	65	123	58	65	110	
		32	33	30	65	35	120	24	19	16	130																		
Size 2		-	34	-	44	-	32	-	70	-	30	-	120	-	24	-	29	-	26	110	50	23	106	64	135	58	64	110	
		34	44	32	70	36	120	24	29	26	130																		
Size 3		-	40	-	46	-	36	-	82	-	40	-	120	-	24	-	30	-	30	110	60	26	103	69	165	58	69	125	
		40	46	36	82	45	120	24	34	30	110																		

6. DESCRIPTION HOW TO ASSEMBLE RACER EVO STROLLER

Remember that stroller needs to be assembled. Manufacturer cannot assemble this stroller due to transportation issues. We will give you stroller which needs to be mounted.

In order to use the stroller please follow these steps:

STEP 1. Unfold stroller (-> look point 7.1, page 14);

STEP 2. Place side pelottes (-> look point 7.9, page 16), seat pillow, covers for folding mechanism (-> look point 7.15, page 19)

7. DETAILED DESCRIPTION HOW TO USE RACER EVO STROLLER

7.1. How to unfold stroller

Follow these steps so as to unfold stroller:

STEP 1. Release clip that regulates backrest's tilt angle (11) (fig.1), pull backrest (2) as backwards as possible (fig.2) and then block it with the aid of clip.



STEP 2. Pull handle (1) backwards until elements of folding mechanism are slammed (12) (fig.3). It is highly important to make sure that safety rail (13) is lifted before folding mechanism is slammed. Next, move safety rail down (fig.4) once folding mechanism is slammed.



STEP 3. Mount safety rail (7): place it on stroller frame and block it by pressing simultaneously buttons that are situated on both sides of rail (fig.5);



7.2. How to fold stroller

Please follow these steps so as to fold the stroller:

STEP 1. Dismantle safety rail (7) by releasing buttons that are situated on both sides of rail. Next, pull out rail from stroller frame (fig.6);

STEP 2. Lift covers for folding mechanism (14) and then pull elements of folding mechanism (12) (fig.7);

STEP 3. Lift safety rail (13) and push simultaneously stroller handle (1) (fig.8,9)



STEP 4. Release clip from backrest's tilt angle (11) (fig.10) and push backrest (2) as forwards as possible (fig.11)



7.3. The adjustment of footplate length (9)

Please follow these steps so as to adjust footplate length:

STEP 1. Remove quick couplings situated on both sides of footplate (fig.12);

STEP 2. Adjust the height of footplate at your discretion (fig.13)

STEP 3. Insert quick coupling into holes.



7.4. The adjustment of tilt angle (2)

Please follow these steps so as to adjust backrest's tilt angle:

STEP 1. Loosen clip from the adjustment for backrest tilt angle (fig.14);

STEP 2. Hold backrest and adjust it at your discretion;

STEP 3. Block clip




7.5. The adjustment of footplate tilt angle (9)

Lift slightly footplate so as to release footplate support from catches. Adjust tilt angle at your discretion and block footplate support in suitable catches (fig.15)



7.6. Feet stabilizing belts

Feet stabilizing belts are equipped with Velcro tapes which make it easier to adjust them to patient's foot (fig.16).

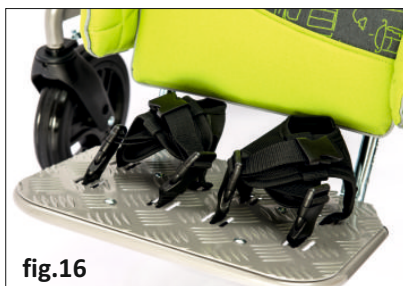

7.7. Headrest (4)

Headrest is equipped with special elastic belt which is fastened behind stroller's backrest (fig.18, fig. 19). It is possible to adjust headrest according to patient's needs .





7.8. Vest (5)

Connect clamps from vest with clamps from adjustable belts that are fastened both at the bottom and top of backrest (4 pcs.) (fig.20). Next, adjust vest to patient's body with the aid of belts.

It is highly important to adjust properly vest due to the fact that it protects the patient from falling out of the stroller and, moreover, patient can move without freely.




7.9. Side pelottes (6)

Side pelottes (2 pcs.) are fastened to backrest with the aid of Velcro tapes. It is possible to adjust side pelottes (fig.22).



7.10. Abduction block (8)

It is possible to adjust abduction block: loosen knobs situated under the seat (fig. 23), place abduction block either along or across seat and tighten knob (fig.24)

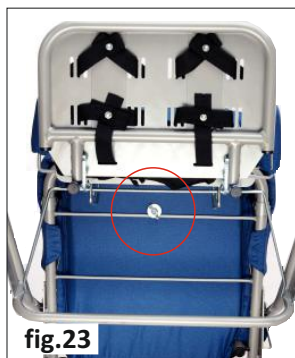

7.11. Wheels (16)(20)

RACERevo special stroller is equipped with rear inflatable wheels and front PU swivel with direction blockade which are blocked with the aid of catch.

It is possible to order PU rear wheels. In case of transportation, rear wheels can be dismantled: undo nuts (17) and remove wheels from axle. Recommended pressure in tyres: 230 KPa (0,23 MPa)



CAUTION! Remember to comply with the guidelines concerning air pressure. If air pressure is not proper, it results in damages or even accident.



CAUTION! It is forbidden to lubricate bearings.

7.12. Hand brake (10)

Stroller is equipped with two independent brakes for rear wheels

Press brake's lever (fig.26) so as to block rear wheels. Raise lever in order to unblock wheels.



7.13. Safety rail (7)

Chapters 'How to fold stroller' and 'How to unfold stroller' describe both assembly and disassembly of safety rail. (→point 7.1, STEP 3, page 11; point 7.2, STEP 1, page 13)



7.14. Adjustable handle (1)

Handle can regulated by pressing the buttons situated on both sides of handle (fig.28). Release these buttons so as to block handle at chosen height.



7.15. Covers for folding mechanism (14)

Covers should be fastened with the aid of zipper situated on stroller frame. Next, put covers on folding mechanism so as to cover it (fig.30,31).



8. ASSEMBLY OF ADDITIONAL EQUIPMENT

8.1. thigh abduction belts , REF RC_001

Follow these steps in order to mount thigh abduction belts:

STEP 1. Place belts on seat and insert fastening belts into hole which is situated on the corner of seat (fig.32).



STEP 2. Reeve fastening belts around backrest's side tubes and insert belts into hole at the side of seat (fig.33).



STEP 3. Interlace fastening belt through regulator which is situated on the belt (fig.34).



STEP 4. Interlace fastening belt through snap fastener (fig.35).



Repeat all steps in order to mount fastening belt from the other side of belts.

8.2. Sunshade, REF RC_002



fig.36



fig.37

In order to mount sunshade it is necessary to loosen clip and place it on handle's frame (fig.36). Next, adjust its position at your discretion and unscrew knob (fig.37). If handle pannels are applied, it is possible to fasten sunshade in any place on stroller's frame.

8.3. Table, REF RC_003

In order to mount table it is necessary to:

STEP 1. Dismantle safety rail (7) (→point 7.2, STEP 1, page 13);

STEP 2. Press simultaneously locks which are placed on the exterior side of table's grip and then insert table into place where safety rail was placed (fig.38).



fig.38



fig.39

8.4. Canopy, **REF**RC_005

In order to mount canopy it is necessary to:

STEP 1. Press catches which are situated on both sides of canopy and then place canopy's catches on stroller's side frame (fig.40);

STEP 2. Unfold canopy (fig.41);

STEP 3. Lock stretchers which are situated on both sides of canopy (fig.42).



8.5.Canopy with cover, **REF**RC_004

Mount canopy in accordance with rules described in 8.4 point and only then cover can be mounted. Next, front side of cover should be fastened at the front of canopy with the aid of zip (fig.43, fig.44).



Back side of cover should be fastened to the back of canopy. Next, fasten back side of cover to the side of cover (fig.45, fig.46).



8.6. Bag, REF RC_006

Bag is out on stroller's handle with the aid of tapes with clasps (fig.47).



8.7. Pelvic belt, REF RC_007

Pelvic belt is used instead of vest. It is fastened with the aid of belts which are placed at the bottom of stroller's support.



8.8. PU wheels, REF RC_008

PU wheels are made of special rubber. They are used instead of standard pumped wheels.



8.9. Cover for headrest, REF RC_010

Cover is made of cotton and it should be put on headrest (fig.52). Regarding material, it has some bands thanks to which material does not slip down.



8.10. Cup holder, REF RC_011

In order to mount cup holder it is necessary to follow these steps:

STEP 1. Loosen clip and place cup holder on stroller's frame (fig.53);

STEP 2. Adjust position of cup holder and tighten knob (fig.54);

STEP 3. You can adjust cup holder with the aid of button. (fig.55)



Cup holder is perfect for small and handy bottles with water



8.11. ELASTICO anti bed sore cushion, REF RC_012

ELASTICO anti bed sore cushion is made of thermo-active foam. Moreover, it is placed in cover which is made of airy material, namely ACTIV Space. Cushion should be put on stroller's seat in a place that is created for cushion from standard equipment.



8.12 Winter footmuff, REF RC_017

In order to mount winter footmuff it is necessary to:


STEP 1. Place winter footmuff on lower part of footrest plate and the fasten it with the aid of Velcro tapes (fig.59,60).



STEP 2. Place upper part of footmuff behind stroller's backrest (fig.61).



STEP 3. Footmuff is equipped with special hole for abduction block and for belts with buckles. It is important to fasten vest to these belts (fig. 62,63).

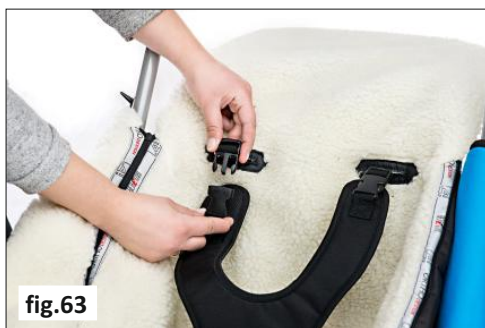

STEP 4. Footmuff is equipped with fasteners with Velcro tapes. These Velcro tapes should be fastened on stroller's frame in a place that is created for rail. (fig.64)





STEP 5. Upper part of footmuff should be secured by means of fastener (fig.65).

8.13 Hood for winter footmuff, REF RC_029

In order to mount hood for winter footmuff it is necessary to:

STEP 1. Fasten winter footmuff (-> point 8.12, Step 1-4, page 26).

STEP 2. Put hood on stroller's frame and then take out belts with buckles through two holes in hood. Next, fasten vest (fig. 66,67)

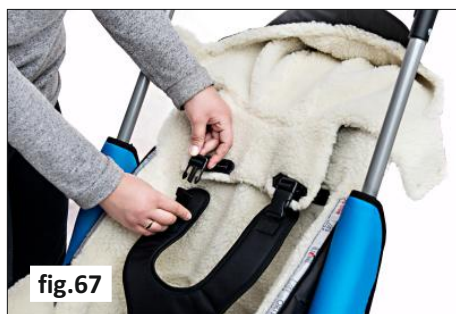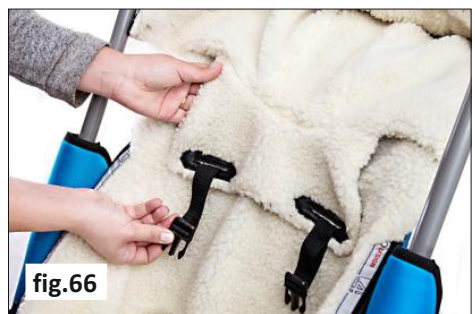

8.14 DeLux Bag, REF RC_028

DeLux bag is equipped with belt thanks to which it is possible to hang it on stroller handle (fig.68).



8.15 Men bag, **[REF]** RC_069

Men bag is equipped with belt thanks to which it is possible to hang it on stroller handle (fig.69).



fig.69

8.16 Lady bag, **[REF]** RC_070

Lady bag is equipped with belt thanks to which it is possible to hang it on stroller handle (fig.70).



fig.70

8.17 Low side cover for head, **[REF]** RC_018

In order to put low side cover for head on stroller it is necessary to place it on frame so as that middle part of cover is situated behind backrest and its endings with Velcro tapes cover side frame (fig.71).



fig.71

8.18 High side cover for head, **[REF]** RC_019

In order to put high side cover for head it is necessary to place it on frame so as that middle part of cover is situated behind backrest and its endings with Velcro tapes cover side frame as well as adjustable stroller handle (fig.72).



fot.72

8.19 Canopy with side covers, REF RC_020

In order to mount canopy with side covers it is necessary to follow these steps:

STEP 1. Place its catches on side frame and then press them (fig.73);

STEP 2. Unfold canopy and simultaneously block stretchers which are situated on both sides of canopy (fig.74);

STEP 3. Side covers should be fastened to canopy with the aid of zipper and catches should be placed on side frame of stroller (fig.75).



8.20 Trunk belt, REF RC_026

Trunk belt (fig.77) is assembled with the aid of Velcro tapes situated behind stroller (fig.78, 79). Trunk belt can be used only when patient is wearing vest!



fig.78



fig.77



fig.79

8.21 Narrowing pads 5 cm, REF RC_065

Narrowing pads 5 cm (fig.80) are really beneficial because they make it possible to adjust the width of seat to patient's measurements. Use Velcro tapes situated on stroller's upholstery so as to assemble them on seat. (fig.81).



fig.80



fig.81

8.22 Narrowing pads 3 cm, REF RC_034

Narrowing pads 3 cm (fig. 82) are really beneficial because they make it possible to adjust the width of seat to patient's measurements. Use Velcro tapes situated on stroller's upholstery so as to assemble them on seat (fig. 83).



8.23 Plexiglas table[®] REF RC_071

Please look at point 8.3. so as to find out how to mount Plexiglas table. (→point 8.3, page 21).




8.24 Calf belt, REF RC_015

Calf belt is assembled on footplate

Please follow these steps so as to assemble belt:



STEP 1. Remove side calf covers which are added to stroller's standard equipment and lay calf belt covers there. Make sure that wider part of cover is placed towards the inside of stroller (fig. 87, fig. 88);

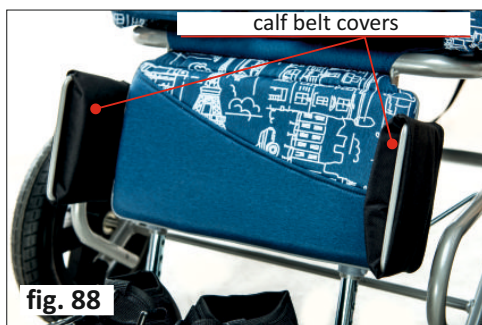

STEP 2. Thread calf belt through the frames of covers in a way that belt's fastening is placed towards the inside of stroller (fig.89, fig.90)



STEP 3. Fasten belt with the aid of Velcro tapes: thread belt through one of two fasteners from calf belt (fig.93). You can choose either first (fig.91) or second fastener (fig.92) - it depends which adjustment is the most comfortable. This activity should be repeated for the second side.



8.25 H harness, REF RC_014

Please look at point 7.8 so as to find out how to assembly vest. (→**point 7.8, page 16**). Initially, patient should not adjust vest tightly. Once vest is put on, it should be fastened with the aid of belts with plastic straps. Vest can be quickly unfastened (fig.95).



8.26 6 points harness, REF RC_025

In order to mount 6 points vest it is necessary to:

STEP 1. 6 points vest should be put on and then it is important to fasten zipper and buckle at the front of vest (fig.97-100);



STEP 2. Adjust vest with the aid of Velcro tapes situated at the back of vest (fig.101-102).

STEP 3. Fasten vest to stroller's frame by means of four buckles. It should be fastened in a place that is created for standard vest (fig.103). So as to fasten it, use not only buckles but also fastening belt that should be threaded behind stroller's backrest and fastened at the front of vest (fig.104)



8.27 Oxygen tank holder, REF RC_048

In order to mount oxygen tank holder it is necessary to:

STAGE 1. Take out two belts from buckles that are situated at cover.

STAGE 2. Take out adjuster no.2 and buckle from belt. It is important to leave adjuster no.1 on the belt. (fig.105,106)

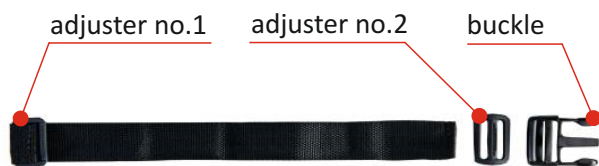


fig.106



fig.105



fig.107

STAGE 3. Thread one of the belts through stroller's handle (fig.107) and then thread the end of belt through adjuster no. 1.



fig.108

STAGE 4. Thread belt through adjuster no.2 (fig.108).

STAGE 5. Put buckle on belt and thread belt through buckle as it is presented in the picture below (fig.109, 110). The end of belt should be inserted into adjuster no. 2.



STAGE 6. Second belt should be fastened on stroller's side frame as it is described in stages 2-5. The picture below shows how belts should be fastened (fig.111).



STAGE 7. It is possible to fasten cover with oxygen tank holder (fig.112,113). The length of belts can be adjusted if necessary.



8.28 Rain cover, REF RC_037

Rain cover can be put on stroller only if canopy is mounted (it concerns folding canopy with covers **RC_004**, Foldin canopy **RC_005** or Folding canopy with side covers **RC_020**).

Follow these steps so as to mount rain cover:



STEP 1. Holes in cover should overlay with handle (fig.115, 116) and stretch cover so that it covers canopy and stroller



STEP 2. Mount cover on stroller's frame (fig.117)



8.29 Side pannels, REF RC_022

Please follow these steps so as to mount side pannels on the stroller:

STEP 1. Fix pannels to stroller's frame in appropriate places.

Pannels should be fixed on stroller's frame in three places (fig.118):

- 1) in upper part of the stroller, next to handle for pushing,
- 2) under folding mechanism,
- 3) next to safety rail.

As far as points 1 and 3 are concerned, side pannels are mounted on rail with the aid of belts with fasteners (fig. 119-122).



Regarding point 2, pannels are mounted on rail by means of belt with Velcro tape (fig.123).



9.THE USE OF RACER EVO STROLLER



CAUTION! Racer evo stroller has to be used with compliance to guidelines included in this user manual. Remember to use the product only with someone who is trained and knowledgeable. Do not forget to prepare both product as well as patient before using Racer evo stroller!

9.1. Preparation of the patient

Before using the product, it is essential to make sure that:

- patient's outfit is light and comfortable sporty clothes are advisable,
- shoes should be comfortable and, moreover, they should stabilize ankle,
- due to hygienic reason and to avoid abrasion it is advisable for the patient to always wear clothes when using the device, skin must not be naked especially in places of regular contact with upholstery,
- drains, bags, catheters etc. if applied, have to be secured to function properly and not to disturb during using the product,
- if it is necessary for a patient to wear orthopedic supply (collar, corset, orthosis, shoes) if it is recommended by doctor.



CAUTION! Outfit should protect patient's skin from direct contact with upholstery as well as other parts of equipment.



CAUTION! Only doctor can make a decision concerning using any kind of orthopedic supply!

9.2. How to use Racer evo stroller

1. Before use it is necessary to:

- Unfold the device;
- Place the device on even ground;
- Make sure that all parts are serviceable, correctly mounted and stable;
- Block brakes;
- Adjust the device to patient's measurements;
- Undo vest, trunk belts etc so as to place easily the patient in the device
- Place patient in the device;
- Fasten all elements which stabilize patient e.g. vest, trunk belts, feet stabilizing belts etc.;
- Mount additional equipment e.g. tray.

2. In case of patients who have problem with bladder, it is necessary to put special cloth on seat. This cloth can be purchased in pharmacy or medical shops.

3. It is essential to hold stroller with the aid of handle during using it otherwise stroller can roll.

4. Remember to place stroller forwards

5. In case of any obstacles such as step or curb, it is necessary to raise front wheels by pressing the handle down . Next, place front wheels on step and then raise back part of stroller with the aid of handle.



CAUTION! Remember to make sure that all belts and locking are correctly adjusted and fastened. Check also if fastenings from adjustable elements are firmly tightened!

Equipment works correctly only when the device is adjusted according to patient's height as well as weight. Remember to take all these measurements into account while purchasing the product. Correct adjustment is the most important due to the fact that only then the product is safe. It is forbidden to force child to keep specific position because he/she will feel uncomfortable and it may lead to wrong posture.

10. SAFETY RULES



CAUTION! Racer evo stroller has to be used in compliance with guidelines in this user manual.

It is highly important to consult either doctor or physiotherapist in order to make right decision concerning the choice of stroller and its size. It is also essential to familiarize with the guidelines described in user manual before using the device.

1. Before use it is necessary to:

- make sure that there are no damages or different faults which could endanger patient's life; check if all parts are working and are correctly mounted and, moreover, check if they are not broken;
- check if all screws, knobs and all glued, sewed or tapped parts are tightened;
- check if brakes are operating correctly;
- check if buckles are operating correctly
- always use safety belts, Velcro tapes should overlap min. 8cm;
- adjust the device according to patient's measurements and needs e.g. the adjustment of pelottes , footplate etc.;
- make sure that all movable parts and knobs for adjustments are stable and locked;
- secure patient with the aid of elements such as vest, foot stabilizing belts, pelottes, abduction block, headrest. It is highly important to secure patient because it safeguards him/her from falling out. If abduction contracture occurs, it is recommended to use abduction block in order to prevent ulcers between knees.

2. It is forbidden to use stroller for different purposes that it is intended to.
3. It is forbidden to use stroller when it is damaged or any part is missing.
4. It is forbidden to leave the patient unattended.
5. Stroller should be used only when there is someone who is trained and knowledgeable.
6. It is forbidden to leave stroller and the patient on unstable ground unattended, even if the brake is engaged.
7. It is forbidden to either go up or down the stairs; it doesn't matter if the patient is placed in the device or not.
8. It's important to make sure that patient doesn't stand on footer during either getting into or getting out the stroller
9. It is important to engage brake during getting into and getting out the stroller.
10. Make sure that all limbs are not close to spokes or other adjustable elements.
11. Although the device and the upholstery are made of fire-resistant materials, please pay your attention when approaching to sources of fire or sources of high temperature during using the device. Moreover, stroller shouldn't be stored in rooms with high humidity.
12. Secure stroller during transportation in order to avoid any possible damages. Manufacturer doesn't take responsibility for damages which appeared due to improper preparation of the device during transportation.



CAUTION! It is forbidden to place the patient in the device while being in car, bus or train.



CAUTION! The product contains small parts which children may swallow.



CAUTION! It is forbidden to make any changes concerning stroller's structure due to the fact that it is dangerous and it results in losing warranty.



CAUTION! The weight of user should not be bigger than maximal weight described on page 10 where you will find information regarding this issue.

- 1) AKCES-MED Sp. z. o. o sells products free from manufacturing defects or faults.
- 2) Manufacturer gives the purchaser 24-month guarantee from the date of purchase. All defects revealed during warranty period will be repaired free of charge.
- 3) All defects which appear after warranty period can be repaired only when the user will pay for it. This rule doesn't apply to refunded products - user can ask National Health Refund for refund.
- 4) Manufacturer claims that the product can be used for 48 months. User can use this product after this period only when he/she will send the device to manufacturer so as to service it. Next, manufacturer determines again how long the product can be used and, moreover, the date of the next service will be determined.
- 5) If product is damaged, please contact our dealer or AKCES-MED customer service (ph. 17/864 04 70)

11. MAINTENANCE AND CLEANING

Stroller should be kept clean and used in compliance with the manufacturer's recommendations. The device should be kept in clean, dry and ventilated room. Maintenance as well as cleaning have to be performed at least once a month. Regular cleaning and maintenance of stroller are indispensable to maintain its correct functionality as well as safety.

It is highly advisable to remember about proper maintenance in order to extend product's aesthetics. So as to do that, it is essential to comply with these guidelines:

1. It is recommended to clean upholstery with the aid of cleaning product, soft sponge and warm water. If you use sponge which is too wet, stain may get bigger, upholstery may be destroyed. It is necessary to remember that the excess of dampness should be removed from fabric before next cleaning. Please rinse upholstery with warm water and clean white cloth once stain is removed.
2. It is forbidden to soak upholstery!
3. It is forbidden to clean upholstery with the aid of hard brushes.
4. It is forbidden to clean upholstery with strong chemical agents such as solvent or caustic chemicals.



CAUTION! Akces-MED. Sp z o. o takes no responsibility for damages caused by use of wrong clearing agent.

5. It is forbidden to soak upholstery during cleaning. Remember that upholstery has to be always dry before using.
6. It is forbidden to clean, make any changes and maintain stroller during using it.
7. All metal as well as plastic elements should be cleaned by hand with the aid of general cleaning products. Remember to familiarize yourself with cleaning product's leaflet.



CAUTION! Remember that upholstery has to be always dry before using!

Before using it is necessary to:

- clean wheels
- check if base as well as wheels are correctly mounted; check if brake stops turning wheel;
- make sure that all wooden parts e.g. tray don't have splinters;
- check if stroller construction is free of breaks or any kinds of deformities.
- check if all movable parts are working and all screws, nuts, knobs etc. are correctly tightened;
- check if belts, vests, trunk belts, feet stabilizing belts etc are not frayed; check if buckles and clamps work correctly




CAUTION! Manufacturer takes responsibility only for hidden faults or damages which appeared despite the fact that user complied with the guidelines.



CAUTION! In case of any damages, it is essential to contact the person who sold you the product.



12. Stroller's marking

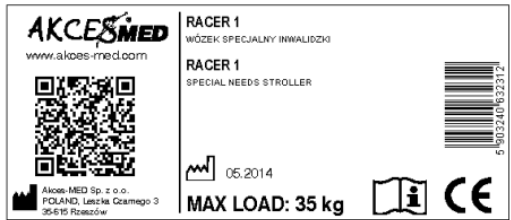

The example of nameplate



The example of sticker with serial number



ORTHOPAEDIC DEVICES MANUFACTURER



Akces-MED. Sp. z o.o.
ul. Leszka Czarneho 3
35-615 Rzeszów
tel. (17) 864 04 70


www.akces-med.com
e-mail: akces-med@akces-med.com

Authorised Dealer



Date of issue: 25.11.2013

The last update: 26.01.2015

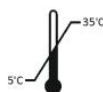

AKCES-MED Ltd. reserve the right to introduce technical and trading changes
in the content of the instruction without warning.