XXL-Rehab Vekseltryksmadras XXL-Rehab Alternating Mattress

DK

UK

MAX 270 KG

DK

Kan med fordel anvendes til:

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DK

UK

Instructions for using XXL-Rehab Alternating Mattress

⚠ CAUTION ⚠ CAUTION ⚠

- This product should not be used without proper instruction from a healthcare professional.
- Using the product for individuals weighing more than the maximum weight-bearing capacity may pose a risk for damaging the product.
- XXL-Rehab assumes no responsibility for any damage or injury caused by improper installation, assembly or use of this product.
- If components are damaged or missing, contact XXL-Rehab immediately. Do NOT use substitute parts.
- XXL-Rehab cannot be held liable for faults or accidents occurring after repairs by service staff without written authorisation from XXL-Rehab.

⚠ To ensure proper use of the XXL-Rehab Alternating Mattress, please read these instructions thoroughly before using the product. In doing so, pay particular attention to the warning notes marked with ⚠. Also remember that in order to prevent the risk of injury to the user, the mattress may be operated only by trained caregivers or aides or by individuals who have been instructed in its proper use.

CONTRAINDICATION

Patient conditions for which the application of pressure relieving therapy on an alternation system is contraindicated as follows:

- Cervical or skeletal traction
- Unstable spinal cord injuries

SAFETY NOTES

When using electrical products, especially when children are present, basic safety precautions should always be followed, including the following:

⚠ Danger

To reduce the risk of electric shock:

- Always unplug this product immediately after use.
- Do not use while bathing.
- Do not place or store product where it can fall or be pulled into a tub or sink.
- Do not place in or drop into water or other liquids.
- Do not reach for a product that has fallen into water. Unplug immediately.

⚠ Warning

To reduce the risk of burns, electrocution, fire or injury:

- This product should never be left unattended when plugged in.
- Close supervision is necessary when this product is used by, on or near children or invalids.
- Use this product only for its intended use as described in this manual. Do not use attachments not recommended by the manufacturer.
- Never operate this product if it has a damaged cord or plug, is not working properly, has been dropped or damaged, or has been dropped into water. Return the product to a service center for examination and repair.
- Keep the cord from heated surfaces.
- Never block the air openings of this product or place it on a soft surface, such as a bed or couch where the openings may be blocked. Keep the air opening free of lint, hair and other similar particles.
- Never drop or insert any object into any opening or hose.
- Connect this product to a properly grounded outlet only.
- Do not use outdoors or operate where aerosol (spray) products are being used or where oxygen is being administered.

⚠ Grounding

Before any connection is made, make certain that this product is connected to a protective earth conductor via the three-wire power cord; the three-blade plug must be inserted only into a socket outlet that provides a protective earth contact.

USE

XXL-Rehab Alternating Mattress can be used in hospitals, in homes, in nursing homes and in institutions.

The mattress is suitable for bariatric users weighing up to 270 kg who need pressure ulcer prevention or treatment grade 0-4.

 If users are weighing more than 270 kg there is a risk that the mattress will break or take damage.

DESCRIPTION

The XXL-Rehab Alternating Mattress is a mattress of high quality suitable for users with medium to high risk of developing pressure ulcers. It is specially designed to help prevent pressure sores and provide 24-hour pressure area care.

Pressure relief of the bariatric user is extra important because they often find it difficult to move and turn themselves in bed.

UNPACKING CHECKLIST

Check inside the package. If any of the following items should be missing or damaged, please contact your dealer.

Pump unit

Make sure the pump is identical to the one you purchased.

Mattress

Make sure the size is identical to the one you purchased.

Cover

Manual

Read this manual before use.

PRODUCT FUNCTIONS

Pump unit

Power switch

The switch is on the right side of the unit. Turn ON/OFF the power, the unit will start/stop operation.

Pressure-selector touch panel

There are seven pressure levels on the touch panel, providing options of pressure value.

Cycle time-selector touch panel

Selects a desired alternating cycle time from the touch panel.
The cycle time value can be 10, 15, 20 or 25 minutes.
There are two LEDs for each time value.

Alarm reset

The alarm will turn on when the low pressure LED is on.
To mute the alarm, simply press the reset button. The alarm LED will flash.
Re-press the reset button to reactivate the audible alarm.

Static/alternating control

Sets the air mattress in static mode or alternating mode.

Auto firm

Sets the air mattress to auto-inflation mode.

Mattress

The XXL-Rehab Alternating Mattress consists of Nylon/PU cells with a cell height of 22 cm.

Brugsanvisning for XXL-Rehab Vekseltryksmadras

USER CUSTOMISATION

1. Place the mattress flatly on the bed frame. The inflation tube should be towards the foot end.
2. Hang the pump unit on the foot end of the bed frame and make sure it is stable.
3. Connect the inflation tubes from the mattress to the pump's inflating nozzles. Make sure they are properly attached.
⚠ Make sure the air hoses are not kinked or tucked under the mattress. Also check to ensure the CPR valve is properly attached.
4. Make sure the CPR valve is correctly connected and CLOSED.
5. Plug the power cord into an electrical outlet with grounded AC power.
⚠ Before inserting the plug into the outlet, make sure the voltage is compatible. Also make sure this product is well grounded.
THE USER SHOULD NOT LIE ON THE MATTRESS IN THE INITIAL PHASE.
6. Turn on the power by pressing the power switch on the right side of the pump unit. A beep sound will begin the operation.
7. The pump will enter static mode. It will inflate the mattress to 30 mmHg and then enter alternating mode.
If 30 mmHg cannot be reached within 30 minutes, the unit will stay in static mode and keep inflating. If 30 mmHg cannot be reached within another 15 minutes, the alarm will beep and its LED will come on. Contact the dealer.
8. The alternating cycle time and pressure level are preset to the middle settings. Select from the touch panel to adjust the cycle time and pressure level.
9. When the mattress is ready (normal pressure LED is on), place a sheet on the mattress.

During normal operation, the unit will monitor pressure. If the pressure is lower than set pressure, the pump will automatically inflate the mattress to the set level and then stop. The alarm will beep and its LED will come on. Turn off the pump, wait 2 minutes and then turn on again. If the alarm starts again within 20 minutes, press the alarm reset button to mute the alarm and contact the dealer.

When activating the Auto Firm button the mattress will automatically inflate to the maximum level (above 70 mmHg), and then returns to the alternating level.

For suitable pressure make a hand check

Check to see if suitable pressure is selected by sliding one hand between the air mattress and the foam base (or bed frame if there is no foam base) to feel the user's buttock.

We recommend a distance between user and mattress of 2 - 4 cm.

When the user does not use the mattress, it is pumped to achieve the set back pressure. When the user then lies down on the mattress the pump releases the excess air.

DURING USE

Make sure that you operate this product in a proper way to optimize its value.

⚠ DO NOT use another pump than Legacy XL. It is dangerous to use a pump with pressure capacity greater than 120 mmHg. This may result in cell-puncture.

⚠ DO NOT change any component by yourself. If there is need for replacement or repair, always contact your dealer.

When the normal pressure LED (green) comes on to indicate the pressure is adjusted to a desired firmness, the user can then lie on the mattress. Please cover the mattress with a cotton sheet to avoid direct skin contact. Do not tighten the cover, this will reduce the effect of the mattress.

For suitable pressure make a hand check

Check to see if suitable pressure is selected by sliding one hand between the air mattress and the foam base (or bed frame if there is no foam base) to feel the user's buttock.

We recommend a distance between user and mattress of 2 - 4 cm.

When the user does not use the mattress, it is pumped to achieve the set back pressure. When the user then lies down on the mattress the pump releases the excess air.

CPR function

If there is an emergency to perform CPR on the user, pull the CPR valves immediately to release the air quickly from the mattress.

The CPR valves are located at the front right-hand side of the mattress.

Pressure set up

XXL-Rehab recommends that the pressure is adjusted by an experienced health care professional. It is easiest to set the pressure if the mattress has been filled to the max before the user lies down on the mattress.

Low pressure

When abnormal pressure occurs, the low pressure LED will come on and the alarm will be activated. Check if the connections are secure and correctly installed.

⚠ If the pressure is consistently low, check for any leakage (tubes or connecting hoses). If necessary, contact the dealer.

MAINTENANCE AND CLEANING

Of pump unit

- ⚠ DO NOT immerse or soak the pump unit.
- Check for external damage and move the pump to the cleaning area.
- Place the pump on a work surface and wipe the outside of the case with alcohol or mild detergent.
- ⚠ DO NOT spray any cleaning solution directly on the surface of the pump.
- ⚠ DO NOT use a hypochlorite or phenolic based cleaning solution as this may cause damage to the case.
- Wipe case with a clean cloth. Make sure all areas are clean.
- Spray cloth with cleaning solution and clean faceplate. DO NOT allow excess cleaning solution on faceplate or control panel. If solution gets inside, damage will occur.
- Allow surface to thoroughly dry after cleaning.
- After the pump is thoroughly cleaned and dried, proceed to plug in the pump and test to see if it runs normally.

Of mattress system

- Brush off or wipe down all surfaces of the cover sheet with soap and water before wetting with any liquid disinfectant.
- Unzip the top cover from the mattress. Brush or wipe down all surfaces with soap and water.
- The air cells are unsnapped from one side and are sprayed on all sides with a disinfectant as are the connections and hoses. Let it sit for a required incubation time and wipe down with a clean cloth.
- Spray the base with the disinfectant, inside and outside.
- Dry the mattress on a sunless area after cleaning.

Of cover

- Covers are immersed and soaked in disinfectant for the required incubation time.
- After pre-soaking, the cover is rinsed through a regular cycle in a washing machine with no soap. Then laundered with mild detergent in washing machine at 90°C.
- Covers are aerated until they are fully dry at 30°-50°C.

Handling and storage

- Unplug the pump and store with proper identification tag.
- Lay the mattress out flat.
- Roll from the foot end towards the head end. The foot-end strap can be stretched around the rolled mattress to prevent unrolling.
- ⚠ Do not fold, crease or stack the mattress.
- Follow the national requirement to dispose the pump unit.

General maintenance

- Check the power cord and plug to see if there are abrasions or excessive wear.
- Check the mattress cover for signs of wear and damage. Ensure the mattress cover and tubes are connected together correctly.
- Plug in the pump unit and check the airflow from the hose connection port. The airflow should alternate between ports every half-cycle time.
- Check the air hoses to see if there are kinks or breaks. For replacement, contact the dealer.
- Make sure the mattress tube is well connected.
- Check the pump unit and make sure both power and power are off when the switch is turned off.

Low pressure

- Examine if there is air leakage between the pump and the mattress connections or from the air mattress tubes.
- Check connectors between the air mattress and pump.
- Check the CPR valves. Ensure their outlets are sealed.
- Check the air connecting tubes.
- Activate Auto Firm and then check each cell for leakage.

CHECKLIST FOR TROUBLESHOOTING

PROBLEM	SOLUTION
The pump does not work	1. Check if the plug is inserted firmly into the outlet. 2. Turn on the power switch again. - If the power LED is ON and the pump still does not work, contact your dealer. - If the power LED is OFF, there may be a faulty outlet. Try to connect the power cord to another outlet. - If the power LED is still OFF, contact a qualified electrician for main power check.
Incomplete inflation (low pressure)	1. For a quick check, adjust the pressure to Auto Firm position. 2. Check to see if the tubes connected to the pump are twisted or there is any leakage occurring. - Always keep the tubes straight. - Change the tubes if there is any leakage. - Ensure the CPR valves are closed. - Ensure every single cell is not broken.
Filter cleaning	A dirty filter may decrease the air flow. Wash the filter with mild detergent to keep it clean. - Check the air filter at the back of the pump unit at least once a month.

WARRANTY

The XXL-Rehab Alternating Mattress is covered by one year warranty from the invoice date for materials and workmanship.

Products which have been used for purposes other than the intended uses; or which have been subject to negligence, abuse, improper storage or handling, improper operation, unauthorized modifications or damages beyond normal wear and tear as determined by XXL-Rehab, are not covered by this warranty.

If weight capacity is exceeded, the warranty will be void.

Any unauthorized repairs to product / part, as well as tampering with any components, will void the warranty.

MANUFACTURER

Roscoe Medical, Inc.
21973 Commerce Parkway
Strongsville, Ohio 44149
USA

DEALER

XXL-Rehab
Fuglebækvej 1D
DK-2770 Kastrup
Denmark
Tel.: +45 70 25 25 22
Fax: +45 70 25 25 44
www.xxl-rehab.com

09/2014/JJP

SPECIFIKATIONER

Maximum weight capacity is	270 kg
Length	200 cm
Width	100 / 200 cm
Height	27 cm
Weight of mattress	14 / 18 cm
Weight of pump	3.8 kg
Pump measurement	35 x 13 x 24 cm
Mattress divided into sections	Yes
Air loss alarm	Yes
Power failure alarm	Yes
Battery back up	No
CPR control	Yes
Static function	Yes
Place directly on bed base	Yes
Suitable for adjustable bed base	Yes
Fire resistant cover	Yes
Incontinence cover	Yes
Allergenic cover	Yes
Washable and disinfectable cover	Yes
Pressure ulcer prevention grade	0-4
Cover	PU
Cells	Nylon / PU
Medical equipment	Class I IXPO AP/APG NO TYPE BF
Applied part	Mattress
Power supply	AC 120 V / 60 Hz, 10 W 1A
Pressure range	10 – 75 mmHg
Cycle time	10, 15, 20, 25 min valgbar
Safety standards	UL, c-UL, CE
ISO – classification	ISO 18 12 18 – 28

XXL-Rehab Alternating Mattress is tested and certified according to these standards: UL60601, CE, FDA.

DK

UK

Strongly engineered and functionally designed exclusively for bariatric users!

XXL-Rehab Minimaxx

XXL-Rehab Minimaxx
Push Motor

XXL-Rehab Manual
Wheelchair - The Eclipse

XXL-Rehab Manual
Wheelchair - The Eclipse Tilt

XXL-Rehab
HD Electric Wheelchair

XXL-Rehab Wheelchair Cushions

XXL-Rehab Shower Bench

XXL-Rehab
Bedside commode

XXL-Rehab
Shower Commode

XXL-Rehab
Shower Commode Wheeler

XXL-Rehab
Shower Commode Rise N Recline

XXL-Rehab
Shower Commode Tilt

XXL-Rehab Barton Transferstol

XXL-Rehab Patient/Dinner chair

XXL-Rehab Rollator King

XXL-Rehab Rollator Support

XXL-Rehab Stand Tall

XXL-Rehab Bariatric walking frame

XXL-Rehab Canes

XXL-Rehab Multifunktional Scale

XXL-Rehab Lift and Slings

XXL-Rehab 400 bariatric hospital bed

XXL-Rehab Home Care Bed

XXL-Rehab Floor raiser

XXL-Rehab
Comfort Foam Mattress

XXL-Rehab
Low Air Loss mattress

XXL-Rehab
Alternating mattress

XXL-Rehab Bariatric suit

BRUGSANVISNING · USER MANUAL

mauryathome.dk

09-2014 XXL-Rehab Vekseltryksmadrasser / XXL-Rehab Alternating Mattress

XXL REHAB®

XXL-Rehab · Fuglebækvej 1D · DK-2770 Kastrup · Denmark
Telephone: +45 7025 2522 · Fax: +45 7025 2544
info@xxl-rehab.com · www.xxl-rehab.com