

Disinfectants for SonoSite Products

Table of Contents

Introduction	1
Systems	2
Standard Transducers, A-H	7
Standard Transducers, I-Z	13
Accessories	26
Cleaning transducers	28

Introduction

SonoSite tests disinfectants and disinfectant devices for use with its systems, transducers, and accessories. Except for TEE and surgical transducers, results are in the tables below. See the ultrasound system user guide for cleaning and disinfecting instructions and for additional warnings and cautions.

For cleaners and disinfectants compatible with TEE and surgical transducers, see the *TEE Care Card* and *Surgical Transducer Guide*.

WARNING: Disinfectants listed are recommended by SonoSite for safe use with SonoSite products.
Refer to instructions on the disinfectant label for guidance on disinfection effectiveness and appropriate clinical use.

This document does not have the following regulatory information for disinfectants:

- EPA registration
- FDA 510(k) clearance (liquid sterilant, high level disinfectant)
- CE approval

Before using a disinfectant, confirm that its regulatory status is appropriate for your jurisdiction and use.

Verify expiration dates on chemicals.

When disposing of chemicals, follow manufacturer recommendations and EPA regulations.

Ultrasound Systems

You can disinfect the exterior surface of the ultrasound system using a recommended disinfectant.

Table 1: Disinfectant Compatibility with Sonosite Ultrasound Systems

Disinfectant	NanoMaxx®	S Series™	M-Turbo® M-OB/GYN Office™	MicroMaxx®	TITAN®	SonoSite® 180PLUS™	iLook®15 iLook®25
AbcoCide 14				✓	✓		✓
Accel Wipes				✓	✓		
Airkem A-33®	✓	✓	✓	✓			✓
Alkacide				✓	✓	✓	
Alkazyme				✓	✓	✓	
Anioxyde 1000				✓			✓
Aquatabs (1000 PPM)				✓	✓	✓	
Ascend				✓	✓	✓	✓
Asepti-HB	✓						
Asepti-Steryl®			✓		✓		
Asepti-Wipes		✓	✓	✓	✓		
Bacillocid rasant						✓	✓
Banicide							✓
Bacoban WB		✓	✓			✓	
Cavicide				✓	✓		
CaviWipes™				✓	✓	✓	

Table 1: Disinfectant Compatibility with Sonosite Ultrasound Systems (continued)

Disinfectant	NanoMaxx®	S Series™	M-Turbo® M-OB/GYN Office™	MicroMaxx®	TITAN®	SonoSite® 180PLUS™	iLook® 15 iLook® 25
Cidalkan®			✓	✓	✓		
Cidalkan® Lingettes				✓	✓	✓	
Cidex®		✓	✓	✓	✓	✓	✓
Cidex® OPA	✓			✓	✓		✓
Cidex® Plus		✓	✓	✓	✓	✓	✓
Cleanisept Wipe		✓	✓				✓
Control III				✓	✓		
Coverage™ Spray					✓		
DentaSept						✓	
DisOPA			✓				✓
Dynacide PA				✓	✓	✓	✓
End-Bac® II			✓	✓	✓		
Endosporine			✓	✓			✓
EndoZime AW® Plus				✓	✓	✓	✓
Envirocide®				✓	✓		
Enzol					✓	✓	✓
Expose™ II				✓	✓	✓	
Glutaraldehyde SDS				✓	✓		

Table 1: Disinfectant Compatibility with Sonosite Ultrasound Systems (continued)

Disinfectant	NanoMaxx®	S Series™	M-Turbo® M-OB/GYN Office™	MicroMaxx®	TITAN®	SonoSite® 180PLUS™	iLook® 15 iLook® 25
Hexanios				✓	✓	✓	✓
Hi Tor Plus						✓	
Hibiclens				✓	✓	✓	✓
Hydrogen Peroxide (3%)			✓				
Korsolex® Basic				✓	✓		
Korsolex® Extra						✓	✓
Korsolex® FF				✓			
LpHse				✓	✓		
Lysol® IC Spray				✓	✓		
MadaCide 1		✓					
MetriCide 14				✓	✓	✓	✓
MetriCide® 28				✓	✓	✓	✓
MetriCide OPA Plus	✓		✓				✓
MetriZyme				✓	✓	✓	✓
Mikrobac forte				✓	✓	✓	✓
Mikrozid Wipes				✓	✓		
Nuclean						✓	✓
PerCept RTU Wipes™	✓		✓	✓	✓		

Table 1: Disinfectant Compatibility with Sonosite Ultrasound Systems (continued)

Disinfectant	NanoMaxx®	S Series™	M-Turbo® M-OB/GYN Office™	MicroMaxx®	TITAN®	SonoSite® 180PLUS™	iLook® 15 iLook® 25
Rely+On™ PeraSafe™	✓				✓		✓
Ruthless®				✓	✓		
Salvanios pH 7				✓	✓	✓	✓
Sani-Cloth® HB		✓	✓				
Sani-Cloth® Plus		✓	✓				✓
Sekusept® aktiv					✓		
Sporicidin®						✓	
Staphene®					✓		
Steranios				✓	✓		✓
Steranios 20%	✓				✓		✓
T-Spray II™	✓	✓	✓	✓	✓	✓	✓
TASK 105				✓	✓	✓	✓
Theracide Plus					✓		
Theracide Plus Wipes		✓	✓	✓	✓		
Transeptic®					✓		
Tristel				✓	✓		
Tristel Solo	✓	✓		✓		✓	✓
Tristel Wipes			✓	✓	✓		

Table 1: Disinfectant Compatibility with Sonosite Ultrasound Systems (continued)

Disinfectant	NanoMaxx®	S Series™	M-Turbo® M-OB/GYN Office™	MicroMaxx®	TITAN®	SonoSite® 180PLUS™	iLook® 15 iLook® 25
Vesphene IIse				✓	✓		
Virex® II 256	✓					✓	
Virox® 5 RTU Wipes		✓	✓	✓	✓	✓	
Virufen				✓	✓	✓	✓
Wavicide -01					✓		✓
Wavicide -06						✓	
Wet Wipe Disinfection				✓	✓	✓	✓
Wex-Cide				✓	✓	✓	✓

✓ - Acceptable for use

Standard Transducers, A-H

You can disinfect the transducer, scanning surface, and cable using a recommended disinfectant or device. Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. See [“To clean a transducer”](#) on page 28.

Table 2: Disinfectant-Device Compatibility with SonoSite Standard Transducers (A-H)

Device	C11 C11e C11x	C11n	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
Nanosonics Trophon EPR*	✓	✓		✓		✓	✓		✓	✓

✓ - Acceptable for use

1- Part number: P01960-xx, P02645-02, P02645-03, or P03361-10

2 - Part number: P00798-03

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer’s user manual.

Table 3: Disinfectant Compatibility with SonoSite Standard Transducers (A-H)

Disinfectant	C11 C11e C11x	C11n	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
AbcoCide 14	✓		✓	✓	✓	✓			✓	
Accel Wipes	✓		✓	✓	✓	✓			✓	
Aidal Plus	✓		✓	✓	✓	✓			✓	
Airkem A-33®	✓	✓	✓	✓	✓	✓	✓	✓	✓	
Alcohol, Ethanol		✓					✓			✓
Alkacide	✓		✓	✓	✓	✓			✓	
Alkazyme	✓		✓	✓	✓	✓			✓	
Anioxyde 1000								✓		
Aquatabs (1000)	✓		✓	✓	✓	✓				

Table 3: Disinfectant Compatibility with SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e C11x	C11n	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
Aquatabs (2000)	✓		✓	✓	✓	✓				
Ascend	✓		✓	✓	✓	✓			✓	
Asepti-HB	✓	✓	✓	✓	✓	✓	✓		✓	✓
Asepti-Steryl®	✓			✓		✓			✓	
Asepti-Wipes	✓		✓	✓	✓	✓			✓	
Bacillocid rasant	✓		✓	✓	✓	✓			✓	
Banicide	✓		✓	✓	✓	✓				
Bacoban	✓		✓	✓	✓	✓		✓	✓	
Bacoban WB	✓		✓	✓	✓	✓		✓	✓	
Bleach	✓		✓	✓	✓	✓			✓	
Cavicide	✓			✓		✓			✓	
CaviWipes™	✓	✓	✓	✓	✓	✓	✓		✓	✓
Chlor-Clean	✓		✓	✓	✓	✓				
Cidalkan®	✓		✓	✓	✓	✓			✓	
Cidalkan® Lingettes	✓		✓	✓	✓	✓			✓	
Cidex®	✓		✓	✓	✓	✓			✓	
Cidex® OPA	✓	✓ ³	✓	✓	✓	✓	✓ ³	✓	✓	✓
Cidex® Plus	✓		✓	✓	✓	✓			✓	
Cleanisept Wipe	✓		✓	✓	✓	✓		✓	✓	
Clorox® Wipes	✓		✓	✓	✓	✓			✓	
Control III			✓	✓	✓	✓			✓	

Table 3: Disinfectant Compatibility with SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e C11x	C11n	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
Coverage™ Spray			✓	✓	✓	✓	✓		✓	
DisCide® Ultra Disinfecting Towelettes	✓			✓		✓			✓	
DisOPA	✓		✓	✓	✓	✓		✓	✓	
Dispatch	✓		✓	✓	✓	✓		✓	✓	
Dynacide PA	✓		✓	✓	✓	✓		✓	✓	
End-Bac® II	✓			✓		✓			✓	
Endosporine	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
EndoZime AW® Plus	✓		✓	✓	✓	✓			✓	
Envirocide®			✓	✓	✓	✓	✓			
Enzol	✓		✓	✓	✓	✓			✓	
Expose™	✓		✓	✓	✓	✓			✓	
Gigasept AF	✓		✓	✓	✓	✓			✓	
Glutaraldehyde SDS	✓		✓	✓	✓	✓				
Hexanios	✓		✓	✓	✓	✓			✓	
Hi Tor Plus			✓	✓	✓	✓			✓	
Hibiclens	✓		✓	✓	✓	✓			✓	
Hydrogen Peroxide (3%)	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Kodan® Tücher	✓			✓		✓			✓	
Kohrsolin® FF	✓		✓	✓	✓	✓				
Korsolex® Extra	✓		✓	✓	✓	✓			✓	

Table 3: Disinfectant Compatibility with SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e C11x	C11n	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
Lem-O-Quat						✓		✓		
LpHse	✓		✓	✓	✓	✓			✓	
Lysol® IC Spray	✓			✓		✓				
Madacide 1				✓		✓	✓		✓	✓
Matar	✓		✓	✓	✓	✓				
MetriCide 14	✓		✓	✓	✓	✓			✓	
MetriCide® 28	✓		✓	✓	✓	✓			✓	
Metricide OPA Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MetriZyme	✓		✓	✓	✓	✓			✓	
Mikrobac forte	✓		✓	✓	✓	✓			✓	
Mikrozid Wipes	✓			✓		✓			✓	
Nuclean	✓		✓	✓	✓	✓			✓	
PerCept RTU Wipes™	✓	✓	✓	✓	✓	✓	✓	✓	✓	
Rely+On™ PeraSafe™								✓		
Ruthless®				✓		✓			✓	
Sagrosept Wipe	✓			✓		✓			✓	
Salvanios pH 7	✓		✓	✓	✓	✓			✓	
Sani-Cloth® HB			✓	✓	✓	✓	✓		✓	✓
Sani-Cloth® Plus	✓		✓	✓	✓	✓			✓	✓
Sekusept®	✓		✓	✓	✓	✓		✓	✓	
Sklar			✓	✓	✓	✓			✓	

Table 3: Disinfectant Compatibility with SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e C11x	C11n	C11 Cast Urethane ¹	C15 C15e	C15 Cast Urethane ²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
Sporicidin®	✓		✓	✓	✓	✓			✓	
Sporicidin® Wipes	✓		✓	✓	✓	✓			✓	
Staphene®	✓			✓		✓				
Steranios	✓		✓	✓	✓	✓			✓	
Steranios 20%								✓		✓
T-Spray™			✓	✓	✓	✓	✓		✓	
T-Spray II™	✓	✓ ³	✓	✓	✓	✓	✓ ³	✓	✓	
TASK 105	✓		✓	✓	✓	✓			✓	
TBQ	✓		✓	✓	✓	✓			✓	
Theracide Plus	✓		✓	✓	✓	✓			✓	
Theracide Plus Wipes	✓		✓	✓	✓	✓			✓	
Tor			✓	✓	✓	✓			✓	
Tristel	✓			✓		✓		✓	✓	
Tristel Solo	✓	✓	✓	✓	✓	✓	✓	✓	✓	
Tristel Wipes								✓		
Vesphene IIse	✓		✓	✓	✓	✓			✓	
Virex® II 256	✓	✓	✓	✓	✓	✓	✓		✓	✓
Virex® TB			✓	✓	✓	✓	✓		✓	✓
Virox® 5 RTU Wipes	✓	✓	✓	✓	✓	✓	✓	✓	✓	
Virufen	✓		✓	✓	✓	✓		✓	✓	
Wavicide -06	✓			✓		✓			✓	

Table 3: Disinfectant Compatibility with SonoSite Standard Transducers (A-H) (continued)

Disinfectant	C11 C11e C11x	C11n	C11 Cast Urethane¹	C15 C15e	C15 Cast Urethane²	C60 C60e C60x	C60n	D2 D2x	HFL38 HFL38x	HFL50x
Wet Wipe Disinfection	✓		✓	✓	✓	✓		✓	✓	
Wex-Cide	✓		✓	✓	✓	✓			✓	

✓ - Acceptable for use

1- Part number: P01960-xx, P02645-02, P02645-03, or P03361-10

2 - Part number: P00798-03

3 - Disinfectant has been tested by SonoSite for disinfection effectiveness on SonoSite transducers.

Standard Transducers, I-Z

You can disinfect the transducer, scanning surface, and cable using a recommended disinfectant or device. Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. See “To clean a transducer” on page 28.

Table 4: Disinfectant-Device Compatibility with SonoSite Standard Transducers (I-Z)

Device	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
Nanosonics Trophon EPR*	✓	✓	✓	✓				✓	✓	✓	✓

✓ - Acceptable for use

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer’s user manual.

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
AbcoCide 14	✓		✓		✓			✓	✓	✓	
Accel Wipes	✓		✓		✓			✓	✓	✓	
Aidal Plus	✓		✓		✓			✓	✓	✓	
Airkem A-33®	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Alcohol, Ethanol		✓				✓	✓				✓
Alkacide	✓		✓		✓			✓	✓	✓	
Alkazyme	✓		✓		✓			✓	✓	✓	
Anioxyde 1000					✓						
Aquatabs (1000)	✓		✓		✓			✓	✓	✓	
Aquatabs (2000)	✓		✓		✓			✓	✓	✓	

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
Ascend	✓		✓		✓			✓	✓	✓	
Asepti-HB	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Asepti-Steryl®	✓		✓		✓			✓	✓	✓	
Asepti-Wipes	✓		✓		✓			✓	✓	✓	
Bacillocid rasant	✓		✓		✓			✓	✓	✓	
Banicide	✓		✓		✓			✓	✓	✓	
Bacoban	✓		✓		✓			✓	✓	✓	
Bacoban WB	✓		✓		✓			✓	✓	✓	
Baxedin Pre-Op							✓				
Bleach	✓		✓		✓			✓	✓	✓	
Cavicide	✓		✓		✓			✓	✓	✓	
CaviWipes™	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Chlor-Clean	✓		✓		✓			✓	✓	✓	
Cidalkan®	✓		✓		✓			✓	✓	✓	
Cidalkan® Lingettes	✓		✓		✓			✓	✓	✓	
Cidex®	✓		✓		✓		✓	✓	✓	✓	
Cidex® OPA	✓	✓ ¹	✓	✓ ¹	✓	✓	✓	✓	✓	✓	✓ ¹
Cidex® Plus	✓		✓		✓			✓	✓	✓	
Cleanisept Wipe	✓		✓		✓			✓	✓	✓	
Clinell Universal Sanitising Wipes							✓				

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
Clorox® Germicidal Wipes							✓				
Clorox® Wipes	✓		✓		✓			✓	✓	✓	
Control III			✓		✓			✓	✓	✓	
Coverage™ Germicidal Surface Wipes							✓				
Coverage™ Spray			✓	✓	✓			✓	✓	✓	✓
DentaSept					✓						
DisCide® Ultra Disinfecting Towelettes	✓		✓		✓			✓	✓	✓	
DisOPA	✓		✓		✓			✓	✓	✓	
Dispatch	✓		✓		✓			✓	✓	✓	
Dynacide PA	✓		✓		✓			✓	✓	✓	
End-Bac® II	✓		✓		✓			✓	✓	✓	
Endosporine	✓	✓	✓	✓	✓		✓	✓	✓	✓	✓
EndoZime AW® Plus	✓		✓		✓			✓	✓	✓	
Envirocide®			✓	✓	✓			✓	✓	✓	✓
Enzol	✓		✓		✓			✓	✓	✓	
Expose™	✓		✓		✓			✓	✓	✓	
Gigasept AF	✓		✓		✓			✓	✓	✓	
Gigasept FF					✓						
Glutaraldehyde SDS	✓		✓		✓			✓	✓	✓	

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
Hexanios	✓		✓		✓			✓	✓	✓	
Hi Tor Plus			✓		✓			✓	✓	✓	
Hibiclens	✓		✓		✓			✓	✓	✓	
Hydrogen Peroxide (3%)	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Incidin Plus (1%)							✓				
Incidin Plus (3%)							✓				
Kodan® Tücher	✓		✓		✓			✓	✓	✓	
Kohrsolin® FF	✓		✓		✓			✓	✓	✓	
Korsolex® Basic					✓						
Korsolex® Extra	✓		✓		✓			✓	✓	✓	
LpHse	✓		✓		✓			✓	✓	✓	
Lysol® IC Spray	✓		✓		✓			✓	✓	✓	
Madacide 1			✓	✓	✓	✓		✓	✓	✓	✓
Matar	✓		✓		✓			✓	✓	✓	
Medzyme							✓				
MetriCide 14	✓		✓		✓			✓	✓	✓	
MetriCide® 28	✓		✓		✓			✓	✓	✓	
Metricide OPA Plus	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
MetriSponge							✓				
MetriZyme	✓		✓		✓			✓	✓	✓	

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
Mikrobac forte	✓		✓		✓			✓	✓	✓	
Mikrozid Wipes	✓		✓		✓			✓	✓	✓	
Nuclean	✓		✓		✓			✓	✓	✓	
PerCept RTU Wipes™	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Rely+On™ PeraSafe™					✓	✓					
Ruthless®			✓		✓			✓	✓	✓	
Sagrosept Wipe	✓		✓		✓			✓	✓	✓	
Salvanios pH 7	✓		✓		✓			✓	✓	✓	
Sani-Cloth® Bleach Wipes							✓				
Sani-Cloth® HB			✓	✓	✓	✓		✓	✓	✓	✓
Sani-Cloth® Plus	✓		✓		✓	✓		✓	✓	✓	
Sekusept®	✓		✓		✓			✓	✓	✓	
Sklar			✓		✓			✓	✓	✓	
Sporicidin®	✓		✓		✓			✓	✓	✓	
Sporicidin® Wipes	✓		✓		✓			✓	✓	✓	
Staphene®	✓		✓		✓			✓	✓	✓	
Steranios	✓		✓		✓			✓	✓	✓	
Steranios (2%)							✓				
Steranios (20%)					✓						
Super Sani-Cloth®					✓						

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
T-Spray™			✓	✓	✓	✓		✓	✓	✓	✓
T-Spray II™	✓	✓ ¹	✓	✓ ¹	✓	✓	✓	✓	✓	✓	✓ ¹
TASK 105	✓		✓		✓			✓	✓	✓	
TBQ	✓		✓		✓			✓	✓	✓	
Theracide Plus	✓		✓		✓			✓	✓	✓	
Theracide Plus Wipes	✓		✓					✓	✓	✓	
Tor			✓		✓			✓	✓	✓	
Trigene Advance Wipes							✓				
Tristel	✓		✓		✓			✓	✓	✓	
Tristel Solo	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓	✓
Tristel Wipes					✓						
Vesphene IIse	✓		✓		✓			✓	✓	✓	
Virex® 256							✓				
Virex® II 256	✓	✓	✓	✓	✓	✓		✓	✓	✓	✓
Virex® TB			✓	✓	✓	✓		✓	✓	✓	✓
Virox® 5 Wipes							✓				
Virox® 5 RTU Wipes	✓	✓	✓	✓	✓	✓		✓	✓	✓	✓
Virufen	✓		✓		✓			✓	✓	✓	
Wavicide -06	✓		✓		✓			✓	✓	✓	
Wet Task Wipes							✓				

Table 5: Disinfectant Compatibility with SonoSite Standard Transducers (I-Z) (continued)

Disinfectant	L25 L25e L25x	L25n	L38 L38e L38x	L38n L38xi	L52 L52e Vet Only	L52x Vet Only	L52n Vet Only	P10 P10x	P17	P21x	P21n
Wet Wipe Disinfection	✓		✓		✓			✓	✓	✓	
Wex-Cide	✓		✓		✓			✓	✓	✓	
Wipes Plus Disinfecting Wipes							✓				
Wood Wyant Ultra Wipes							✓				

✓ - Acceptable for use

1 Disinfectant has been tested by SonoSite for disinfection effectiveness on SonoSite transducers.

Specialty Transducers

You can disinfect the transducer, scanning surface, and cable using a recommended disinfectant or device.

Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids. See [“To clean a transducer”](#) on page 28.

Table 6: Disinfectant-Device Compatibility with SonoSite Specialty Transducers

Device	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
Nanosonics Trophon EPR*	✓		✓	✓	✓	

✓ - Acceptable for use

* For information and guidance on disinfection effectiveness and appropriate clinical use, see the manufacturer’s user manual.

Table 7: Disinfectant Compatibility with SonoSite Specialty Transducers

Disinfectant	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
AbcoCide 14	✓		✓		✓	✓
Accel Wipes	✓		✓		✓	✓
Aidal Plus	✓	✓	✓		✓	✓
Airkem A-33®	✓		✓	✓	✓	
Alcohol, Ethanol				✓		
Alkacide	✓		✓		✓	✓
Alkazyme	✓		✓		✓	✓
Aquatabs (1000)	✓		✓		✓	✓
Aquatabs (2000)	✓		✓		✓	✓
Ascend	✓		✓		✓	✓
Asepti-HB	✓		✓	✓	✓	✓
Asepti-Steryl®	✓		✓		✓	✓

Table 7: Disinfectant Compatibility with SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
Asepti-Wipes	✓		✓		✓	✓
Bacillocid rasant	✓		✓		✓	✓
Banicide	✓	✓	✓		✓	✓
Bacoban	✓		✓		✓	✓
Bacoban WB	✓		✓		✓	✓
Baxedin Pre-Op				✓		
Bleach	✓	✓	✓		✓	✓
Cavicide	✓		✓		✓	✓
CaviWipes™	✓		✓	✓	✓	✓
Chlor-Clean	✓		✓		✓	✓
Cidalkan®	✓		✓		✓	✓
Cidalkan® Lingettes	✓		✓		✓	✓
Cidex®	✓		✓	✓	✓	✓
Cidex® OPA	✓	✓	✓	✓	✓	✓
Cidex® Plus	✓		✓	✓	✓	✓
Cleanisept Wipe	✓		✓		✓	✓
Clinell Universal Sanitising Wipes				✓		
Clorox® Wipes	✓		✓		✓	✓
Clorox® Germicidal Wipes				✓		
Control III	✓		✓		✓	✓
Coverage™ Germicidal Surface Wipes				✓		

Table 7: Disinfectant Compatibility with SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
Coverage™ Plus Germicidal surface Wipes				✓		
Coverage™ Spray	✓		✓	✓	✓	✓
DentaSept						✓
DisCide® Ultra Disinfecting Towellettes	✓		✓		✓	✓
DisOPA	✓		✓		✓	✓
Dispatch	✓	✓	✓		✓	✓
Dynacide PA	✓		✓		✓	✓
End-Bac® II	✓		✓		✓	✓
Endosporine	✓		✓	✓	✓	
EndoZime AW® Plus	✓		✓		✓	✓
Envirocide®	✓		✓	✓	✓	✓
Enzol	✓	✓	✓		✓	✓
Expose™	✓		✓		✓	✓
Gigasept AF	✓		✓		✓	✓
Gigasept FF		✓				✓
Glutaraldehyde SDS	✓		✓		✓	✓
Hexanios	✓		✓		✓	✓
Hi Tor Plus	✓		✓		✓	✓
Hibiclens	✓		✓		✓	✓
Hydrogen Peroxide (3%)	✓		✓	✓	✓	✓

Table 7: Disinfectant Compatibility with SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
Incidin Plus (1%)				✓		
Incidin Plus (3%)				✓		
Kodan® Tücher	✓		✓		✓	✓
Kohrsolin® FF	✓		✓		✓	✓
Korsolex® Basic						✓
Korsolex® Extra	✓		✓		✓	✓
LpHse	✓		✓		✓	✓
Lysol® IC Spray	✓		✓		✓	✓
Madacide 1	✓		✓	✓	✓	✓
Matar	✓		✓		✓	✓
Medzyme				✓		
MetriCide 14	✓		✓		✓	✓
MetriCide® 28	✓	✓	✓		✓	✓
Metricide OPA Plus	✓		✓	✓	✓	
MetriSponge				✓		
MetriZyme	✓	✓	✓		✓	✓
Mikrobac forte	✓		✓		✓	✓
Mikrozid Wipes	✓		✓		✓	✓
Nuclean	✓		✓		✓	✓
PerCept RTU Wipes™			✓	✓	✓	
Ruthless®	✓		✓		✓	✓

Table 7: Disinfectant Compatibility with SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
Sagrosept Wipe	✓		✓		✓	✓
Salvanios pH 7	✓		✓		✓	✓
Sani-Cloth® Bleach Wipes				✓		
Sani-Cloth® HB	✓		✓	✓	✓	✓
Sani-Cloth® Plus	✓		✓		✓	✓
Sekusept®	✓		✓		✓	✓
Sklar	✓		✓		✓	✓
Sporicidin®	✓		✓		✓	✓
Sporicidin® Wipes	✓		✓		✓	✓
Staphene®	✓		✓		✓	✓
Steranios	✓		✓		✓	✓
Steranios (2%)				✓		
Super Sani-Cloth®						✓
T-Spray™	✓		✓	✓	✓	✓
T-Spray II™	✓	✓	✓	✓	✓	✓
TASK 105	✓		✓		✓	✓
TBQ	✓		✓		✓	✓
Theracide Plus	✓		✓		✓	✓
Theracide Plus Wipes	✓		✓		✓	
Tor	✓		✓		✓	✓
Trigene Advance Wipes				✓		

Table 7: Disinfectant Compatibility with SonoSite Specialty Transducers (continued)

Disinfectant	C8 C8e	HST	ICT ICTx	ICTn	SLA SLAx	SLT
Tristel	✓		✓		✓	✓
Tristel Duo	✓					
Tristel Solo	✓		✓	✓	✓	✓
Tristel Wipes						✓
Vesphene IIse	✓		✓		✓	✓
Virex® 256				✓		
Virex® II 256	✓		✓		✓	✓
Virex® TB	✓		✓	✓	✓	✓
Virox® 5 Wipes				✓		
Virox® 5 RTU Wipes	✓		✓		✓	✓
Virufen	✓		✓		✓	✓
Wavicide -01		✓				
Wavicide -06	✓		✓		✓	✓
Wet Task Wipes				✓		
Wet Wipe Disinfection	✓		✓		✓	✓
Wex-Cide	✓		✓		✓	✓
Wipes Plus Disinfecting Wipes				✓		

✓ - Acceptable for use

Accessories

You can disinfect the exterior surface of the accessories using a recommended disinfectant.

Table 8: Disinfectant Compatibility with SonoSite Accessories

Disinfectant	H-Universal™ Stand V-Universal™ Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook® Stand, MDSe, MDSm, S Series™ Stand, ScanPack, SiteCharge® SiteStand®,	System battery	iLook® docking station Mini-dock	NanoMaxx® dock	PowerPack	PowerPark	ECG cable	SonoRemote™ control	Field Case (NanoMaxx® system)
Airkem A-33®	✓					✓	✓	✓			✓
Alcohol, Ethanol Denatured, 3A	✓					✓	✓				
Alcohol, Isopropyl (100%)	✓						✓				
Alcohol, Isopropyl (70%)		✓	✓	✓	✓	✓			✓		
Anios Wipes	✓					✓	✓	✓			
Aniosyme DD1 (5%)	✓					✓	✓	✓			
Anioxyde 1000						✓	✓	✓			
Asepti-HB						✓	✓	✓			✓
CaviWipes™		✓	✓	✓	✓	✓	✓	✓	✓		
Cidalkan®	✓					✓	✓				
Cidalkan® Wipes						✓					
Cidex® OPA	✓					✓	✓	✓			✓
Control III	✓						✓	✓			
Coverage™ Spray	✓					✓	✓				
Coverage™ Wipes	✓							✓			

Table 8: Disinfectant Compatibility with SonoSite Accessories (continued)

Disinfectant	H-Universal™ Stand V-Universal™ Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook® Stand, MDSe, MDSm, S Series™ Stand, ScanPack, SiteCharge® SiteStand® ,	System battery	iLook® docking station Mini-dock	NanoMaxx® dock	PowerPack	PowerPark	ECG cable	SonoRemote™ control	Field Case (NanoMaxx® system)
Discide Ultra Wipes						✓	✓				
Dynacide PA	✓						✓	✓			
Echo Clean Lingettes	✓					✓	✓	✓			
Endosporine	✓					✓	✓	✓			
Envirocide®	✓					✓	✓	✓			
Hydrogen Peroxide (3%)	✓					✓	✓	✓		✓ ¹	✓
Incidin Plus (1%)								✓			
Incidin Plus (3%)	✓							✓			
Madacide 1						✓	✓				
Metricide OPA Plus	✓					✓	✓	✓			✓
Nu-Cidex	✓						✓	✓			
PerCept RTU Wipes™	✓					✓	✓	✓			✓
Rely+On™ PeraSafe™						✓	✓	✓			
Sani-Cloth® HB						✓	✓			✓	
Sani-Cloth® Plus		✓	✓	✓	✓	✓	✓	✓	✓		✓
Sporicidin®	✓					✓	✓	✓			
Steranios 2%	✓						✓	✓			

Table 8: Disinfectant Compatibility with SonoSite Accessories (continued)

Disinfectant	H-Universal™ Stand V-Universal™ Stand	MDS Lite, MDS Lite II	Basic Stand, Footswitch, iLook® Stand, MDSe, MDSm, S Series™ Stand, ScanPack, SiteCharge® SiteStand® ,	System battery	iLook® docking station Mini-dock	NanoMaxx® dock	PowerPack	PowerPark	ECCg cable	SonoRemote™ control	Field Case (NanoMaxx® system)
Steranios 20%	✓					✓	✓	✓			
Super Sani-Cloth®	✓		✓	✓	✓	✓	✓		✓	✓ ¹	
T-Spray™	✓						✓				
T-Spray II™	✓					✓	✓	✓			✓
Tristel Solo						✓	✓	✓			✓
Virex® 256						✓	✓	✓			✓
Virex® TB	✓					✓	✓				
Virox® 5 Wipes	✓					✓	✓	✓			
Wex-Cide	✓					✓	✓	✓			

✓ - Acceptable for use

¹ Not for use on the microphone clip. (The microphone clip is detachable from the microphone.)

Cleaning transducers

Clean the transducer prior to disinfecting to remove any particulate matter or bodily fluids.

To clean a transducer

- 1 Disconnect the transducer from the ultrasound system.
- 2 Remove any transducer sheath.
- 3 Clean the surface using a soft cloth lightly dampened in a mild soap or detergent cleaning solution.
Apply the solution to the cloth rather than the surface.
- 4 Rinse with water or wipe with water-dampened cloth; then wipe with a dry cloth.

H-Universal, iLook, M-OB/GYN Office, M-Turbo, MicroMaxx, NanoMaxx, S Series, SiteCharge, SiteStand, SonoRemote, SonoSite, SonoSite 180PLUS, TITAN, V-Universal, and the SonoSite logo are registered (in some jurisdictions) and unregistered trademarks owned by SonoSite, Inc.

The SonoSite ultrasound system(s) referenced in this document may be covered by one or more of the following U.S. patents: 5722412, 5817024, 5893363, 6135961, 6203498, 6364839, 6371918, 6383139, 6416475, 6447451, 6471651, 6569101, 6648826, 6575908, 6604630, 6817982, 6835177, 6962566, 7169108, 7449640, 7534211, 7549961, 7588541, 7591786, 7604596, 7643040, 7686766, 7694814, 7727153, 7740586, 7804970, 7809400, 7819807, 7841575, 7849250, 7867168, 7883276, D456509, D461895, D509900, D538432, D544962, D558351, D559390, D591423, D592750, D592760, D625014, D625015, and by the following counterpart foreign patents: AU727381, AU730822, CA2372152, CA2372158, CA2373065, CN ZL 97113678.5, CN ZL 98106133.8, CN ZL 98108973.9, CN ZL 200830007734.8, DE60021552.0, DE60029777.2, DE60034670.6, DE69730563.5, DE6980539.6, DE69831698.3, DE60 2004 23 816.3-08, FR0815793, FR0875203, FR0881492, FR1175713, FR1180970, FR1589878, GB0875203, GB0881492, GB1175713, GB1180970, GB1180971, GB1589878, IT0815793, IT0881492, IT1175713, IT1589878, KR528102, KR532359, NO326202, NO326814, NZ542968, RCD000897368-0001, SP0815793, SP0881492, SP1589878. Patents pending.

SonoSite, Inc.

21919 30th Drive SE
Bothell, WA 98021 USA
T: 1-888-482-9449 or
1-425-951-1200
F: 1-425-951-1201

SonoSite Ltd

Alexander House
40A Wilbury Way
Hitchin, Herts
SG4 0AP UK
T: +44-1462-444800
F: +44-1462-444801

P06703-06 05/2011
Copyright 2011 by
SonoSite, Inc.
All rights reserved.

