



Healthcare Solutions

Important Technical Bulletins Please Read Immediately Retain in your WatchMate® User Manual

December 9, 2010

Dear WatchMate System User,

It has come to our attention that some facilities are using the Watchlet™ Tester (part number 803A1901) for weekly testing of Watchlet bracelets for the WatchMate system. **Every Watchlet bracelet must be tested weekly at an exit protected by a WatchMate monitor.**

IMPORTANT! Please take the time to review these procedures and implement any change(s) to your protocol(s) that may be necessary.

The attached bulletin “Proper Procedure for Watchlet Bracelet Weekly Testing” provides the detailed test procedure for weekly testing of Watchlet bracelets. The Watchlet Tester is a device to facilitate management of your Watchlet bracelets. The attached bulletin “Evaluating the Watchlet Bracelet Battery Status with a Watchlet Tester” outlines proper use of this device.

WARNING! FAILURE TO PROPERLY TEST YOUR WATCHLET BRACELETS AT A MONITORED EXIT COULD RESULT IN INJURY TO, OR THE DEATH OF, A RESIDENT/ PATIENT IN YOUR CARE.

Please train affected staff immediately to implement the procedures attached, and retain them as written in your WatchMate System User Manual. If you have any questions, please contact your distributor or Stanley Healthcare Solutions at 866-559-6275, ext. 154.

Sincerely,

Long Term Care
Stanley Healthcare Solutions

Attachments (970-804006-000, 970-804007-000)



Healthcare Solutions

www.stanleyhealthcare.com

© 2010 Stanley Security Solutions, Inc. All Rights Reserved. December 2010. 970-804008-000 Rev A.

