

USER'S MANUAL

MODLE RF3.6

Multi-Polar Radio Frequency Slimming Machine



MODEL:RF3.6

Warning

To dear Users,Beco advice you follow suggestion as bellow:

- 1.Read user's manual before any operation.
- 2.Do not remove or change any parts of machine
- 3.Contact with Beco if any malfunction or breakdown.
- 4.people with heart disease,or cardiac pacemaker,vascular disease are not recommended to use the machine.

Machine function:

- 1.Rebuild and increase collagen,improve skin's elasticity.
- 2.Tighten skin,reduce skin aging
3. Radio frequency creat heat,dissolve body fat,stimutale blood circulation,smooth lympt,burn fat.
- 4.Vacuum to suck and move cellulite,to consume fat.

machine's feature

- 1.4D frequency conversion,low energy consumption,converted frequency
- 2.multi-polar RF,dissolve deep cellulite stockup,stable,regular output.
- 3 a.triple-polar handle (small) for eye
b.triple-polar handle(big) for face
c.7 polar handle for body
d.8 polar RF+Vacuum for body
- 4.combine RF and Vacuum for suck,move and dissolve cellulite.
- 5.multi media touch screen
(music,film display, usb connect to computer)

USB connector

Touch Screen

8 polar RF + vacuum handle

7 polar RF handle

tripolar RF handle for eye

tripolar RF handle for face

voltage: AC110V/220V 50-60hz

power: 650W

RF : 5Mhz

Vacuum : 0-100kpa

4 handles

can set work time on touch screen.

intensity adjustable,10 different gears for each handle

weight :45KG

Specification :48*70*140CM



MODEL:RF3.6

the interlocution about electric wave pull skin treatment

Dispart the face to two parts, then use the electrobe probe head and follow the lymph line to nurse the skin from inside to exterior outside, and it should nurse from one part to one part, each part should be nursed 5-8 times, but it can't operate too heavy.



- 1 Nurse the chin by the circulation way.
- 2 It should nurse from the chin to the both side face and to the earlap by the drawing line way.
- 3 Do the removing wrinkle treatment on the cheek by the circulation way. and the operation steps should be from inside to outside and down to up.
- 4 Tighten and upgrade the cheek skin by the beeline way.
- 5 Nurse the eyes parts by the circulation way. The electrode probe head should leave the eyepit about 1cm, and it should move to the temple from inside to outside, but be careful and don't let the ointment into the eyes.
- 6 Upgrade the side canthus and eyebrow by the drawing line

way. The probe head should follow the wrinkle line and be slightly done on the heavy wrinkle parts, which can strengthen local treatment. Please take attention to the eyes in operating, because the eyes parts are very sensitive.


- 7 Dispart the forehead to two parts, then nurse the forehead wrinkle by the circulation way, and nurse the temple from inside to outside. The probe head should follow the wrinkle line and be slightly done on the heavy wrinkle parts, which can strengthen local treatment. Then upgrade the skin by the beeline way. Please select weak energy when it is operating on the upper eyebrow side eyes, because it is the sensitive region.

- 8 Stimulate the lymph node of ears.
- 9 Nurse from the lymph nobe of chin to the neck and to the lymph nobe of clavicle by the come -and -go way.
- 10 Delaminate the neck and dispel the wrinkle by curve way.

- 11 Please reduce the energy when it is operating on the forehead and chin, because both their skin are very

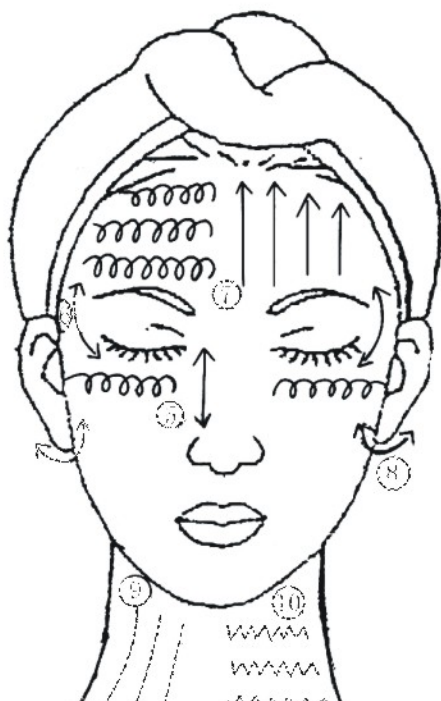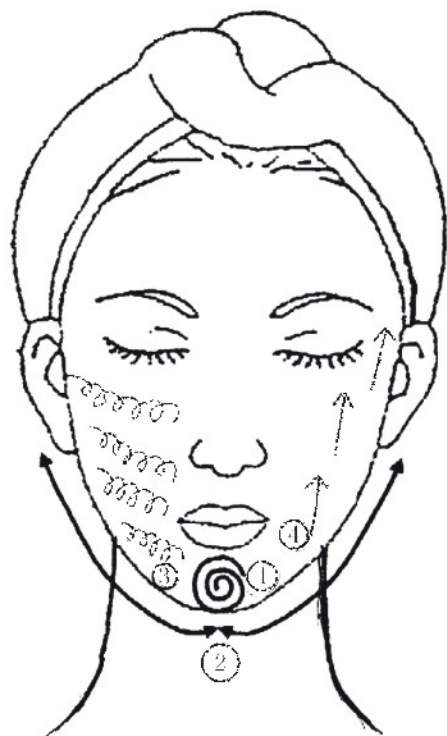


The high frequency electric current constantly change the electrode and make the electron of body organization move fleetly, Which will produce heat energy and heat up the deep layer organization ,and which will make the cell more active ,and which will improve the lymph circulation system.,and which will help to lose fatness and dispel the honeycomb organization. All of that will reach the effect of tightening skin and slimming body. .

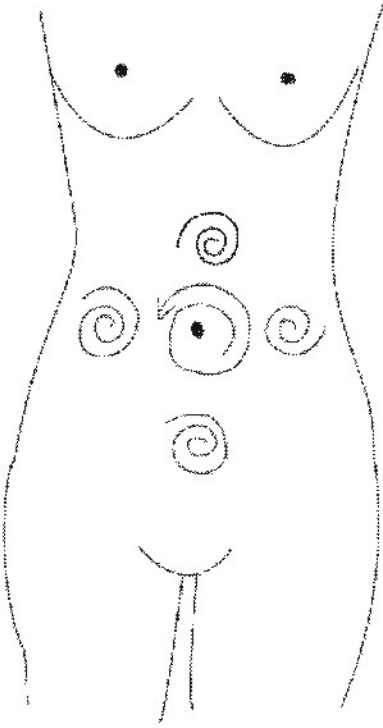

Nurse the abdomen

Nurse the abdomen point by the rotation way.

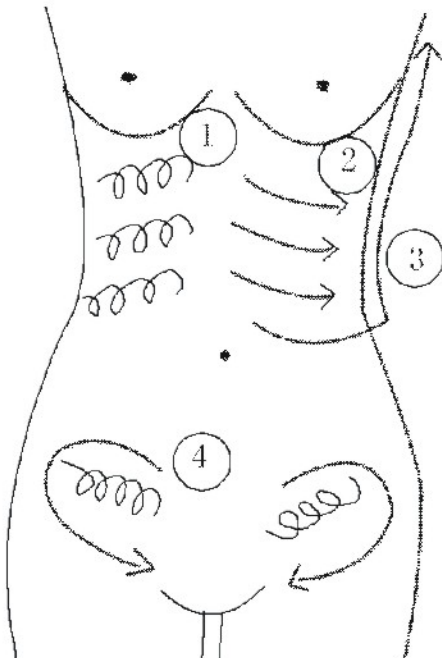

Dispart the abdomen to two parts when it is operated, and operate them from one part to other part , top to down and inner to exterior.


(1) Nurse the midsection by the rotation way

(2) Pull to the sides by the come -and go and drawing line way.

(3) Pull to the armpit and follow the fringe of sides.


(4) Nurse the underbelly by the rotation way, and pull to the groin by the drawing the way.

* It can increase the radio frequency energy when it is operating on the fat part.

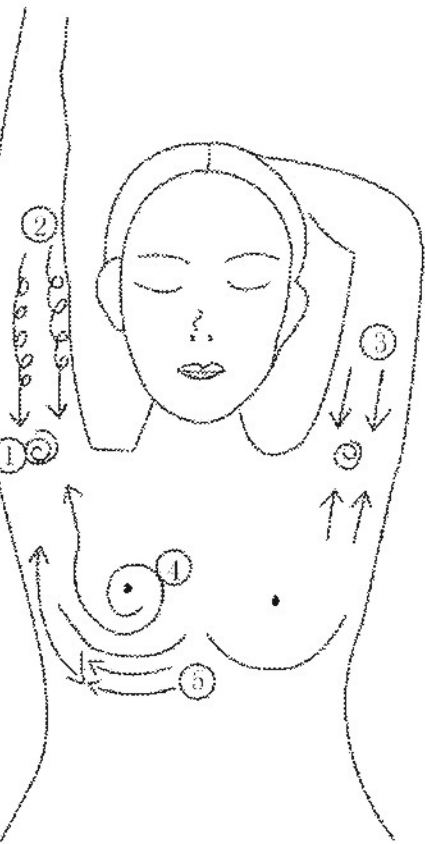


Nurse the breast and arms

- 1 Massage the outer part by the circle way
- 2 Nurse the arms toward the outer by the rotation way.
- 3 Tighten the inside arms by the beeline way.
- 4 Massage the breast by the circle way and upgrade it toward the outer.
- 5 Pull the breast toward the outer by the drawing way, and it should be pulled from inside to outside.

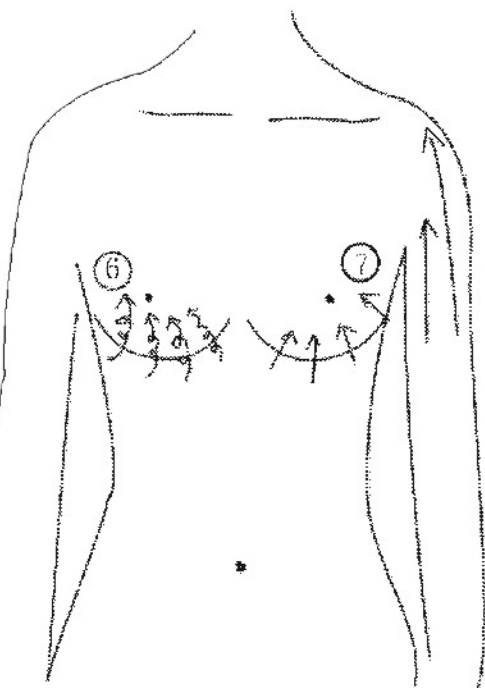


- 6 Massage the breast by the rotation way
(It is applicable to the big and loose breast)
- 7 Tighten the breast outline by the beeline way.

Nurse the back

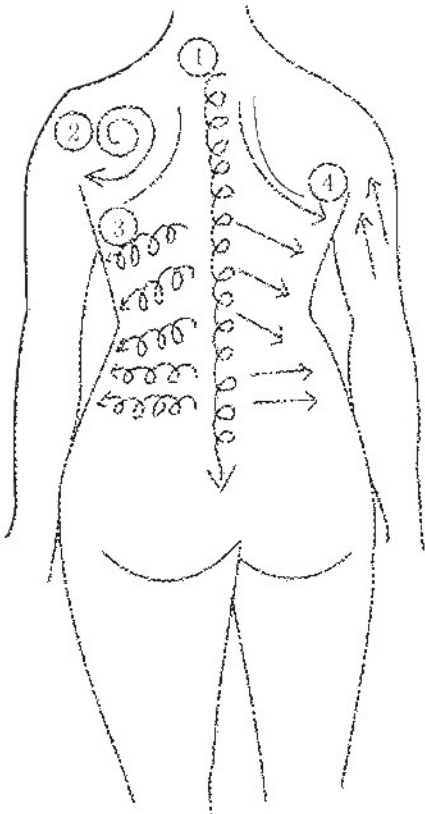

- 1 Nurse spinal column by the rotation way
- 2 Nurse the bladebone by the circle way
- 3 Nurse the back by the rotation way, and it should move from top to down and inside to outside.


Nurse the buttocks and feet

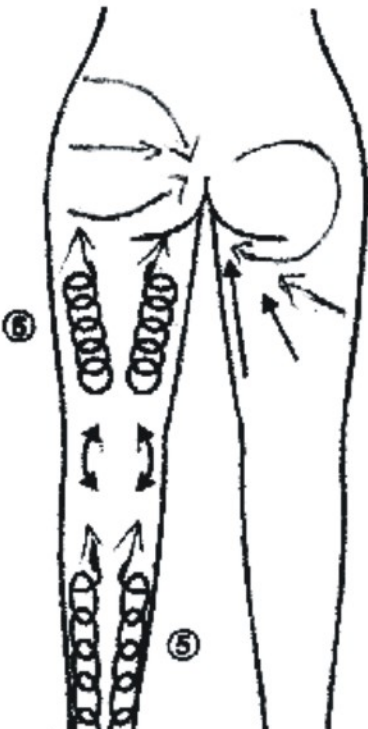


- 1 It should pull from the crus to the nest by the rotation way.
- 2 Nurse the nest by the drawing line way
- 3 Nurse the thigh muscle and buttocks by the rotation way, then pull to the buttocks stock ditch.

Subcutaneous tissue and muscular tissue are heated up, which will strengthen the local blood flow and the elasticity of collagen fibre organization. This treatment will reduce the pain and hardness of muscle ligament and arthrosis. Also the treatment will relax the muscle hyperkinesias.

Nurse the cervical vertebra

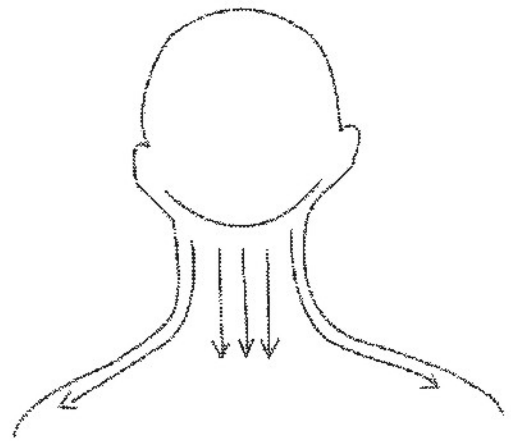

Massage from the part of belowing hair 2cm to the seventh cervical vertebra.

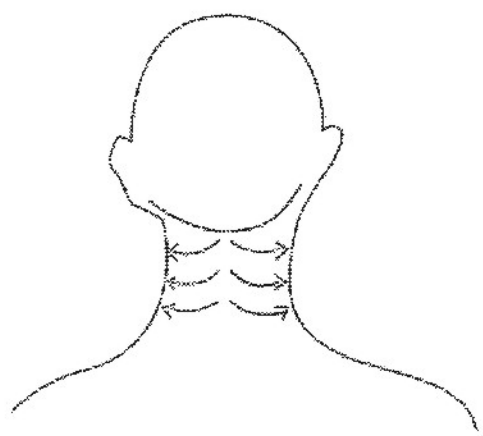


Massage from the middle of cervical vertebra to the sides of cervical vertebra.

Notices:

If the customer has the emergencies of high blood pressure , cerebral hemorrhage or others, please don't operate on the back of neck and ears. .

Nurse the feet

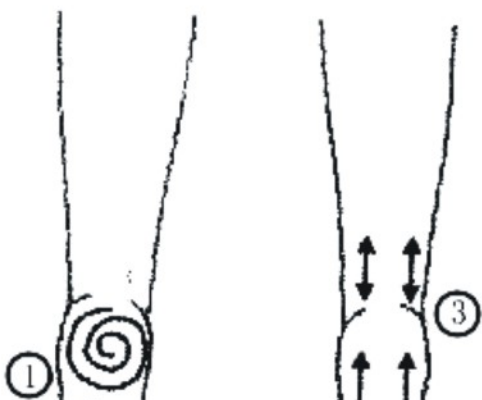

- 1 Nurse the whole baseboard by the rotation way, which can make blood smooth.
- 2 Massage the toe
- 3 Massage the heel by the beeline way

The interlocution about electric wave pull skin

Q: What's the radio frequency?

A: Radio frequency is a transmittable electromagnetic wave, shortly, it is RF radio frequency. There are monopole and double pole in the current time, and our company adopts the monopole in the electric wave pull skin catena. The monopole radio frequency can collect more energy, so it can more deeply penetrate to the skin than that of double pole and make more internal energy. So the effect will be more obvious.

Q: How long does it take to see the effect of the treatment?

A: Most people can feel the effect of tightening skin after trial treatment, and whose skin will become more whitening and ruddy. Also the pore will be more thin. However, the effect of treatment will be more obvious after 2-6 months of treatment. The skin will be more tightening and smooth after 2 months of treatment. And after one period of treatment, the skin will be fully rubby, tender and flexible. In other words, the skin comes back to the youth skin, the facial outline and body will be more standard.

Q: What parts does it aim at?

A: For an example, the dispelling wrinkle treatment, the effect of which will be obvious on many parts, such as the forehead, eyes and so on. They can be improved in one time.

Q: How long does it take to do the treatment?

A: It is according to what is the part and how big the region and how degree of the ageing. Each part should be operated about 20-30 minutes at each time.

Q: How does it work to dispel the wrinkle?

A: When the radio frequency electric current turns into the human body, which will produce heat energy, and the heat energy will diffuse from inside to outside, but it will collect to the corium layer. So when the temperature arrived at 45°C - 60°C , the collagen and albumen will produce two phenomena: 1 Tighten the collagen and albumen at once, 2 Increase the collagen and albumen and fill up the space of the collagen organization, which upgrades the skin again and makes the skin flexible and smooth, so the wrinkle will lose gradually.

Q: How long does it last for?

A: The effect will be related with personal constitution and ageing degree. Usually, it can maintain more than two years, but it also will be effected by the environment.

Q: Why does it need to keep the skin moisture and sun-resistant?

A: Because the ageing skin is caused by the outside environment and its energy reduction. Especially, the ultraviolet radiation will cause the collagen reduction and fibre dissolution. So it needs to keep the skin moisture and sun-resistant.