

# Netti 4U Comfort | CED



# **CONTENTS**

	Quality and durability	4 5
	he environment and waste disposal	5
	bout this manual	5
1.4 V	ital measures	6
2. QUICK R	EFERENCE	7
B. DESCRIP	TION	8
4. FEATURE	S OF NETTI 4U COMFORT CED	9
5. ACCESSO	ORIES	10
5.1 A	Assembly of Hip-Belt	12
6. ASSEMB	LING AND ADJUSTMENT	
	Unpacking	12
6.2	Main wheel	12
6.3	Front castors	13
6.4	Seat height adjustment	13
6.5	Back rest	14
6.6	Seat depth adjustment at the rear	15
6.7		15
6.8	1	16
6.9		16
	Back rest cushion	16
	Adjusting velcro back	17
	Foot support	17
	Head support	19
6.14	Arm support	21

6.15 Adjusting the brakes	21
6.16 Push bow	23
6.17 Seat angle/tilt	23
6.18 Back rest angle	24
7. MANOEUVERING THE WHEELCHAIR	
7.1 General about driving and parking	24
7.2 Driving techniques - Step up	24
7.3 Driving techniques - Step down	25
7.4 Driving techniques - Slope	25
7.5 Driving techniques - Up stairs	25
7.6 Driving techniques - Down stairs	26
7.7 Transfer	26
7.8 Lifting the wheelchair	27
8. TRANSPORT	
8.1 Folding for transport	27
8.2 Transport in a car	27
9. MAINTENANCE	
9.1 Maintenance instructions	28
9.2 Cleaning and washing instructions	28
-	
10. TROUBLESHOOTING	29
11. TESTS & WARRANTY	30
12. MEASURES & WEIGHT	31



### 1. INTRODUCTION

Netti 4U Comfort CED is a comfort wheelchair meant for both indoor and outdoor use. It is tested and approved to EN 12183:1999. The tests were carried out by TÜV Product Service GmbH in Germany. It is also tested and approved to crash test ISO/FDIS 7176/19.

In Alu Rehab we believe that wheelchairs should be chosen based on a thorough assessment focusing on the needs of the user and demands from the environment. Therefore it is important to know about the possibilities and restrictions of the wheelchair. Netti 4U Comfort CED is a wheelchair designed for users with the need for comfort and relief. The combination between the seating system and the ergonomical solutions in the frame construction, offers many possibilities for adaptation and adjustments.

The wheelchair offers the possibility to vary the sitting position from activity to rest using tilt and recline functions.

Max user weight: 140 kg.

Specifications varies between countries.

### 1.1 Quality and durability

The Netti 4U Comfort CED wheelchair is tested at TÜV Product Service GmbH in Germany, following the European Standard EN 12183:1999. As manufacturer, Alu Rehab A.S evaluates the test to be equal to 5-6 years of normal use of the chair. The disability of the usage as well as the level of maintenance done foremost decides the durability of the wheelchair. Thus, the durability will vary depending on these two factors.

# 1.2 The environment and waste disposal

Alu Rehab and its suppliers wish to protect the environment.

This means:

- That we avoid using environmentally harmful substances and processes to the greatest extent possible.
- That Alu Rehab's products are ensured a long service life and a high degree of flexibility – to benefit the environment and economy.
- That all packaging can be recycled.
- That the wheelchair was designed to be separated into its component materials to make recycling easier.
- That the wheelchair was designed with many different auxiliary functions - to improve the working environment for staff at instutions and hospitals.

#### 1.3 About this manual

In order to avoid damages while using the Netti 4U Comfort CED wheelchair, please read this manual carefully before starting to use the chair.

- Symbol of forbidden actions.
  No warranty can be claimed whenever these actions are implemented.
- Symbol of warning.
  Whenever this symbol is used, caution has to be taken.
- Symbol for important information.
- Symbol for useful tips.
- Symbol for tools.

#### Anti-tip

Correctly fitted, the anti-tip will secure the chair from tipping backwards. We strongly recommend use of the anti-tips.

#### 1.4 Vital measures

Netti 4U Comfort CED is a comfort wheelchair designed for both outdoor and indoor use. It is tested and approved to EN 12183:1999. The tests were carried out by TÜV Product Service GmbH in Germany.

Specifications varies between countries.

Seat width:



35, 40, 43, 45, 50, 55 & 60 cm

Seat height:



46,5 cm\*

\*By changing position of main wheels, it is possible to achieve seat height of 50 cm.

Seat depth:



(From back rest cushion to front of seat plate)

42,5, 45, 47,5, 50 cm

Back rest height:



48 cm\*

\* Using back rest extender gives 60 cm

### 2. QUICK REFERENCE

The content of this page is a summary of the whole manual. It gives a brief introduction to the use and care of the Netti 4U Comfort CED wheelchair.


- The quick reference is not a replacement for the manual, only a reminder/check list.
- Unpack the wheelchair (Chapter 6.1)
- Mount the main wheels (Chapter 6.2)
- Mount the front castors (Chapter 6.3)
- Put the back rest back, and mount the recline gas strut to the back rest using the locking bolt. (Chapter 6.5)
- Mount the cushions (Chapter 6.9 and 6.10)
- Mount the foot supports (Chapter 6.12)
- Mount the head support (Chapter 6.13)
- Set anti-tip in active position (Chapter 6.8)
- Mount accessory. (See chapter 5 for more information. Mounting descriptions will accompany the accessory.)
- For troubleshooting, see chapter 10. For mounting and adjustments see chapter 6.


- ▲ Drive carefully!
- Never stand on the foot supports due to risk of tipping forwards.
- A Never lift the wheelchair by the foot supports or arm supports.
- ★ When the chair is tilted rearwards, the anti tips and brakes should always be in use.
- ▲ Be sure to lock all handles properly.
- ⚠ The anti-tips should always be used for the safety of the user.
- Product configuration may vary between different countries.
- If in doubt contact your dealer!

### 3. DESCRIPTION

#### Standard version\*

- 1. Lock for push bow
- 2. Release handle
- 3. Push bow
- 4. Head support bracket
- 5. Angle and depth adjustment for head support
- 6. Head support
- 7. Back rest
- 8. Arm support with pad
- 9. Seati cushion
- 10. Angle adjustable foot support
- 11. Calf support bracket
- 12. Calf support
- 13. Foot plate
- 14. Front castors
- 15. Front fork
- 16. Brakes
- 17. Anti tip
- 18. Main wheel
- 19. Knee upholstery

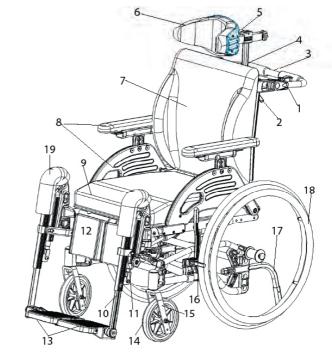

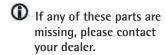

\* Product configuration may vary between different countries.

## 4. FEATURES OF NETTI 4U COMFORT CED

## **Standard**

#### **SEAT**

- Pressure distributive cushion
- Tilt -5° to +20°
- Adjustable depth of 7.5 cm

#### **WHEELS**

- 24" x 1" Puncture proof main wheels with quick release axle
- Push rim: Oval plastic
- 7" Puncture proof front castors with quick release axle
- \* Standard main wheels may vary between countries.

#### **BACK REST**

- Angle: 90° to 135°
- Height: 48 cm
- The back rest cushion has integrated lumbar support and side support
- Push how

### **FOOT SUPPORT**

- Angle adjustable foot support
- Height- and angle adjustable foot plates
- Removable

#### **ARM SUPPORT**

Height and depth adjustable

### **HEAD SUPPORT**

- Height, depth and angle adjustable
- Removable

### Accessories

#### **SFAT**

- Trays & reading stand for trays (See chapter 5)
- Hip belts (See chapter 5)

#### WHFFIS

 Puncture proof wheels 12", 16" and 24" with drumbrake (See chapter 5)

#### **BACK REST**

- Back rest extender (See chapter 5)
- Lumbar support and wedge (See chapter 5)
- Different back cushions

#### **FOOT SUPPORT**

Universal foot support

#### **ARM SUPPORT**

Hemi cushions (See chapter 5)

#### **HEAD SUPPORT**

 Different head support models (See chapter 5)

## 5. Accessories

#### **Belts**

model.

Several variants: Hip belts with or without upholstery and with plastic lock or car lock.



(See chapter 5.1 for mounting)

#### **Backrest Cushions** Many to choose from. Please contact your dealer.



Spoke protectors For 20", 22" and 24". Black or transparent is optional.



**Trays** 3 models: Swing-able, lockable and standard "push on"



Foot box Upholstered



Upholstery for tray Offers a soft base for the arm resting on the tray.



Foot supports:



Wedge Increases side support.



Universal Adjustable in fixed positions between 33° to 105° using an adjustment wheel.



Lumbar support Increases lumbar curvature.



**Amputation support** 



Vital Base Integral Pelvic stabilizer.



Upholstery for calf support bracket Reduces pressure.



**Seat Cushions** Many to choose from. Please contact your dealer.



Head support Support C Large. Support A Side support Support B Small



Side support Correction Meant for correction of bad postures in the upper trunk.



Hygiene cover Protects the core of the head support.



Pad for side support Correction



Head cushion 40x40 cm cushion with Kospoflex filling and rubber band.



Side support Stable Meant for users with decreased stability of the upper trunk. For otimal function use together with Stable cushion.



**Head cushion Comfort** Cushion with Kospoflex filling to pull onto head rest.



Pad for side support Stable



Hemi cushion An accommodating support for hemiplegic users.



Calf pad hinged The user doesn't have to lift the leg when mounting or dismounting the foot supports.



Backrest extender 12 cm extender. To be used together with 60 cm back rest cushion.



Abduction block The block reduces abduction. Small: 80 mm



width 140 mm

width





width Medium: 120 mm



Main wheels 12". 16" and 24" with drumbrake



Anti tip Anti tip with tramp pedal

Large:



Foot board with lock The foot support can be swung to the side like standard foot supports.



Tool set



### 5.1 Assembling of hip belt

#### Illustration 5.1.1

• Pull the belt through the hole in the hip belt bracket.



III. 5.1.1

#### Illustration 5.1.2

• Thread the belt back through the belt clamp.



III. 5.1.2

#### Illustration 5.1.3

• Fix the hip belt bracket to the back rest hinge in the rearmost hole, using the enclosed screws and nuts.



III. 5.1.3

2 pcs13 mm open-end spanner

## 6. ASSEMBLING AND **ADJUSTMENT**



### 6.1 Unpacking

- 1. Unpack all the parts, and check that everything is there according to the packing list.
- 2. Mount the main wheels (Chapter 6.2)
- 3. Mount the front castors (Chapter 6.3)
- 4. Check and adjust the seat dept (Chap. 6.6)
- 5. Mount the backrest (Chapter 6.5)
- 6. Mount the seat and back cushion (Chapter 6.9 and 6.10)
- 7. Mount the foot supports (Chapter 6.12)
- 8. Mount the head support (Chapter 6.13)
- 9. Mount any accessories (Chapter 5).

Tools needed are described under each chapter. Accessories described in chapter 5 is a presentation of options and will be delivered with sepearate mounting descriptions

#### 6.2 Main wheel

To mount the main wheel remove the quick release bolt from the hub bushing. lead it through the centre of the main wheel and into the hub bushing while pressing the knob in center (Illustration on next page)



▲ To check that the main wheel is properly attached to the hub, remove the finger from the central knob and pull at the main wheel.

A If the main wheel doesn't lock, don't use the wheelchair but contact your dealer.

### 6.3 Front castors

To take of

• Press the release button under the front fork.



#### To mount

• Lead the quick release axle into the bearing house. Pull the fork slightly to ensure that the fork is fully locked.



## 6.4. Seat height adjustment

The seat height at the rear depends on:


- Size of main wheel.
- Position of main wheel.
- Using 24" main wheels in the upper hole, the seat height is 46.5 cm from the floor to the seating plate.
- Using 24" main wheels in the next upper position, the height will measure 50 cm to the seating plate.
- Using 12" and 16" main wheels the seat height is 46.5 cm from the floor to the seating plate.

If it is required to change position of the main wheels or to change to a different size of main wheels, unfix the hub bushing including washer and nut. Remove the hub bushing and mount it in the required position.



2 pcs. 24 mm open-end spanner.

Main wheels and front castors should be mounted according to the description below.

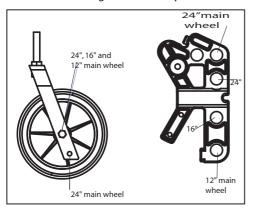

- When the seat height is changed ensure that the front castors are placed so that the lower frame tube is parallel with the ground.
- A Check the position of the anti tip and readjust the brakes after mounting the main wheels or changing the main wheel position.

#### 6.5 Backrest

- To mount the gas spring, lift the push bow with one hand, and lead the gas spring locking head into the plastic bracket (III. 6.5.1).
- Check that the hole in the locking head is parallel with the open holes in the plastic bracket.



III. 6.5.1.

• Lock the back rest by pushing the locking bolt through the plastic bracket and gas spring locking head (III. 6.5.2.)



III. 6.5.2

- To check that the back rest is locked, grip the push bow and press the back rest forward. If the back rest falls forward repeat the locking procedure or contact your dealer.
  - The wheelchair is set to a standard seat depth, and the plastic bracket has 4 holes of which three are temporarily blocked with plastic plugs (III. 6.5.3).



III. 6.5.3.

• The back rest hinge has 4 holes (III. 6.5.4) The hole positions are in accordance with the holes in the plastic bracket. If the locking head of the gas spring is mounted in the inner hole of the plastic bracket, the back rest hinge should also be mounted in the inner hole etc.



III. 6.5.4.

# 6.6 Seat depth adjustment at the rear.

- If the seat depth should be adjusted at the rear, release the locking bolt from the plastic bracket (III. 6.5.2.).
- Find the required position for the locking head in the plastic bracket, and remove the plastic plug from this hole.

- Lock the back rest by pushing the locking bolt through the plastic bracket and the gas spring locking head.
- After changing the hole position in the plastic bracket, the hole position in the back rest hinge must be changed into the parallel position.
- Check that the hole in the back rest hinge and the plastic bracket are mounted in parallel position.
- **%** 6 mm Allen-key

# 6.7 Seat depth adjustment in front

It is possible to adjust the seat depth with up to 10 cm in front. Do the following.

- Screw out the screws in the adjustment piece (III. 6.7.)
- Place the adjustment piece in the wanted position
- Replace and tighten the screws



III. 6.7

**%** 6 mm Allen-key

A If the user has spastic tendencies the adjustment piece should not be pulled out more than 5 cm.

### 6.8 Anti tip

The anti tips should be mounted according to the mounting description which is enclosed with the chair upon arrival.

#### Use of the anti tip

- Pull the anti tip out
- Turn it up or down 180°.
- Lock it in position



#### Adjusting the height of the anti tip

The anti tip can be adjusted in to fixed positions. The short position is for 12" & 16" main wheels and 24" in upper position. The long position is for 24" main wheels in lower position.

- Unfix the screw in the adjustable extension piece as shown in picture below.
- This extension piece has two holes. Place it in the required position and tighten the screw





▲ The anti tip should always be used for the safety of the user.

### 6.9 Seat cushion

The seat cushion is attached to the seat plate with velcro.



It is very important to place the cushion in the wheelchair before use.

#### 6.10 Back rest cushion

The back rest cushion is attached to the back rest with the velcro.



The seat and back rest cushion covers are washable and thereby reuseable. Follow the instruction on the cushions for correct maintanance and washing of the cushions.

# 6.11 Adjusting the velcro back



- Loosen the straps and place the back rest cushion so that the user gets room for the bottom and the integrated lumbar support in correct position.
- Tighten the straps so that they follow the curvature of the spine and gives a little extra support at the top of the sacrum.

### **6.12** Foot supports



#### Angle adjustable foot support

The Angle adjustable foot support is freely adjustable in angle. It is also swingable and removeable. The foot plates are foldable and can be angled in fixed positions. Delivered with height and depth adjustable calf support



#### Mounting of foot support

Grip the foot support in the hinge point.
 Hold it in an angle of app. 20° to the side frame. Put it into the plastic lock attached to the extractable adjustment bar. Swing it in and push down



#### Adjusting the angle

- The angle of the foot support can be adjusted using the star wheel.
- Loosening this star wheel enables you to adjust the foot support to the required angle.
- Fix the angle by tightening the star screw.



#### Adjusting the length of the foot support

- Untighten the screw (A) to make the adjustment tube move
- Adjust the foot support in required position and fix the screw properly.



#### Adjusting the angle of the footplate

- The foot plates can be adjusted in angle.
- Untighten the screw (B) and adjust the foot plate to the required angle. Fix the screw properly





#### Locking and releasing the foot plates

- The foot plates come with a locking bolt which makes the plates stronger
- To lock the foot plates pull the plastic lock on the right foot plate and place the lock over the bolt on the left foot plate.
- To release the foot plate pull the plastic lock and lift the right foot plate up.

For outdoor use, there should be a clearance of 4-5 cm between the plate and the ground.



#### Removing the foot support:

- Pull the plastic lock on the foot plate rearwards, so that the pin is released, and the foot plate can be folded up.
- Raise the foot support a few degrees.
- Release the foot support by pulling it slightly straight up.
- Swing the foot support outwards.
- Lift and remove the foot support.



Adjusting the calf support
The calf support is height and depth
adjustable. To adjust height unfix the nut
on the outside of the calf support bracket,
find the required height and fix the nut
again (III. A).




To adjust in depth, the calf pad is removed from the bracket by using an open-end spanner between the pad and the bracket. Find the required position and refix it (III.B)



A Never stand on the foot supports!

A Never lift the wheelchair by the foot supports.

### 6.13 Head support

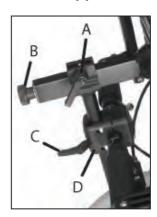

A - Lever for depth adjustment

B - Wheel for angle adjustment

C - Lever for height adjustment

D - Head support bracket





- Place the squared nut in the trace of the head support bracket as shown above.
- Place the head support in the head support bracket.
- The height and the depth of the head support is set to the required positions and tightened.



 The head support bracket is fixed by tightening the four screws two by two diagonally so the bracket is fixed with the same strength divided on the four screws.

#### Adjusting the head support in depth:

- Release the locking lever on top of the vertical bar (A).
- Adjust the head support and fix it in required position.

#### Adjusting the head support in height:

- Release the locking lever on the head support adapter (C).
- Adjust the head support and fix it in required position.

#### Adjusting the head support in angle:

- Release the adjustment wheel at the rear of the horizontal bar (B).
- Adjust the head support and fix it in required position.

#### Adjusting the head support sideways:

- The head support adapter can be moved both to the right and left, giving the possibility to accommodate special needs for head support.
- Untighten the four screws holding the adapter together.
- Move the adapter to the required position and fix the adapter by tightening the screws diagonally.
- A Remember to release the levers when adjusting the head support.
- If the head support stand does not fit the bracket perfectly the bracket is probably fixed too tight or unevenly.
- After fitting the head support fix it properly by tightening the little set screw in the centre on top of the head support bracket using an Allen key.
- If the head support seems to short in height, it can be turned 180° by releasing the adjustment wheel at the rear of the horizibtal bar (B)

## UREHAB

### 6.14 Arm support



• The arm support can be swung backwards



• Press the red handle to release the arm support for to swing it backwards



Adjusting the depth of the arm support

• Press the red knob for to adjust the depth of the arm support.



#### Adjusting the height of the arm support

- Untighten the screw on the arm support using a 4 mm Allen key.
- · Raise or lower stem
- Tighten the screw



**▲** When side support is mounted on the wheelchair, it will not be possible to revolve this arm support.

## 6.15 Adjusting the brakes

Netti 4 U Comfort CED is delivered with two diffrenet models of brakes.

• The brakes are freely adjustable along the frame tube.



III. 6.15.1

• To release the brake, pull the handle rearwards (III. 6.15.2).



III. 6.15.2

#### Netti 4U brake

• To activate the brake, push the handle forward (III 6.15.3



III. 6.15.3

• To release the brake, pull the handle rearwards (III. 6.15.4).



III. 6.15.4

 To reposition the brake, loosen the two screws on the inside of the brake clamp (III. 6.15.5).




III.6.15.5

- For fine adjustment, loosen the upper screw on the inside of the brakes (III. 6.15.6)
- Adjust the brake position and tighten the screws.



III. 6.15.6

**%** 5 mm Allen key

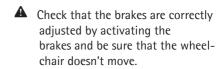

⚠ The brakes are constructred as parking brakes and shall not be used as driving brakes

#### Drum brake

If the wheelchair is mounted with 12" or 16" main wheels, these will be equipped with drum brakes.



#### If the brake doesn't brake properly:

To adjust the wire on one or both sides, adjust the foot screw 2-4 rounds out. Then re-check the brakes.

#### If the wire is too loose:

Adjust the foot screw all the way in. Tighten the wire by loosening the wire clamp before pulling the wire further through it. Tighten the wire clamp, and adjust the foot screw out again



1 pc 10mm open-end spanner.

#### 6.16 Push bow

The push bow is handle adjustable.



III. 6.16.1

• Open the handle for to change the angle of the push bow (III. 6.16.1)



III. 6.16.2

 Lock the handle after placing the push bow in the required position. (III. 6.16.2)

## 6.17 Seat angle/tilt

The seat angle is regulated using the release handle mounted on the push bar. The seat unit can be tilted from -5° to +20



### 6.18 Back rest angle

The back rest angle is regulated using the release handle mounted on the push bar. The angle can be regulated from 90° to 135°.

#### The release handles have the following label





Recline

A Risk for tipping. Check the position of anti tip.



A Don't leave a user in the wheelchair when it is tilted forward

## 7. MANOFUVRING

### 7.1 General techniques

The weight and balance of the chair influences the manoevring qualities of the wheelchair. The weight, size and sitting position of the user are also influencing factors. The position of the wheels will in addition influence the manoevring qualities. The more weight placed over the main wheels, the easier it is to manoevre. If heavy weight is placed over the front castors, the chair will be heavy to maneuvre.

#### Companion:

If the user is left alone in the wheelchair. always lock the brakes and check that the anti tips are turned down.

#### Parking:

Increase the underneath support of the wheelchair by moving the chair about 10 cm rearwards making the front castors turn forward.

### 7.2 Driving techniques - Step up-

Companions, drive up a step forwards:

- Check that the anti tip is turned up.
- Angle the wheelchair backwards.
- Balance the chair on the mainwheels and push it forward until the front castors are on the step.
- Lift the push handles while pushing the chair onto the step.
- Turn the anti tip downwards.

### Users, drive up a step backwards:

This technique is only useful if the step is very low. It also depends on the clearance between the footplates and the ground.

- Check that the anti tip is turned up.
- Drive the chair backwards towards the step
- Make a firm grip on the push rims and move the body forward while pulling.
- Turn the anti tip downwards.

# Companions, drive up a step backwards:

- Check that the anti tip is turned up
- Pull the chair backwards next to the step
- Angle the wheelchair backwards, moving the front castors slightly up in the air.
- Pull the wheelchair up the step and go backwards long enough to put down the front castors on the step.

# 7.3 Driving techniques - Step down-

#### Companions, drive down a step forwards:

- Check that the anti tip is turned up
- Angle the wheelchair backwards, moving the front castors slightly up in the air.
- Drive carefully down the step and angle the wheelchair forward putting the front castors back on the ground.

**A** Turn the anti tip downwards.

# Companions, drive down a step backwards:

- Check that the anti tip is turned up.
- Move the wheelchair backwards to the step.
- Drive carefully down the step and move the wheelchair backwards on the main wheel until the front castors have come away from the step.
- Put the front castors down on the ground.

Turn the anti tip downwards.

# 7.4 Driving techniques - Slope -

Important advise for driving down and up hill avoiding the risk of tipping.

Avoid turning the wheelchair in the middle of a slope.

Always drive as straight as possible.

A It is better to ask for assistance than taking risks.

#### Driving uphill:

Move the upper part of the body forwards in order to maintain the balance of the chair.

#### Driving downhill:

Move the upper part of the body backwards to maintain balance of the chair. Control the speed of the chair by clutching the push rims. Do not use the brakes.

# 7.5 Driving techniques - Up stairs -

Always ask for assistance.

A Never use escalators, even if assisted by a companion.

#### With assistance, backwards.

- Check that the anti tip is turned up,
- Pull the wheelchair backwards to the first step of the stairs.
- Angle the wheelchair backwards on the main wheels.
- Pull the wheelchair slowly up the stair, one step at the time keeping the balance on the main wheel.

 Reaching the top of the stair, pull the wheelchair backwards far enough to put the front castors safely down on the floor.

▲ Turn the anti tip downwards.

If two companions are present, one person can assist lifting in the front of the frame.

**A** Do not lift the wheelchair holding onto the foot supports.

**A** Do not lift the wheelchair holding onto the arm supports.

The companions should use the strength in their legs carrying the chair, avoiding unnecessary stress on the back.

# 7.6 Driving techniques - Down stairs -

A Never use escalators, even if assisted by a companion.

#### With assistance, forwards

- Check that the anti tip is turned up
- Drive the wheelchair forward to the first step of the stair.
- Angle the wheelchair backwards on the main wheels.
- Have a firm grip on the push bow, and keep the balance on the main wheel taking one step at the time
- Reaching the bottom of the stair, put the front castors safely down on the floor.

A Turn the anti tip downwards.

If two companions are present, one person can assist lifting in the front of the frame.

**A** Do not lift the wheelchair holding onto the foot supports.

**A** Do not lift the wheelchair holding onto the armrests.

#### 7.7 Transfer

Techniques for transfering to/from the wheelchair should be practiced well with the persons involved. Here, we give some important advices for preparation of the chair.

# With or without companion – sideways. Before transfer:

- The wheelchair should be placed as close as possible to the destination of the transfer.
- Pull the wheelchair backwards 5-10 cm in order to make the front castors turn forward.
- Lock the brakes.
- Remove foot support and arm support on the side of the transfer.

# With or without companion – forwards. Before transfer:

- The wheelchair should be placed as close as possible to the destination of the transfer.
- Pull the wheelchair backwards 5-10 cm in order to make the front castors turn forward.
- Lock the brakes.
- Tilt chair forward

#### Using a lift:

Before transfer to chair:

- Tilt the chair back
- Remove the head support
- Remove the foot supports
- Open the back rest angle slightly
- Replace the components when transfer is finished.

### 7.8 Lifting the wheelchair

• The wheelchair should be lifted in the frame only and it is marked with symbol (ill. 7.8.1) where it should be lifted.





III. 7.8.1

III. 7.8.2



A Never lift the wheelchair with in the the push bow,. This is marked with symbol (ill.7.8.2).



A Never lift the wheelchair in the foot supports or arm supports



A Never lift the wheelchair with a user in it.

### 8. TRANSPORT

Netti 4U Comfort CED is tested and approved to crash test ISO/FDIS 7176/19.

### 8.1 Folding for transport

- Remove the head support (Chapt. 6.13)
- Turn the anti tips upwards (Chapt. 6.8)
- Remove the foot supports (Chapt. 6.12)
- Pull out the locking bolt for the back rest, and place the backrest forward in the seat (Chapt. 6.5)
- Remove the main wheels (Chapt. 6.2)
- Remove the front castors (Chapt. 6.3)

### 8.2 Transport in car

It is important to notice that correctional belt systems used in the chair are not replacing the need for a safety belt when transported in a car.

If the head support is mounted correctly it is very stable but does not replace the need for an external neck support mounted in the car.

The chair is marked with stickers in 4 positions, showing where to fix the straps



In front: Use hook or strap attachment.



A 3-point belt system is recommended. This secures the user well during transport. In the pictures straps of the type Q'Straint is used.



Mount an "eye-bolt" in one of the holes in the

wheel frame bracket. Hook on a hook/carabine hook in the "eye-bolt"

The angle of the straps should be  $0-45^{\circ}$ 

Frequency	Weekly	Monthly
Check defects/damages E.g. breakage/missing parts	Х	
Washing of wheelchair		Х
Oiling of bearings*		Х
Washing of cushions		Х
Check anti tip function		Х
Check brake adjustment		Х
Check tyre wear		Х

\* As a rule of thumb, use oil on movable parts and all bearings. Alu Rehab recommends use of ordinary bicycle oil

### 9. MAINTENANCE

# 9.1 Maintenance instructions

Extra parts/replacement of single parts
The Netti chairs are built of modules. Alu
Rehab carries stock of all parts and is
ready to supply these on short notice.
Necessary instructions for mounting will
follow the parts.

- A You can order original paint from Alu Rehab to repair scratches and minor damages to the paint: Please contact your dealer to order.
- ▲ If defects or damages occur, please contact your dealer.
- Check/re-adjust screws and nuts at regular intervals.

### 9.2 Cleaning and washing

- Remove seat cushion before washing the wheelchair.
- 2. Clean cushions and covers according to instructions printed on cushions.
- 3. Clean frame using water and a rag.
- 4. If jet water washing, avoid pointing directly towards gas springs, labels and back cushion.
- 5. We recommend using soft soap.
- 6. Wash the wheelchair well using clean water to remove all the soap.
- 7. Use methylated spirit to remove any dirt eft
- For small damages to the surface, original surface paint can be ordered from the manufacturer.

## 10. TROUBLESHOOTING

Symptom	Reason/Action	Reference in manual
The wheelchair is going askew	<ul> <li>The main wheel hubs might be incorrectly mounted</li> <li>One of the brakes might be too tight</li> <li>The user is sitting very askew in the chair</li> <li>The user might be stronger on one side than the other</li> </ul>	6.4 6.12
The wheelchair is heavy to manoeuvre	<ul> <li>The main wheel hubs might be incorrectly mounted</li> <li>Clean the front castors and forks for dirt</li> <li>Too much weight over the front castors</li> </ul>	6.4
The wheelchair is hard to turn	<ul> <li>Check that the front castors are not fixed too hard</li> <li>Clean the front castors and forks for dirt</li> <li>Check that the front castors are placed in correct position</li> </ul>	6.3 6.3
The main wheels are difficult to take off and put on.	<ul> <li>Clean and grease the quick release</li> <li>Adjust hub bushing further out from the frame</li> </ul>	6.4 6.4
The brakes are not functioning well	Adjust the brake	6.12
The wheelchair feels "shaky"	Check screws and adjustment points in general	

Please contact your dealer for information about authorized service facilities that can give support if solution is not reached in this form.

When in need of spare parts, please contact your dealer.

# 11. TESTS & WARRANTY

The wheelchair is tested and has been approved for usage both indoors and outdoors. The chair is CE marked.

Maximum user weight: 140 kg

TÜV Product Service GmbH.

EN 12183: 1999.



Tested and approved to crash test: ISO 7176/19.

#### **CE** marking



The CE mark is a proof that the chair meets the requirements of the EU directive for medical devices.

### Warranty

#### Frame

Five years against material and/or mistakes/damages in manufacturing and workmanship.

Back rest, armrest and foot support One year against material and/or mistakes/ damages in manufacturing or workmanship.

Cushions, tires and spokes No quarantee.

Damages covered by the warranty will be repaired/replaced. Please contact your dealer. If in doubt, please contact your dealer.

#### Special adjustments/adaptations

Special adaptations/adjustments are defined as all adjustments that are not included in this manual. When wheelchairs are specially adjusted/adapted by the customer, the warranty given by Alu Rehab A.S Norway will not be valid. If any uncertainty about special fitting and adaptations, please contact Alu Rehab A.S.

Combinations of Netti and other products not manufactured by Alu Rehab A.S Generally in these cases, the CE mark of all the products involved will not be valid. However, Alu Rehab A.S has made agreements with some manufacturers about some combinations.

For further information, please contact your dealer or Alu Rehab A.S Norway directly.

## 12. MEASURES & WEIGHT

Seat width*	Seat depth**	Back rest heigh ***	Total width	Transport width	Weight
35cm	42,5-50 cm	48(60) cm	53 cm	46 cm	28,0 kg
40cm	42,5-50 cm	48(60) cm	58 cm	51 cm	28,5 kg
43cm	42,5-50 cm	48(60) cm	61 cm	53 cm	29,0 kg
45cm	42.5-50 cm	48(60) cm	63 cm	54 cm	29.0 kg
50cm	42,5-50 cm	48(60) cm	68 cm	61 cm	29,5 kg
55cm	42,5-50 cm	48(60) cm	73 cm	68 cm	31,5 kg
60cm	42,5-50 cm	48(60) cm	78 cm	71 cm	32,0 kg

<sup>\*</sup> Measured between skirt guards.

The weight is including main wheels, front castors, foot supports and arm supports. No cushion.

Max user weight is 140 kg.

Dealer:	
Serial no.:	
Date supplied:	

<sup>\*\*</sup> Measured from front of seat plate to back rest hinge without cushion. Using standard Smart back rest cushion subtract app. 3 cm.

<sup>\*\*\*</sup> Measured from from seat plate to top of back rest velcro.



Bedriftsveien 23 NO-4353 Klepp Stasjon, NORWAY

Tel. +47 51 78 62 20 Fax +47 51 78 62 21 E-mail: post@alurehab.com www.alurehab.com