STORZ

KARL STORZ—ENDOSKOPE

Anleitung LAPAROSKOPIE 1/2007

Instructions LAPAROSCOPY 1/2007

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Anleitung

Manual

Manual

Anleitung LAPAROSKOPIE 1/2007

Instructions LAPAROSCOPY 1/2007

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Important information for users of KARL STORZ instruments

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, these products are the result of years of experience and great care in manufacture. You and your organization have decided in favor of modern, high-quality items of equipment from KARL STORZ.

This manual contains information for personnel authorized to prepare instruments for laparoscopy. The instruments and accessories manufactured by KARL STORZ are, as a rule, sensitive precision instruments which require particular care during preparation. To prevent damage during preparation, support is provided here for disassembly, preparation and assembly of the KARL STORZ endoscopic instruments.

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KARL STORZ is constantly working on the further development of all products. Please appreciate that changes to the scope of supply in form, equipment and technology are possible for this reason. Therefore, no claims may be deduced from the information, figures and descriptions in this manual.

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**IMPORTANT INFORMATION FOR USERS
OF KARL STORZ UNITS AND
INSTRUMENTSU 2**

ABOUT THIS MANUAL..... 5
Using this manual..... 5
Symbols and hazard information..... 6
Safety information 7
General information on cleaning and sterilization 9

TROCARS FOR LAPAROSCOPY 16

Cones for open laparoscopy..... 16
General information for trocars and accessories 16
VERESS pneumoperitoneum cannulas..... 17
Rigid trocars with silicone lip valve, 2.5/11 mm 18
Flexible trocars with silicone lip valve 9 mm 18
Trocars with multifunction valve 20
Trocars with automatic flap valve < 5 mm..... 23
Trocars with automatic flap valve 6 - 13 mm .. 24
Instructions for LUER LOCK stopcocks..... 25
Reduction cannulas / Extractors 27
Spiral cannulas 27
Distance holders..... 28
Dilation set..... 29
Preparation overview..... 30

DISSECTING AND GRASPING

FORCEPS 31

General information for forceps and scissors .. 31
CLICK^{line} instruments with and without connector pin for unipolar coagulation 32
CLICK^{line} grasping forceps, angulating..... 35
Take-apart instruments with U-spring handle..... 37
Insulated instruments with U-spring handle..... 39
Surgical sponge holder 40
General information on coagulation instruments..... 43
Take-apart® forceps with double sheath for bipolar coagulation..... 45
RoBi™ instruments 48
Preparation overview..... 51

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INSTRUMENTS FOR UNIPOLAR, AND BIPOLAR COAGULATION 55

General information on coagulation instruments.....	55
Dissecting electrodes with exchangeable tip... ..	57
MANHES high-frequency needle	59
Coagulation and dissecting electrodes	60
Bipolar needle	61
Take-apart® bipolar coagulation suction tube ..	62
GORDTS-CAMPO coagulation suction tube ...	63
Adaptor for coagulation and dissecting electrodes.....	64
REMORGIDA bipolar instrument	65
Preparation overview.....	69

RETRACTORS, DISSECTORS, PALPATION PROBES AND EXTRACTORS 70

CUSCHIERI liver retractor	70
Take-apart fan retractor, 5 mm and 10 mm.....	72
Deflecting retractor model 26173 FP.....	73
Retractors for gastric banding.....	74
Anal dilators.....	75
Take-apart CUSCHIERI variable curvature dissector	76
Preparation overview.....	77

SUTURE AND LIGATURE 78

Needle holder and suture instrument with ring handle	78
Needle holder with straight and curved handle, with and without ratchet.....	78
SCARFI needle holder	79
DUBECQ/PRINCETEAU needle holder	80
CUSCHIERI integrated knot pusher and thread catcher	82
Take-apart CUSCHIERI variable curvature suture and sling passer	83
Clip applicator model 30444 LR.....	85
Preparation overview.....	87

MODULAR HANDLES AND INSTRUMENTS FOR SUCTION AND IRRIGATION..... 88

Handle with piston valve	88
Handle 30810 for irrigation, suction and coagulation.....	89
Handle for irrigation and suction models 37112 T, 37113 T	92
Metal suction and irrigation tubes.....	95
Suction and irrigation tubes with two-way stopcock.....	95
Preparation overview.....	96

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INGUINAL CANAL RETRACTOR, INSTRUMENTS FOR MINI-LAPAROTOMY AND FOR HAND-ASSISTED LAPAROSCOPIC SURGERY.....

DUNDEE multitool.....	98
CUSCHIERI inguinal canal retractor.....	98
Instruments for hand-assisted laparoscopic surgery.....	98
Preparation overview.....	100

MORCELLATORS	101
STEINER morcellator.....	101
SAWAHLE morcellator.....	106
CHARDONNENS morcellation knife.....	113
KARL STORZ ROTOCUT G1.....	115
Preparation overview.....	116

INTRAOPERATIVE CHOLANGIOGRAPHY AND CHOLEDOCHOSCOPY	119
OLSEN cholangiography fixation forceps.....	119
BERCI CLICK ^{line} grasping forceps, with rubber-coated jaws.....	119
Micro scissors.....	120
Endotom.....	122
Preparation overview.....	123

ANAL SURGERY	124
Video operating rectoscope.....	124
Preparation overview.....	126

LAPAROSCOPIC AORTIC SURGERY, INSTRUMENTS FOR HAND-ASSISTED LAPAROSCOPIC SURGERY.....

Trocars See <i>Trocars for laparoscopy</i> CLICK ^{line} scissors and forceps See <i>Dissecting and grasping forceps</i> Instruments for hand-assisted laparoscopic surgery See <i>Inguinal retractor, instruments for mini-laparoscopy</i>	127
--	-----

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INSTRUMENTS FOR VIDEO-ASSISTED THYROIDECTOMY.....	128
Dissecting and grasping forceps <i>See Dissecting and grasping forceps</i>	
Bipolar coagulation forceps <i>See Instruments for unipolar and bipolar coagulation</i>	
GYNECOLOGY.....	129
Intrauterine probes and uterus elevators	129
COHEN intrauterine probe	129
Tenaculum forceps / Uterus grasping forceps.....	130
SWOLIN cervix cannula.....	131
HOHL uterine manipulator.....	132
CLERMONT-FERRAND model uterine manipulator	135
SAWAHLE uterine manipulator.....	142
C.C.L. vaginal extractor.....	146
Ovarian biopsy forceps	147
Ring applicator for 2/4 rings 26173 RA/26174 RA.....	148
REMORGIDA double bipolar forceps.....	151
GYNFORCE III Take-apart punch.....	154
Preparation overview.....	161
MAINTENANCE.....	162
Operating/Storage conditions	162
Disposal.....	162
Directive compliance.....	162
Limitation of liability.....	162
Warranty	163
Important information.....	163
Servicing and repair.....	164
Repair and replacement program.....	164
APPENDIX.....	165
Cleaning agents and disinfectants.....	165

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Using this manual

This manual is not an instruction manual, but instead provides general information regarding disassembly, preparation and assembly of laparoscopic instrument sets from KARL STORZ. This manual presumes knowledge of the 'Cleaning, sterilization and care of KARL STORZ instruments' manual, in which the various preparation methods are described in detail. If you do not yet have this manual, it can be requested without charge from KARL STORZ.

Particularly sensitive product groups, such as telescopes or fiberscopes, have their own manuals. Separate instruction manuals and service manuals are available for units. Other general regulations (normal use, user qualification, safety precautions when using instruments, etc.) must be heeded.

Note: For the most part, all of the methods described in the section 'General notes on cleaning and sterilization' can be used for the instrument set described in these instructions. Where this is true, the suitable methods are no longer specified separately in the descriptions of the individual instruments.

When certain methods (e.g., ultrasound cleaning, steam sterilization) are not suitable for particular instruments, the necessary restrictions are specified in the descriptions of the individual instruments.

In addition, a table can be found at the end of each section giving an overview of the individual instruments and the preparation methods suitable for each.

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Symbols and hazard information

Please read this manual and follow its instructions carefully. The words **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To make the words **Warning** and **Caution** stand out more clearly, they are accompanied by a pictogram.

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result an injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the instrument.

Note: A Note indicates special information about operating the instrument or clarifies important information.

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Safety information

Warning: Make certain before every application that the instruments are in perfect condition. If there is any sign of damage, the instrument must not be reused.

Warning: This instrument is delivered **unsterilized** and must therefore be cleaned, disinfected and/or sterilized before the initial application as well as before any further use.

Warning: During the application, the tip of the instrument must remain in the user's field of view.

Warning: The listed cleaning, disinfection and sterilization methods alone do not guarantee disinfection or sterility. This can only be achieved if the staff follows recognized and validated preparation methods.

Warning: For all cleaning and disinfection procedures on contaminated endoscopes and instruments, the guidelines of the Employer's Liability Insurance Association and equivalent organizations must be observed.

Warning: Remember that sterilization is only then successful if:

- the instruments have been freed of all organic materials and cleaning residues. Successful sterilization is only possible on clean, disinfected surfaces;
- work proceeds according to recognized and validated methods;
- the recommended sterilization parameters are maintained. These are only valid with sterilization equipment that is properly maintained and calibrated.

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Caution: Use of preparation methods other than those listed here may have a negative effect on the function and service life. All damages resulting from this are not covered by the guarantee.

Caution: The following information on cleaning and sterilization only applies to the thermostable instrument set described in these instructions. For HOPKINS telescopes and fiberscopes, special rules apply (see instructions for 'telescopes' and 'fiberscopes').

Caution: For the production and application of solutions, precisely observe the information from the manufacturer regarding concentration and exposure times. A maximum immersion time of 60 minutes, including sterile water, must not be exceeded. Prolonged immersion can lead to corrosion and/or material damage.

Caution: Constant change between various preparation methods is especially hard on the materials and must therefore be avoided. A one-time change of method, due to procurement of new cleaning and sterilization equipment for example, is however possible.

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Observe the guidelines of the Employer's Liability Insurance Association and equal ranking organizations.

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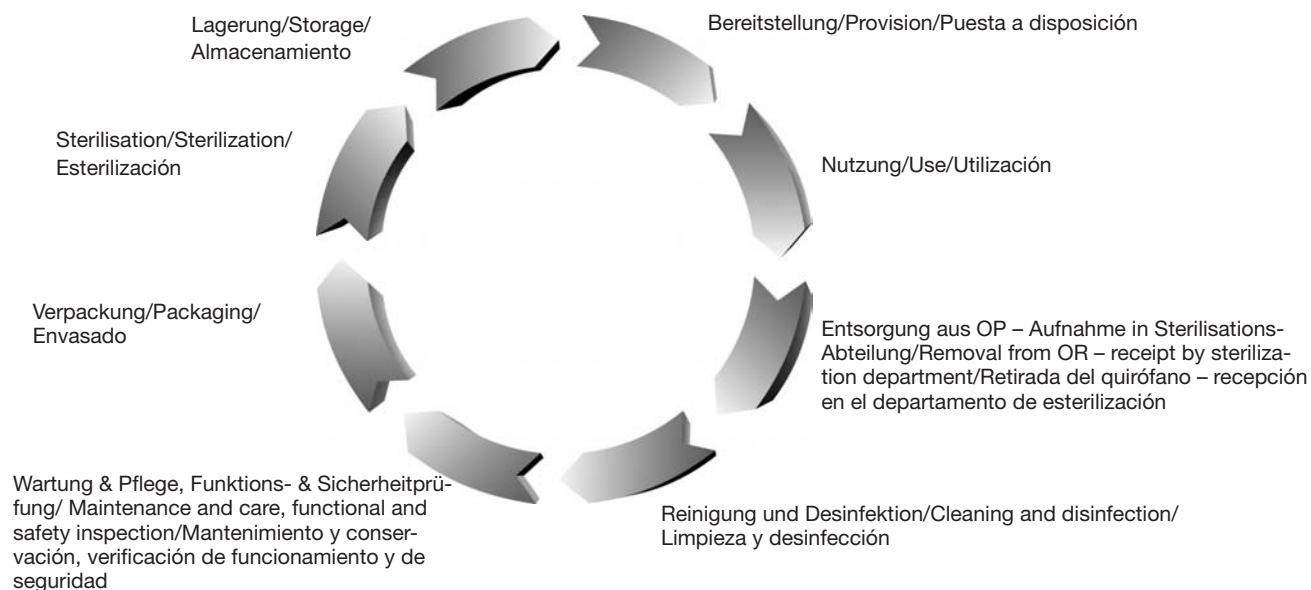
General instructions on cleaning, disinfection and sterilization

The medical device cycle shown below illustrates the sequence of steps for advisable instrument preparation:

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According to the instrument cycle, preparation of the instruments described here generally comprises the following 6 steps:

Caution: The instructions for cleaning and sterilization in the manual 'Cleaning, sterilization and care of KARL STORZ instruments' must be observed. The methods for cleaning, disinfection and sterilization are explained therein in detail.

Note: Recommended cleaning accessories and sterilization containers can be found in the catalog Section 'Cleaning, sterilization and care'.

1. Manual preliminary cleaning

The instruments may require manual preliminary cleaning on account of their design (e.g., slits and long, narrow lumens) and depending on the type and degree of soiling.

In addition, a long waiting period before preparation (= which allows contaminations to dry out) may require a preliminary cleaning to be performed in the operating room first, whereby organic residues, e.g., from pharmaceuticals, can be removed with a moistened cloth (no rapid-action disinfectant containing alcohol) or rinsed off with the help of an injection aid filled with active cleaning solution. Used and contaminated instruments should always be prepared with the minimum possible delay.

Take-apart instruments must be disassembled following the instructions. Articulating instruments and scissors must be treated whilst open.

Tissue remnants sticking or caked to the jaws must be removed with a cleaning brush. Incrustations should be removed as quickly as possible with a disposable surgical pad moistened with a 3% hydrogen peroxide solution. Special care must be taken when cleaning the jaws, as they are precision-engineered parts.

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2. Ultrasound treatment

All non-take-apart scissors, coagulation instruments and microinstruments must be pre-cleaned with ultrasound (3 - 5 minutes exposure time). The cleaning effect of the ultrasound bath reaches even inaccessible hollow spaces. For this reason, all non-take-apart scissors and forceps, with or without irrigation channel, rail-sheath instruments, aspirators, circular punches, coagulation instruments and microinstruments must be preliminary cleaned with ultrasound.

Ultrasound has no cleaning effect on soft plastic parts because the oscillations are attenuated. Hollow instruments (aspirators, etc.) should be held at an angle in the ultrasound bath so that the hollow spaces can fill. Position large instruments in the bath so that no acoustic shadows are formed. The following items, amongst others, are not suitable for cleaning in an ultrasound bath: all rigid and flexible telescopes, electrical devices, cables, prisms, rubber articles, magnetic discs and fiberoptic light cables.

After ultrasound treatment, rinse thoroughly with completely demineralized water.

Place the instruments in a suitable instrument carrier (e.g., a tray holder).

Note: If manual preliminary cleaning is mandatory, the necessary steps are explained in this manual for each instrument.

3. Manual/Machine cleaning and disinfection

For preparation, multi-part instruments must be dismantled, and seals, cock plugs and protective caps removed.

Manual cleaning/Disinfecting cleaning

For manual cleaning or disinfecting cleaning, use active-cleaning, non-protein-fixing chemicals, where necessary with a verified range of action (DGHM (German Association of Hygiene and Microbiology) listing or comparable).

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Caution: When preparing and using the solutions, the manufacturers instructions concerning concentration and exposure time must be strictly adhered to. Prolonged immersion can lead to damage.

Note: An extract from the list of products approved by KARL STORZ for manual cleaning and disinfection can be found in the appendix of this manual and on the Internet (www.karlstorz.com).

When immersing instruments in cleaning and disinfectant solutions, ensure that air can escape from hollow spaces so that all the internal surfaces are completely wetted. It is best to immerse the instruments at an angle until no more air bubbles can be seen. During cleaning, always ensure that all hollow spaces (sheath lumens, stopcocks, valves etc.) are also cleaned and that there are no obstructions (flow test, for example with a syringe).

Thoroughly clean surfaces, interiors and jaws with a suitable disinfected soft brush, sponge or cleaning gun, and rinse. Take care when cleaning small parts.

The cleaning gun with accessories (Art. no. 27660) is ideal for rinsing out hollow spaces with water and drying them with sterile compressed air.

After cleaning, rinse the instruments copiously with microbiologically pure/sterile water and dry with sterile compressed air.

Thermal cleaning and disinfection in a machine

As far as possible, thermal cleaning/disinfection should be carried out in a machine, followed by steam sterilization at 134°C. The advantage of preparation by machine is that it reduces the risks for personnel and standardizes the cleaning procedure (validatable or validated processes are demanded).

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As there is sometimes strong movement of the water, it is necessary to secure all parts in holding devices to avoid damage. To ensure that the instrument is adequately rinsed out, it is essential to connect up the hollow spaces appropriately to irrigation tubes or irrigation devices.

It is not advisable to use lubricants, etc. in the machine because they may cause deposits to form on the instruments and may damage plastics. Selective manual care and lubrication of joints, etc., is safer and more effective.

Adequate final rinsing with completely demineralized water is required to remove all chemical residues.

Note: A list of products approved by KARL STORZ for machine cleaning/disinfection can be found in the appendix of this manual and on the internet (www.karlstorz.com).

4. Functional testing

All instruments should be checked for correct functioning before sterilization, at the latest. This functional check should include the following aspects in particular:

- Smooth operation of the joints
- General signs of wear
- Exact closing of forceps, clamps and punches, etc.
- Secure closing of valves and seals
- Sharpness of cutting instruments
- Condition of telescopes
- Condition of the insulating sheaths of cables, including cold light cables
- Broken fibres in cables

Damaged instruments must be separated out and sent for expert, qualified repair. For this purpose we recommend the KARL STORZ Repair Service, which ensures consistent quality and guaranteed safety of your instruments and units.

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Prior to sterilization, instruments with a closure mechanism or joint must be treated with lubricants capable of withstanding steam sterilization (e.g., instrument oil Art. no. 27656 B).

Note: Apply a thin coat of special grease to sealing and running surfaces, such as stopcocks and plungers (Art. no. 27657). This thin layer of grease not only serves to lubricate the metal surfaces which are in contact with one another, but also makes the stopcocks leakproof. Oil is not suitable as a sealant.

Warning: Only use special silicone- or oil-based lubricants, as the sterilants cannot penetrate all silicones and oils.

5. Sterilization

For almost all instruments, steam sterilization by the fractionated vacuum method at 134°C, gas sterilization (formaldehyde, ethylene oxide), plasma sterilization and chemical sterilization (STERIS) are possible. Steam sterilization at 134°C is usually the gentlest and most economical method of sterilization. This also applies to rigid telescopes because the longer exposure time for steam sterilization at 121°C can have a negative effect.

Sterilization is only ever possible on surfaces that have been cleaned and disinfected.

Almost all instruments can be sterilized without being dismantled. Only tighten screws and closures gently, so that they are easy to undo later. Insert the valves into the stopcocks and tighten the screws only gently before sterilization.

Note: If the design of the instrument necessitates sterilization in the dismantled state to ensure effectiveness, this is expressly stated for the article in question in this manual.

For steam sterilization (with saturated steam at 134°C + 3°C), the process defined in EN 285 or DIN 58946 Part 1, Number 3.25.2 should be used. Use prevacuum methods for instruments with a lumen. The steam must fulfill the requirements of EN 285, Appendix B, in order to avoid discoloration and stains.

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These procedures are also permissible for telescopes labeled 'AUTOCLAVE'.

Caution: When sterilizing telescopes, please follow the instructions in the telescopes manual.

Warning: Long lumens, slits and channels, and assembled instruments can only be sterilized completely with the fractionated prevacuum method.

As there are various types of cleaning, disinfection and sterilization, it is essential to ensure that recognized and validated (tested) preparation methods are used.

Refer to the manual 'Cleaning, sterilization and care of KARL STORZ instruments' for more detailed information.

6. Storage/Transportation

To avoid damage during transportation and storage, place the instruments carefully in suitable holding devices.

Suitable accessories, storage systems, and sterilization trays for special instrument sets can be found in the relevant Section of the TELEPRESENCE catalog.

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Cones for open laparoscopy

Disassembly

Take off the sealing cap. Check the sealing cap for damage, replace if necessary.
Unscrew the spring cap ①.
Screw the plastic cone out of the attachment point and pull out.

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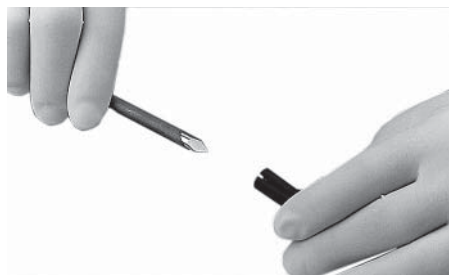
Assembly

Screw the plastic cone into the attachment point.
Screw on the spring cap ①.
Put on the sealing cap.

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General information for trocars and accessories

Warning: For pointed trocar tips, we recommend the use of protective caps or the protective sleeve 30100 Z for trocars sized 6-13 mm (all except 30105 K, 30106 K) for protection during transportation and to prevent injuries.

Protective caps may be placed on the individual tip or over the complete trocar.

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In the 'Laparoscopy' catalog, you will find the article numbers in the Section 'Spare parts' for protective caps for trocars, while the protective sleeve Art. no. 30100 Z (metal) is listed in the 'Trocars' Section. Check the trocar tips for damage and sharpness, and replace if necessary.

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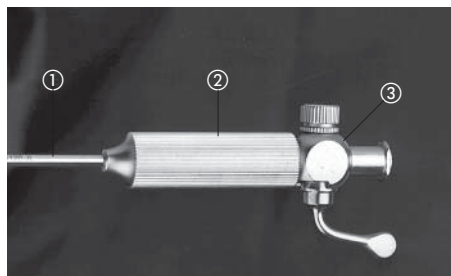
Sterilization

Sterilize trocars and trocar accessories in their assembled condition.

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cannulas

Disassembly

Note: Disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 25)

Unscrew the outer tube ① by rotating the knurling ② on the stopcock housing ③ and pulling off towards the front.

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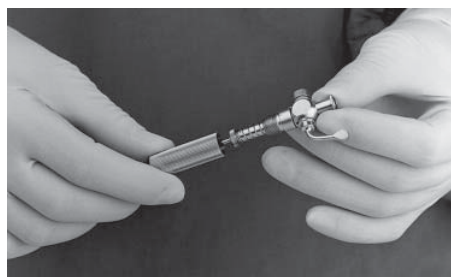
For manual cleaning, the stopcock is only disassembled after the inside tube has been rinsed out.

Manual preparation

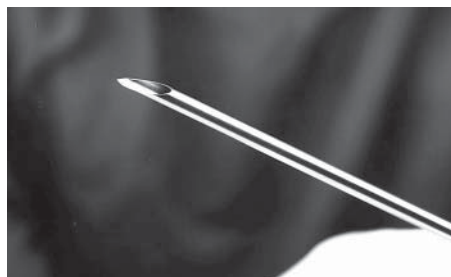
Connect the LUER connector and rinse out the inside of the sheath tube until clear water flows out the distal end.

Dry with sterile compressed air before sterilization.

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Machine cleaning/disinfection

The tubes must be rinsed out on the inside. To do this, an irrigation hose from the machine is connected at the LUER connection. Slide a suitable hose onto the knurling over the outer tube.

Assembly

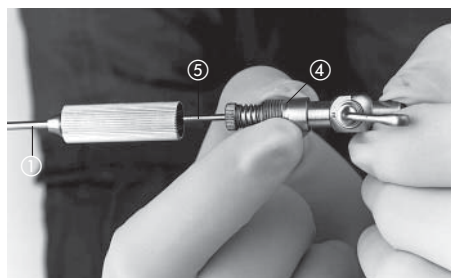
Check the needle tip for damage. Blunt, bent or broken off needles must be replaced.

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Lightly grease the cock plug and assemble. Check the movement of the spring mechanism ④. Slide the outer tube ① onto the inner tube ⑤ from the front and screw together. Again, check the movement of the spring mechanism.

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Rigid trocars with silicone lip valve, 2.5 mm and 11 mm

Note: Disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 25)

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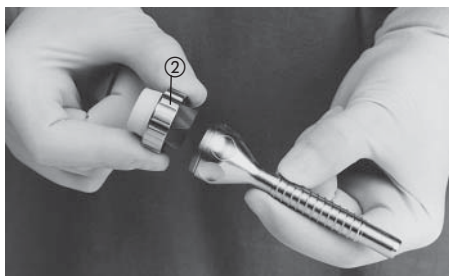
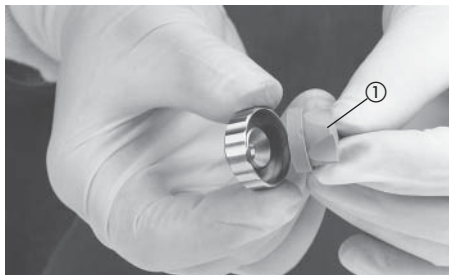
To remove the silicone lip valve ①, unscrew the knurled ring ②. Take out the silicone lip valve, check for damage and replace if necessary.

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Assembly

Insert the silicone lip valve ① into the knurled ring and screw on.
Put on sealing cap.
Insert the trocar tip.
Put on the trocar protective cap.

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Flexible trocars with silicone lip valve, 9 mm

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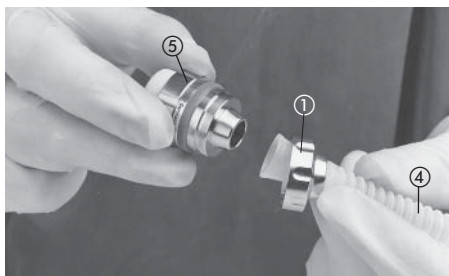
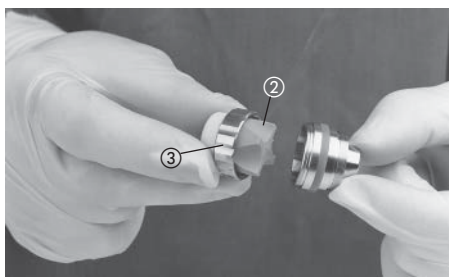
Turn the knurled ring ① clockwise (left-handed thread) and remove with the top of the valve ⑤ from the flexible trocar cannula ④.
Replace the flexible trocar cannula.

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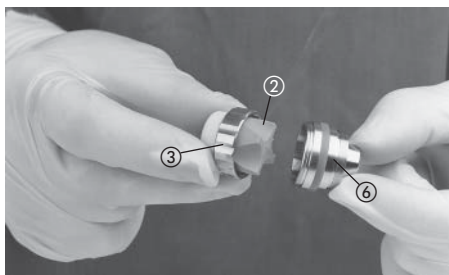
To remove the silicone lip valve ②, screw on the knurled ring ③. Take the silicone lip valve ② out of the knurled ring ③, check for damage, replace if necessary.

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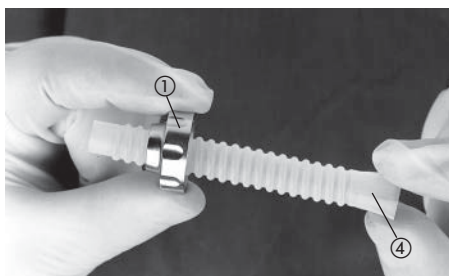
Assembly

Insert the silicone lip valve ② into the knurled ring ③ and screw onto the top part ⑥.

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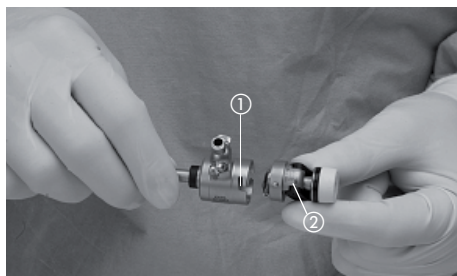
Place the knurled ring ① over the flexible trocar cannula ④ and screw together with the top of the valve ⑤ in place.

Put on the sealing cap.
Insert the trocar tip.
Put on the trocar protective cap.

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Trocars with multifunction valve

Note: Disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 25)

Disassembly

Remove trocar tip.
Take off sealing cap, check for damage; replace if necessary.
Loosen bayonet mount ① on the trocar cannula.
Take off the top of the valve ②.

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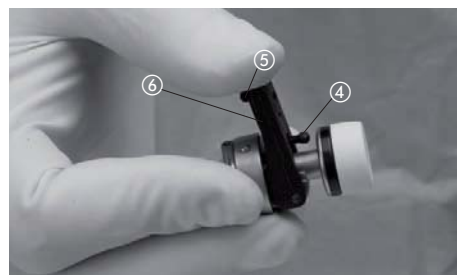
The top of the valve ② may be further dismantled in order to replace the small sealing cap, the plunger ④ or the inside seal ⑩. To do this, pull out the pusher ⑤ on the lever ⑥ on the top of the valve ②.

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Keep the lever ⑥ pressed down and pull out the plunger ④ towards the back.
For trocars size 6 mm (color code: black) and size 7 mm (color code: yellow), first unscrew the rear top part.

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Note: It is easier to unscrew the rear top part if the sealing cap is in place.

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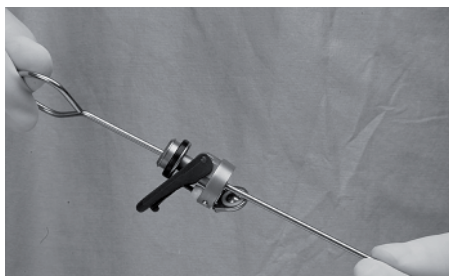
Check the inside seal ⑩ for damage, replace if necessary. To do this, turn the screw ⑪ over the seal clockwise (left-handed thread).

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Manual cleaning

Hold the valve flap open when cleaning the top of the valve.

Caution: Clean the valve flap carefully, since misalignment of the valve flap prevents smooth functioning.

Machine cleaning

Fix the flap in an open position using the opening rod (Art. no. 26 G) or a suitable holding device.

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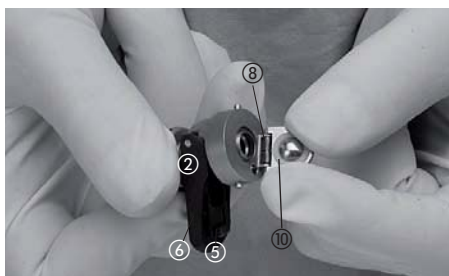
Assembly

Oil the joint on the flap valve ⑧ and lateral attachment point of the thumb plate ⑨ precisely.

If the top of the valve has been further disassembled:

Put on the small sealing cap. Insert the inside seal ⑩ into the recess in the flap and fix in place with the screw ⑪ by turning counterclockwise. Tighten the screw only slightly.

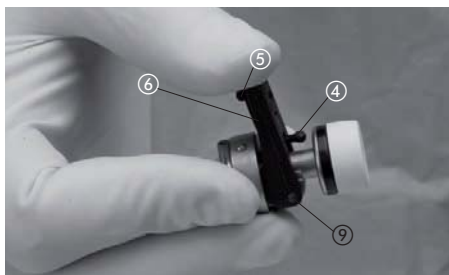
Check the flap valve for correct seating in its closed condition.

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With the thumb plate ⑥ depressed, insert the plunger ④ from behind.

Release the thumb plate ⑥ until the pusher ⑤ can be slid over the ball-shaped holder. For 6 and 7 mm size trocars, screw on the rear top part.

Note: It is easier to screw on the rear top part if the sealing cap is in place.

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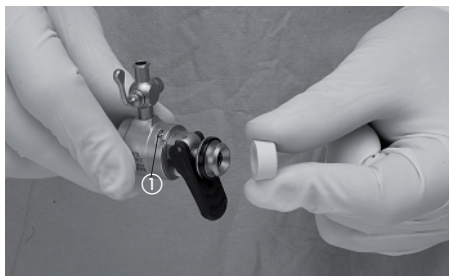
Note: To ensure the correct positioning of the tip of the REMORGIDA trocar tips, the positioning aid 12 must lock into place on the insufflation stopcock.

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Trocars with automatic flap valve < 5 mm

Note: Disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 25).

Disassembly

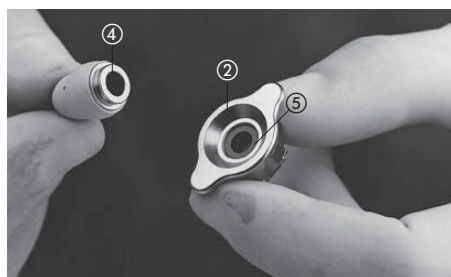
Remove the trocar tip.
Take off the sealing cap, check for damage; replace if necessary.
Loosen the bayonet mount 1 on the trocar

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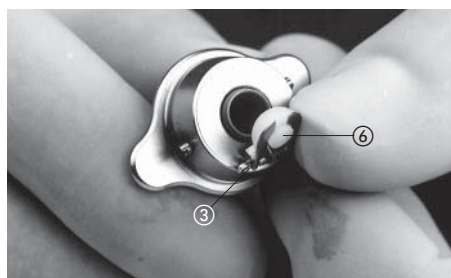
cannula. Remove the top of the valve 2.
Unscrew the back part 4 of the top of the valve 2 by turning clockwise, open the valve flap 3.
Check the gray seal 5 in the top of the valve 2 for damage, replace if necessary.

Note: It is easier to unscrew the back top part if the sealing cap is in place.

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Check the mushroom-shaped seal 6 on the flap valve 3 for damage, replace if necessary.

To do this, hold the flap open and pull the

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Assembly

Oil the joint on the flap valve precisely. Screw on the rear part of the valve top.

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Insert the top of the valve (2) into the trocar cannula and close the bayonet mount (1).

Put on sealing cap (3). Insert the trocar tip.

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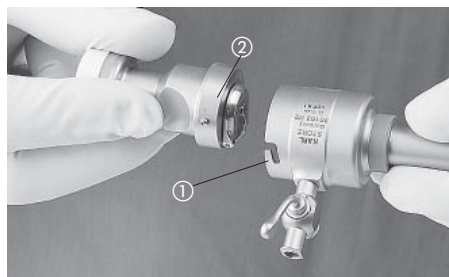
Trocars with automatic flap valve 6 mm - 13 mm

Note: For disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 25).

Disassembly

Remove the trocar tip.

Take off the sealing cap.

Loosen the knurled ring (1) on the trocar cannula.

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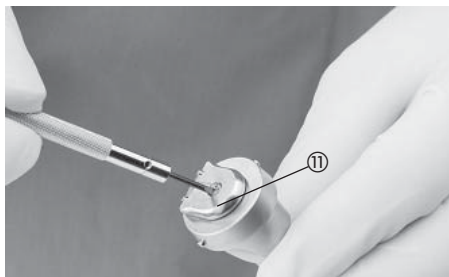
The top of the valve may be further dismantled. Check the inside seal (10) for damage, replace if necessary.

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To do this, turn the screw ⑪ over the seal ⑩ clockwise (left-handed thread).

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Machine cleaning

Fix the flap valve in an open position using the opening rod (Art. no. 26 G) or a suitable holding device.

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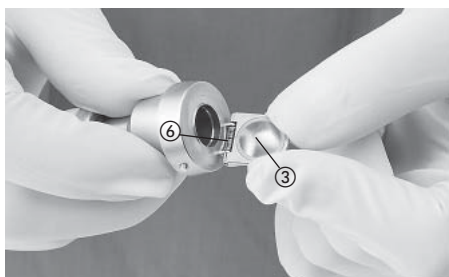
Assembly

Oil the joint ⑥ on the valve flap ③ precisely.

If the top of the valve has been further dismantled:

Insert the inside seal ⑩ in the flap recess and fix in place by turning the screw ⑪ counterclockwise. Tighten the screw only slightly!

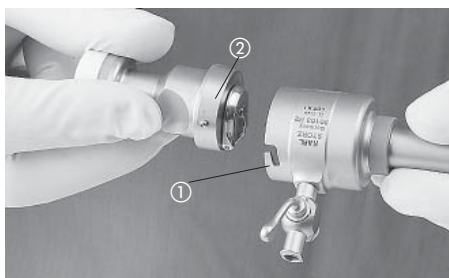
Check the flap valve ⑧ for correct seating in its closed condition.

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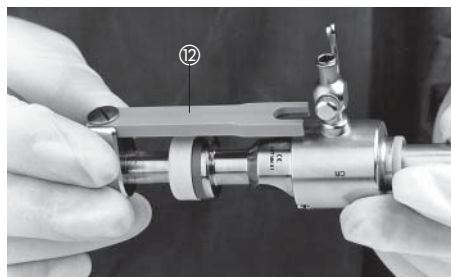
Insert the top of the valve ② into the trocar cannula and close the bayonet mount ①.

Put on the sealing cap.
Insert the trocar tip.

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Note: To ensure the correct positioning of the tip of the REMORGIDA trocar tips, the positioning aid (12) must lock into place on the insufflation stopcock.

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Instructions for LUER-lock stopcocks

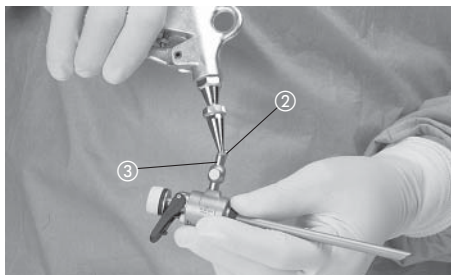
Cleaning before disassembly

Note: First clean the nozzle and sheath lumen with the stopcock open; use the cleaning gun if necessary.

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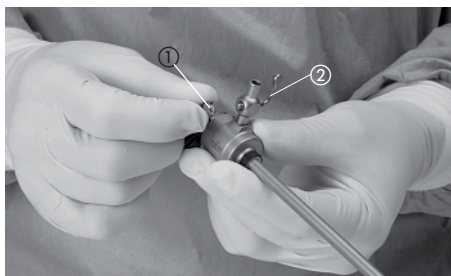
Set the stopcock to Open (cock plug (2) parallel to the nozzle (3)) and rinse out. Make certain that rinsing continues until clear water flows out of the front end.

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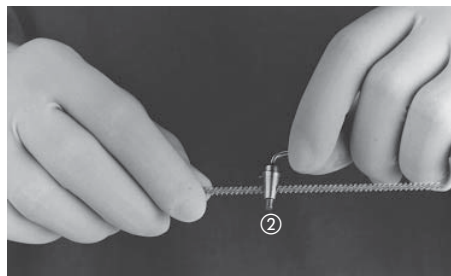
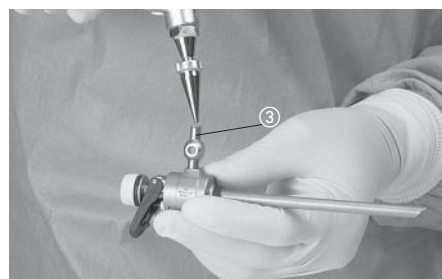
Cleaning
Carefully clean the stopcock ② and ...

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... connector ③ with a brush. The connector ③ and the sheath lumen must be rinsed out well inside at every cleaning, and then dried with sterile compressed air.

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Assembly
Lightly grease the contact surfaces on the stopcock ② (special grease for stopcocks; Art. no. 27657).

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Insert the stopcock ② and fix in place using the thumbscrew. Tighten slightly.

Test the cock plug for proper operation.

Note: Take stopcock apart again for sterilization.

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**Reduction cannulas / Extractors
Disassembly**

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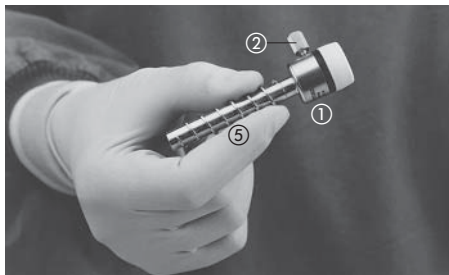
Assembly

Check the sealing cap for damage, replace if necessary, then put on.
Oil the swivelling reduction cannula at the joint ① precisely.

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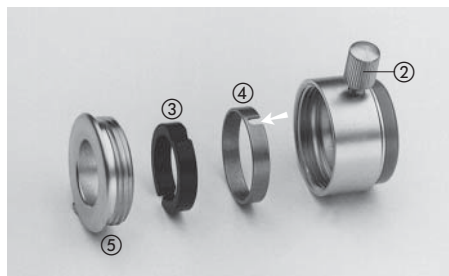
**Spiral cannulas
Disassembly**

Take off the sealing cap. Check the sealing cap for damage, replace if necessary.
Unscrew the top part ① of the spiral cannula from the threaded part ⑤.
Unscrew the knurled screw ② up to the limit stop.

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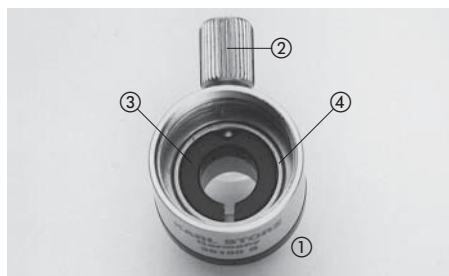
Take out the plastic inside ring ③.
Screw on the knurled screw ② somewhat, take out the metal inside ring ④.
Unscrew the knurled screw ②.

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Assembly

Screw the knurled screw ② halfway into the top part ①. Place the metal inside ring ④ with the groove toward the back over the knurled screw ②. The metal inside ring ④ lies flat in the top part ①.

Unscrew the knurled screw ② up to the limit stop. Place the plastic inside ring ③ with the recess under the knurled screw ②.

Note: The opening of the plastic inside ring is now facing the knurled screw.

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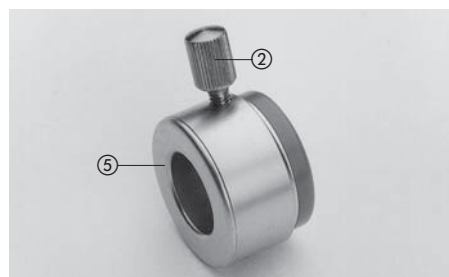
Screw the threaded part ⑤ into the top part ①.

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**Distance holders
Disassembly**

Unscrew the backing plate ⑤.

Note: Place the spacing sleeve with the backing plate onto the tip of the finger, press and turn.

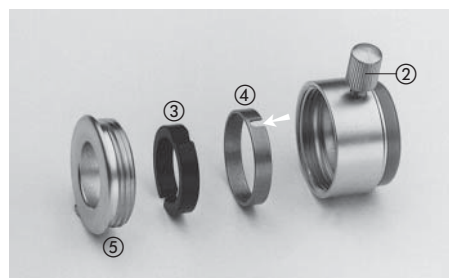
Unscrew knurled screw ② up to the limit stop.

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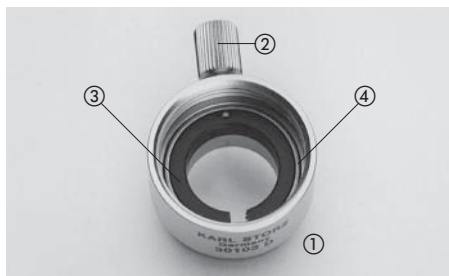
Take out the plastic inside ring ③. Screw on the knurled screw ② somewhat, take out the metal inside ring ④. Unscrew the knurled screw ②.

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Assembly

Screw the knurled screw ② halfway into the top part ①. Place the metal inside ring ④ with the groove toward the back over the knurled screw ②. The metal inside ring ④ lies flat in the top part ①.

Unscrew the knurled screw ② up to the limit stop. Place the plastic inside ring ③ with the recess under the knurled screw ②.

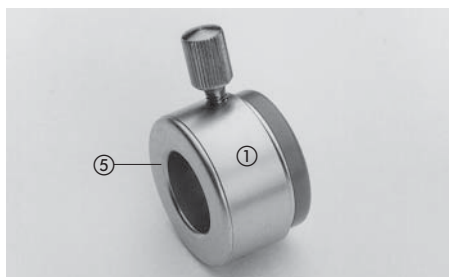
Note: The opening of the plastic inside ring is now facing the knurled screw ②.

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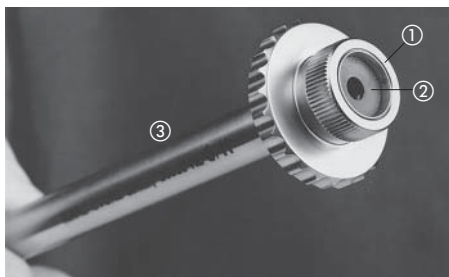
**Dilation set
Disassembly**

Remove dilation pin. Unscrew the knurled ring ① on the dilation cannula ③, take out the sealing disk ②.

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Assembly

Check the sealing disk ② for damage, replace if necessary, then insert into the dilation cannula ③.

Fix the sealing disk ② in position with the knurled ring ①. Insert the dilation pin.

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General information for forceps and scissors

Disassembly and preparation for cleaning

Preparation should be started as soon as possible after using the instruments, since incrustations may cause irreparable damage. Preliminary cleaning must be performed promptly.

To prevent damage during transportation and storage, the instruments must be carefully placed in suitable holders.

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Cleaning

Clean surfaces, inside cavities and jaws thor- oughly with a suitable brush, sponge or cleaning gun. Give particular attention to the jaws when cleaning since these are precision mechanical parts.

All scissors and forceps which cannot be com- pletely dismantled must be cleaned ultrasonically. After cleaning, rinse thoroughly and dry with sterile compressed air.

Instrument channels are especially easy to rinse with water using the cleaning gun and accesso- ries (Art. no. 27660) and dried with sterile compressed air.

Manual cleaning

Note: For heavy soiling, we recommend using an ultrasound bath for a maximum of five minutes.

Remove sticky or caked tissue remnants with a cleaning brush.

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CLICK!linē instruments

CLICK!linē instruments can be dismantled very easily and may therefore be cleaned especially simply and thoroughly. Due to its modular de- sign, the user may choose at any time among various handles. Additionally, in case of damage to heavily used parts such as the jaws, only the defective element must be replaced instead of the complete instrument. This not only saves money, but is also more sensible ecologically.

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**CLICK'line™ instruments with and
without connector pin for
unipolar coagulation**

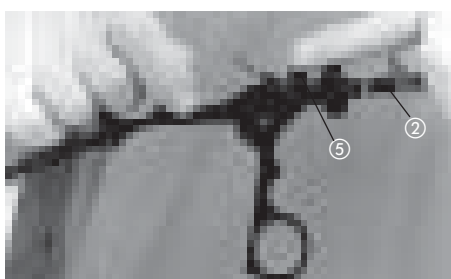
Disassembly

Open handle completely. If necessary, unlock the locking mechanism on the handle.

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Press and hold the locking button ⑤ (do not turn).
The handle is now opened completely in order to push the sheath tube ② out towards the front.

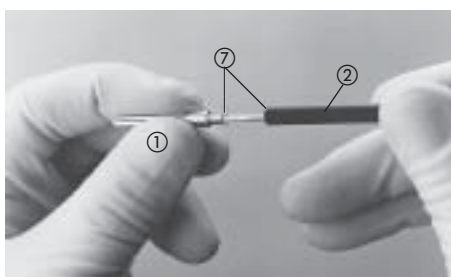
Caution: Do not turn locking button ⑤, as this may cause it to snap off.

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To remove the internals, hold the jaws ① tightly and open the bayonet mount ⑦ by rotating the sheath tube ② ¼ turn. Pull out the internals.

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Caution: Please note that in CLICK'line instruments in the 2 and 3 mm sizes and in curved instruments the internals cannot be removed from the sheath tube.

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Cleaning

Note: For heavy soiling, we recommend using an ultrasound bath for a maximum of five minutes.

Manual cleaning

Remove tissue residue with a cleaning brush. Brush and rinse out sheath tubes on the inside. Blow dry with medically clean compressed air.

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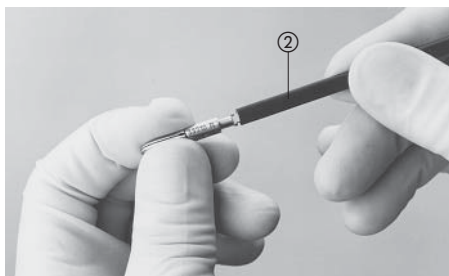
Machine cleaning

Make certain all parts are positioned securely so that damage during cleaning is prevented. The sheath tube must be rinsed out on the inside. By means of the LUER-lock irrigation connection (4), easy attachment to a syringe or cleaning machine is also possible.

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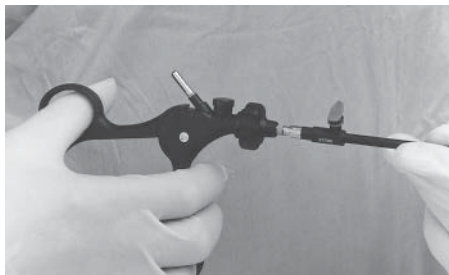
Assembly

Check all parts for damage. Replace if necessary.
Check jaws and joints especially carefully, and also the insulation on instruments for HF use.
Insert the internals from the front into the sheath tube (2).
Lock the internals by rotating ¼ turn in the bayonet mount on the sheath tube (2).

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Insert the sheath with the jaws closed as far as it will go into the completely opened handle.

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Then lock the sheath tube by closing the handle completely.

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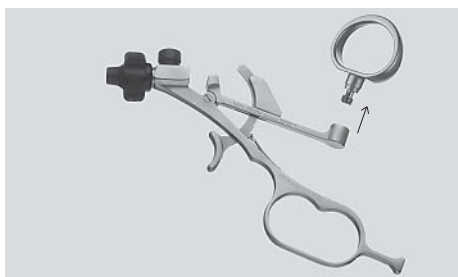
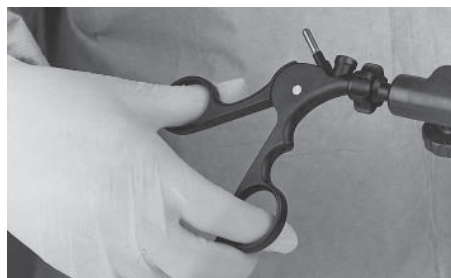
In the case of handle 33140, the thumb grip can be removed. For this purpose, pull the thumb grip toward distal and remove by screwing in clockwise direction.

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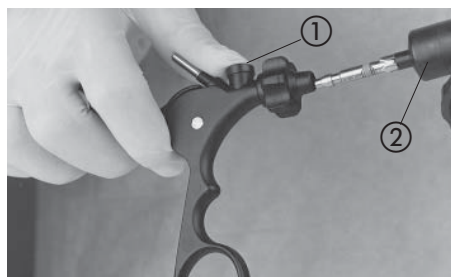
**CLICK'line™ grasping forceps, scissors
and biopsy forceps, angulating,
with connector pin for unipolar
coagulation**
Disassembly

Open handle completely. If necessary, unlock
the locking or spring mechanism on the handle.
Press and hold the locking button ①.

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The handle is now opened completely in order to
push the sheath tube ② out towards the front.

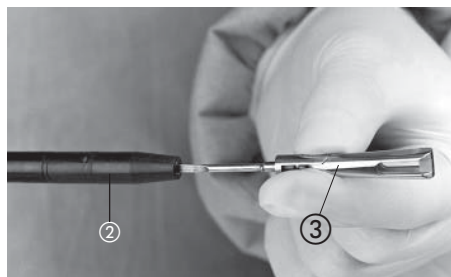
Caution: Do not turn locking button ①,
as this may cause it to snap off.

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Hold the jaws ③ tightly and open the bayonet
mount by rotating the sheath tube ② ¼ turn.
Pull out the internal part.

Warning: Grasp pointed or sharp jaws
with a cloth.

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Angulate the inner sheath ④ by turning the
wheel.
Hold the inner sheath ④ tightly by the distal end
and open the bayonet mount by rotating the
outer sheath ② ¼ turn.
Pull out the inner sheath toward distal.

Caution: The inner sheath ④ must
always be angulated when it is pulled out
of, or inserted into, the sheath tube

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Cleaning

Note: For heavy soiling, we recommend using an ultrasound bath for a maximum of five minutes.

Manual cleaning

Remove tissue residue with a cleaning brush. Brush and rinse out sheath tubes on the inside. Blow dry with sterile compressed air.

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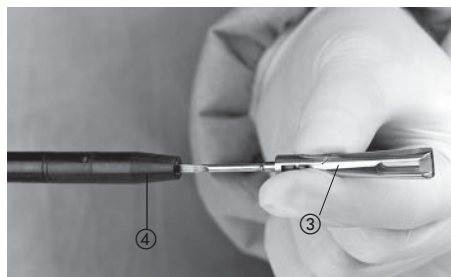
Assembly

Check all parts for damage. Replace if necessary. Check jaws and joints especially carefully. Angulate the inner sheath (4) by hand and insert it into the sheath tube (5) from the front. Lock the inner sheath in the bayonet mount of the sheath tube (5) by rotating 1/4 turn. Straighten the inner sheath (4) by turning the wheel.

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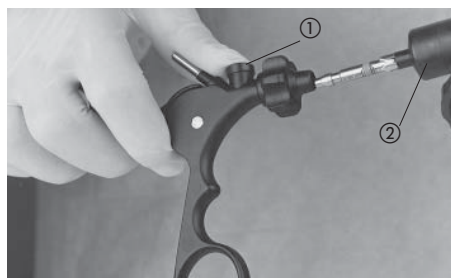
Insert the internal part (3) into the inner sheath (4) from the front. Lock the internal part in the bayonet mount of the inner sheath (4) by rotating ¼ turn.

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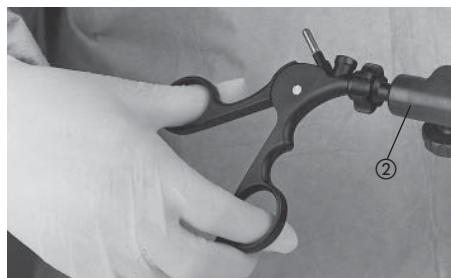
Insert the sheath tube (2) with the jaws closed as far as it will go into the completely opened handle (1).

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Then lock the sheath tube ② by closing the handle completely.

Place a drop of instrument oil (Art. no. 27656 B) in the joints.

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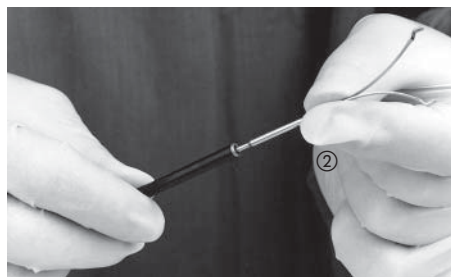
**Take-apart instruments with
U-spring handle
Disassembly**

Press the guide wire ① downwards and then detach from the grip.

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Pull out the insert ② towards the front.

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Cleaning

Clean with a disinfected sponge or brush

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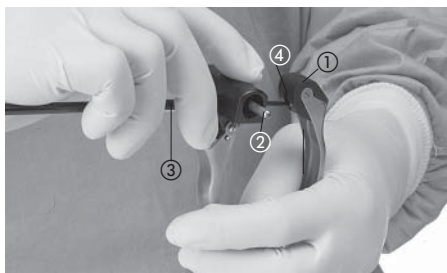
Assembly

Reassemble the instrument in reverse order.

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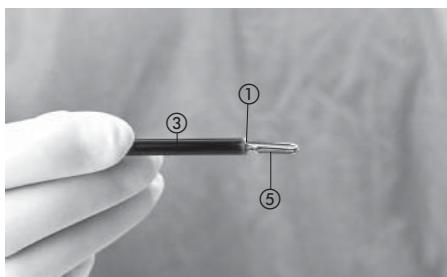
**Insulated instruments with U-spring
handle
Disassembly**

Detach the rear end of the internals ① from the handle. To do this, hold the sheath tube ③ directly in front of the high-frequency cable connector ② and press the rear end of the handle upwards until the rear end of the internals ① can be removed from the short leg of the L-shaped notch ④. Pull the internals ① out towards the front.

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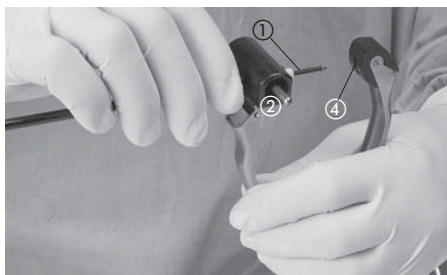
Assembly

Check the sealing cap and internals for damage, replace if necessary. Insert the internals ① from the front into the sheath tube ③ up to the limit until the jaws ⑤ are completely closed.

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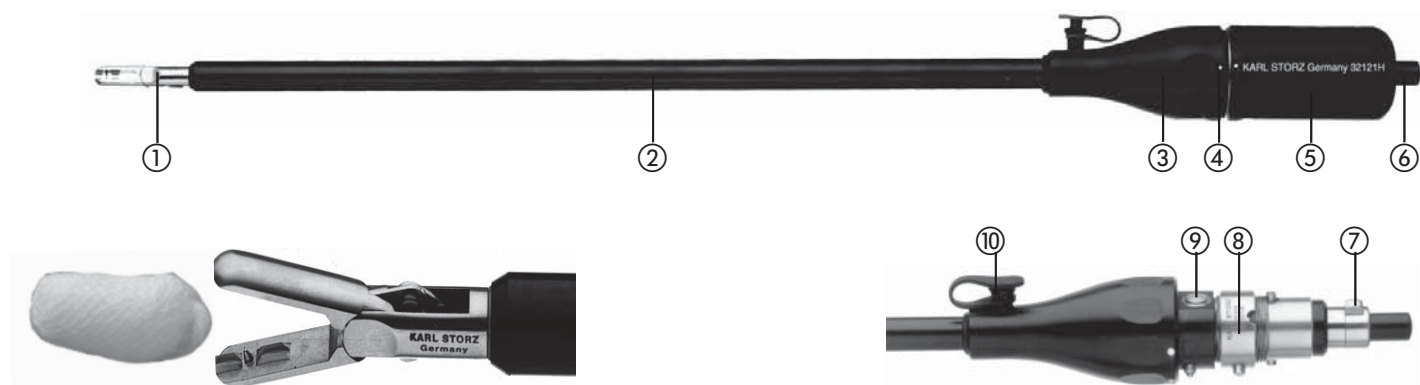
Hold the sheath tube ③ directly in front of the high-frequency cable connector ② and press the rear end of the handle upwards until the rear end of the internals ① can be inserted into the short leg of the L-shaped notch ④ and slid upwards in the long leg.

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- ① Jaws (sponge holder)
- ② Outer sheath, insulated
- ③ Handle, fixed
- ④ ○ Locked
● Replace sponge
▶ Disassembly
- ⑤ Handle, rotatable
- ⑥ Button
- ⑦ Jaw release (rectangular)
- ⑧ Threaded ring
- ⑨ Outer sheath release (round)
- ⑩ Irrigation port

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Recommended spherical sponges

In accordance with DIN 61630/VM24, or spherical sponges of a similar size and weight.

Surgical sponge holder	Spherical sponges
Size 5 mm	4 cm x 4 cm
Size 10 mm	8 cm x 8 cm

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Application

Inserting or replacing spherical sponges
Place handle ⑤ in the '● Replace sponge'
position.
Press the button ⑥: The jaws open.
Insert the spherical sponge.
Release the button ⑥: The jaws close
and hold the spherical sponge securely.
Place the handle ⑤ in the '○ Locked' position.
The sponge holder is ready to use.

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Disassembly

Caution: When disassembling the
instrument, do not hold it by the outer
sheath ② or by the irrigation port ⑩.

Grip the instrument by the recesses on the fixed
handle part ③, then turn the rotatable handle
part ⑤ into the '► Disassembly' position and
pull off.

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First press the unlock jaws button ⑦ then the
unlock outer sheath button ⑨.

Remove the handle ⑤ from the outer sheath ②.

To remove the jaws ①, hold them firmly and
open the bayonet mount by rotating the outer
sheath ② ¼ turn.

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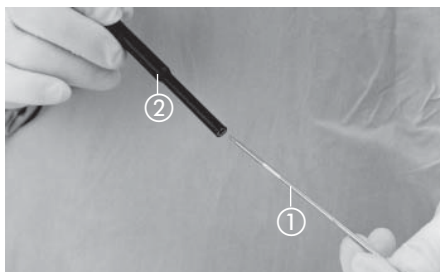
A cleaning machine or syringe can be connected
to the LUER-lock irrigation port ⑩.
Remove tissue residue with a cleaning brush.
Brush and rinse out the outer sheath ② on the
inside. Dry with sterile compressed air.
Lubricate movable parts precisely with
instrument oil (Art. no. 27656 B) at the points of
movement.

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Assembly

Check all parts for damage.
Replace if necessary.

Outer sheath with jaws preassembly

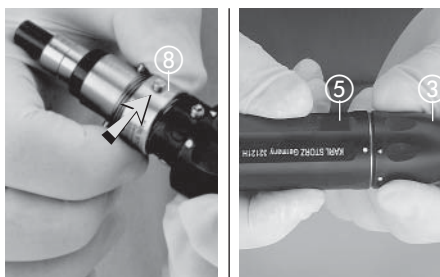
Insert the jaws ① from the distal end into the outer sheath ②.
Lock the jaws ① by rotating ¼ turn in the bayonet mount of the outer sheath ②.

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Handle preassembly

Caution: When assembling the instrument, do not hold it by the outer sheath ② or by the irrigation port ⑩.

If necessary, turn the threaded ring ⑧ as far as it will go (see arrow in picture).

Grip the instrument by the recesses on the fixed handle part ③, then attach the rotatable handle part ⑤ and turn to the “● Replace sponge” position.

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Final assembly

Slide the assembled handle over the premounted outer sheath with jaws until the outer sheath lock clicks into place.

Press the button ⑥ so that the jaws lock clicks into place.

Test the instrument for proper operation.

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**General information on coagulation
instruments**

**Cleaning of coagulation instruments
during surgery**

The coagulation surfaces on coagulation instruments should be wiped off with a moist compress between uses during surgery. Incrustations may be loosened by briefly immersing in a 3% H₂O₂ solution. Before reusing, rinse with sterile water and wipe off again.

Manual cleaning

Remove sticky tissue remnants on the tip carefully with a cleaning brush. Sheath tubes with lumen must be rinsed out well on the inside.

Dry with sterile compressed air, particularly in the area of the plug.

Note: The HF cable and electrode connectors must be clean and dry. Moisture and dirt or residue from disinfectants may lead to malfunctions during operation.

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Assembly and care of the instrument

Incidents during high-frequency surgery which have resulted in damage to the instrument set or even injuries to the user or patient, may be traced back, under closer examination, to incorrect use or defective equipment. A frequent source of errors here proved to be damaged cable and/or electrodes. A further cause may lie in the use of unsuitable accessories from other manufacturers which are not compatible with KARL STORZ units. Safe and proper use of KARL STORZ instruments is only possible using KARL STORZ high-frequency cables and electrodes.

When testing for proper operation, check the insulation material especially for damages such as cracks and chipped material!

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Cable connections

Warning: The use of damaged HF cables may lead to very serious dangers. Check the cable before every use for visible damage. Damaged cable must never be reused.

HF cables are subject to a certain degree of wear, particularly with improper handling and maintenance. For example, damage to the cable terminal or cable insulation may lead to HF voltage flashovers on the working element and therefore to the metallically conductive parts of the instrument. The parts of the instrument at HF potential then become dangerous during patient application to the patient and surgeon.

Careful inspection of the HF cable used is therefore absolutely necessary before every application.

Particular attention must be paid to:

- burn marks on the cable terminal;
- defects on the connectors;
- breaks in the cable or cable insulation;
- missing pin for closing the second hole on the cable terminal.

If damage is found or suspected on HF cable, it must be discarded.

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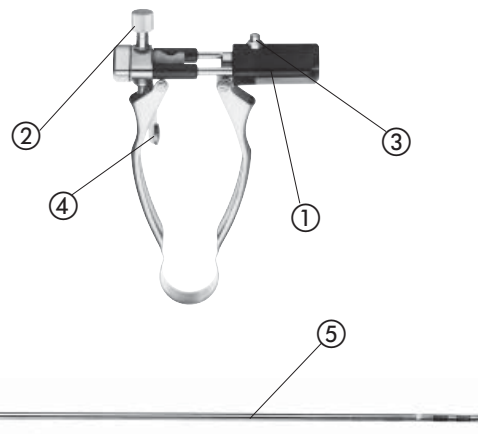
All KARL STORZ high frequency cables may be autoclaved at up to 134°C. After immersing the cable in solution, carefully dry it out in order to avoid any malfunctions due to residual moisture at the mating contacts. After cleaning, steam sterilization should preferably be performed at 134°C by the fractionated prevacuum method.

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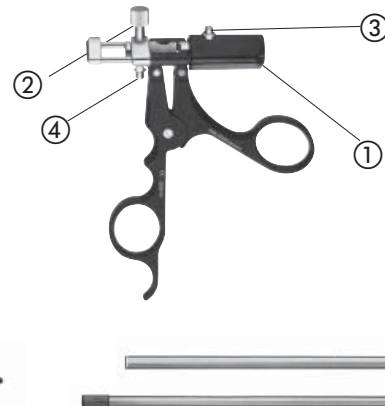
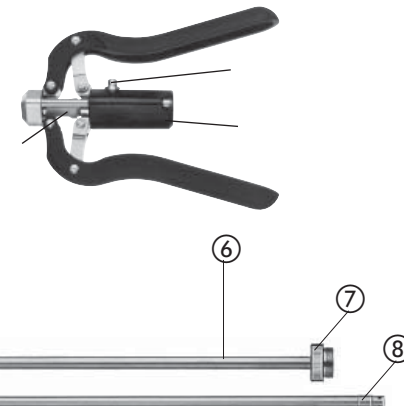
**Take-apart® forceps with double
tube for bipolar coagulation
Models 26276 xx, 26277 xx**

- ① Connecting piece
- ② LUER-lock irrigation connection
- ③ Unlock button outer sheath
- ④ Unlock button inner sheath
- ⑤ Working insert
- ⑥ Outer sheath
- ⑦ Knurled screw
- ⑧ Inner sheath

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Disassembly

Hold the Take-apart® forceps by the jaws of the working insert and by the connecting piece ① behind the handle.

Press the unlock button ③ on the connecting piece ① and simultaneously pull out the working insert towards the front.

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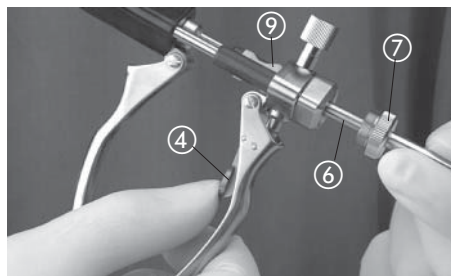
By pressing the unlock button ④, detach the inner sheath and remove toward the front.
Loosen the outer sheath ⑥ with the knurled screw ⑦, unscrew and remove toward the front.

Then remove the sealing cap ⑨ from the handle, check for damage, replace if necessary.

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Manual cleaning

Remove tissue residue with a cleaning brush. Brush the lumen and rinse through before drying with sterile, compressed air, especially in the plug-in connection.

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By means of the LUER-lock irrigation port ②, easy attachment to a syringe is also possible.

Machine cleaning

Make certain all parts are positioned securely so that damage during cleaning is prevented. By means of the LUER-lock irrigation port ②, easy attachment to a cleaning machine is also possible.

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Sterilization

Take-apart® instruments can be steam sterilized at 134°C ± 3°C and, if necessary, for up to 18 minutes.

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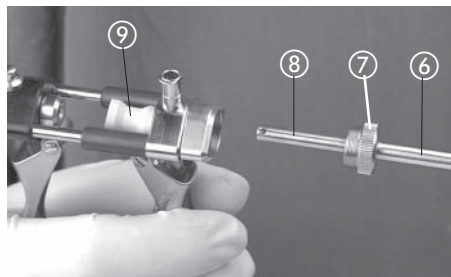
Caution: Attention must be paid to the information on cleaning, sterilization and storage in the 'Cleaning, sterilization and care of KARL STORZ instruments' instructions. The methods for cleaning, disinfection and sterilization are explained therein in detail.

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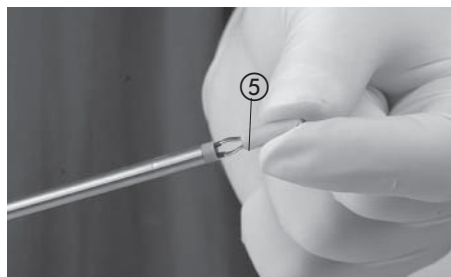
Assembly

Put the sealing cap ⑨ onto the handle. Then attach the outer sheath to the handle and tighten the knurled screw ⑦. Slide the inner sheath ⑧ into the outer sheath ⑥. Hold the inner sheath at the front and slide into the holder in the sliding piece up to the limit. Twist the inner sheath until it clicks into place.

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Insert the working insert ⑤ with the jaw closed through the outer sheath ⑥ up to the connecting piece ① and put in the correct position. (The unlock button ③ clicks into place). The working insert ⑤ may no longer be turned in the sheath.

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Caution: The jaws should not touch when closing.

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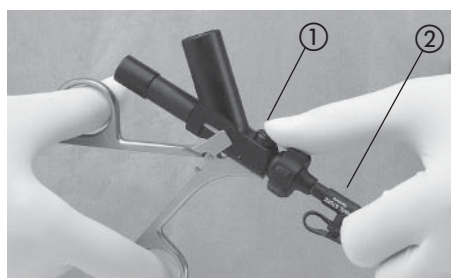
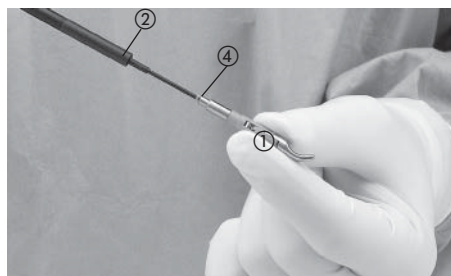
Caution: Do not turn the locking button ①, as this may cause it to snap off.

Press and hold the locking button ①. The handle is now opened completely in order to pull the outer tube ② out towards the front.

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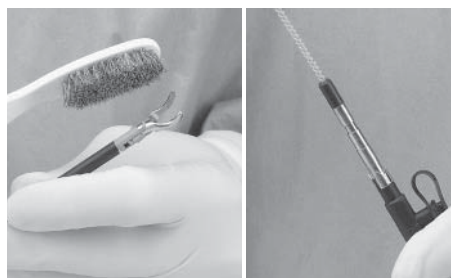
To remove the internals, hold the jaws ① tightly and open the bayonet mount ④ by rotating the sheath tube ② ¼ turn. Pull out the internals.

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Manual cleaning

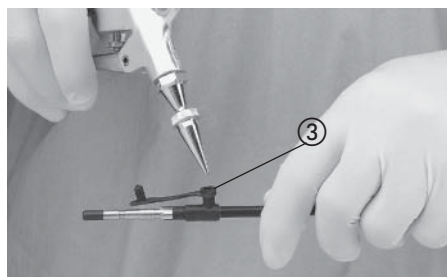
Remove tissue remnants with a 2x disinfected brush. Brush and rinse out sheath tubes on the inside with a cleaning brush (Art. no. 27650 G). Blow dry with sterile compressed air.

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By means of the Luer-lock irrigation port ③, easy attachment to a syringe is also possible.

Machine cleaning

Make certain all parts are positioned securely so that damage during cleaning is prevented. By means of the Luer-lock irrigation port ③, easy attachment to a cleaning machine is also possible.

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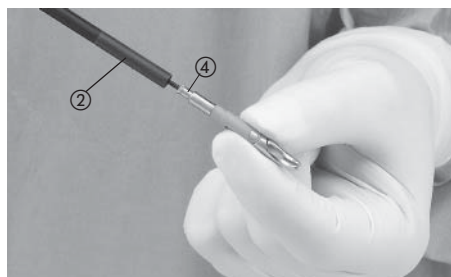
Sterilization

RoBi® instruments can be sterilized using steam at up to 134°C $\pm$ 3° and, if necessary, for up to 18 minutes.

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Assembly

Check all parts for damage. Replace if necessary.

Check jaws and joints and the insulation on instruments for HF use especially carefully.

Insert the internals from the front into the sheath tube ②.

Lock the internals by rotating ¼ turn in the bayonet mount ④ on the sheath tube ②.

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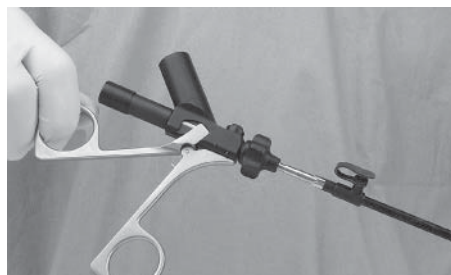
Insert the outer tube, with the jaws closed, as far as it will go into the completely opened handle. Then lock the outer tube by closing the handle completely.

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Scissors, punches and biopsy forceps

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KOH micro guillotine

Disconnect the union nut and outer sheath from distal.

Warning: Instrument has a sharp cutting edge.

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Cleaning:

Remove any remaining and dried on tissue residue using a brush. Clean the small parts carefully. Clean the dismantled instrument in an ultrasound bath.

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Loosen the special LUER-lock cap (Art. no. 6225895) from the outer sheath.

Fit the cap in the opposite direction at the proximal end of the outer sheath. Rinse the outer sheath through with the cap attached.

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Note: Adequate cleaning of the inner sheath cannot be ensured if the outer sheath is rinsed without the cap attached.

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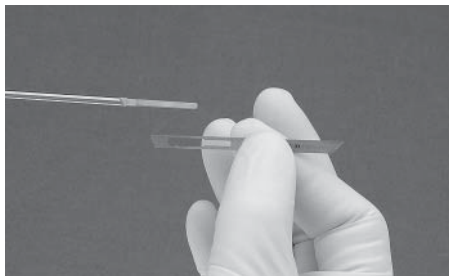
Assembly:

If necessary replace the knife insert.

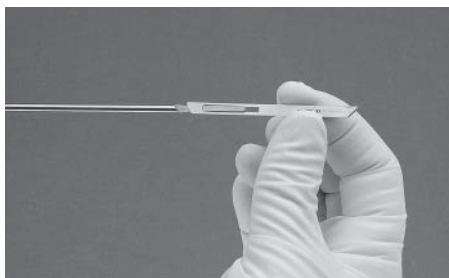
Warning: Risk of injury! The scalpel
blade has a sharp cutting edge. Grasp
with a cloth or something similar when
removing.

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Insert the handle piece with knife insert into the
outer sheath. When doing so, ensure that the
knife insert is in the correct position. The outer
sheath can only be locked in one position.

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Turn the union nut counterclockwise and tighten.

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Instruments for unipolar and bipolar coagulation

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General information on coagulation instruments

Cleaning of coagulation instruments during surgery

The coagulation surfaces on coagulation instruments should be wiped off with a moist surgical pad between uses during surgery. Incrustations may be loosened by briefly immersing in a 3% H₂O₂ solution. Before reusing, rinse with sterile water and wipe off again.

Manual cleaning

Remove tissue remnants adhering to the tip carefully with a disinfected cleaning brush. Sheath tubes with lumen must be rinsed out well on the inside.

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When testing for proper operation, check the insulation material especially for damages such as cracks and chipped material.

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Cleaning high frequency cables

Clean the high frequency cable with a brush or a sponge.

All KARL STORZ high frequency cables may be autoclaved up to 134°C.

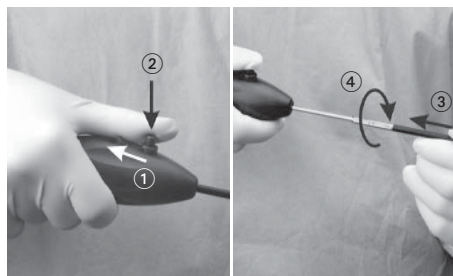
After immersing the cable in solution, carefully dry it out in order to avoid any malfunctions due to residual moisture at the mating contacts. After cleaning, steam sterilization should preferably be performed at 134°C by the fractionated prevacuum method.

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**Dissecting electrodes with
exchangeable tip**

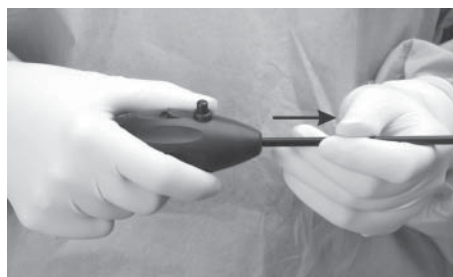
Assembly

Pull the slide backward ① and press the button ②. Then push the outer sheath, metal end first, onto the inner sheath as far as it will go ③. At the same time slightly turn the outer sheath ④ to find the lock-in position.

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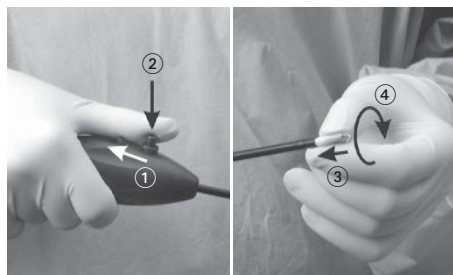
In the lock-in position, release the button and the slide. Pull out the outer sheath far enough so that it locks into place. The button springs back into its original position.

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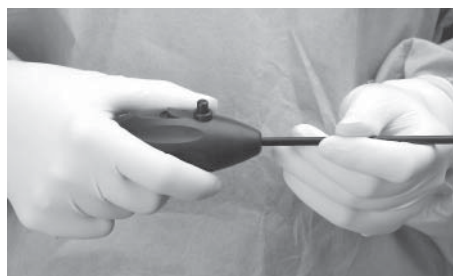
Pull the slide backward ① and press the button ②. Then insert the electrode into the outer sheath ③ and turn clockwise ④, until it reaches its lock-in position.

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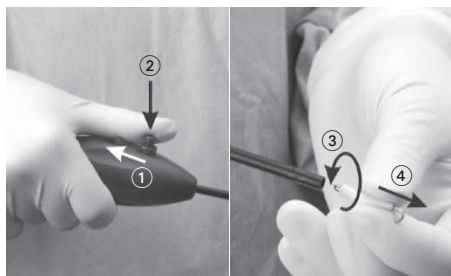
In the lock-in position, release the button and the slide. The button springs back into its original position.

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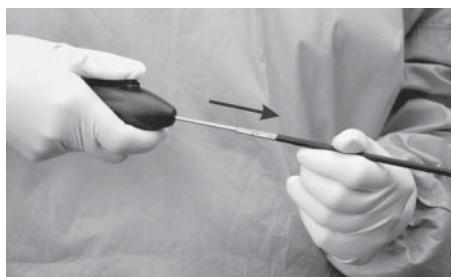
Disassembly

Pull the slide backward ① and press the button ②. Then turn the electrode counterclockwise ③ and pull it out of the outer sheath ④.

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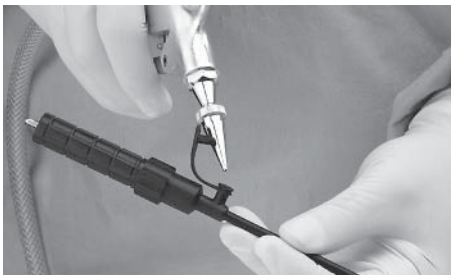
Then pull the outer sheath away from the inner sheath while keeping the button pressed down.

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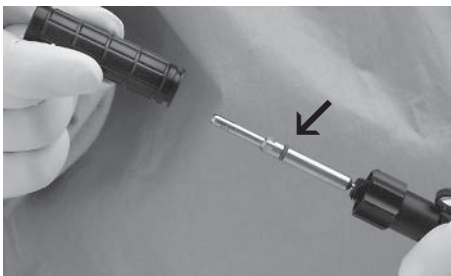
Angulating monopolar hook

Clean the exterior with a sponge or brush. Rinse out the lumen via the irrigation port.

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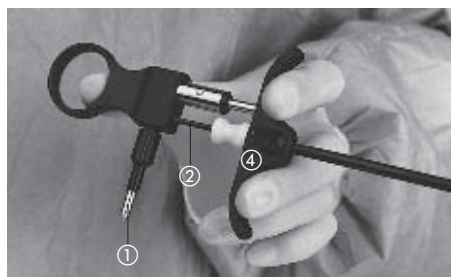
The handle can be removed for cleaning. Check the O-ring and, if necessary, fit a new one.

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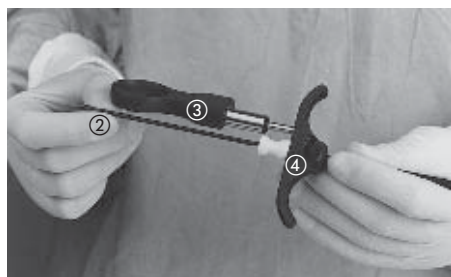
MANHES high-frequency needle/ KOH ultramicro high frequency electrode Disassembly

Unscrew the connector for the high-frequency
cable ①. Extend the inner part (needle) ② by
pulling the handle ④,

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Release the grip elements ③ + ④ and swing the
grip ring ③ to the side.
Pull the inner part ② out towards the back.
Take off the sealing cap.
Check the sealing cap and inner part (needle) ②
for damage, replace if necessary.

Assembly and functional test

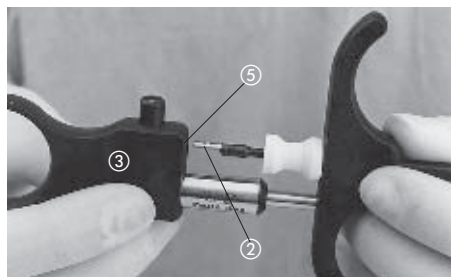
Put on the sealing cap. Insert the inner part
(needle) ② carefully from the back, until the tip
extends out of the sheath tube.

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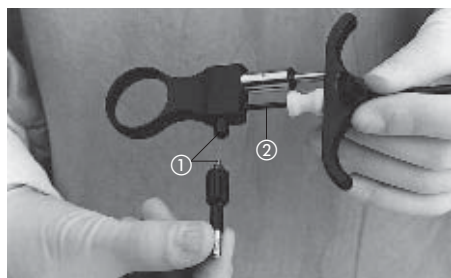
Note: Hold the instrument vertically while insert-
ing the inner part and insert the inner part from
above.

Turn the back end of the inner part (needle) ②
until it can be inserted in the socket ⑤ on the
ring grip ③ provided for this.

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Fix the inner part (needle) ② into position by
screwing in the connection for the high-
frequency cable ①.

Check the mechanical function. The inner part ②
must extend from the front when the handle is
activated and, when released, automatically
retract into the sheath tube.

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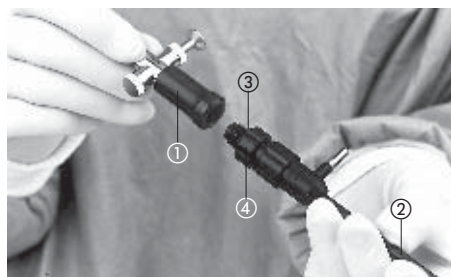
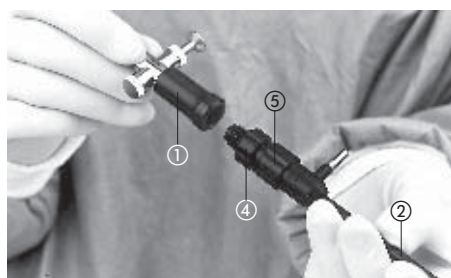
Coagulation and dissecting electrodes Disassembly

Separate the electrode ② from the handle ① by loosening the knurled nuts ④.
Check the sealing ring ③ for damage, replace if necessary.

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Assembly

Screw the electrode ② into the handle ①.
Fix in place with the knurled nut ④.
The knurled nut ⑤ is used to fix the rotatable sheath in position.

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Handles for suction and irrigation
See Section *Modular handles and instruments for suction and irrigation*

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Bipolar needle

Disassembly

Press the unlock button and pull out the needle holder toward distal.

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Remove the rubber seal. Check for damage and, if necessary, fit a new seal.

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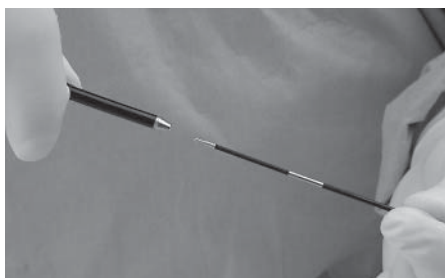
Cleaning

Clean the exterior with a disinfected sponge or brush. Rinse lumen and finally dry with compressed air.

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Assembly

Fit the rubber seal.
Insert the needle holder into the sheath from distal.
Press the unlock button and insert the needle holder into the mount until you hear the locking mechanism engage.

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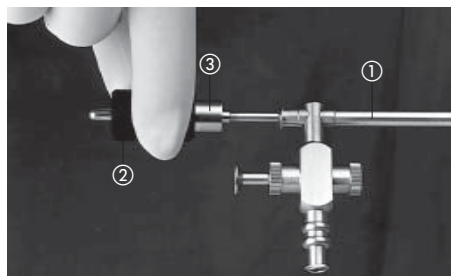
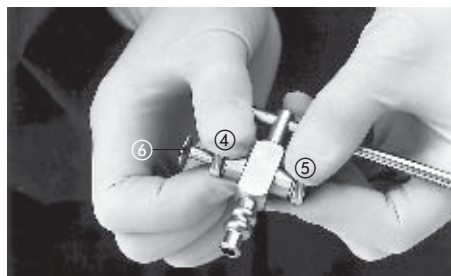
Take-apart® bipolar coagulation suction tube Disassembly

Pull the coagulation electrode ② out of the outer sheath ①. To do this, open the lock ③ and turn the connector counterclockwise.

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Loosen both knurled screws ④, ⑤.
Remove the piston ⑥.
Take the spring out of the piston.

Note: The spring should remain on the knurled screw, since this makes assembly easier.

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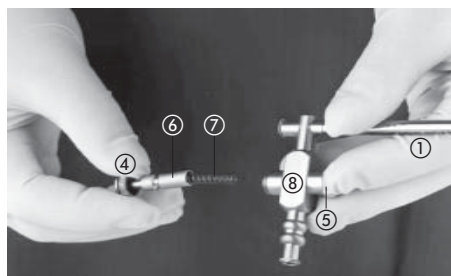
Cleaning

After manual or if necessary machine cleaning and disinfection, thoroughly dry with sterile compressed air, particularly in the area of the plug.

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Assembly and functional test

Lightly grease the piston ⑥. Insert the spring ⑦ into the piston ⑥. Screw the knurled screw ⑤ tight. Insert the piston with spring into the valve housing ⑧. Tighten the knurled screw ④ with the piston slightly depressed.
Test the operation of the valve. Disassemble again for sterilization.

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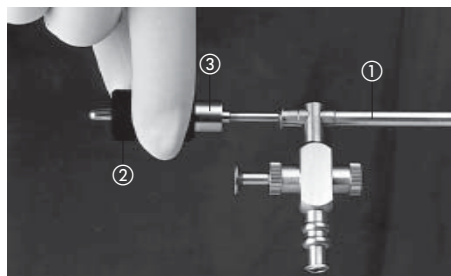
Insert the coagulation electrode ② into the outer sheath ①. Close the lock ③ by turning the connector clockwise.

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GORDTS-CAMPO coagulation suction tube

The coagulation suction tubes are suitable for use with the following KARL STORZ handles: Models 30803, 30804, 30810, 37112, 37113

Connection to a suction pump

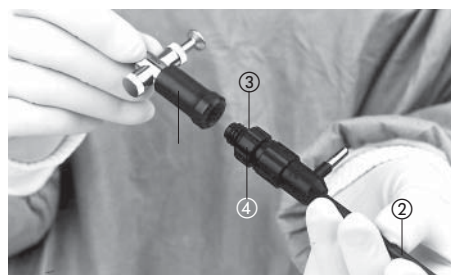
Screw the threaded part ① into the KARL STORZ handle. Then connect the handle to the suction pump (cf. suction pump instruction manual).

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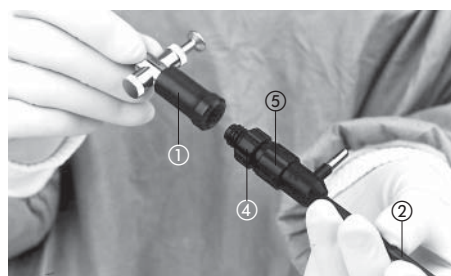
Disassembly

Separate the electrode ② from the handle ① by undoing the knurled nuts ④.
Check the sealing ring ③ for damage, replace if necessary.

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Assembly

Screw the electrode ② into the handle ①. Fix in place with the knurled nut ④.
The knurled nut ⑤ is used to fix the rotatable sheath in position.

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Adaptor for coagulation and dissecting electrodes

An adaptor is required to use 5 mm sized coagulation or dissecting electrodes on handles intended for 10 mm suction/irrigation tubes.

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Cleaning

Remove tissue residue adhering to the electrode with a disinfected cleaning brush.

Sheath tubes with a lumen must be well rinsed through on the inside.

Then dry with compressed air, particularly the contact surfaces.

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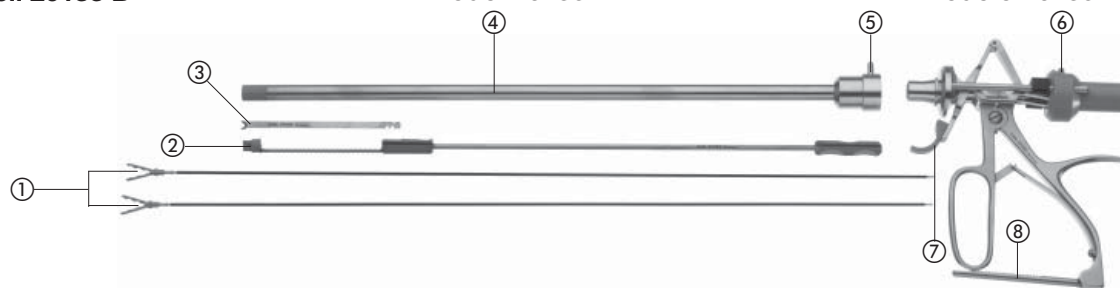
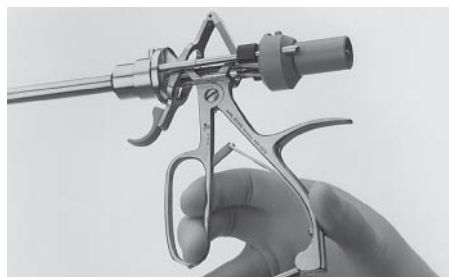
- ① Forceps inserts
- ② Knife insert
- ③ Knife
- ④ Outer tube
- ⑤ Sheath lock
- ⑥ Unlock button for forceps insert
- ⑦ Pusher
- ⑧ Stop, fold-down

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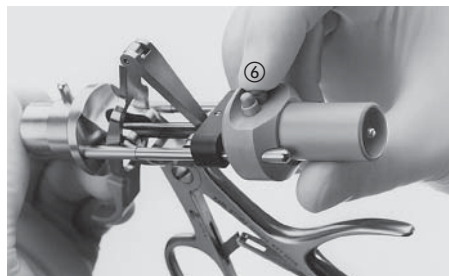
Dismantling the instrument
Open handles by releasing the stop.

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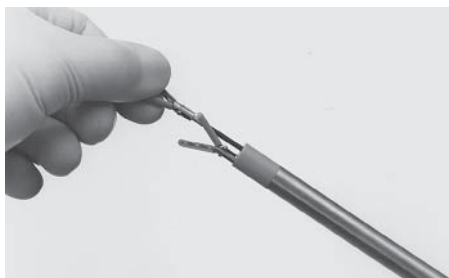
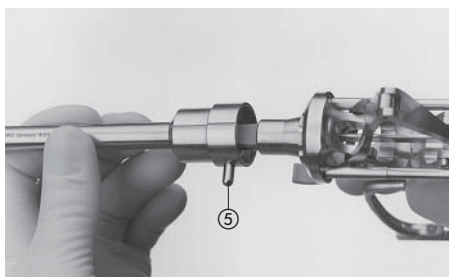
... rotate the jaws 90° clockwise (as seen from the back). The forceps insert can now be pulled out toward the front. Then remove the second forceps insert in the same manner.

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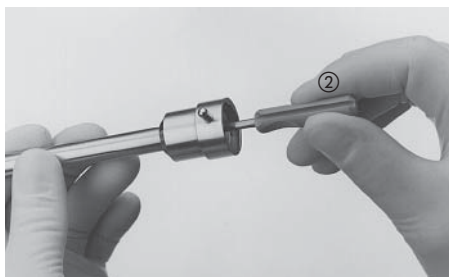
Undo sheath lock ⑤ and remove the handle.

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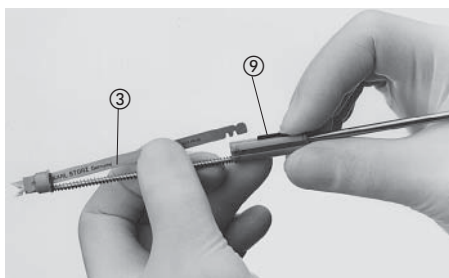
The knife insert ② can now be pulled out toward the back. Make certain when pulling out that the knife insert is not bent or turned too sharply, otherwise the knife will be jammed.

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Pull back the locking pusher ⑨. Lift the knife ③ out of the holder at the back and slide out toward the front.

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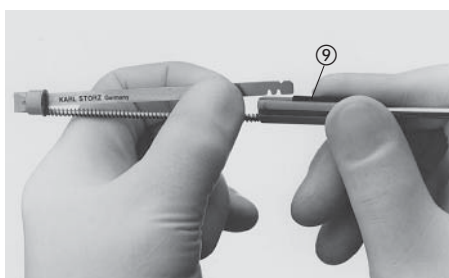
Cleaning and sterilization

Clean the jaws and knife of adhering tissue remnants with a disinfected brush. Brush out instrument channels, then rinse and dry with sterile compressed air (especially around the plug). All instrument parts can be steam sterilized up to 134 °C, gas sterilized or disinfected in solution. Open the handle for sterilization.

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Assembly

Insert the knife from the front into the front holder. Make certain that the individual notches on the knife point upward.

Place the knife in the rear holder in such a way that the two lower recesses on the knife are over the pins in the rear holder.

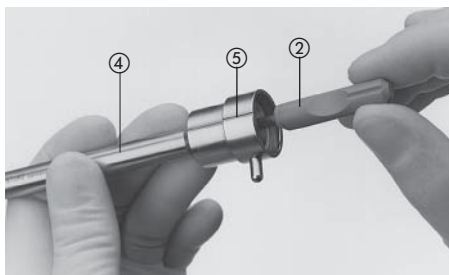
First slide the locking pusher ⑨ towards the front until it clicks into the upper recess of the knife.

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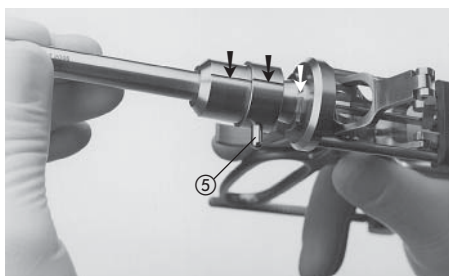
Insert the knife insert ②, with the recess on top, into the outer tube ④ from behind. Make certain that insertion is parallel to the axis so that the knife insert does not jam.

The correct position is attained if the marking on the sheath lock ⑤, on the outer tube ④ and on the knife insert ② are all in alignment.

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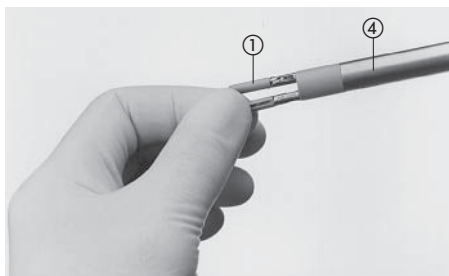
Attach the outer tube to the grip (marking and recess aligned) and fix in place with the sheath lock ⑤.

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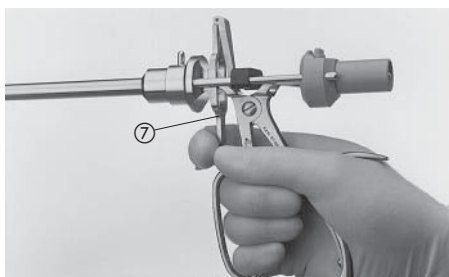
Insert the forceps insert ① with the handle closed into the outer tube ④ and lock by turning 90° counterclockwise (as seen from the back).

If the stop on the hand grip is released, the grip opens and the forceps inserts automatically click into place.

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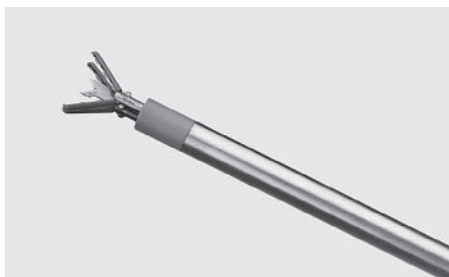
Handles for suction and irrigation
See Handles and instruments for suction and
irrigation

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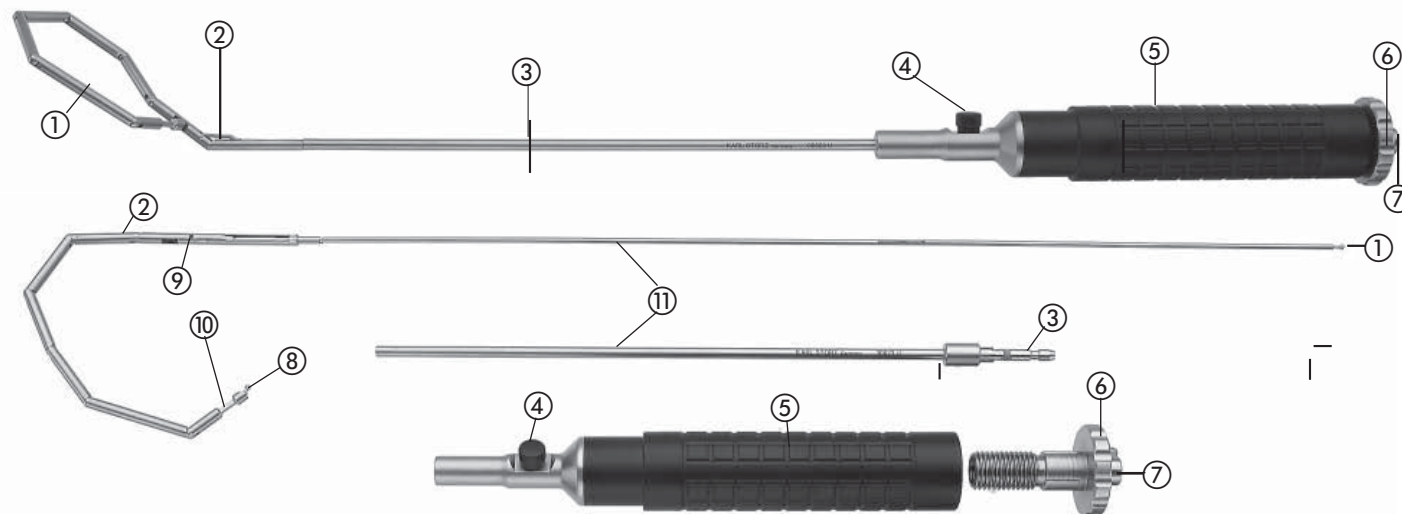
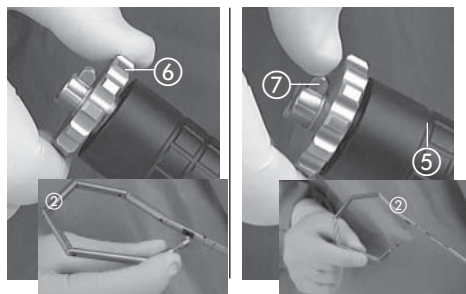
- ① Inner part (with flexible end)
- ② Hinge
- ③ Outer sheath
- ④ Insert release
- ⑤ Handle
- ⑥ Rotating wheel
- ⑦ Inner part release
- ⑧ Fixing pin
- ⑨ Fixing hole
- ⑩ Grasping bridge
- ⑪ Inset (components ①+③)

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Dismantling for cleaning

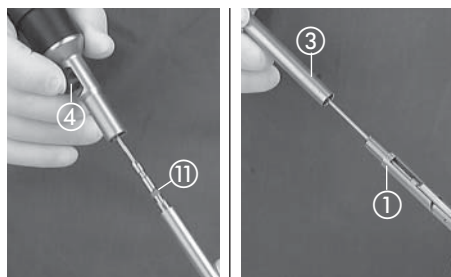
Slightly angle the hinge ② using the rotating wheel ⑥.

Press the release bolt (inner part) ⑦ and stretch out the hinge ② (do not angle any more, otherwise the bolt will reengage).

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Then press the insert release bolt ④ and pull the insert ⑪ distally.

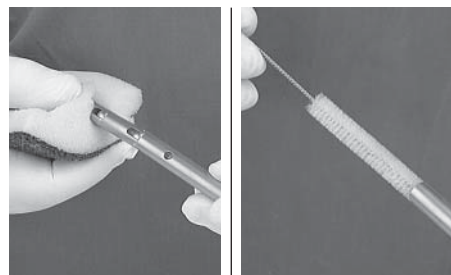
Turn the outer sheath ③ a ¼ turn clockwise and pull it off the internal part (with flexible end) ①.
Open the loop if the occasion should arise.

Twist the rotating wheel ⑥ off the handle ⑤.

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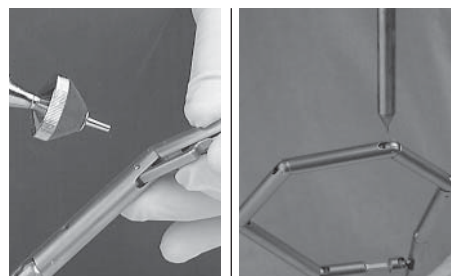
Remove any signs of soiling with a disinfected sponge or brush.

Clean the outer sheath with a disinfected brush.
A higher level of cleaning efficiency can be attained using ultrasound procedures.

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Following this, rinse out thoroughly with completely demineralized water and dry with sterile compressed air.

Oil all movable parts at their contact points with instrument oil (Art. no. 27656 B) in a well-directed manner.

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Retractors, dissectors, palpation probes and extractors

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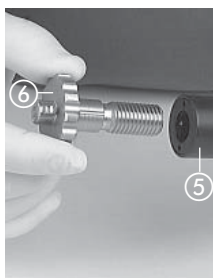
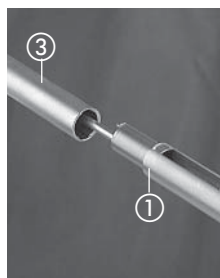
This instrument can be steam sterilized at temperatures of up to 134°C.

The fully assembled instrument can be sterilized using the fractionated vacuum procedure. With other procedures, the instrument must first be dismantled.

The instructions for use provided by the sterilization device manufacturer must be followed exactly.

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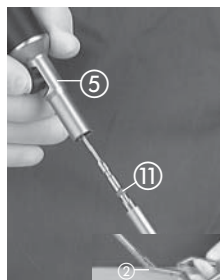
Assembly

Check all the parts for damage. If necessary, replace.

Preassembly of insert

Push the outer sheath (3) over the internal part (with flexible end) (1) from proximal and lock the bayonet mount with a ¼ turn.

Preassembly of hand grip

Fit the wheel (6) into the hand grip (5) by turning it until it stops, and then open by approx. 1 turn.

Final assembly

Insert the inset (11) into the hand grip (5) until the release bolt engages.

Then, angle the hinge (2) cautiously until the (inner part) release bolt engages.

Turn the rotating wheel (6) until it stops.

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Take-apart fan retractor, 5 mm and 10 mm Disassembly

Pull the inner part (1) on the grip out of the sheath tube (2).

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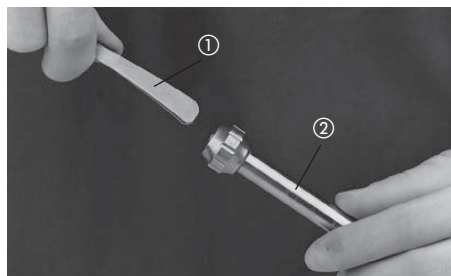
Cleaning

Remove sticky tissue remnants with a disinfected cleaning brush. Clean the small parts carefully. Rinse the sheath tubes with a cleaning gun. Clean the instrument in an ultrasound bath.

Dry with sterile compressed air.

Assembly

Press the fan together on the front end of the inner part ① and insert into the sheath ② from the back.

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Disassembly

Open the fan.

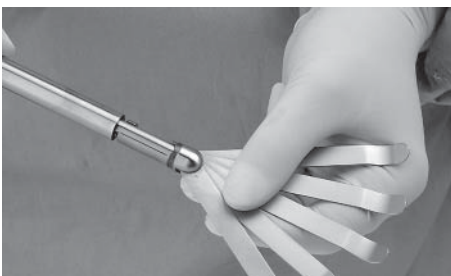
To do this, turn the handwheel at the proximal end until the fan has opened completely.

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Unscrew the sheath tube and remove it from the internal part with the fan by turning counterclockwise.

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Unscrew the handle and remove it from the sheath tube by turning counterclockwise.

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Cleaning

Remove sticky tissue remnants with a disinfected cleaning brush. Clean small parts carefully. Rinse the sheath tubes with a cleaning gun. Clean the instrument in an ultrasound bath.

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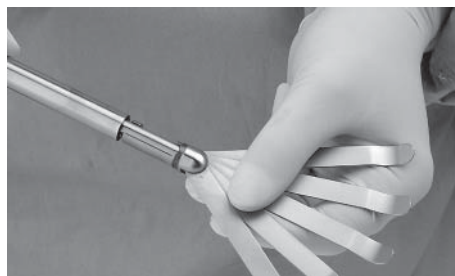
Assembly

Carefully lubricate the moving parts with oil.
Screw the handle onto the outer sheath.
Insert the internal part into the sheath tube and screw on by turning clockwise.
Turn the handwheel to close the fan.

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Retractors for gastric banding Models 30623 G/GB

Clean the exterior with a disinfected sponge or brush.

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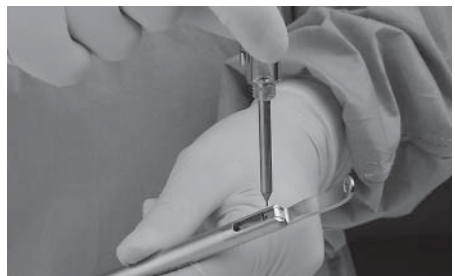
Rinse out LUER-lock irrigation port, then clean with ultrasound and dry with sterile pure compressed air.

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Oil the moving parts with instrument oil (Art. no. 27656 B).

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Anal dilators

Manual cleaning: Clean any heavy soiling on the surface of the tube with a disinfected sponge or brush.

Thoroughly brush the lumen, then rinse with microbiologically clean/sterile water and dry with compressed air.

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Clean the obturator with a disinfected sponge or brush, then rinse and dry. Check the plastic tip for damage. If damaged, the obturator must no longer be used.

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Remove both adaptors from the light inlet piece in order to have unobstructed access to the fibers.

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Caution: The anal dilator must not be cleaned in an ultrasound bath. This could damage the fiberoptic light cable.

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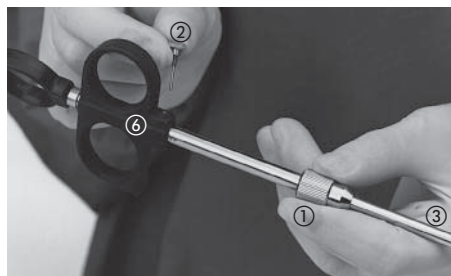
The fibers and optical ends can be dried with alcohol after manual or machine cleaning.

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Take-apart CUSCHIERI variable curvature dissector Disassembly

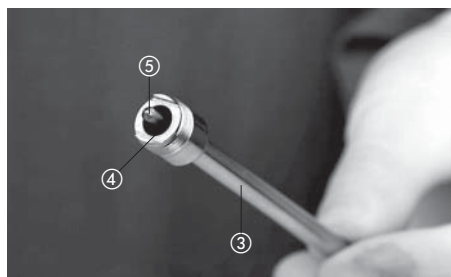
Unscrew knurled ring ① on the sheath ③.
Unscrew and remove the knurled screw ②
on the double ring handle ⑥.

Remove hand grip element toward the back.

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Pull out the inner part ⑤ toward the front.
Remove the sealing ring ④ (inside) from the
sheath ③.

Cleaning

Remove any remaining tissue with a disinfected
cleaning brush. Clean small parts carefully and
rinse out the sheath tube. Clean the instrument
in an ultrasound bath.

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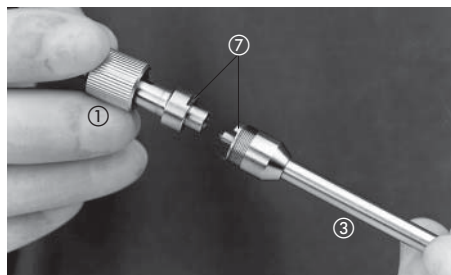
Assembly

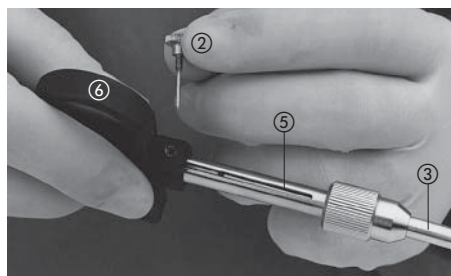
Lightly grease the inside part ⑤.
Insert the sealing ring
Insert the inner part into the sheath tube ③ from
the front and align in such a way that it may be
entirely inserted into the sheath tube ③.

Connect the hand grip element and sheath ③ by
aligning the positive lock ⑦ on the sheath tube
③ with the handle element and screwing on with
the knurled ring ①.

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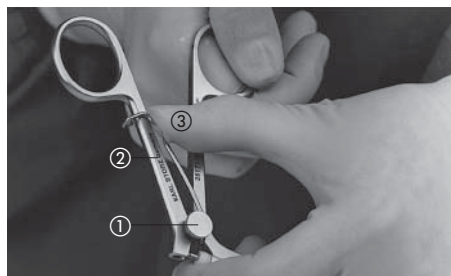
Slide back the inner part ⑤ in the sheath ③ until
the bore in the inner part ⑤ is visible in the
groove of the handle element.
Insert the knurled screw ② in the threaded
bore of the double ring handle ⑥ and align with
the bore in the inner part ⑤. Screw in knurled
screw ② completely.
Test for proper operation.

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Needle holder and suture instrument with ring handle

Disassembly

Check the sealing cap for damage, replace if necessary.

To do this, detach the spring (3) on the movable part of the handle (2).

Unscrew the knurled screw (1) on the handle.

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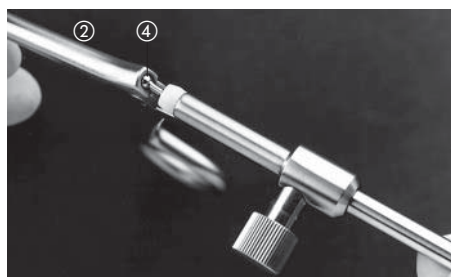
Swing back the movable part on the handle (2) and remove.

Cleaning

Use the LUER-lock connection for cleaning. Additionally clean with ultrasound.

Assembly

Attach sealing cap.

Hook the movable part of the handle (2) in the ball on the end of the internal part (4) and swing forward.

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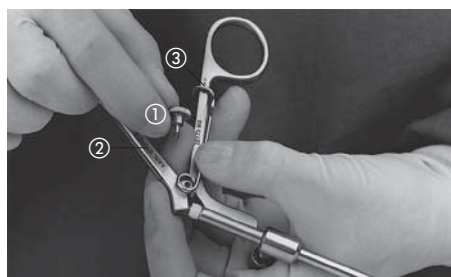
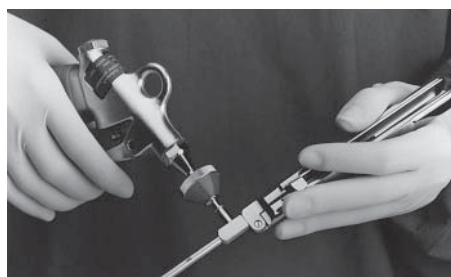
Fasten the handle with the knurled screw (1).

Attach spring (3) to the movable part of the handle (2).

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Needle holder with straight and curved handle, with and without ratchet

Note: Straight handles cannot be dismantled.

Cleaning:

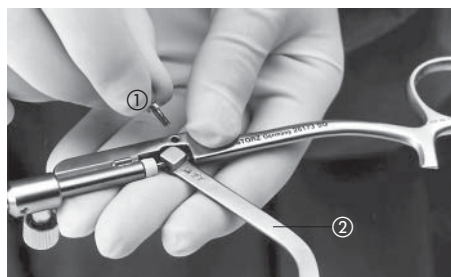
Use the irrigation port for cleaning. Additionally clean with ultrasound.

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SCARFI needle holder

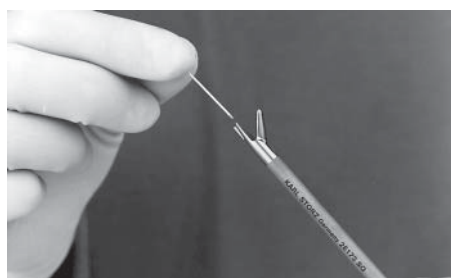
Disassembly

Check sealing cap for damage, replace if necessary.
To do this, unscrew the wing bolt ① on the handle. Swing down the movable part of the handle ② and remove.

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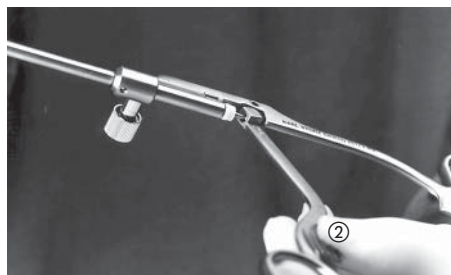
Cleaning

Run a pipecleaner for cannulas or the like through the eyelets on the jaw and handle of 26173 SQ.
Additionally clean with ultrasound.

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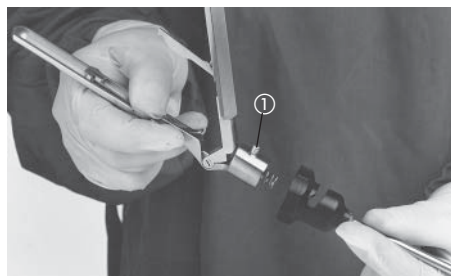
Assembly

Oil joints precisely.
Check jaws and joints in particular. Check the lock for a secure close.
If the sealing cap was replaced, insert the movable part of the handle ② and fix in place with the wing bolt.
Then test for proper operation.

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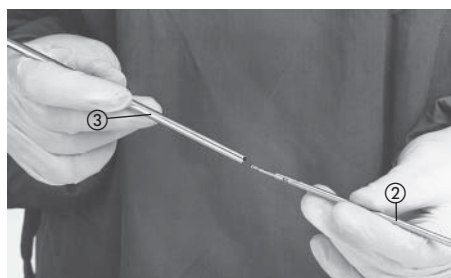
DUBECQ/PRINCETEAU angulating needle holder
Disassembly

Open the handle as far as it will go. Press and hold the locking button ①. Open the handle completely and remove.

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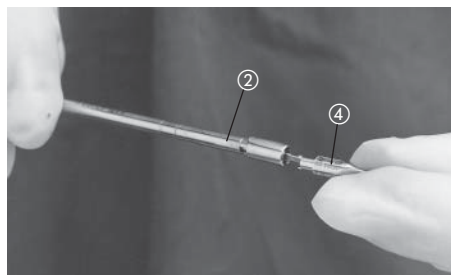
Pull the inner sheath ② out of the outer sheath ③.

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Hold onto the jaws ④ tightly and open the bayonet mount by rotating the inner sheath ② ¼ turn. Pull out the internal part.

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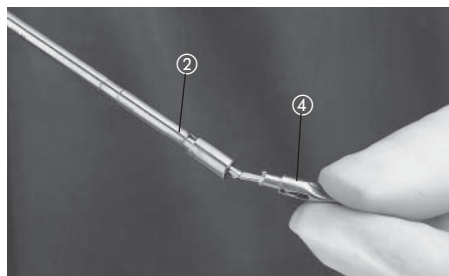
Cleaning

Remove tissue remnants with a cleaning brush. Brush and rinse out the inside of sheath tubes. Ultrasound cleaning increases cleaning efficiency. Blow dry with sterile compressed air.

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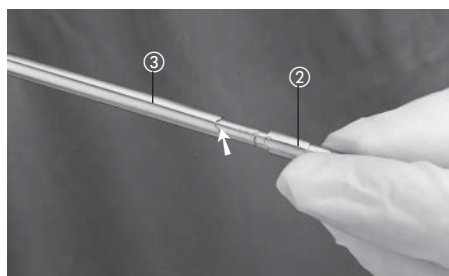
Assembly

Insert the internal part ④ into the inner sheath ② from the front. Lock internal part ④ by giving it a ¼ turn in the bayonet mount of inner sheath ②.

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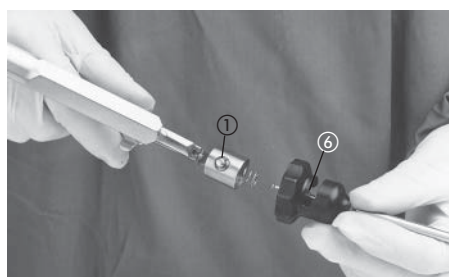
Insert the inner sheath ② into the outer sheath ③. Ensure that the recess on the inner sheath sits properly in the notch on the outer sheath ③.

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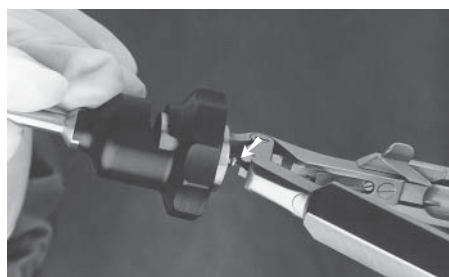
Hold the outer sheath with the internal part distally and the mount on the completely opened handle.

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Note: Ensure that the groove ⑥ on the set wheel and the locking button ① are aligned with one another.

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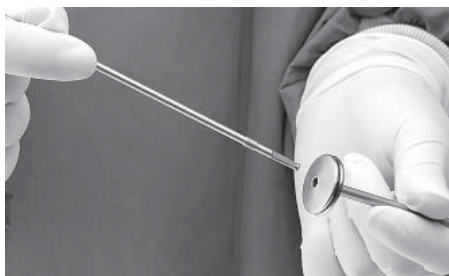
The ball at the rear end of the internal part must lie in its holder on the handle. Lock the outer sheath by completely closing the handle.

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CUSCHIERI integrated knot pusher and thread catcher

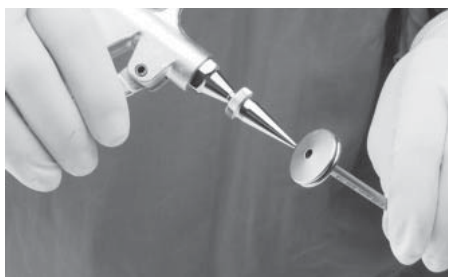
Disassembly

Pull the internal part out of the sheath tube toward the back.

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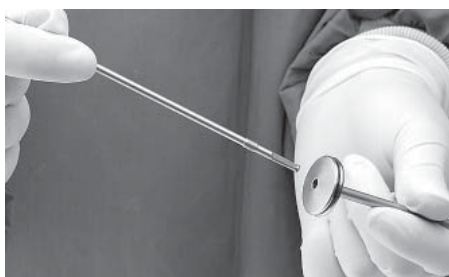
Cleaning

Remove sticky or caked tissue remnants with a disinfected cleaning brush. Rinse out the sheath tubes. Clean the instrument in an ultrasound bath. Dry with sterile compressed air.

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Assembly

Insert the internal part into the sheath tube from proximal and press forward until you hear it click into place.

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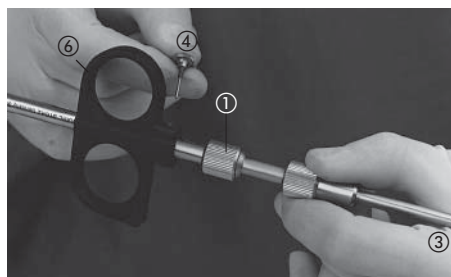
Take-apart CUSCHIERI variable curvature suture and sling passer
Disassembly

Extend the inner part by pushing the double ring handle (6) towards the front.
By turning the front knurled ring (2), loosen the outer sheath tube (3) until it can be moved freely.

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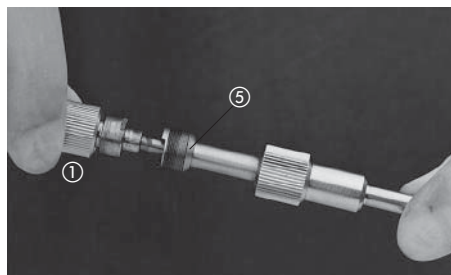
Loosen the back knurled ring (1) between the outer sheath tube (3) and double ring handle (6).

Unscrew and remove the knurled screw (4) on the double ring handle (6).

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Screw the back knurled ring (1) all the way down.

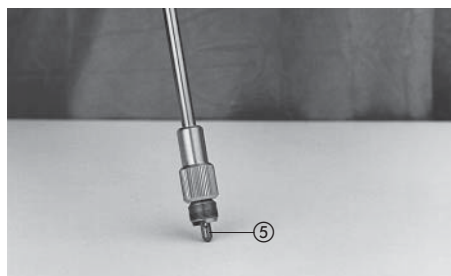
Remove the handle element towards the back.

Pull the inner part (5) out towards the front.

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Note: Press the back end of the inner part (5) onto a hard surface until it pops out towards the front.

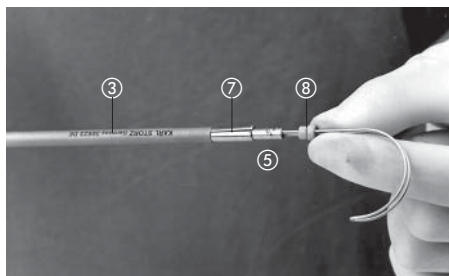
Cleaning

Remove sticky tissue remnants with a cleaning brush. Clean small parts carefully. Clean the instrument in an ultrasonic bath. Rinse out sheath tubes. Dry with sterile compressed air.

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Disassembly

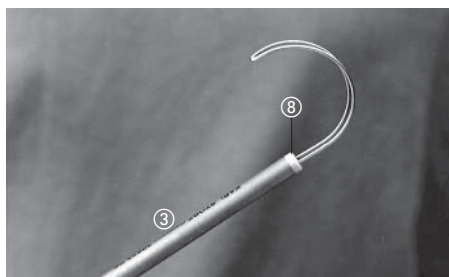
Lightly grease the sheath of the inner part (5).
Insert the inner part (5) from the front into the inner tube (7).
Insert the plates (8) running along the wires of the inner part (5) into the claw of the inner tube (7).

Note: At the same time, the outer sheath tube (3) must be slid out towards the back up to the limit stop.

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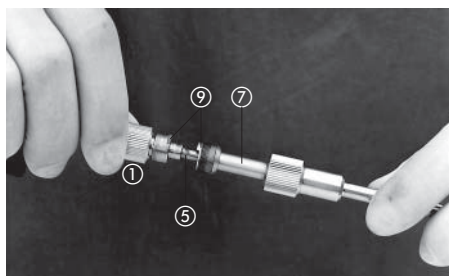
By turning the front knurled ring counterclockwise, fix the outer sheath tube (3) in place, until the gap on the front end between the plates (8) and sheath (3) is completely closed.

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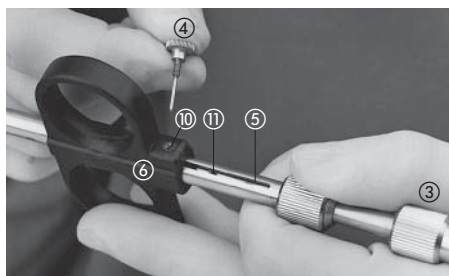
Slide the handle element onto the inner part (5).
Align the positive lock (9) from the inner tube (7) with the handle element and screw on with the back knurled ring (1).

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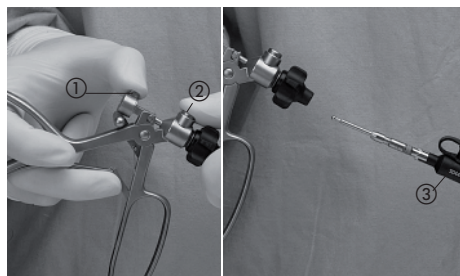
Slide back the inner part (5) in the sheath (3), until the bore (11) is visible in the inner part (5) in the groove of the handle element.
Insert the knurled screw (4) in the threaded bore (10) of the double ring handle (6) and align with the bore (10) in the inner part (5). Screw in knurled screw (4) completely.
Test for proper operation.

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Clip applicator model 30444 LR/GA

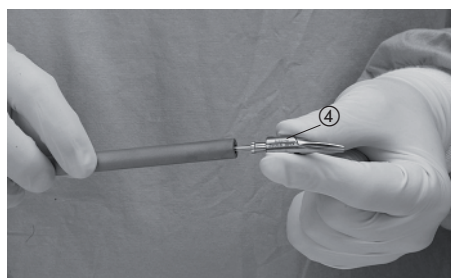
Disassembly

Press both locking buttons ① and ② and pull out sheath tube ③ distally.

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Hold firmly to remove the inner section ④ and open the bayonet mount by a ¼ turn of the sheath tube.

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Cleaning

Remove tissue remnants with a disinfected brush. Brush the inside of the sheath tube and rinse out. Dry with sterile compressed air

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Assembly

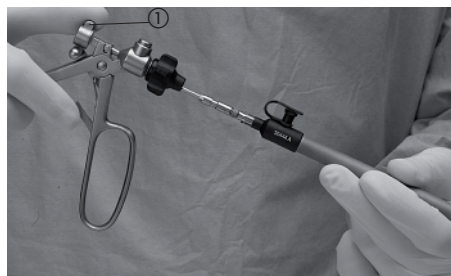
Inspect all parts for damage, if necessary replace.

Introduce inner section from the front into the sheath tube ③ and lock in the bayonet mount of the sheath tube by a ¼ turn.

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Press locking button ① and introduce the sheath with closed jaws into the completely opened handle as far as it will go. The locking must be heard to click into place.

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Loosen the stop and lock the sheath tube by completely closing the handle.

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Add a drop of instrument oil into the joints (Art. no. 27656 B B).
Test for proper operation.

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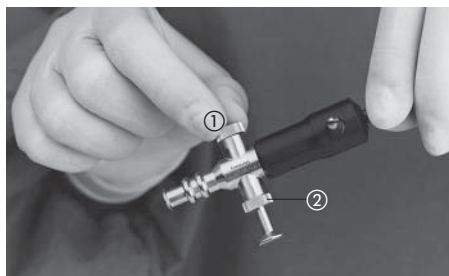
SEVINC CLICK'liné suture grasper
MANGESHIKAR CLICK'liné needle holder with
integrated scissors
MANGESHIKAR CLICK'liné endo-knotting device
See Dissecting and grasping forceps

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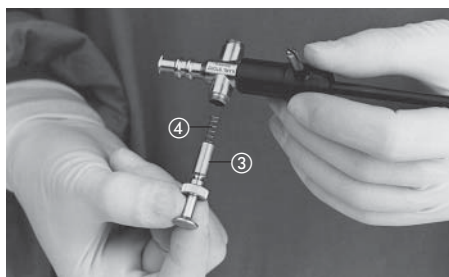
**Handle with trumpet valve
Disassembly**

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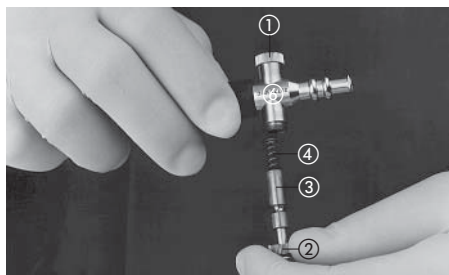
Assembly and functional test

Lightly grease piston ③.
Insert spring ④ into piston ③.
Screw knurled nut ① tightly.

Insert piston with spring into valve ⑥.
Screw knurled nut ② tightly with piston ③
slightly depressed.

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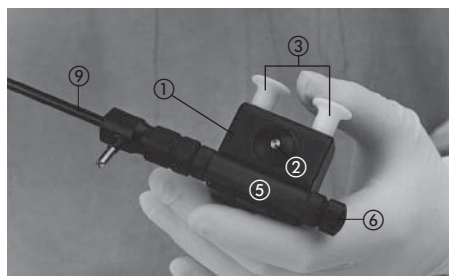
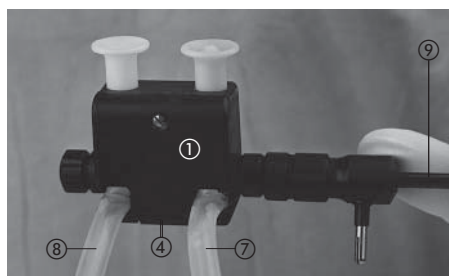
Modular handles and instruments for suction and irrigation

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Handle 30810 for irrigation, suction and coagulation

- ① Handle
- ② Locking bolt
- ③ Trumpet valve
white = suction
blue = irrigation
- ④ Lock
- ⑤ Hose connector
- ⑥ Cap
- ⑦ Outflow hose, white valve
- ⑧ Inflow hose, blue valve
- ⑨ Instrument

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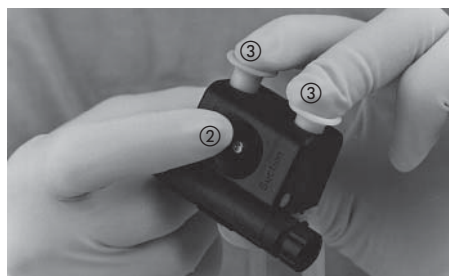
Disassembly

Unscrew the instrument ⑨ clockwise – the cap ⑥ counterclockwise. Loosen the hose connector ⑤ by pressing the lock ④.

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Lock the trumpet valve ③ on the handpiece in the open position. To do this, both trumpet valves are pressed down at the same time and secured with the locking bolt ②.

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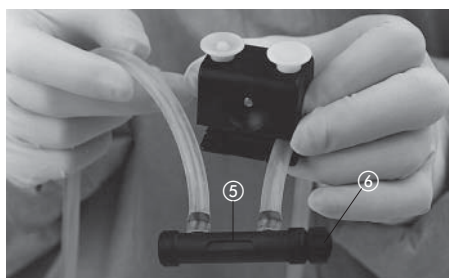
Pull the hose connector (5) approx. 5 cm out of the handle.

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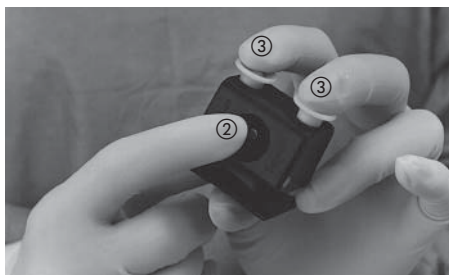
Take the hoses on the right and left out of the handle guide.
Take the hoses off of the hose connector.
Screw the cap (6) off of the hose connector (5).

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Assembly

Lock the trumpet valve (3) on the handpiece in the open position. To do this, both trumpet valves are pressed down at the same time and secured with the locking bolt (2).

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Lay the hoses into the guides on the handpiece (pay attention to color coding).

Note: When placing the irrigation and suction hoses, make certain the trumpet valves operate clockwise, i.e., by pressing the blue valve, the hose under the white valve is released and vice-versa.

Push the hose connector (5) up to the handle.

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Press the hose connector ⑤ into the handle until the lock clicks audibly into place.

Release the lock for the trumpet valves ③ by depressing.

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Place the cap ⑥ onto the hose connector ⑤.

Note: The cap may be attached on both sides of the hose connector, depending on whether right- or left-handed use is required.

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Connect the instrument ⑨ and join the handle to the suction/irrigation unit.

Blue valve = irrigation
White valve = suction

Note: Make certain that the inserted hoses, especially around the squeeze valves, are undamaged.

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**Handle for
irrigation and suction
Models 37112T, 37113T**

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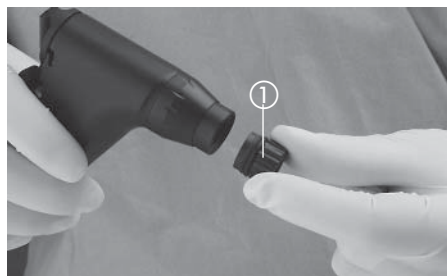
- ① Adapter for 5 mm sheath
- ② Locking ring
- ③ Head piece
- ④ Handle
- ⑤ Removable plug
- ⑥ Suction port
- ⑦ Irrigation port
- ⑧ Suction valve
- ⑨ Irrigation valve
- ⑩ Sealing ring
- ⑪ Spherical seal

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Disassembly

Pull tube out of suction port ⑥. Loosen and remove tube on irrigation port ⑦ by turning counterclockwise.
Preparation of the silicone tubing set is described in the suction/irrigation pump instruction manual.

Unscrew suction/irrigation sheath.

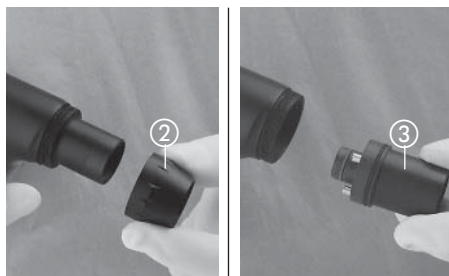
Unscrew adaptor① for 5 mm sheath, if fitted.

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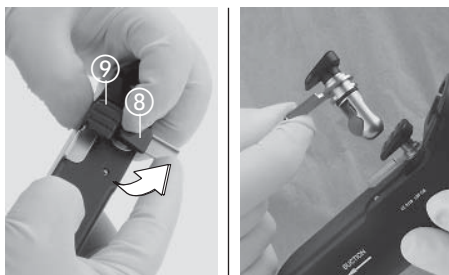
Additionally for pistol-shaped handle:

- Loosen and pull out removable plug ⑤ by turning counterclockwise.

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! Caution: If the button is not pressed down during rotation of the valve, the spherical seal can be damaged.

Hold down the suction valve ⑧ button while rotating the metal plate outwards.
Pull out the suction valve ⑧ toward proximal.

Remove the irrigation valve ⑨ in the same way.

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Remove the spherical seal ⑪
Check all seals ⑩+⑪.

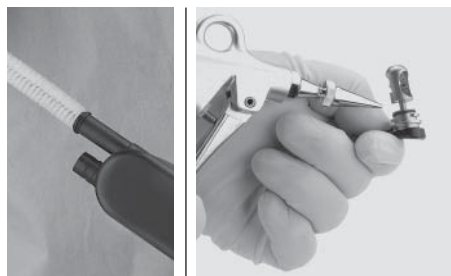
Replacement seals	Art. no.
10 spherical seals for suction valve	37112VS
10 spherical seals for irrigation valve	37112VI
1 set (all sealing rings and spherical seals)	
for 37112T	37112V
for 37113T	37113V

Damaged seals must be replaced with new seals, make sure to observe the various sizes.

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Cleaning

If necessary, clean the dismantled parts in an ultrasound bath. Thoroughly clean the lumen and suction/irrigation channels with a brush. Carefully clean the surfaces with a disinfected sponge or disinfected brush. Clean the valves ⑧+⑨ with button pressed down. Then, rinse thoroughly with completely demineralized water and dry with sterile compressed air. Thermal machine disinfection at 93 °C is possible, taking into consideration the chemicals approved by KARL STORZ.

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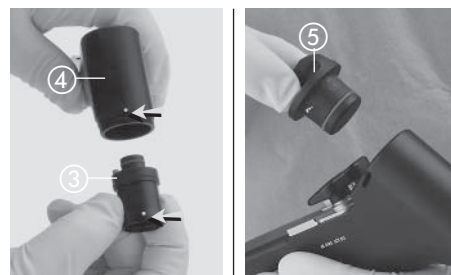
Lightly grease sealing rings ⑩ with special grease (Art. no. 27657).

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Assembly

Insert head piece ③ correctly in the handle ④ (observe white markings).

Screw on locking ring ②.

Additionally for pistol-shaped handle:

- Insert removable plug ⑤ correctly and fix by turning clockwise.

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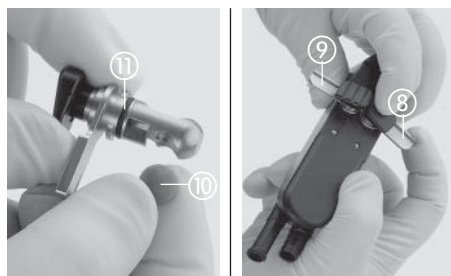
Place the spherical seal ⑪ in the suction valve ⑧ and in the irrigation valve ⑨; make sure to observe the various sizes.

Place the suction valve ⑧ and irrigation valve ⑨ in handpiece ④ and then lock into position by holding down the button: rotate the metal plate inwards.

Screw in suction/irrigation sheath (if necessary, use the adapter ① for the 5 mm sheath).

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Metal suction and irrigation tubes

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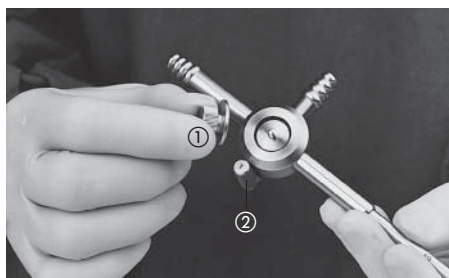
Suction and irrigation tubes/handles with two-way stopcock

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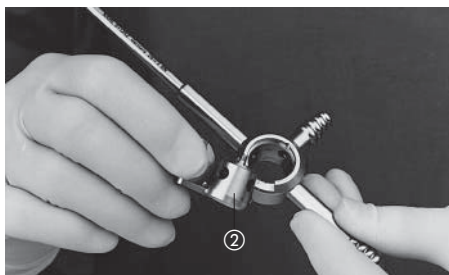
Assembly

Lightly grease the contact surfaces on the cock plug (2) (Special grease for stopcocks, Art. no. 27657). Insert the cock plug (2) and fix with the knurled-screw (1). Tighten slightly. Test for proper operation.

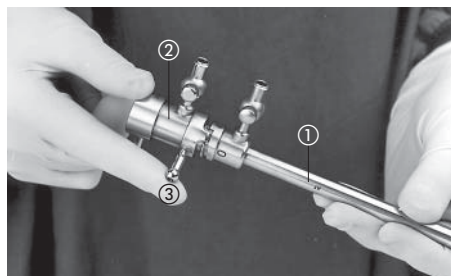
Note: This thin layer of grease not only serves to lubricate the metal-to-metal surfaces in contact with one another, but also prevents the stopcocks from leaking. Oil is not suitable as a sealant.

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**Sheaths for CO₂ LASERS
Disassembly**

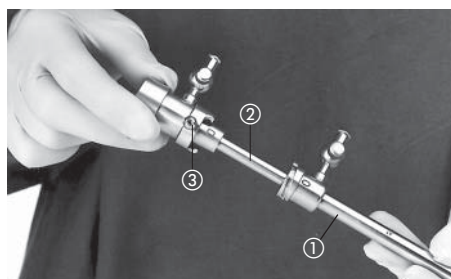
Note: Disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 27).

After loosening the locking ring ③, remove the outer sheath ① from the inner sheath ②.

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Cleaning

Remove sticky or caked tissue remnants with a cleaning brush. Clean small parts carefully. Rinse out sheath tubes. Clean the instrument in an ultrasound bath. Dry with sterile compressed air.

Assembly

Slide the outer sheath ① onto the inner sheath ②, fix in place with the locking ring ③. Disassemble again for sterilization.

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**CO₂ laser coupler
Disassembly**

No disassembly is necessary. The brake ① for the mirror lever ② must not be completely unscrewed.

Cleaning

Caution: Do not allow any moisture to enter the coupler to prevent deposits on the optical components.

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Manual cleaning

The laser coupler can be wiped off from the outside with a cloth soaked in cleaning and disinfectant solution or alcohol-free rapid action disinfectant. In doing so, no fluid must be allowed to penetrate into the motor. The exterior glass surfaces can be cleaned with a cotton applicator soaked in alcohol.

Machine cleaning is not permitted.

Adaptor for CO₂ laser coupler

The adaptors may be cleaned and disinfected like a normal instrument set.

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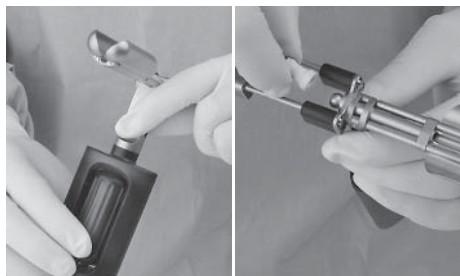
CUSCHIERI inguinal canal retractor

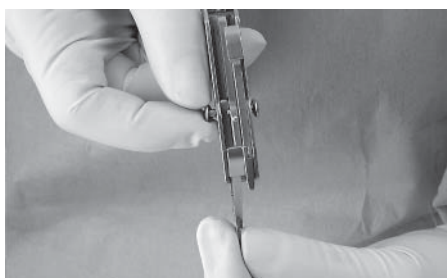
To clean the retractor, open it by turning the handle. Remove the seal, check it and, if necessary, fit a new one. Brush out the lumen, rinse, and dry with compressed air.

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DUNDEE multitool

To remove the working insert, pull the side- mounted locking button outward and pull the working insert out toward the front.

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Then clean the guide rail with a brush or the like.

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Instruments for hand-assisted laparoscopic surgery

Manual cleaning:

Clean the jaws with a disinfected brush. Clean the exterior of the needle holder in a cleaning solution.

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Rinse out the LUER-lock irrigation port.

Finally rinse with microbiologically clean/sterile water and dry with compressed air.

The instrument is suitable for cleaning in an ultrasound bath.

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Machine cleaning/disinfection:

The needle holder can be thermally cleaned or disinfected in the washer and disinfectant.

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Add a drop of instrument oil into the joints (Art. no. 27656 B).

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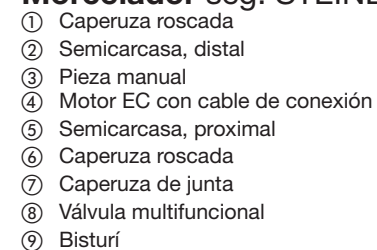
CLICK'liné scissors and forceps
See Section Dissecting and grasping forceps
bipolar forceps
See Section Instruments for unipolar and bipolar coagulation

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Dismantling the morcellator handle for cleaning

Caution: Never remove the EC motor from the handle by pulling on the connecting cord.

Pull the EC motor ④ out of the handle ③.

Caution: Do not immerse the EC motor and connecting cable in liquid!

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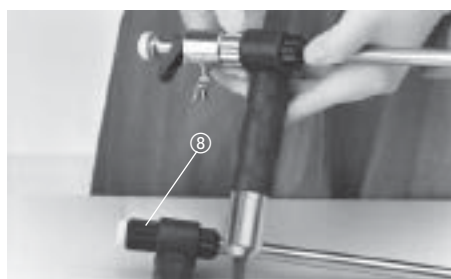
Undo and remove the screwcap ⑥ (with multi-function valve on 2671012).

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Undo the thumbscrew ⑩ (turn counterclockwise). The knife may now be pulled out towards the back.

Warning: Danger of injury! Sharp edged cutting tube. Do not touch the cutting edge with bare fingers during replacement.

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Undo the front screwcap ①. Both gear shells ② ⑤ may now be removed.

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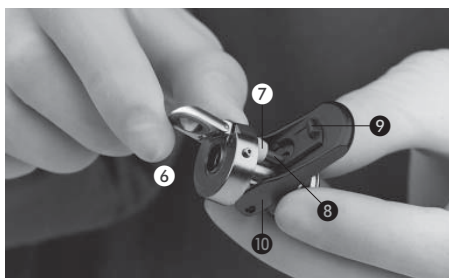
Dismantle multifunction valve further
Remove sealing cap.
Undo bayonet mount (5) on the valve sleeve.
Remove valve head (6).
Loosen thumbscrew on cock plug, remove cock plug.

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The valve head (6) may be dismantled even further to replace the small sealing cap (7) or the tappet (8).
To do this, pull out the pusher (9) on the thumb plate (10) of the valve head (6).

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Press down and hold the thumb plate (10) and remove the tappet (8) from the back.
Remove the small sealing cap (7).

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Cleaning/disinfection

Manual cleaning

Hold the valve flap open when cleaning the valve head (cleaning rod 26G).

Caution: Clean the valve flap carefully, since maladjustment of the valve flap impairs smooth functioning.

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Cleaning/Disinfection

Clean the dismantled parts (except the EC motor and connecting cable) mechanically with warm water or a cleaning and disinfectant solution (see Appendix) and a soft brush. Rinse with completely demineralized water and dry with sterile compressed air.

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Machine cleaning/disinfection

Hold the valve flap in the open position using the opening rod (Art. no. 26 G) or an appropriate holding device.

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Note: Consult the instruction manual 'STEINER electromechanical morcellator Model 267121 12/267121 13/267121 14 - Unidrive unit 20712120'.

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Assembly

Reattach both gear shells ② ⑤. Tighten the front screw cap ①.

Insert knife from the back and turn the smaller knurled screw using slight pressure until the knife audibly clicks into place in the groove.

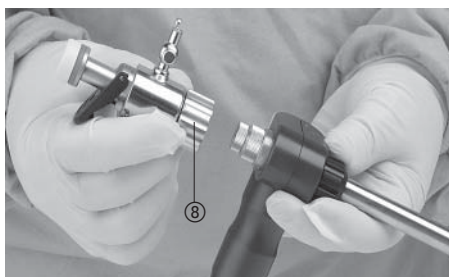
For assembly of the multifunction valve, see Section *Trocars with multifunction valve*.

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Attach the screw cap ⑧ (with multifunction valve on 267121 12/267121 14) and tighten.

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Slide in the motor ④ until it audibly clicks into place.

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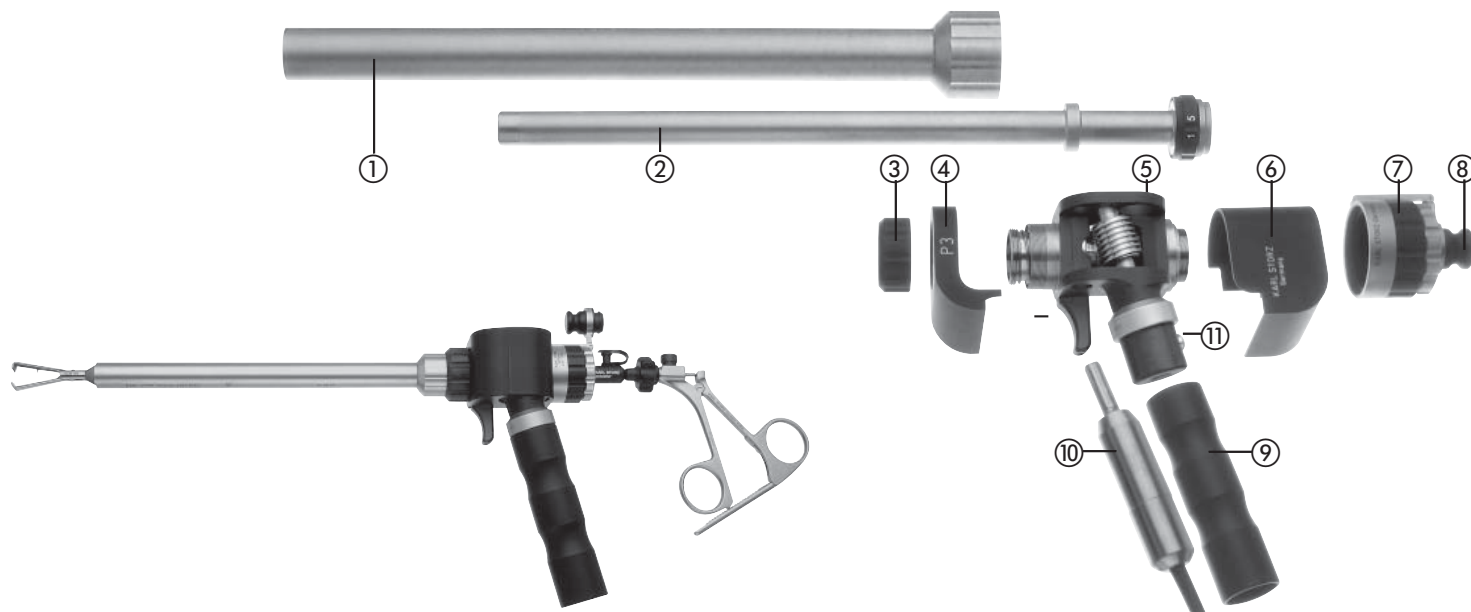
Perform a pulling test to ascertain whether the motor is solidly in place.

Insert the trocar tongue (while operating the thumb plate on 267121 12/267121 14).

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SAWALHE morcellator

- ① Outer tube
- ② Knife
- ③ Screw cap
- ④ Housing cover
- ⑤ Gear
- ⑥ Housing
- ⑦ Sealing cap
- ⑧ Plug
- ⑨ Handle
- ⑩ EC motor with connecting cable
- ⑪ Locking button

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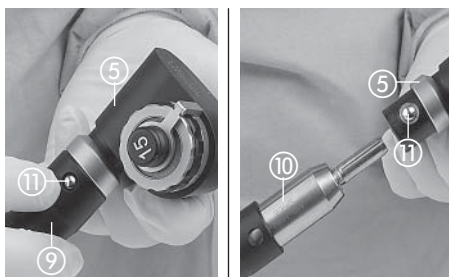
Unlock handle ⑨ at the locking button ⑪ and pull off at the same time.

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Remove handle ⑨ from the EC motor ⑩.

Caution: Do not immerse the EC motor or connecting cable in liquid!

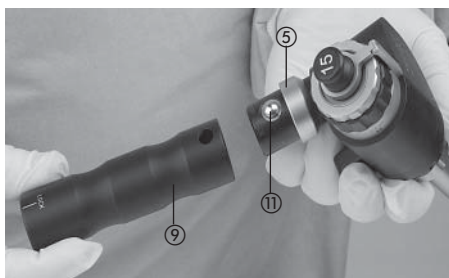
Prepare the EC motor and connecting cable in accordance with the 'UNIDRIVE® II motor system' instruction manual.

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Put the handle ⑨ onto the gear ⑤ again and latch the locking button ⑪ into place.

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Loosen the sealing cap ⑦ by turning ¼ counterclockwise, and pull off.

Dismantle sealing cap ⑦ and prepare.

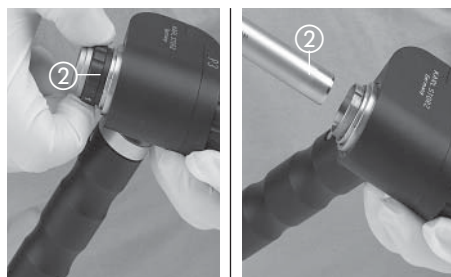
Caution: Housing ⑥ is unfastened on the gear.

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Loosen profile nut on the knife and remove knife ② to proximal from the handle.

Warning: Danger of injury
Distal end of the knife is sharp!
Do not touch with bare fingers.

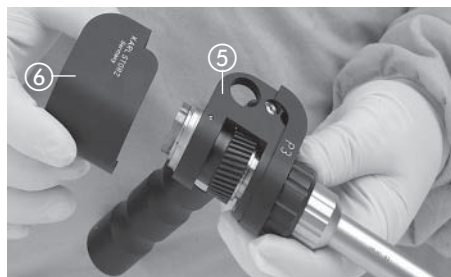
Note: Send blunt knives for sharpening to KARL STORZ (For address see Section Maintenance).

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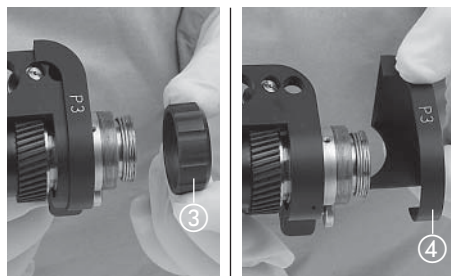
Cleaning and sterilization

Immerse the individual parts (except for the EC motor and connecting cord) in a cleaning solution and mechanically clean with a soft, disinfected brush.

Rinse with completely demineralized water and dry with sterile compressed air.

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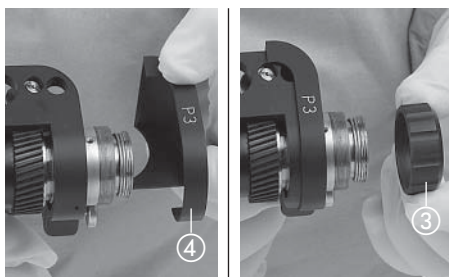
Assembly

Push handle ⑨ onto the gear ⑤ until the locking button ⑪ clicks into place.

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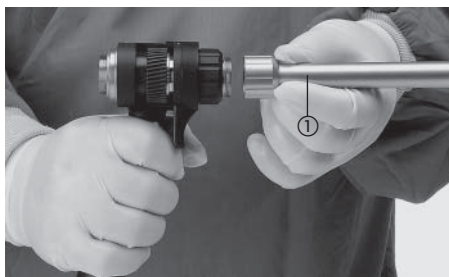
Attach housing cover ④ and fasten with screw cap ③.

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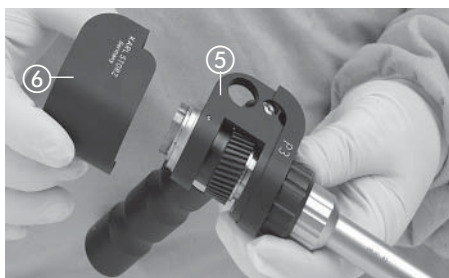
Screw on outer tube ① and fasten tightly.

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Attach housing ⑥.

Caution: Housing ⑥ is loose on the gear ⑤.

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Warning: Danger of injury!
Distal end of the knife is sharp!
Do not touch with bare fingers.

Insert knife ② from proximal into the handle and fasten at handle ⑨ with profile nut.

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Attach sealing cap ⑦ (also fastens housing ⑥) and fasten by turning ¼ clockwise.

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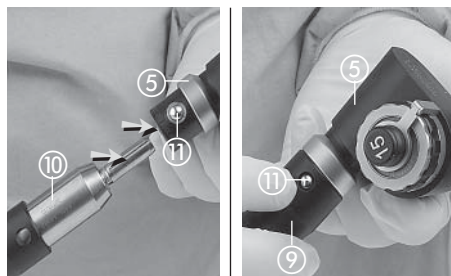
Slide handle ⑨ over the EC motor with the connecting cable ⑩

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Align EC motor ⑩: The groove in the motor shaft must be aligned with the marking on the gear ⑤.
Push in the EC motor ⑩ until it clicks into place in the gear ⑤.

Slide handle ⑨ over the motor onto the gear ⑤ until the locking button ⑪ clicks into place.

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Warning: Follow the 'UNIDRIVE® II motor system' instruction manual!

Connect connecting cable of the EC motor to the UNIDRIVE® II supply unit.

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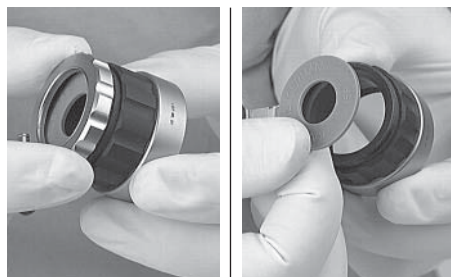
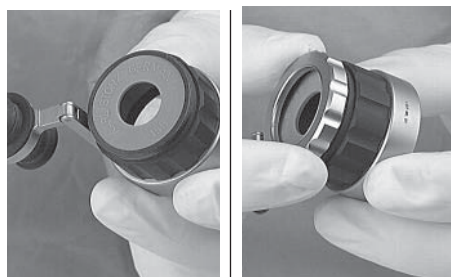
Replacing the sealing
Unscrew and remove the screw cap.

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Slide the grip forwards as far as it will go and then press simultaneously the unlock buttons. The knife insert can now be pulled off toward distal together with the outer sheath.

Warning: Danger of injury!
The scalpel blade has a sharp edge. When removing, grasp with a cloth or the like.

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Manual cleaning

Remove tissue residue with a disinfected cleaning brush.

Brush and rinse out sheath tubes on the inside.

Blow dry with sterile compressed air.

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Assembly

Press the scalpel blade into the holder on the knife insert.

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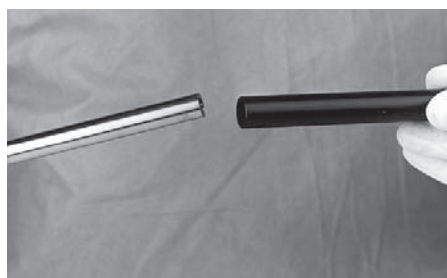
Introduce the knife insert into the outer sheath. The insert only fits in one position in the grooves at the distal end of the outer sheath. Pull back the knife insert far enough until the scalpel blade has completely disappeared within the outer sheath.

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Place the protective cap on the distal end of the outer sheath to prevent injuries due to the scalpel blade.

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Insert the outer sheath and proximal section of the knife insert into the handle. Lock the handle and outer sheath. Make certain that the 0 marks are in alignment.

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Press the unlock button and slide the handle slightly forward until the knife insert can click into place. Then test for proper operation.

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KARL STORZ ROTOCUT G1
hollow shaft motor 26711030

Separate the hollow shaft motor from the accessories for cleaning.

The hollow shaft motor must not be machine cleaned. Sterilization can be performed at 134°C.

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The hollow shaft motor must be sprayed with universal spray (see accessories) before each sterilization. For this purpose, place each of the two spray slits on either side of the hollow shaft motor onto the spray insert in turn, and spray for approx. 1 second each side.

Caution: Do not kink the motor cable.

Warning: Let the hollow shaft motor cool down after autoclaving.

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Dismantling the morcellator handle for cleaning or changing the instrument/cutter
(different diameters)

Remove forceps and prepare separately.

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Remove the cutter

Turn the handle counterclockwise and remove it from the motor housing together with the cutter. If the 90° handle was used, it too can be removed.

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Undo the cutter

Holding the cutter in one hand and the handle in the other, pull them apart. The cutter will detach from the handle with a click.

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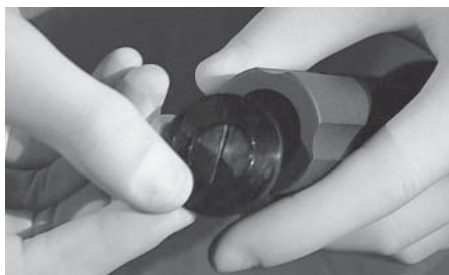
Caution: Take care with the sharp edge of the blade!

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Remove the sealing cap

The sealing cap is still fitted at the back end of the handle. Remove it from the handle by pulling gently.

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Note: The sealing cap is delivered unsterilized. It must be sterilized before being used for the first time. Check for damage before and after cleaning/sterilization. It is recommended that the sealing cap be replaced every time the unit is used.

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Detach outer tube

Detach the outer tube from the motor housing by turning it counterclockwise.

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Take out motor valve

Carefully turn the motor housing with the flat side (see arrow) facing downward and remove the motor valve by hand.

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Cleaning

Immerse the individual parts in a cleaning solution and mechanically clean with a soft disinfected brush

Rinse with completely demineralized water and dry with sterile compressed air.

The dismantled parts can then be thermally* disinfected by machine, steam sterilized at 134°C (+ 3°C) or sterilized with gas.

* except hollow shaft motor and connecting cord

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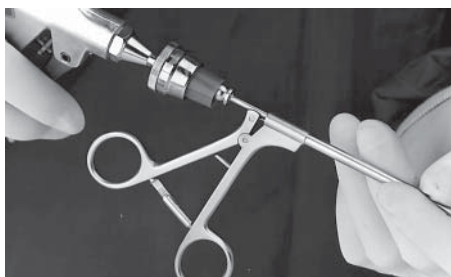
Machine cleaning/disinfection

The dismantled parts can be thermally cleaned/disinfected by machine.

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forceps

Cleaning

Clean canal with a disinfected brush and then rinse with a cleaning gun.

The forceps can also be immersed in an ultrasound bath.

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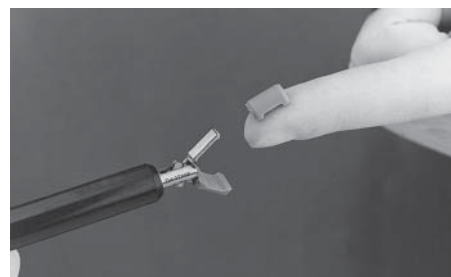
**BERCİ CLICK'line® grasping forceps,
with rubber-coated jaws**

Disassembly/Assembly

The silicone casings on the jaws may be taken off, cleaned and replaced on the jaws.

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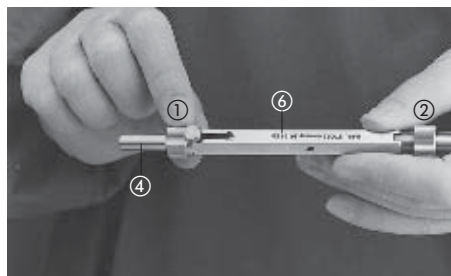
As an alternative, or if there is visible damage, the silicone casings should be disposed of after each use and replaced by new ones. Replacement silicone casings are available under article number 30510 PX 'Silicone cover, for use on forceps 33510 PG, autoclavable, pack of ten'.

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**Micro scissors
Disassembly**

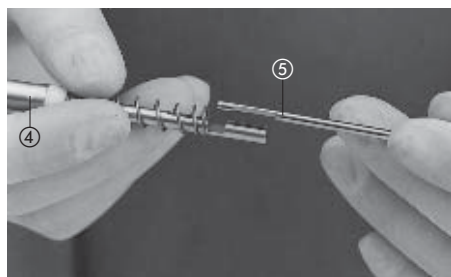
Unscrew the rear knurled screw ① from the handle ⑥.

Pull out the internals ④ with the knife ⑤ towards the back.

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Lift the knife ⑤ out of the holder in the internals ④.

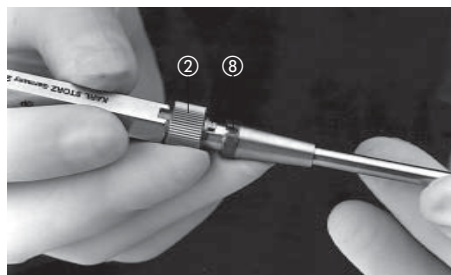
Cleaning

Remove any remaining or dried on tissue residue using a brush. Clean the small parts carefully. Rinse sheath tubes. Clean the instrument in an ultrasound bath. Dry with sterile compressed air.

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Open the front thumbscrew ② on the handle ⑥ and take the handle off of the sheath.

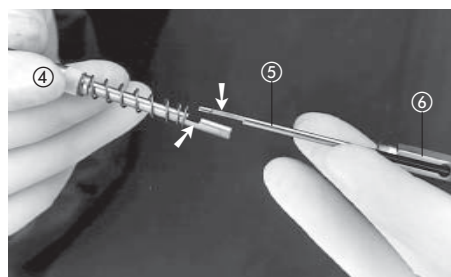
Assembly

Check the components, particularly the knife, for damage. Replace if necessary. Insert the lug on the handle into the groove on the sheath tube ⑧ and screw onto the sheath tube using the front knurled screw ②.

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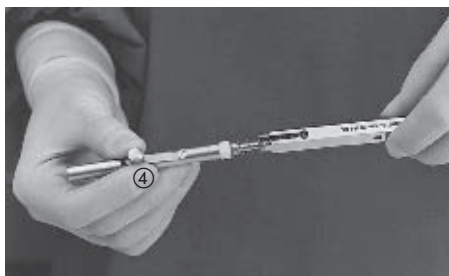
Carefully insert the knife ⑤ into the handle ⑥ from behind so that the blade is not damaged. Place the proximal end of the knife ⑤ (rounded surface facing up) into the holder provided for this on the internals ④.

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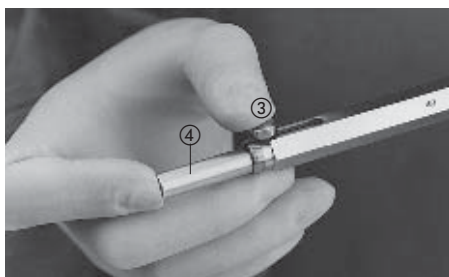
! Warning: When performing the following work procedures, the knife projects out of the front of the sheath tube.
Insert the internals ④ into the handle.

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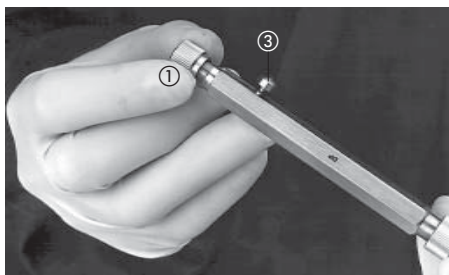
At the same time, the locking button ③ must be inserted into the notch on the handle while depressed.
Insert the internals ④ completely until the knife projects out the front and the locking button ③ pops up.

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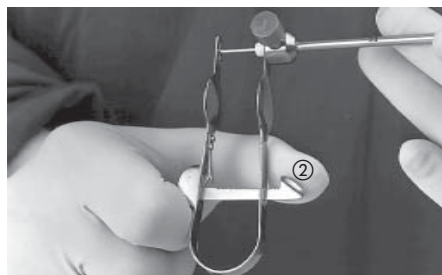
Screw the rear knurled screw ① onto the handle.

When the locking button ③ is depressed, the knife must pop back into the sheath tube completely.

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Sperre ② am U-Feder-Handgriff lösen.

**Endotom
Disassembly**

Warning: Instrument has a sharp blade.

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Detach the rear end of the internals ① from the handle. To do this, hold the sheath tube ③ firmly in front of the handle and press the rear part of the handle upwards until the rear end of the internals ① can be removed from the short leg of the L-shaped notch ④. Pull out the internals ① towards the front.

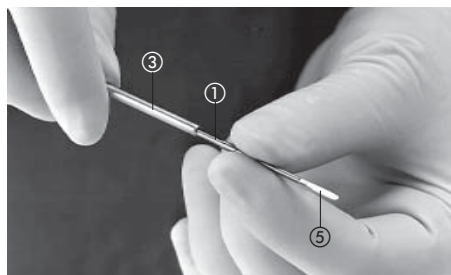
Cleaning

Clean through the irrigation connection ⑥. For heavy soiling, rinse out during surgery; always rinse out immediately after surgery.

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Assembly

Check the sealing cap and blade ⑤ of the internals for damage, replace if necessary. Insert the internals ① from the front into the sheath tube ③. The blade ⑤ should face down.

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Bring the notches on the rear end of the internals ① into the correct position in order to slide into the L-shaped notch ④ on the handle.

Hold the sheath tube ③ firmly in front of the handle and press the rear part of the handle upwards until the rear end of the internals ① can be inserted into the short leg of the L-shaped notch ④ and slid upwards in the long leg.

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**Video operating rectoscope
Disassembly**

Release the union nut by turning it counterclockwise and remove the attachment.

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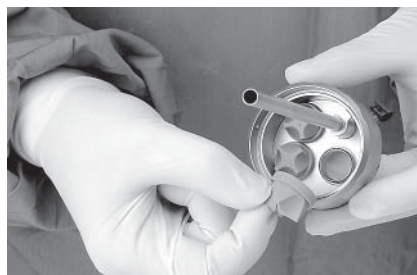
Take off sealing cap, check for damage, replace if necessary.

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Take out the silicone lip valve, check for damage and replace if necessary.

Note: We recommend replacing the silicone lip valve after every use.

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Inspect the red seal ring on the surgical sheath for damage and, if necessary, fit a new one.

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Cleaning

Clean any heavy soiling on the obturator and rectoscope tube with a disinfected sponge or brush.

Thoroughly brush the rectoscope tubes, LUER-lock connectors and telescope channel, then rinse and blow dry with sterile compressed air.

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Assembly

Place the silicone lip valves onto the instrument channels.

Put on the sealing caps.

Insert the obturator or attachment.

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CLICK'linē forceps

See Section 'Dissecting and grasping forceps'

CLICK'linē scissors

See Section 'Scissors, punches, biopsy forceps'

Coagulation suction tubes

See Section 'Instruments for unipolar coagulation'

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**Laparoscopic aortic surgery,
instruments for
hand-assisted ...**

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See Section Trocars

CLICK'liné Scissors and forceps

See Section Dissecting and grasping forceps

**Instruments for hand-assisted laparoscopic
surgery**

*See Section Inguinal canal retractor, instruments
for mini-laparoscopy, instruments for hand-
assisted surgery*

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Dissecting and grasping forceps

see Section Dissecting and grasping forceps

Bipolar coagulation forceps

*see Section Instruments for unipolar and bipolar
coagulation*

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Intrauterine probes and uterus elevators

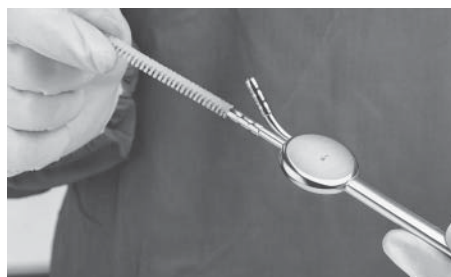
Note: Disassembly, cleaning and assembly of LUER-lock stopcocks see 'Instructions for LUER-lock stopcocks' (page 22)

Clean outer surfaces with a disinfected sponge or brush.

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Clean all lumen channels with a disinfected flexible brush.

Then rinse out with water, clean in an ultrasound bath and dry with sterile compressed air.

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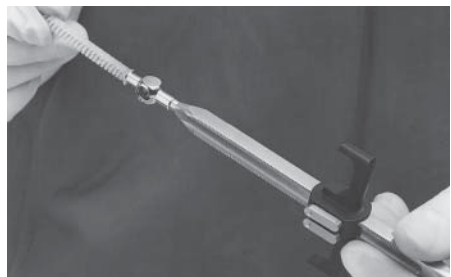
COHEN intrauterine probe 26168 UN

The cones can be pulled off toward distal to clean the instrument.

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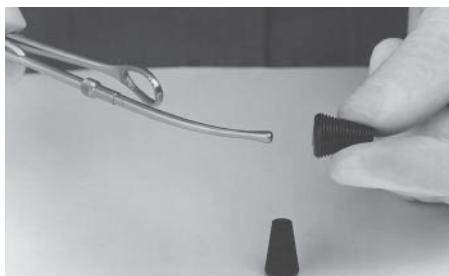
Clean lumen from sheath and cones with a disinfected flexible brush.

Then rinse out thoroughly with water, clean in ultrasound bath and dry with sterile compressed air.

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**Tenaculum forceps / Uterus
grasping forceps**

Attached cones can be pulled off toward distal to clean instrument. Then clean all outer surfaces with a sponge.

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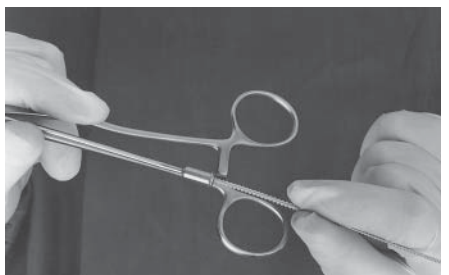
Thoroughly clean jaws and cones with a disinfected brush.

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Clean the perturbation channel with a brush and rinse out with a cleaning gun. Then rinse out with water, clean the instrument in an ultrasonic bath and dry with sterile compressed air.

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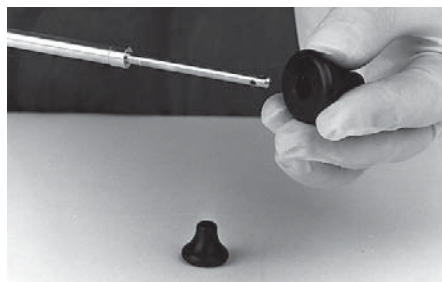
Oil the joint and, at the same time, hold the instrument in such a way that oil may flow into it.

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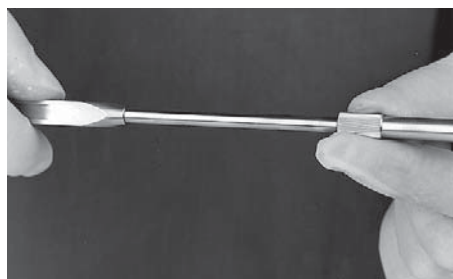
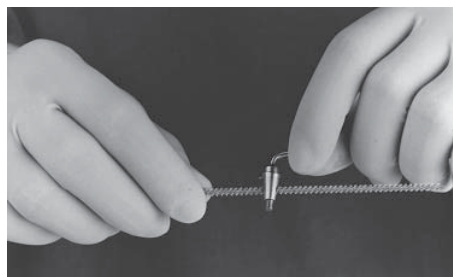
To remove the adjusting sleeve, turn the knurled screw clockwise until the sleeve can be pulled off toward distal.

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Cleaning

Take apart and clean the stopcock (see the Section on Trocars, manual for LUER-lock stopcocks)

Brush out the lumen of the sheath and cones. Clean in an ultrasound bath. Rinse and dry with sterile compressed air.

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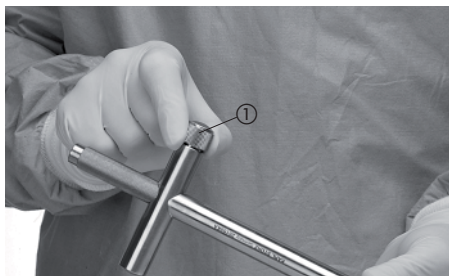
- ① Uterus manipulator probe
- ② Handle
- ③ Fixing screw for locking the uterine manipulator probe
- ④ Union nut
- ⑤ Cone
- ⑥ Spiral insert
- ⑦ Opening for inserting the key ⑨
- ⑧ Attachment
- ⑨ Key

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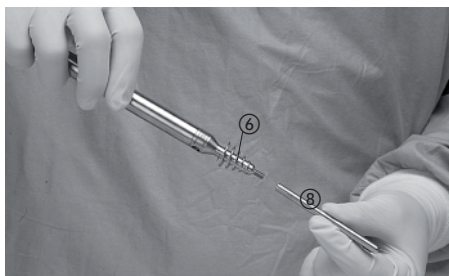
Disassembly

Loosen fixing screw ③ and pull uterine manipulator probe ① distally out of the handle ②

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Unscrew attachment ⑧ from the spiral insert ⑥ by turning clockwise.

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Loosen spiral insert ⑥ with the aid of the key ⑨ and remove.

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Loosen union nut ④ on the handle and remove cone ⑤.

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Cleaning

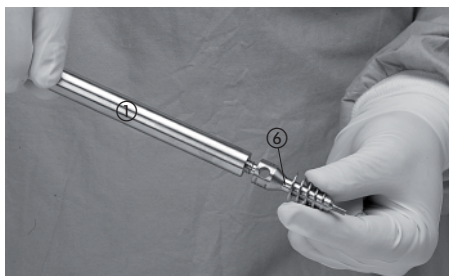
If necessary, clean the parts (except the cone) in an ultrasound bath. Thoroughly clean the lumen and suction/irrigation channels with a brush. Carefully clean the surfaces with a sponge or soft brush. Then, rinse thoroughly with completely demineralized water and dry with sterile compressed air.

All parts can be sterilized at a temperature of 134°C by steam.

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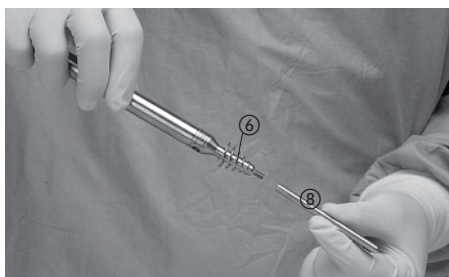
Assembly

Screw one of the spiral inserts ⑥ into the uterine manipulator probe ① and fasten with the accompanying key ⑨.

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Screw one of the attachments ⑧ onto the thread of the spiral insert ⑥ used.

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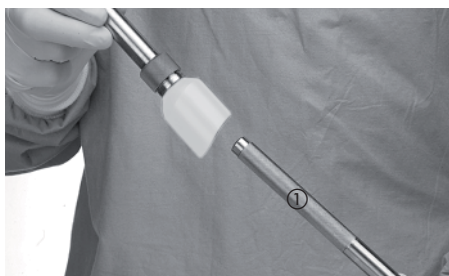
Screw one of the cones ⑤ onto the handle ② with the aid of the union nut ④. Ensure that the projection on the cone engages in the groove on the handle.

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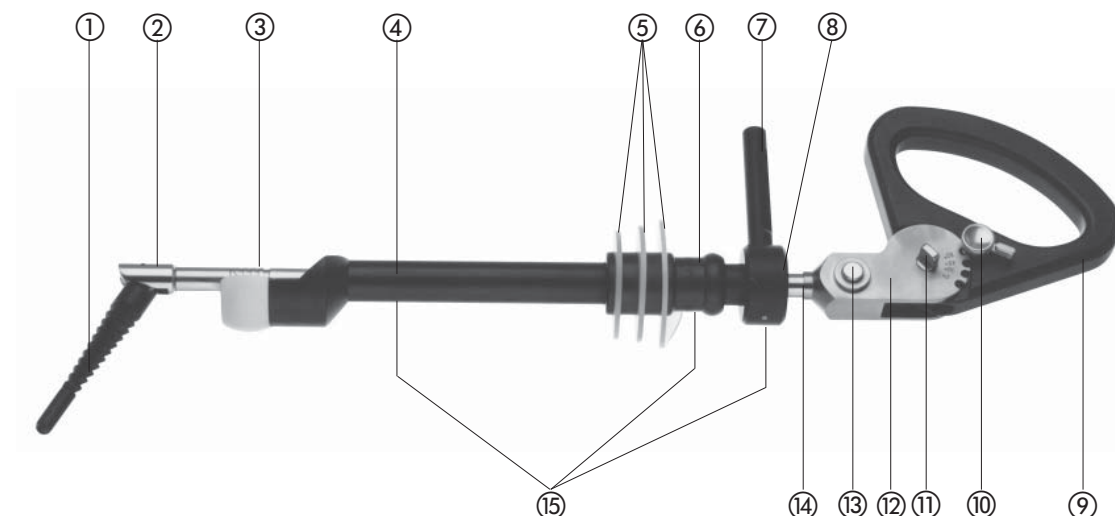
Introduce the preassembled uterine manipulator probe ① from distal into the handle ② and fix in the desired position with the fixing screw ③.

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- ① Insert
- ② Forked parts
- ③ Bayonet safety device
- ④ Anatomic blade
- ⑤ Silicone sealing rings
- ⑥ Seal cylinder
- ⑦ Rod handle
- ⑧ Rod sleeve
- ⑨ Handle
- ⑩ Locking button
- ⑪ Handle screw
- ⑫ Handle holder
- ⑬ Sheath unlock
- ⑭ Sheath

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**CLERMONT-FERRAND
uterine manipulator model**

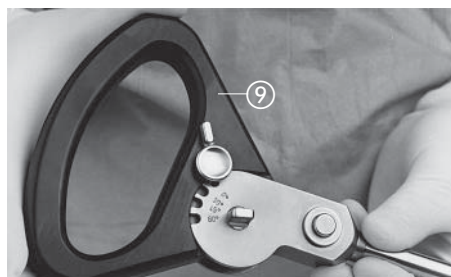
Disassembly

Unlock locking button ⑩: depress and turn counterclockwise.

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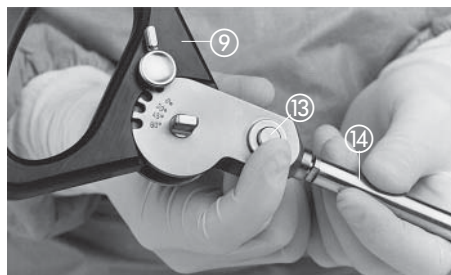
Caution: Loose parts of the instrument may fall and thus become damaged. Hold on to disassembled parts firmly and lay aside safely.

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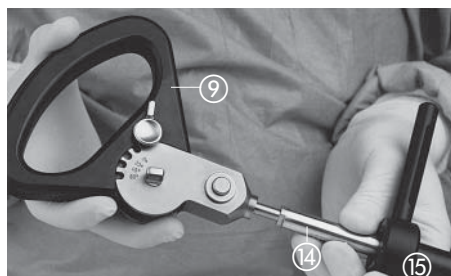
Press the sheath unlock ⑬ and pull the complete shaft ⑭ approx. 1 cm out of the handle ⑨.

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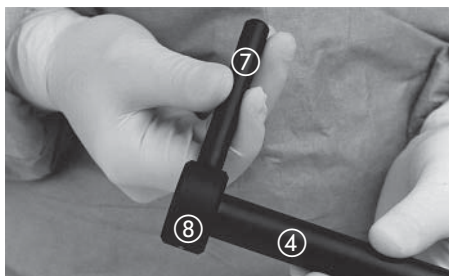
Move the handle ⑨ beyond the 0° mark up to the limit. Pull out the sheath ⑭ completely.

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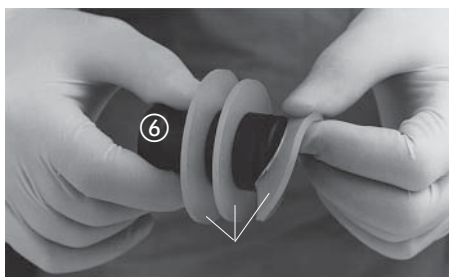
Disassembly of the attachment

Turn the rod handle ⑦ counterclockwise and screw off.
Pull the rod sleeve ⑧ off of the anatomic blade ④.

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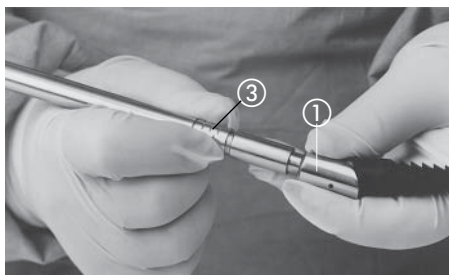
Pull the sealing cylinder ⑥ off the anatomic blade ④ toward the back.
Pull the sealing rings ⑤ down from the sealing cylinder ⑥.

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Disassembly of the sheath and insert

Slide the bayonet safety device ③ up to the other limit and hold there tightly.
Turn the insert ① counterclockwise and pull out.

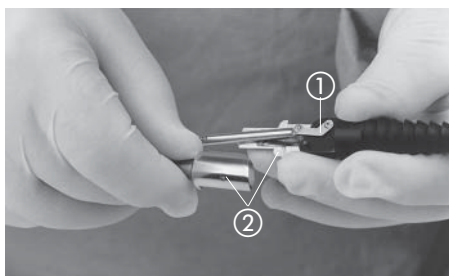
Caution: Loose parts of the instrument may fall and thus become damaged.
Hold on to disassembled parts tightly and lay aside safely.

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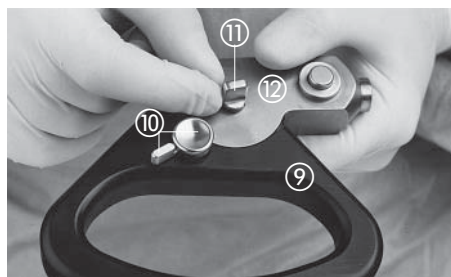
Remove the two forked parts ② from the insert ① sideways.

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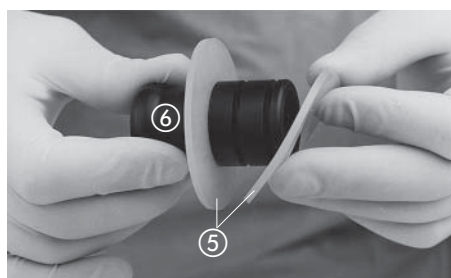
Disassembly of the handle

Lock the locking button ⑩: turn clockwise if necessary.
If necessary, move the handle holder ⑫ back and forth until it clicks into place.
Undo the handle screw ⑪ and pull out.
Remove the handle ⑨ from the handle holder ⑫.

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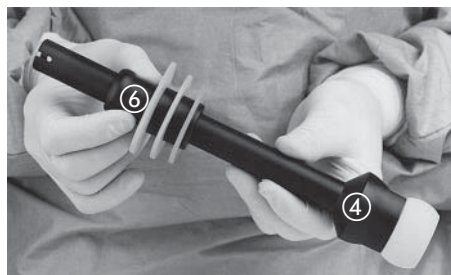
Assembly Assembly of the attachment

Insert the sealing rings ⑤ into the grooves on the sealing cylinder ⑥.
First insert the sealing ring with the largest diameter next to the writing, finally the sealing ring with the smallest diameter on the outside.

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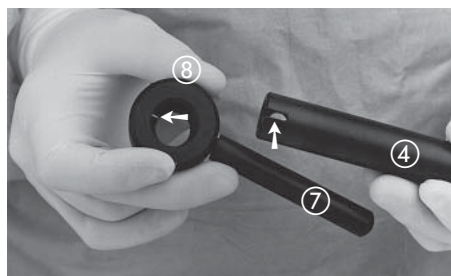
Slide the sealing cylinder ⑥ with the silicone sealing rings toward the front, onto the anatomic blade ④.

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Screw on the rod handle ⑦ approx. 3 rotations into the threaded bore on the rod sleeve ⑧.

Attach the rod thus mounted onto the anatomic blade ④ and, if necessary, turn until the pin on the rod sleeve ⑧ clicks into place in the groove on the anatomic blade ④.

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Fix in place: Screw in rod handle ⑦ completely.
The attachment ⑮ is mounted.

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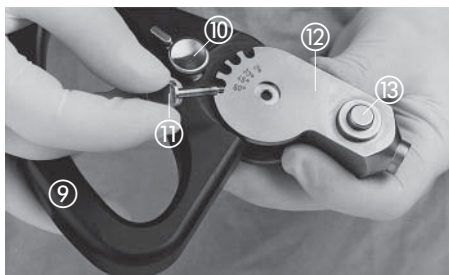
Assembly of the handle
Lock the locking button ⑩: turn clockwise if necessary.

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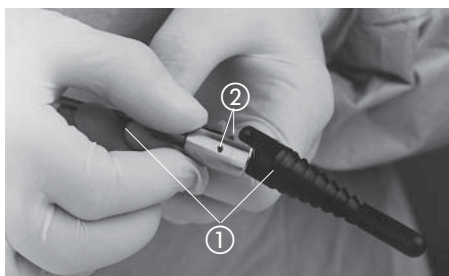
Insert the handle ⑨ into the handle holder ⑫ (the
locking button ⑩ and sheath unlock ⑬ are on
the same side).
Insert the handle screw ⑪ and tighten.
The handle is mounted.

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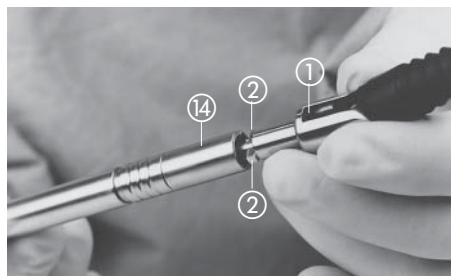
Assembly of sheath and insert
Attach both forked parts ② sideways onto the
pin of the straightly aligned insert ① and hold
tightly.

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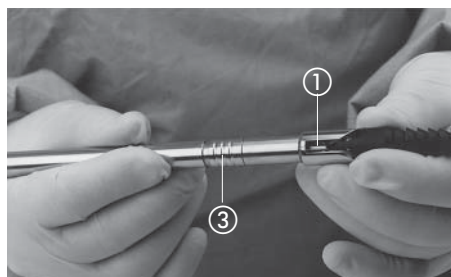
Slide the insert ① with the ball-shaped end into the bayonet on the sheath ⑭.

Note: Both forked pieces ② have projections of varying widths, which fit the sheath grooves in only one position.

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Push in the insert ① against the spring tension of the bayonet safety device ③ up to the limit and turn clockwise simultaneously.

Check whether the bayonet is securely locked.

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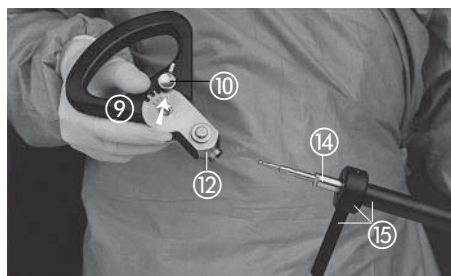
Check whether the insert functions and operates smoothly.

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Final assembly

Slide the attachment ⑮ over the sheath ⑭.

Unlock the locking button ⑩.
Set the handle ⑨ at the 0° mark.
Lock the locking button ⑩.

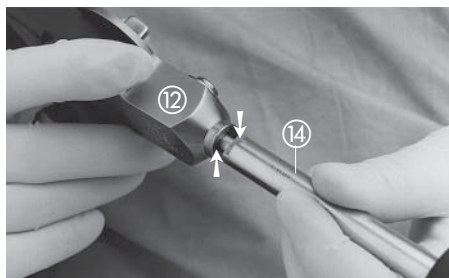
Slide the sheath ⑭ with the ball-shaped end in front into the handle holder ⑫ and hold tightly.

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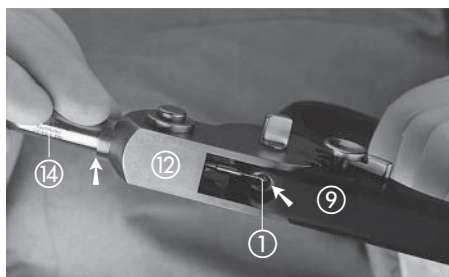
Align the surface of the sheath (14) with the surface of the handle holder (12).
Press the sheath (14) and handle holder (12) lightly against one another and, at the same time: Unlock the locking button (10).
Move the handle (9) up to the limit beyond the 0° marking and then back again.
The sheath (14) clicks into place in the handle holder (12).

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Check whether the ball on the insert (1) is in the holder for the handle (9).
Check whether the sheath (14) has properly clicked into place in the handle holder (12).

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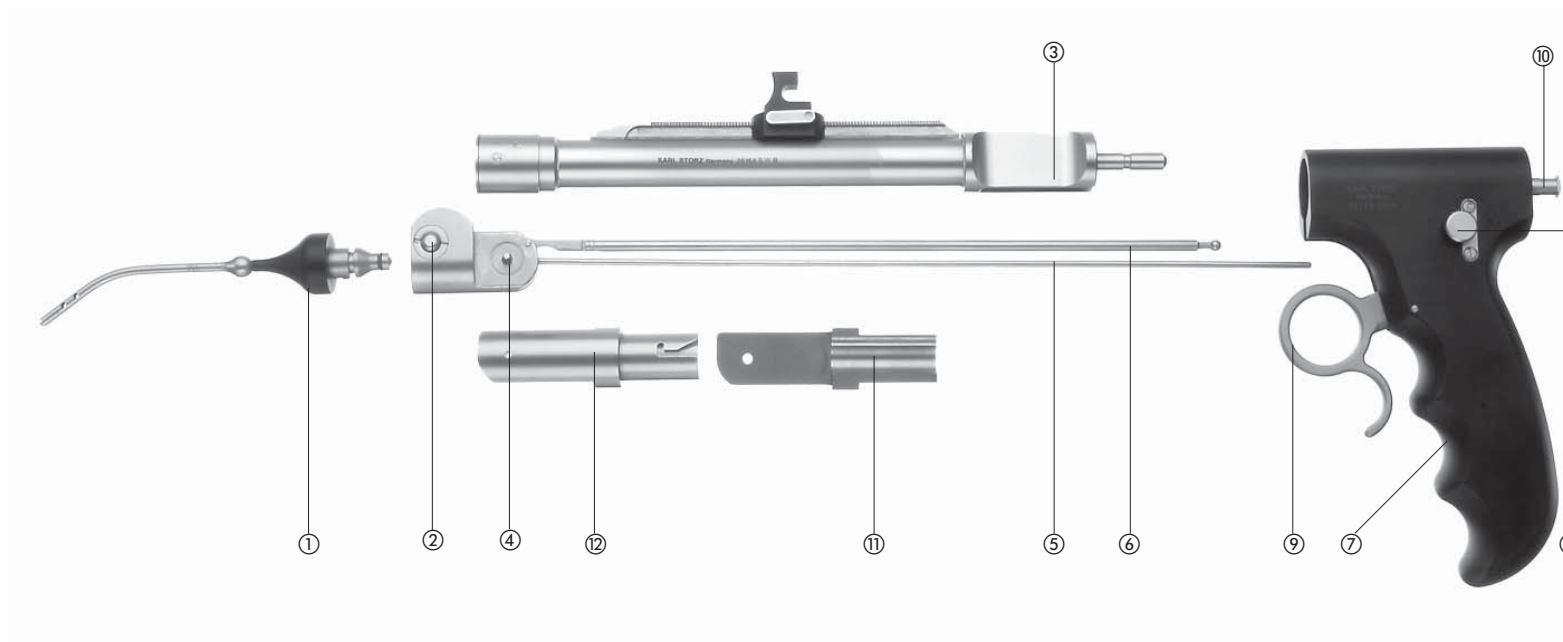
All components must be checked for complete functional capability.
All moveable parts must operate smoothly and be free of defects.
Check the correct alignment of all parts on the instrument.

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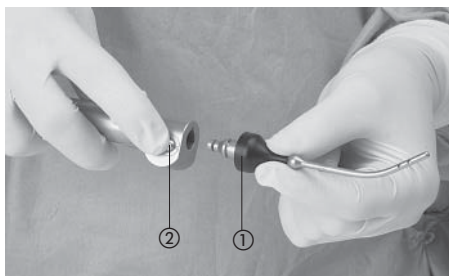
- ① Intrauterine probes
- ② Unlock button
- ③ Outer sheath
- ④ Working insert
- ⑤ Pertubation channel
- ⑥ Plunger
- ⑦ Handle
- ⑧ Catch for operating lever
- ⑨ Operating lever
- ⑩ Irrigation channel
- ⑪ Left half-shell (large bore)
- ⑫ Right half-shell (small bore)

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Disassembly

Press the unlock button ② on the distal end of the working insert and pull out the intrauterine probe ①.

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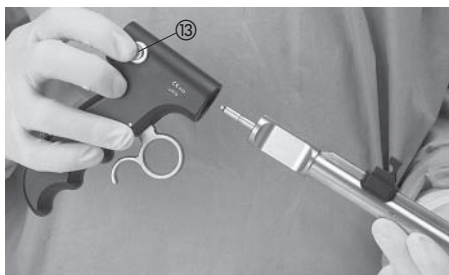
Loosen catch for the operating lever ⑧ (push in locking button; the operating lever must be able to move freely).

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Press the locking button ⑬ for the outer sheath and pull off outer sheath distally.

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Caution: Do not place your fingers between the handle and the operating lever when loosening the outer sheath. Risk of trapping fingers!

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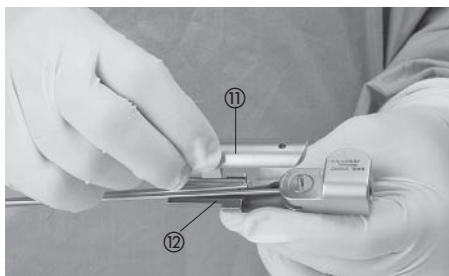
Loosen working insert ④ by turning the marked sleeve ⑭ (unlock) in the direction of the arrow and pull out of the outer sheath distally.

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Pull half-shells ⑪, ⑫ off the working insert.

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Cleaning and sterilization

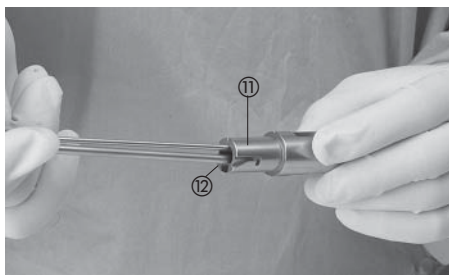
Thoroughly clean the lumen of the sheath with a brush, rinse and dry with compressed air. Rinse out irrigation channel ⑩ and perturbation channel and dry with compressed air. Carefully clean the jaws and surfaces with a disinfected sponge or brush.

The SAWAHLE uterine manipulator can be steam sterilized at temperatures of up to 134°C.

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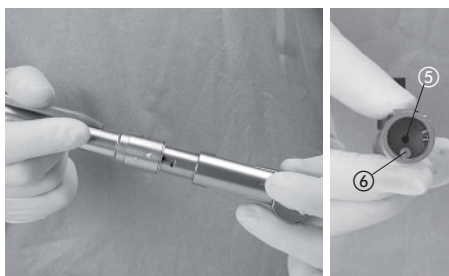
Assembly

Place half-shells ⑪, ⑫ onto the working insert. The pertubation channel and plunger must be inside the half-shells.

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Introduce the working insert ④ oriented with the half-shells into the outer sheath (pertubation channel into the central bore ⑤), plunger into the lower bore ⑥). The working insert must be heard to click into place.

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Prepare the handle for holding the outer sheath: Loosen catch for the operating lever ⑧ by pressing in the locking button (the operating lever must be able to move freely).

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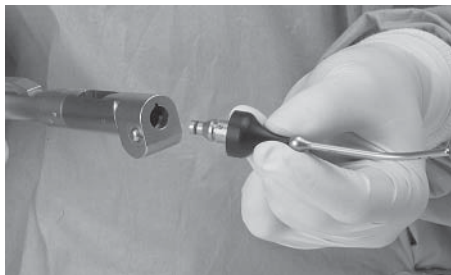
Introduce outer sheath ③ into the handle by pulling the operating lever ⑨ gently against the handle, until the first stop is reached. After this, release the operating lever and introduce the outer sheath until it locks into place. The distal end must now be able to be moved by the operating lever.

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Introduce the intrauterine probe into the distal end of the working insert until it clicks into place. Ensure the correct position of the direction pin.

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C.C.L. vaginal extractors

Turn the knurled screw on the top of the extractor and remove the top.

For disassembly, cleaning and assembly of the extractor top, see Section *Trocars with automatic flap valves, 6 mm - 13 mm*.

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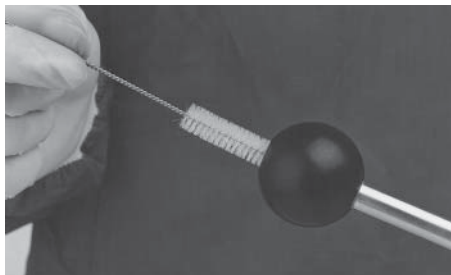
Clean all outer surfaces with a disinfected sponge or brush.

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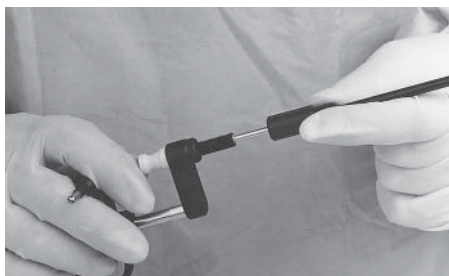
Carefully brush out the lumen. Then rinse out the instrument with water and dry with sterile compressed air.

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Ovarian biopsy forceps 26173 PN

Disassembly

Turn knurled screw on outer sheath and remove outer sheath.

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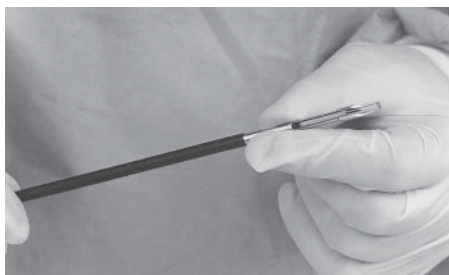
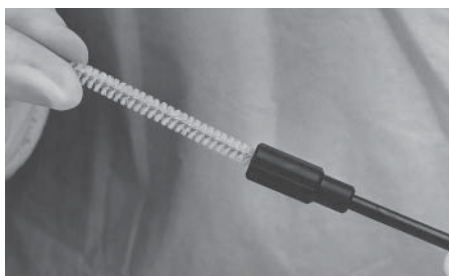
Cleaning

Remove sticky or caked tissue remnants with a cleaning brush. Clean outer surfaces with a disinfected sponge. Rinse out sheath tubes.

Dry with sterile compressed air.

Assembly

To assemble, proceed in reverse order.

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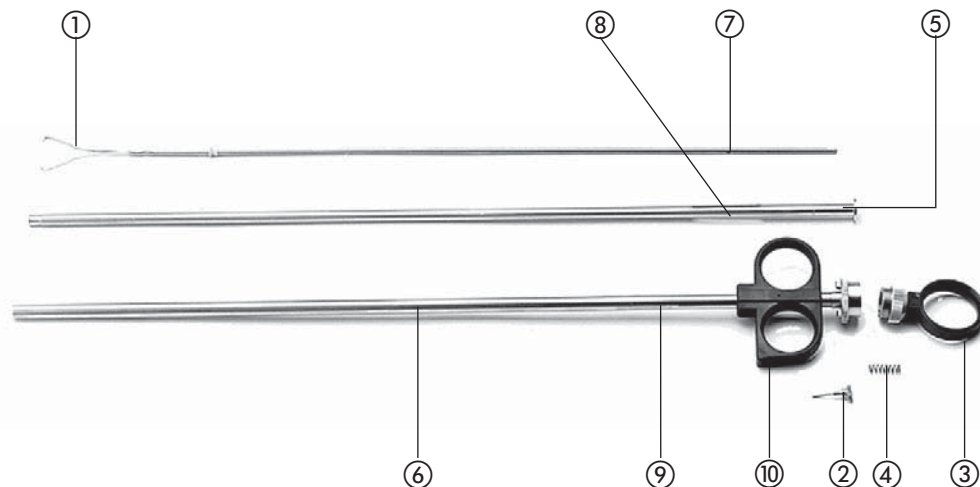
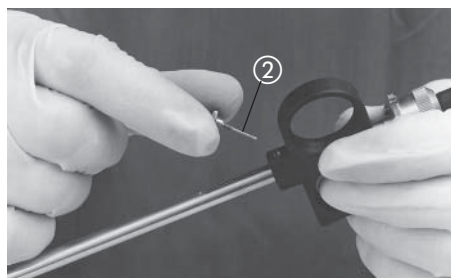
- ① Gripper
- ② Guide screw
- ③ Finger ring
- ④ Spring
- ⑤ Inside tube
- ⑥ Outer tube
- ⑦ Bore
- ⑧ Slot inside tube
- ⑨ Slot outer tube
- ⑩ Grip

- ① Pinza
- ② Tornillo guía
- ③ Anillo para el dedo
- ④ Resorte
- ⑤ Tubo interior
- ⑥ Tubo exterior
- ⑦ Orificio
- ⑧ Ranura - tubo interior
- ⑨ Ranura - tubo exterior
- ⑩ Pieza del mango

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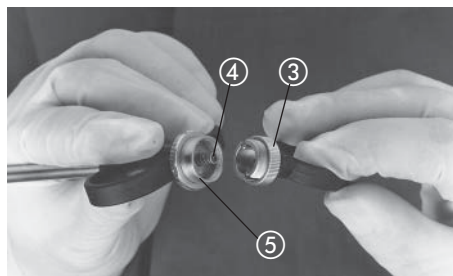
Disassembly

Extend the gripper ①.
Screw out the guide screw ② and pull the
gripper out toward the front.

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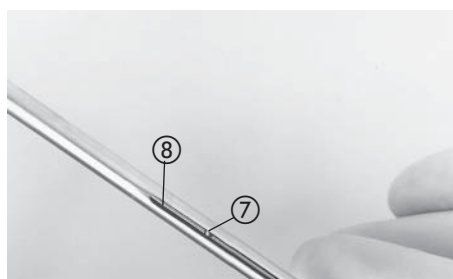
Screw off the finger ring ③ and remove the spring ④. Pull out the inside tube ⑤ toward the back.

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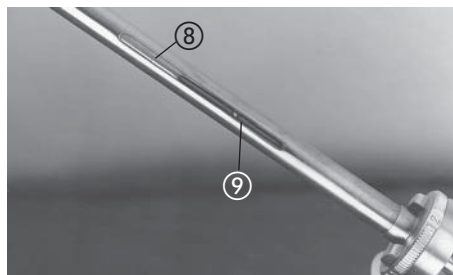
Assembly

Insert the gripper ① into the inside tube ⑤ completely and make certain that the bore ⑦ is aligned with the slot ⑧.

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Slide the inside tube into the outside tube ⑥ completely and make certain that the slots ⑧ and ⑨ are in alignment.

Insert spring ④ and screw on finger ring ③.

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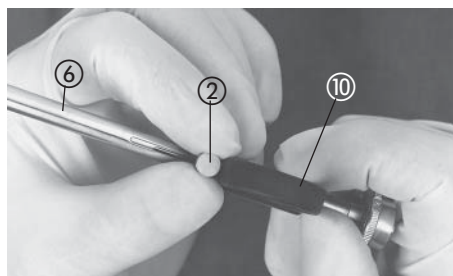
Caution: Carefully insert the gripper ① and inside tube ⑤ without force, so that the seals are not damaged.

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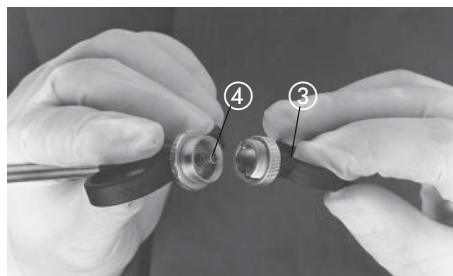
Place the grip ⑩ over the outside tube ⑥ in such a way that the hole for the screw ② is aligned with the bore ⑦.

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Place the spring ④ on the end of the gripper ① and screw on the finger ring ③.

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Loading the plastic rings

Note: The applicator 26174 RA must be fixed in place in the surgical laparoscope to load the rings. The applicator, with the rings loaded, is introduced through the trocar cannula together with the surgical laparoscope.

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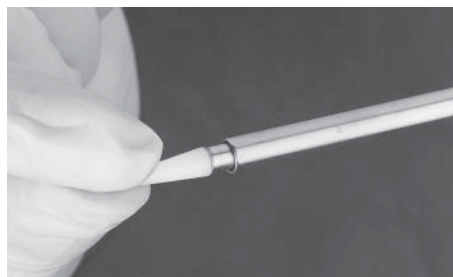
Pull back the double finger grip as far as it will go and slide forward about 5 mm. This blocks the spring mechanism and the rings can be loaded more easily.

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Insert the cone part as far as it will go and place the plastic rings by hand on the cone part.

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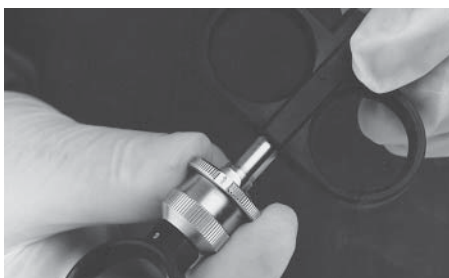
Using the pusher, slide the plastic rings individually, one after another, completely onto the retaining part of the applicator. At the same time, hold the pusher briefly and squeeze together.

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The knurled ring on the grip is engraved with a 1 and a 2. It is necessary to be certain that the knurled ring is always at position 1 to apply the first ring. To place the second ring, the knurled ring must be turned to position 2.

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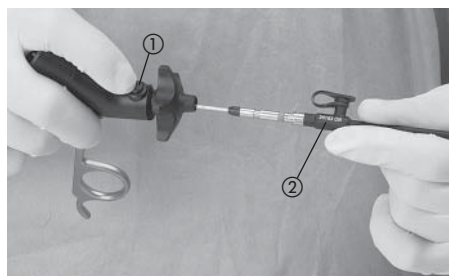
Press and hold the locking button ①. The handle is now opened completely in order to pull the outer sheath ② out towards the front.

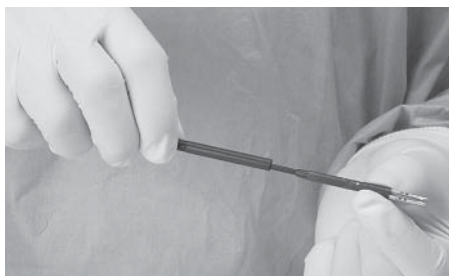
Warning: Grasp pointed or sharp jaws with a cloth.

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Hold the working insert behind the jaws and turn ¼ clockwise. The working insert can now be pulled out distally.

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Cleaning

Note: For heavy soiling, we recommend using an ultrasonic bath for a maximum of five minutes.

Manual cleaning

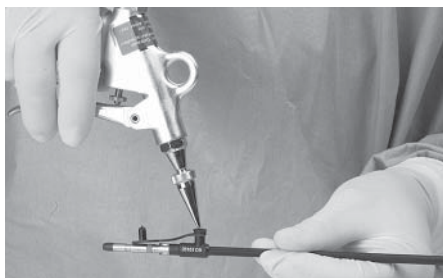
Remove tissue residue with a disinfected cleaning brush. Rinse and dry with sterile compressed air.

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By means of the LUER-lock irrigation port ③, easy attachment to a syringe is also possible.

Machine cleaning

Make certain all parts are positioned securely so that damage during cleaning is prevented. By means of the LUER-lock irrigation port ③, easy attachment to a cleaning machine is also possible.

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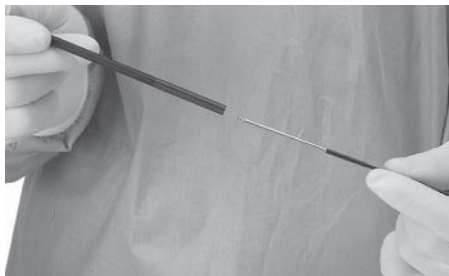
Assembly

Check all parts for damage. Replace if necessary. Check jaws and joints and the insulation on instruments for HF use especially carefully.

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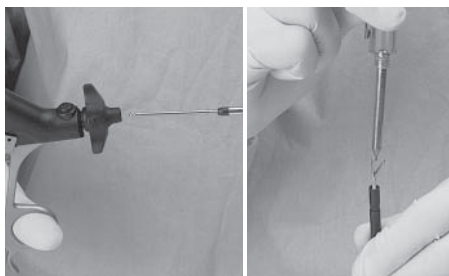
Introduce the working insert from distal into the outer sheath and lock by turning ¼ counter-clockwise.

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Insert the outer sheath, with the jaws closed, as far as it will go into the completely opened handle.

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Then lock the working insert by closing the handle completely.

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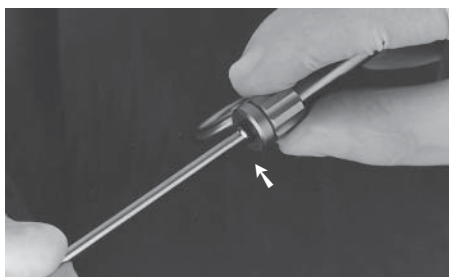
**GYNFORCE III Take-apart punch
26166 SA**

Thoroughly clean the outer surfaces with a disinfected sponge and the jaws with a brush.

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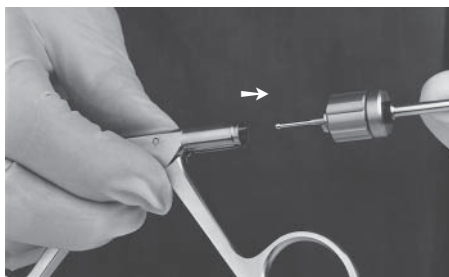
Disassembly

Rotate the knurled nut on the forceps tip in the direction of the arrow and hold tightly in the end position.

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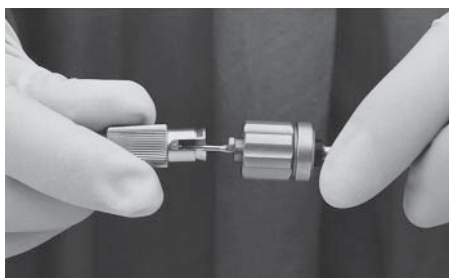
Open the handle completely.
Take off the forceps tip in the direction of the arrow.

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Slide the cleaning adapter 26166 LL onto the forceps tip; it automatically clicks into place. At the same time, make certain that the tab on the forceps tip is aligned with the slot on the cleaning adapter.

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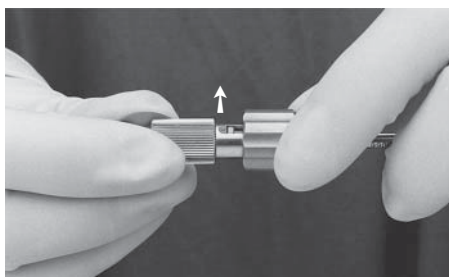
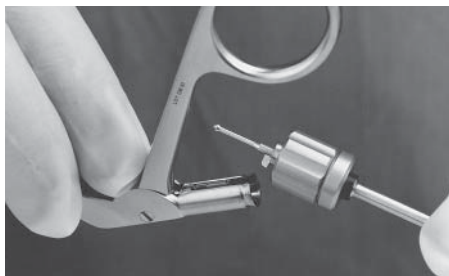
The cleaning adapter is removed by rotating the knurled nut on the forceps tip in the direction of the arrow. Hold the knurled nut in this position and pull off the cleaning adapter.

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Then clean the forceps in an ultrasound bath.
Rinse out with water and dry with sterile compressed air.

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Assembly

Open the handle as far as it will go. The slot for holding the forceps tip must point at 12 o'clock.

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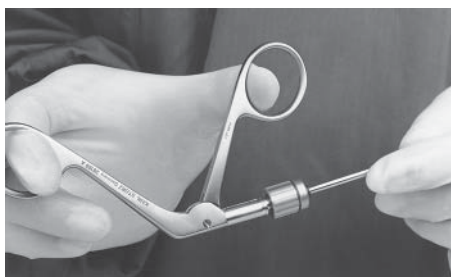
Attach the handle to the forceps tip at 6 o'clock as seen in the figure. At the same time, make absolutely certain that the tab on the forceps tip points to 12 o'clock.

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Align the handle and forceps tip. By closing the handle, the forceps tip is automatically fixed into place.

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CLICK'liné claw forceps and CHARDONNENS morcellation knife
See Section Dissecting and grasping forceps, scissors, punches, biopsy forceps.

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Open the front knurled screw ② on the handle ⑥ and take the handle off of the sheath.

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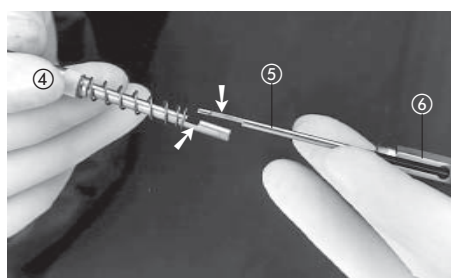
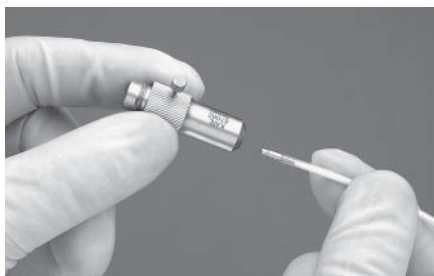
Ureter light probe

Manual cleaning:

Disassemble the knurled screw on the light port and pull out the ureter light probe.

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Carefully wipe the light probes with a cloth soaked in a cleaning/disinfecting solution (no solutions containing alcohol).

Caution: The probe must not have too many kinks in it as the fiberoptic light cables could break.

Then clean the optical ends with an alcohol-soaked cotton swab.

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Machine cleaning:

Caution: The ureter light probe must not be cleaned in an ultrasound bath. This could damage the fiberoptic light cable.

The ureter light probe can be cleaned in the washer/disinfector at up to 65°C.

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Sterilization

Gas or plasma sterilization is possible at a maximum temperature of 65°C.

Caution: The ureter light probe must not be steam sterilized under any circumstances.

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CLICK!liné instruments
 See Section Dissecting and grasping forceps

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Disposal

Standard batteries and rechargeable battery inserts may contain toxic substances so dispose of in accordance with the environmental regulations.

To dispose of the instrument set, no special measures are necessary.

National laws and regulations must be observed.

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Directive compliance

This medical product bears the CEMark in accordance with the Medical Device Directive (MDD) 93/42/EEC. A code number after the CEMark indicates the responsible notified body.

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Limitation of liability

KARL STORZ shall be liable for failure or deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the conditions that all assembly operations, system expansions, readjustments, modifications, or repairs to same have been performed by KARL STORZ, and that the instrument has been used in accordance with its operating instructions at all times.

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KARL STORZ GmbH & Co. KG
Postfach 230
D-78503 Tuttlingen

Warranty

For **two** years as of delivery to the end customer, we agree to replace the goods free of charge if proof can be provided of faulty materials or faulty workmanship. In doing so we cannot accept the cost of transportation or the risk of shipment. The warranty referred to in our Standard Conditions of Business shall apply.

Please fill out the attached warranty card on the last page and return it as soon as possible to:

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Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall release us from any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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Important information

To avoid the spread of infectious illnesses among hospital staff or among KARL STORZ employees, telescopes, instruments and equipment must be cleaned and sterilized/disinfected before they are sent for repair. We reserve the right to return contaminated instruments/equipment to the sender.

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Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ genuine KARL STORZ parts.

Repairs, changes or expansions which are not carried out by KARL STORZ or by experts authorized by KARL STORZ will invalidate all warranty rights.

Repair and replacement program

KARL STORZ maintains a repair and replacement warehouse which is normally adequate to ensure prompt replacement of damaged telescopes and instruments. Under the repair and replacement plan, you receive an identical as-new instrument and are only charged the repair costs for the defective instrument.

For telescopes, you receive a warranty of 1 year, and for instruments 6 months.

For fiberscopes and equipment, individual repair is necessary.

To bridge the repair period, you will receive a unit on loan which you then return to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to

KARL STORZ GmbH & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Straße 34
D-78532 Tuttlingen

In other countries please contact your local KARL STORZ branch or authorized dealer.

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