
Hydro Microdermabrasion

Instruction

HGH-05




Accessories List

1. Host	1pc
2. Vacuum Spa handle	1pc
3. Vacuum Spa work head	2pc (8pc)
4 .Liquid bottles	1pc
5 .Wastewater recycling bottle	1pc
6 .Power supply line	1pcs
7. User manual	1copy
8 .Seal ring of liquid bottles	4pc
9. Hard transparent tube	1pcs

Catalogue

Parts instruction.....	1
Accessories.....	2
General introduction.....	3
Operation.....	3
Effect.....	5
Caution.....	5
Forbidden group.....	6
Technical parameters.....	6
Accessories list.....	7

Parts introduction

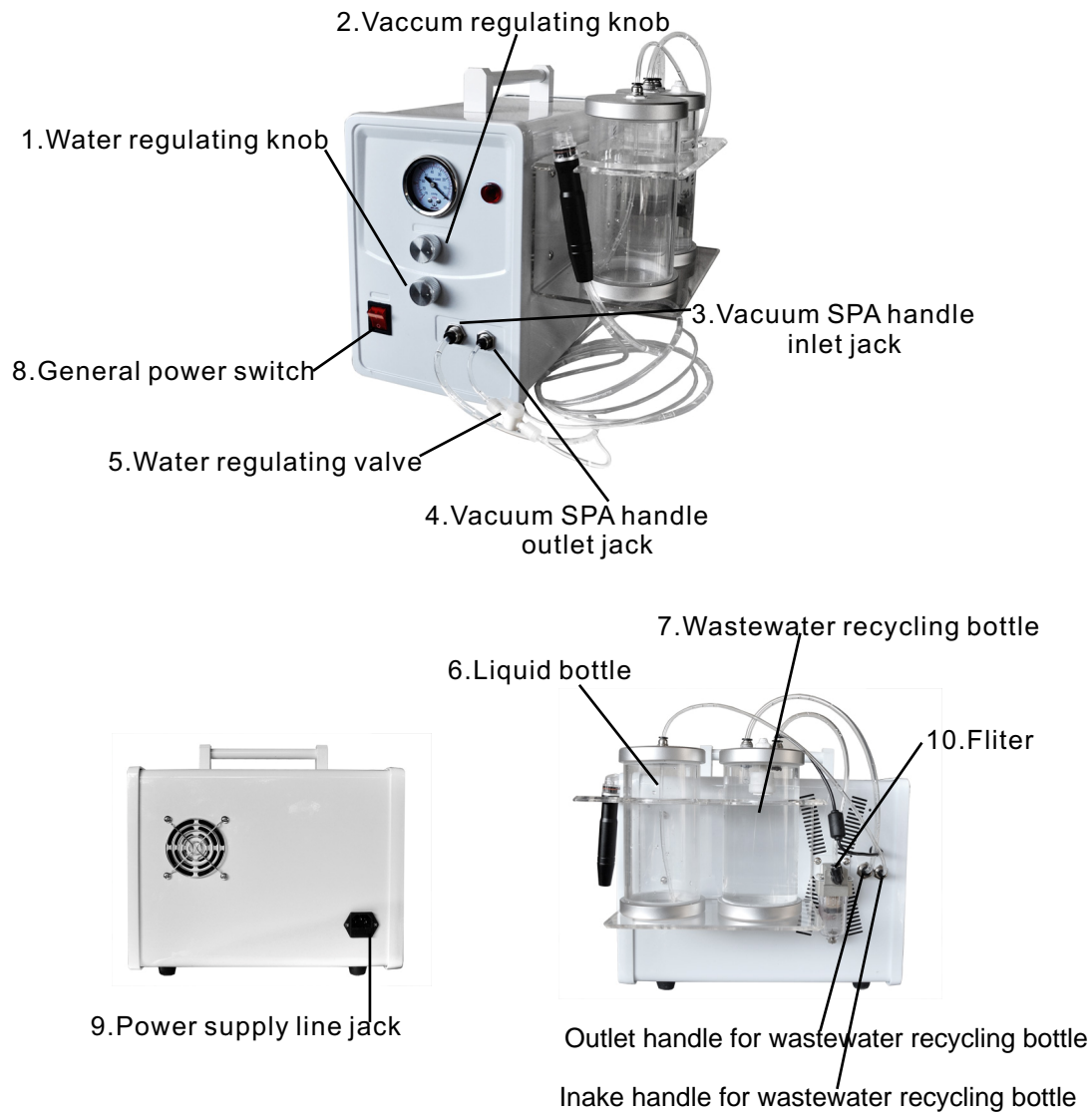

Forbidden Group

1. People who has fever, infectious diseases, acute diseases.
2. People with hemorrhagic disease, trauma, vascular rupture, skin inflammation, skin disease.

Technical Parameter

Input: 220V / 50HZ ☐ 110V / 60HZ ☐
 Power: <150VA
 Vacuum: 5~70Kg/m²

Effect

Dermabrasion: Deep cleaning, clear skin stratum corneum layer, clearing blackhead.

Caution


1. Do not use on wound.
2. Pour out the water in the waste recycling bottle in time, to avoid damage caused by overflowing.
3. Different size of vacuum work head can be flush with water directly, Then soaked with 75% alcohol.

Accessories



Vacuum SPA Handle

Vacuum SPA Heads

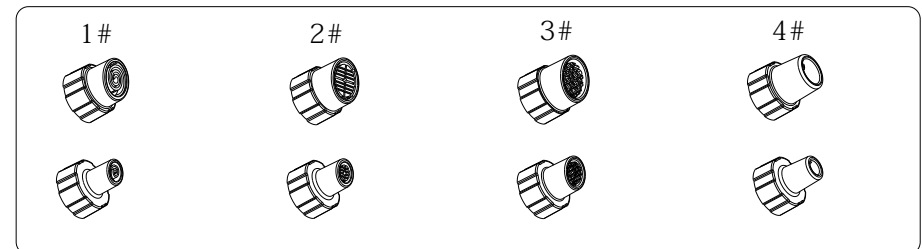

General Introduction

SPA Skin beauty machine completely changed the traditional way, which is hand-cleaning skin relying on individual's practice skills, SPA Skin beauty machine, using vacuum suction mode controlled by intelligent process, through the combination of products and equipment, deep cleaning the skin and pores off horny, acne, blackheads and other impurities in very short period of time. And improve the deep absorption of nutrition products, promote pores tightening, smooth skin, increase skin moisture, and make your skin whitening, moisturizing and good texture.

Operation

1. Connect the accessories to the host and place them well.
 - (1) Insert the inlet transparent tube of the Vacuum SPA handle to [3].
 - (2) Insert the outlet transparent tube of the Vacuum SPA handle (with water regulating valve) to [4].
 - (3) Insert the power supply line to [9].
2. Unscrew the liquid bottle in a clockwise direction, and put appropriate liquid into the bottle.
3. Fasten the liquid bottle in a anticlockwise direction, notice the sealing ring.
4. Choose the vacuum work head, and connect to the vacuum spa handle.
 - 1# work head for clearing comedones, Whitehead and acne.
 - 2# work head for deep cleaning of normal skin, uneven color and dark skin, dry aging skin.
 - 3# work head for thick cuticle, coarse pores, oil over secretion skin.
5. Turn on the general power switch [8].

6. Adjust the vacuum regulating knob [2] (left-small, right-big) to the appropriate vacuum.
7. Adjust the water regulating knob [1] (left-small, right-big) to the appropriate amount of water.
8. Beautician hold the vacuum spa handle on skin for a while until the water flow into the work head, and then drop some water on skin by points, to make the skin surface wet, and then apply cleansing product on the skin, and move the vacuum work head on skin back and forth.
9. Adjust the water regulating valve [5], the instruction as follows:



Opened



Closed

10. Check the Wastewater recycling bottle [7] before turn on the Machine, if the bottle is full, please clean it.

Cleaning Method:

Unscrew the wastewater recycling bottle in a clockwise direction, cleaning the wastewater, then fasten it in a Anticlockwise direction.