

**SAVE THESE INSTRUCTIONS**

*NOTE: Check all parts for shipping damage. In case of damage, DO NOT use. Contact Dealer/Carrier for further instructions.*

**SAFETY SUMMARY**

To ensure the safe operation of the ROLLATOR BASKET, these instructions **MUST** be followed:

**GENERAL WARNINGS**

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

DO NOT carry or place weight exceeding 10 pounds in the rollator basket when attached to rollator.

NEVER use the rollator basket as a means of support.

The seat must be in the DOWN position when using the rollator for ambulatory assistance.

When using the rollator in a stationary position, the hand brakes **MUST** be locked.


Always test to see that the rollator and attachments are properly and securely locked in place before using.

**INSTALLING THE BASKET (FIGURE 1)**

*NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY in this instruction sheet.*

1. Place the seat in the UP position.
2. Install basket between the side frames and the seat supports so the hooks are secured onto the seat supports as shown in FIGURE 1.
3. Grasp the handle and lower the seat, pushing DOWN until the seat supports fully engage the side braces.

*NOTE: it is necessary to lift the seat in order to place items into or take items out of the basket.*

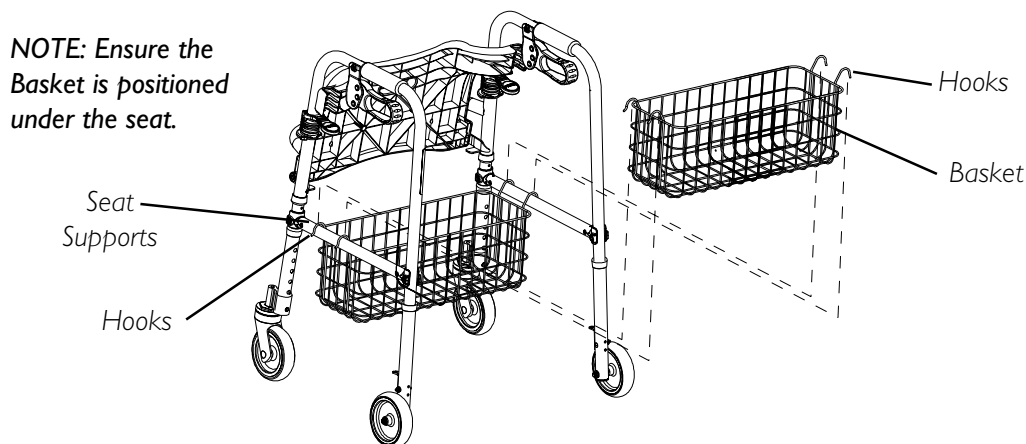

**FIGURE 1 - INSTALLING THE BASKET**

---


## STORING THE BASKET (FIGURE 2)

NOTE: Refer to INSTALLATION WARNINGS in the SAFETY SUMMARY in this instruction sheet.

NOTE: If desired the basket may be stored on the front of the rollator when the rollator is in the folded position.

1. With the rollator in the closed position, and holding the basket at an angle to the rollator, align two (2) of the hooks with the front brace as shown in DETAIL "A" of FIGURE 2.
2. Secure the two (2) hooks onto the front brace and lower the bottom end of the basket until it rests against the rollator as shown in DETAIL "B" of FIGURE 2.

NOTE: Reverse steps 1-2 to remove the basket from the stored position.

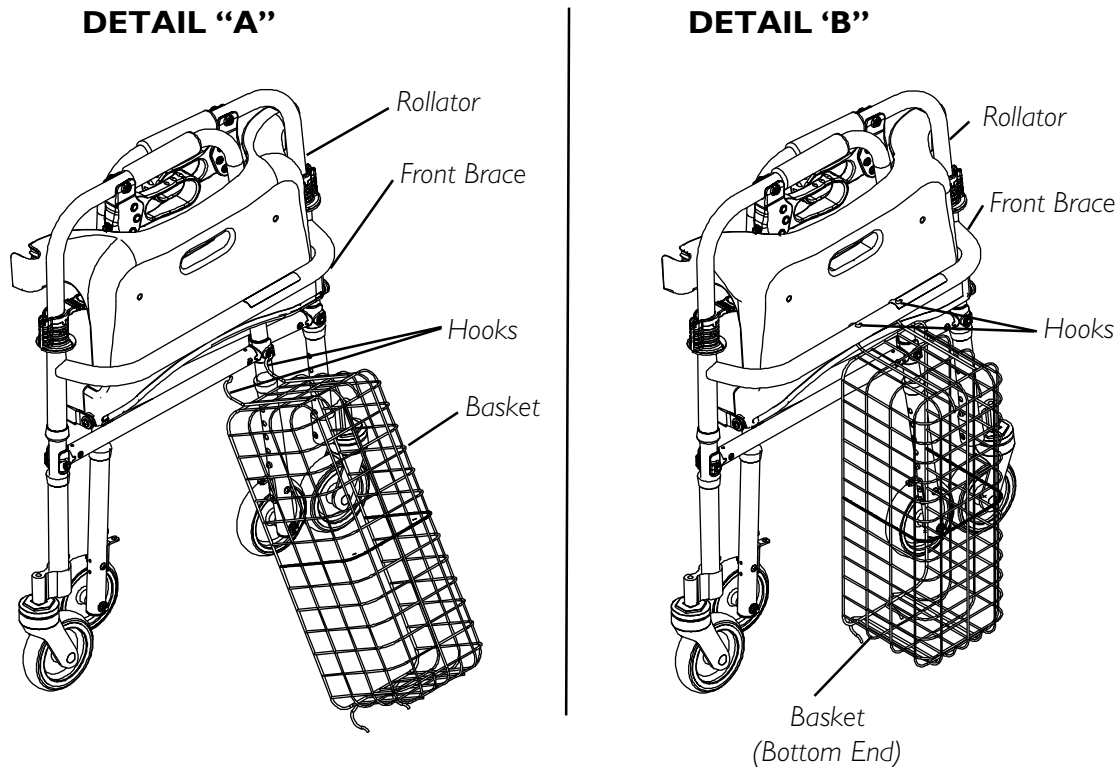

NOTE: Back rest not shown for clarity.

**FIGURE 2 - STORING THE BASKET**

---

## **LIMITED WARRANTY**

**PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.**

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for three (3) years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address on the back page. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

**LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.**

**THE FOREGOING WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE.**

**IT SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESSED WARRANTY PROVIDED HEREIN AND THE REMEDY FOR VIOLATIONS OF ANY IMPLIED WARRANTY SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

**THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.**



**Yes, you can.™**

**Invacare Corporation**    [www.invacare.com](http://www.invacare.com)

---

**USA**  
One Invacare Way  
Elyria, Ohio USA  
44036-2125  
800-333-6900

Invacare is a registered trademark of  
Invacare Corporation.  
Yes, you can. is a trademark of Invacare  
Corporation.  
© 2001 Invacare Corporation  
Form No. 01-234  
Part No. 1108904    Rev A (1) 9/01