

rev. 02/2010

ORTHO P.A.S.

Quick User's Guideline



www.eurosets.it

Innovation and Technology for Better Patient's Outcome

This document is not intended to replace the ORTHO P.A.S. user's manual, it is for quick reference only. For complete ORTHO P.A.S. set up and procedure please refer to its User's manual.


SET UP

- 

Transfer Ortho P.A.S. to the sterile field.

Remove the green pouch. Keep the transparent pouch
- 

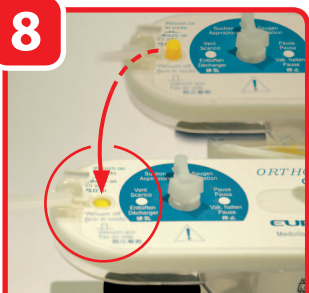
cut the fingers connector according to the drainage size in use
- 

Connect the drainage to the fingers connector
- 

Move Ortho P.A.S. out of the sterile field. **Remove the transparent pouch. Close the clamp.**
- 

Connect the vacuum pump: holding the black ring down slide the pump up
- 

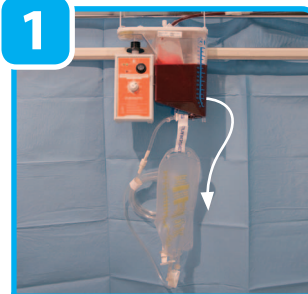
write down the blood collection start date and time
- 


Turn the pump on and adjust the vacuum
- 

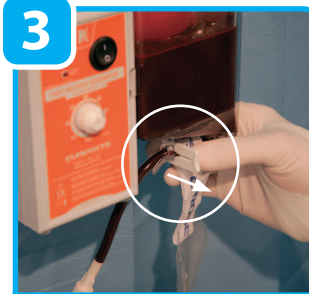
The yellow indicator disappears (vacuum ON)
- 

Hang Ortho P.A.S. to the bed. System working


REINFUSION


- 

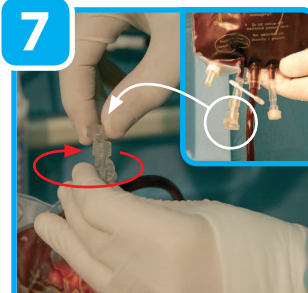
Slide the bag out from the pouch at the back of the reservoir
- 

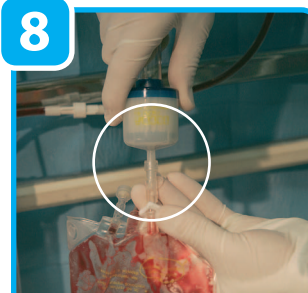
Turn the lever to the left ('vent' position). The yellow indicator becomes visible
- 

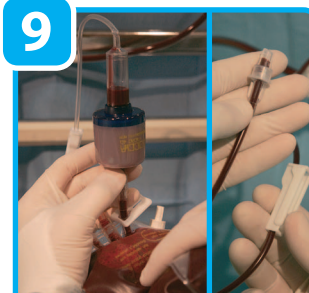
Open the clamp
- 

The blood moves into the bag. 50 ml (supernatant and fat) remain in the reservoir
- 

Close the clamp
- 

Turn the lever back to 'suction' position. Yellow indicator disappears
- 

Take the bag and open the connector
- 

Insert the Goccia 40µm filter
- 

Prime the Goccia 40µm filter and the reinfusion line. Reinfuse the blood*

In case of **multiple reinfusion**, repeat the complete sequence **without disconnecting** the Goccia filter, neither from the bag, nor from the Patient

*The blood should be reinfused according to your Hospital's Protocol