

Hospital	_____
Department	_____
Clinician	_____
Contact information	_____

## Provox® LaryButton™ When and How to Use

The Provox LaryButton sits inside the stoma by itself and prevents the tracheostoma from shrinking. It is used to hold the Provox HME's, Provox ShowerAid and the Provox FreeHands HME®.




### 1. Sizing of the Provox LaryButton:

- Your ability to use the LaryButton depends on the size and shape of your stoma. It should fit well into your stoma without causing irritation. Your clinician needs to size you using a Provox LaryButton Sizing Kit.

### 2. Inserting the Provox LaryButton:

- Lightly lubricate the small end of the LaryButton that fits into your stoma. Use only a water soluble lubricant.
- Fold the narrow end of the LaryButton in half with your fingers or hemostat and hold it firmly folded.
- Insert the narrow end into your tracheostoma and release your fingers.
- Gently tug on the LaryButton tabs to insure the LaryButton is seated in the correct position.
- Confirm that the LaryButton is in the correct position and is not pushing on the voice prosthesis.
- The LaryButton can be secured with the Provox LaryClip™ or a Provox TubeHolder.
- Insert a Provox Heat and Moisture Exchanger (HME) or Provox FreeHands HME into the opening of the LaryButton.
- Determine the wear schedule with your clinician.



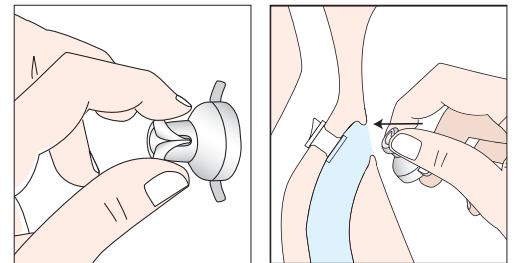


**TIP:** A wear schedule will gradually build up tolerance to the Provox LaryButton preventing stomal irritation.

Hospital	_____
Department	_____
Clinician	_____
Contact information	_____

## Provox® LaryButton™ When and How to Use

3. Removing the Provox HME from the Provox LaryButton:
  - Hold Provox LaryButton in place with one hand.
  - Hold the Provox HME and gently remove the HME cassette while keeping LaryButton in place.
4. Removing the Provox LaryButton:
  - Hold the Provox LaryButton by the sides and gently squeeze to allow the narrow end to collapse.
  - Remove the LaryButton by tilting it out of the tracheostoma.
5. Cleaning the LaryButton:
  - Rinse with tap water and clean with the Provox TubeBrush.

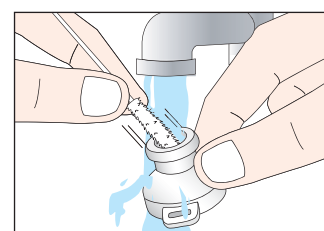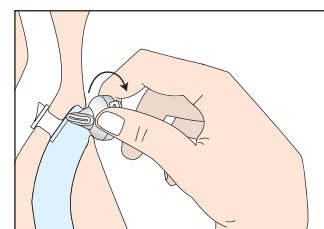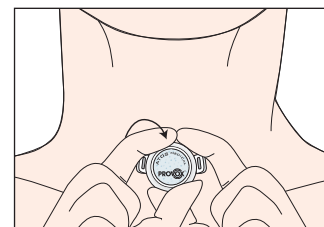

Provox® ShowerAid™, Provox® LaryClip™ and Provox® LaryButton™ are trademarks owned by Atos Medical AB, Sweden. Provox® and Provox® FreeHands HME® are registered trademarks owned by Atos Medical AB, Sweden. Provox® HME, Provox® FreeHands HME® and Provox LaryClip are protected by patents, patents pending and design patents and design patents pending.

This text does not replace nor does it set forth the complete contents of the User Manual and/or Prescriber Information for the products in this text, and is not a substitute for reviewing and understanding that important information. Therefore, before prescribing and/or using any of the products included in this text, please review the entire contents of the respective User Manual and/or Prescriber Information.  
For more information contact us at: [info.us@atosmedical.com](mailto:info.us@atosmedical.com) • [www.atosmedical.us](http://www.atosmedical.us) • 1.800.217.0025

