

Shower chair 302015

User Manual



MOBILEX A/S
Noerskovvej 1
8660 Skanderborg
Denmark
www.mobilex-care.com

IMPORTANT!

Before you use the shower chair, please read this user manual carefully. Pay attention to all instructions, especially those regarding safety, and follow them at all time.

By queries please contact your dealer or Mobilex A/S direct:

Mobilex A/S
Noerskovvej 1
8660 Skanderborg
Denmark

Telephone: +45 87932220
Telefax: +45 87931777
E-Mail: info@mobilex-care.com

1. General instructions

Introduction

This operating instruction contains information and instructions regarding the safe and proper use of your shower chair.

Proper use

The shower chair is designed only as a seating option for bathroom and showers and can also be used as a toilet. Any other use is prohibited. Don't use this chair as a climbing aid.

Guarantee terms

The producer offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

CE marking

The product complies with EU Directive 93/42/EEC for medical equipment.

Uses symbols

The following symbols are used in these user manual:



Warning - This symbol is used for security references, that applies to points that can lead to risk of damages is not followed.



Reference – This symbol applies to reference as to how the product should be used.

2. Security–General references



- Only use the shower chair in good technical condition.
- If you detect any crack or damages, please contact your dealer immediately.
- Observe the information on the label. Don't overload the shower chair.
- Only use the shower chair for the purpose it is intended for.
- Don't climb on to the shower chair.
- The shower chair must only be used on stable surface.
- When using the shower chair the user must take care not to squeeze the fingers or other body parts

3. Description

Product description

The Shower Chair comes with all the standard features of the regular shower chair and is specifically designed for easy carrying and storage. It is designed for individuals who require seating support when bathing or showering. Made from powder coated steel tubing they are warm to the touch and corrosion resistant. No tools are required for assembly.

Technical data

| | |
|-------------------------|--------------|
| Overall height: | 900 – 950 mm |
| Seat width: | 480 mm |
| Seat length: | 860 mm |
| Seat height: | 880 mm |
| Weight of chair: | 15,9 kg |
| Maximum weight of user: | 130 kg |

Product label

The product label is placed on the back of the backrest.



4. Usage description

Using the breaks



- You should always engage the breaks when you don't plan to move it.
- Never leave chair without breaks engaged when there is someone sitting in it!
- Never leave chair on slopes without breaks engaged!



- To engage the breaks step down with your toes on the pedals attached to the rear wheels until they are locked (picture 1).
- To disengage the breaks pull up with your toes the pedals attached to the rear wheels until they will unlock (picture 2).



Picture 1



Picture 2

Armrest elevating



- This function helps to transfer the user in to and out off the chair.
- It also helps to wash all the parts of user's body during the shower
- Never leave chair unattended with armrests lifted when there is someone in it.



- To elevate the armrest turn the lock handle, so it is directed up and pull up the armrest (picture 3).
- To lock the armrest back push it down, so the end of the tube will slide in to the plastic locking bracket and turn the lock handle, so it is directed down (picture 4).



Picture 3



Picture 4

Footrest attachment and detachment



- This function together with elevatable footplates helps to transfer the user in to and out off the chair.
- It also helps to wash all the parts of user's body during the shower
- Never move the chair with footrests detached when there is someone in it. Always put user's feet on foot plates when moving the chair



- To unlock and detach the footrest push with your thumb on the lock lever and pull the footrest to the outer side of the chair (Picture 5).
- Grab the top of the unlocked footrest and pull it up, off the hole in the end of the frame tube. (Picture 6).
- To put the footrest back on put it back in to the hole in the end of the frame tube and turn it to the inner side of the chair until it locks it self.



Picture 5



Picture 6

Adjusting the footrests to desired length



- The length of the shower chair's footrests should be adjusted to suit the user.
- As a rule, the footrests' length height is correctly adjusted when the user is sitting on the chair and upper legs' parts fully touch the seat and both feet fully touch the footplates.



- Take the footrest out of the chair
- Loosen the locking bolt and take it off (picture 7).
- Adjust the footrest to the desired length by pulling or pushing the inner tube inside the outer one (picture 8).
- Align holes in the inner and outer tube.
- Put the locking bolt back on and tighten it (Picture 9).



Picture 7



Picture 8



Picture 9

Using the excrements' bucket



- Excrements bucket can be used by users who have difficulties to use regular toilets.
- Never leave bucket not empty or unwashed.



- For the bucket to be used take the middle part of the seat off (picture 10).
- Take off the lid of the bucket (picture 11)
- When finished put the lid back on the bucket and pull the bucket out of the chair (picture 12).

- Empty the bucket, wash it and slide it back under the chair's seat
- Put the middle part of the seat back on the chair (picture 13).



Picture 10



Picture 11



Picture 12



Picture 13

Care and maintenance

Cleaning

Regularly clean the shower chair and keep it clean. Use renewal purpose neutral detergent and warm water. Don't use harsh abrasives or bleach based cleaners with may cause damage or discolouration. Dry the parts with a cloth.

Maintenance

The shower chair is maintenance free.

Deposit of the product

The shower chair cannot be deposited with normal garbage. It must be delivered to the public station for re-use.