



USER MANUAL

ZT-0500H

HEAVY DUTY COMMODORE

FEATURES

Heavy duty commode, with swing away arm, removable backrest, Lift up seat and lid with removable pail.

Height adj. 43cm/17" - 54cm/21".

Maximum user weight 127kg/22 stone.



ASSEMBLY

1. Take all parts out of the packaging.
2. Lift off the pail from the base.
3. Undo the bolts on the rear of the base (Photo 1).
4. Place the seat and lid onto the frame, place the bolts through the chrome seat fitting and the base frame.
5. Tighten the bolts up with the clear plastic nuts.
6. Swing the arms up until they lock in position by pulling out the spring loaded plunger and letting it return to the locking position.
7. Replace pail into the base frame.
8. Fit the backrest by inserting into the base frame, to locate into the locking holes press the spring loaded plunger in and slide the backrest in until you hear it click into place.
9. To adjust the height of the commode each leg has to be adjusted this is done by pressing in the spring loaded plunger and sliding the leg downwards until the desired height is met, make sure that the spring plunger is located in one of the height adjustment holes each leg has to be at the same height for safety.
10. Your commode is now ready for use.



PHOTO 1

MAINTENANCE

1. Periodically check that the feet are serviceable.
2. Check that the seat and lid bolts are tight.
3. Check the backrest is secure.
4. Make sure that the armrests are locked into position and serviceable.

WARNING

1. Do not use as a step.

Please ensure that this leaflet is read and understood before using your Commode

Z-Tec Mobility, Unit 1B, Albion Works, Moor Street, Brierley Hill, West Midlands, DY5 3SZ

(April.07/01)