

"Alligator" Ramps

LR/TR- Ramps

User's manual



MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark
www.mobilex.dk

1. General information

Dear customer,

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these.

We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The Mobilex Alligator Ramps are medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (f.e. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside.

Any other use is prohibited

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S

Grønlandsvej 5
DK-8660 Skanderborg
Denmark

Phone: +45 87 93 22 20
Telefax: +45 87 93 17 77
E-Mail: info@mobilex.dk

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Ramps must not be loaded with more than the max load described in Technical Data and on the label.
- The ramp can be used at maximum inclination of:
LR/TR-150: 30 cm
LR/TR-210: 40 cm
TR-300: 50 cm
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- When unfolding the ramp the user must take care not to squeeze the fingers or other body parts.
- The ramp must only be used on a stable surface.


3. Description

Product description

Mobilex LR/TR-Ramp set consists of 2 separate rails made of aluminium. The surface is covered with special anti slipping material.

Ramps are supplied in cartons. The contents are:

- ✓ 1 set, consisting of 2 pieces of Alligator-Ramp Type TR/LR,
- ✓ 1 User manual

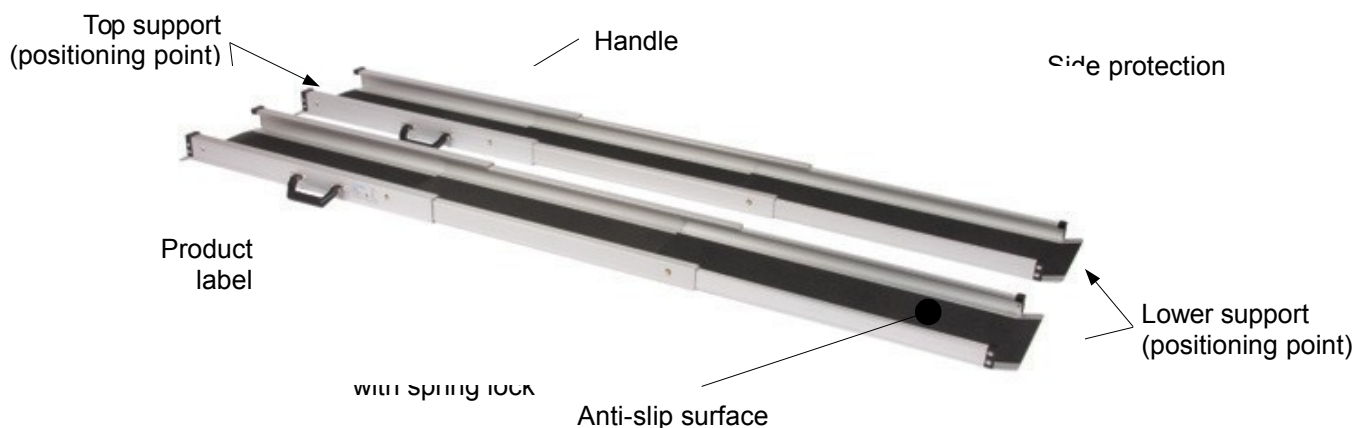

Set Up & Function description

- The ramps must be locked in “open” position with springs.
- The ramps must be placed on a stable surface.
- Ramp halves must be always placed in parallel position.
- Check that the ramps are stable before driving on them.

Transport recommendations

Alligator ramps type TR can be folded (by sliding function) for transport and locked. Always assure that ramp during transportation is safely secured against unintended movements.

Accessories

There are no accessories provided for Alligator Ramps.

4. Technical data

„Alligator“ Ramps LR, TR type	TR-150	TR-210	TR-300	LR-150	LR-210
Length	2 x 150 cm	2 x 210 cm	2 x 300 cm	2 x 150 cm	2 x 210 cm
Width	2 x 26cm	2 x 26 cm	2 x 26 cm	2 x 16 cm	2 x 16 cm
Surface width	2 x 19 cm	2 x 19 cm	2 x 19 cm	2 x 11 cm	2 x 11 cm
Height of side protections	Outer & Inner: 5 cm	Outer & Inner: 5 cm	Outer & Inner: 5 cm	Outer & Inner: 3 cm	Outer & Inner: 3 cm
Max. elevation / Height difference	12° / 30 cm	12° / 40 cm	12° / 60 cm	12° / 30 cm	12° / 40 cm
Total weight	2 x 5,5 kg	2 x 7,5 kg	2 x 9,5 kg	2 x 3 kg	2 x 4 kg
Max. load	225 kg	225 kg	225 kg	180 kg	180 kg
Remarks	Used as a pair.	Used as a pair.	Used as a pair.	Used as a pair.	Used as a pair.

Product label

The product label is located on one of the ramp sides

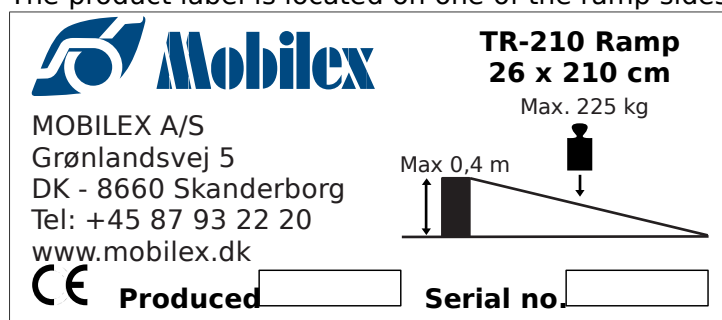


Image of the product label (not original)

5. Maintenance & Cleaning

Maintenance

Ramps are basically maintenance free.

Although it is recommended to check always before use if the anti-slip surface isn't dirty or torn. In case of any doubts concerning technical condition of your ramp stop using it and contact your dealer. Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

Cleaning

- Lighter dirt and dust can be removed with a wet cloth.
- Heavier types of dirt can be removed with a non-corroding, mild washing agent.
- You can also use garden hose to clean the ramp.

6. Most common problems and solutions

<i>Symptoms</i>	<i>possible cause</i>	<i>what to do?</i>
Ramp doesn't open	The spring lock was not unlocked	Unlock springs and open the ramp
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it on flat, smooth surface
Ramp is not stable	Ramp is putted on the surface upside down	Check if the A label is in the upper end and B label in the lower end of the ramp. Correct position if needed.

7. Disposal of the product

The product can not be disposed of with waste but must be brought to the local recycling centre.

8. Guarantee terms

Mobilex A/S offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

9. Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark declares that the product

Mobilex Alligator Ramp - TR and LR series

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment
Annex I of the June 14, 1993**

Lars Nygaard,
Managing Director
Mobilex A/S

Skanderborg, the 03.02.2005


.....
signature