



**ZT-9266**  
**‘L-Type**  
**Rollator**

**USER MANUAL**



**Available in : Shining Silver & Dark Grey**

Item No.	Width inside handgrips	Overall width	Overall depth	Seat width	Seat depth	Seat height	Adjustable handle height	Folding dimension (W'D'H)	Weight capacity	N.W.
<b>ZT- 9266</b>	45.2cm	62. 8cm	68.2cm	38cm	29cm	57.2cm	82.4-95cm	62.8×31.5×75	136kg/300lbs	7.2kg

**Please ensure that this manual is read and understood before using the rollator.**

## Contents

- |  |                                    |
|--|------------------------------------|
| 1. Introduction                        | 6. Transportation                  |
| 2. Parts Description                   | 7. Maintenance                     |
| 3. Personal Safety                     | 8. Warranty                        |
| 4. Assembling the Rollator             | 9. Weights/Dimensions (front page) |
| 5. Brake Operation & Brake Maintenance |                                    |

### 1. Introduction

The ZT-9266 is a high quality four-wheel rollator available in shining silver & grey painted colours. The rollators are designed to aid the user to walk and, as an option, carry items such as shopping.

The rollators are suitable for everyday indoor & outdoor use. They are lightweight & portable whilst offering stability & security to the user.

The ZT-9266 rollators are manufactured to the highest quality standards & are CE marked as Medical Devices.

### 2. Parts Description




### 3. Personal Safety

- Ensure that the handles are at equal heights & securely locked in position.
- Adopt a comfortable posture so that you can operate the braking mechanism with ease.
- Ensure the rollator is always fully opened out & locked before using.
- Ensure that the braking mechanism is adequate & test before using.
- Ensure that all the wheels are stable & are on the ground before using.
- Ensure that the brakes are always in the locked position before sitting on the seat.
- Ensure that the user sits in the centre of the rollator seat.
- Do not use the rollator on gradients of 7° / 1 in 8 or above
- Ensure that the latest health & safety practice is used for carrying & lifting items.


## 4. Assembling the Rollator

1. Remove all packaging from the rollator.

2. Fold out the rollator by first releasing the Red Locking Tab (a), then lay the rollator down. Next lift up the front section of the rollator until the underseat locking mechanism is engaged (c). Fold down the seat (d) and set the front castor wheels to suit by flipping the Red Tab. Free Wheel (e) or Fixed (f).

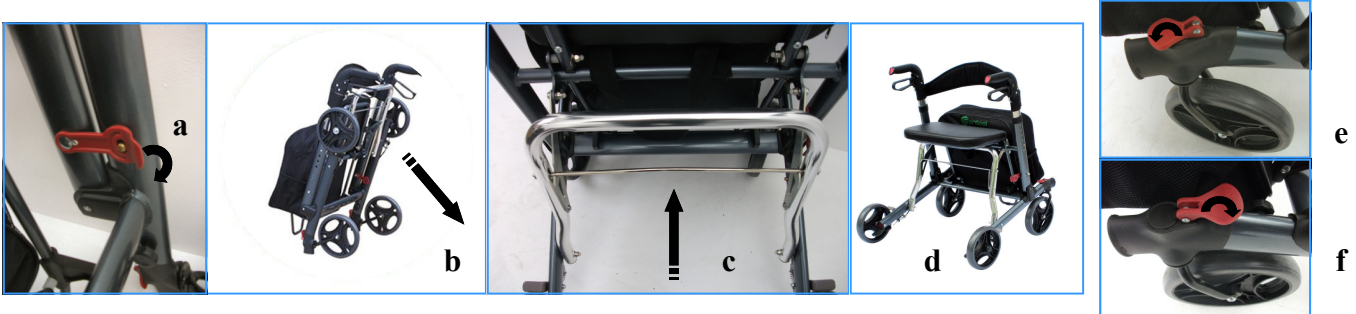

3. To re-adjust height of handles, depress the locking button (g), then re-adjust to desired height. Ensure both handles are set to same height before using rollator.

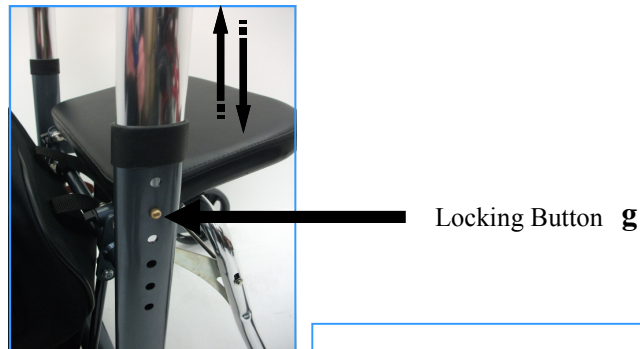

4. Your rollator should now look like (h) and is ready for use once you have read the rest of this manual.



## 5. Brake Operation & Brake Maintenance


NOTE: The brakes are factory set & may not require initial adjustment. For technical assistance on the brakes, contact your local Z-Tec dealer.



To operate the locking loop brakes follow points 1 to 3.


1) Brake in walking position.

2) Squeeze both brake levers up at the same time to engage the brakes.

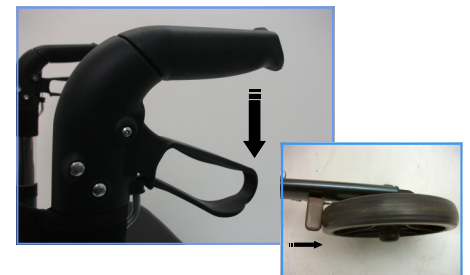

3) To lock the brakes push down both brake levers at the same time until they lock. To unlock pull up brake levers until the lock disengages.

To adjust the braking system release the locking nut & screw out the cable adjuster, located under the frame near the rear wheel, until the brake arm clears the wheel by approx. 1mm. Retighten locking nut & repeat on other brake mechanism. With both wheels off the ground & the brakes disengaged check that they spin freely & do not rub the brake pad.

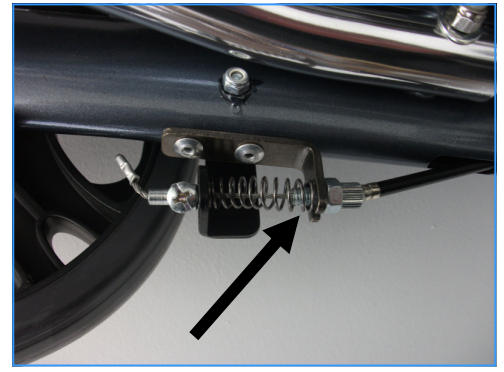

## 6. Transportation

To fold the rollator for transportation, pull up the release bar under the seat. Take care with the shopping bag and lift away if required while folding. When folded secure with Red Locking Tab.



## 7. Maintenance

- Ensure the rollator is used only as an aid for mobility
- Ensure the rollator is kept clean. Wipe down with a damp cloth on a monthly basis.
- Check the moving parts (e.g. wheels & height adjustment) monthly.
- Ensure that the brakes are working correctly.
- All the fasteners (screws, nuts & bolts) are to be checked monthly to ensure they are securely tightened.
- Check that the front castor wheels swivel freely. Withdraw from use if they do not work.
- Only use an approved Z-Tec dealer for any maintenance & repairs on your 4 wheel rollator.

## 8. Warranty

To improve the safety & lifespan of your rollator, it is recommended that you have it serviced on an annual basis by an authorised dealer. Z-Tec rollators are guaranteed for 12 months from the date of purchase, this warranty is not transferable. Should any part of the rollator require repair or full or part replacement, as a result of a manufacturing or material defect within the warranty period, parts will be supplied free of charge. The guarantee excludes tyres & all items that have been subjected to undue wear & misuse. Under normal circumstances, no responsibility will be accepted where the rollator has not been maintained in accordance with the manufactures recommendations. This does not affect your statutory rights.

**Disposal**—Contact your local authority to dispose of the rollator when no longer required.

## Z-Tec Mobility

Unit 1B, Albion Works, Moor Street, Brierley Hill, West Midlands, DY5 3SZ