

Instruction card

Elexo

Safety Instructions

- Read the user manual carefully before use.
- Maximum load: 180 kg.
- Ensure that the brakes are applied to the chair during patient transfers to and from the chair.
- Armrests closed during height adjustment.

1a Operation

Up




Down



Activate




2 seconds

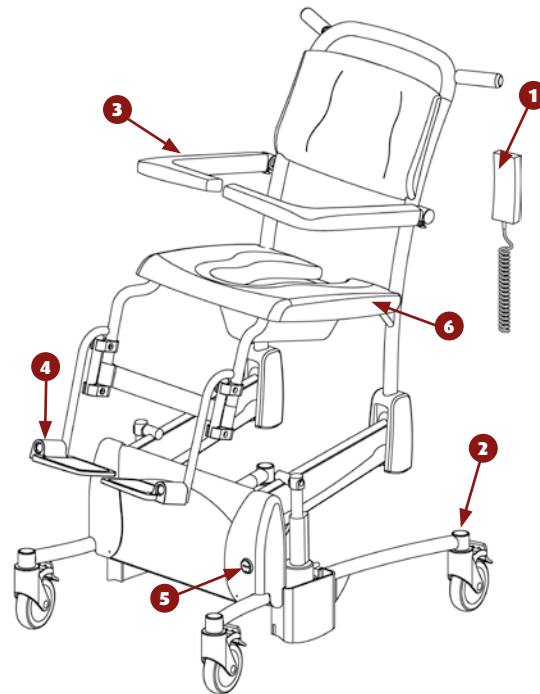


1b Use

1. Activate the Elexo by holding down the 'up' button for 2 seconds.
Caution: After use, the Elexo will go into stand-by mode within 60 minutes.
2. Set the chair to the correct position.
3. Apply brakes on the chair!
4. Open the armrests and, if necessary, fold up.
5. Support patient when they sit down.
6. Close the armrests.
Getting out of chair occurs in reverse order

2 Brakes

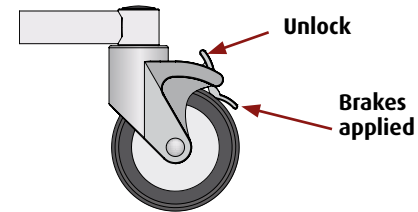


3 Armrests

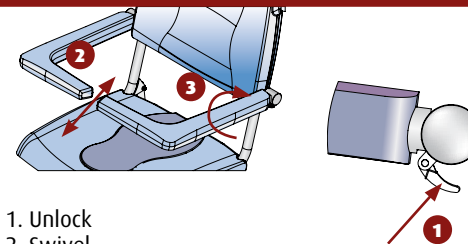

1. Unlock
 2. Swivel
 3. Fold up
- Close in reverse order.

4 Foldable foot supports

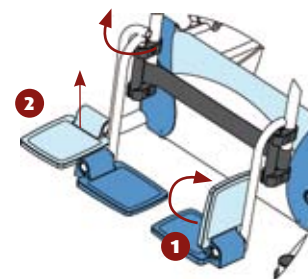

Both foot supports need to be folded away when the patient sits down on the chair. This can be done by folding up the footplates (1) or folding up and swivelling the complete support outwards (2).
Ensure that the patient has both feet on the foot supports when manoeuvring.

5 Charging

After use, attach chair to the charger for upkeep of the batteries.



Off	Stand-by
Green	Chair operational
Orange	Battery almost empty
Red flashing	Battery empty, charge soon
Red	Battery empty, charge immediately

The chair cannot be operated during recharging.



6 Bucket or bedpan holder

When using a bucket or bedpan, the seat cover needs to be removed and the bucket or bedpan inserted from behind, under the seat, in the special holder before the user sits down on the chair.

Cleaning

After use each time, the Elexo must be cleaned with hot water and a normal household detergent. Do not use a scourer or any other aggressive solvent. Check the wheels regularly for any hairs, dirt and remains of soap and remove if necessary.

In the event of defects, please contact your dealer.

Important!

This instruction card should not be seen as a replacement for the user manual.