



Access Your World

Lopital Marina Shower Gurney / Bath Trolley

Battery-Powered Models Quick Use Instructions


Safety Tips:


- Read the user manual carefully before use (find it at www.lopital.com).
- Maximum load: 396 lbs.
- Make sure the brake is on during patient transfers.
- Check shower bed before each use: mattress intact, no bent or broken parts, and all moving elements function properly.

Use






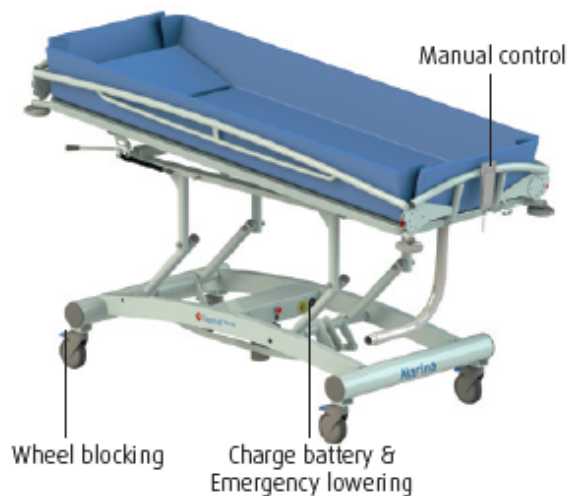


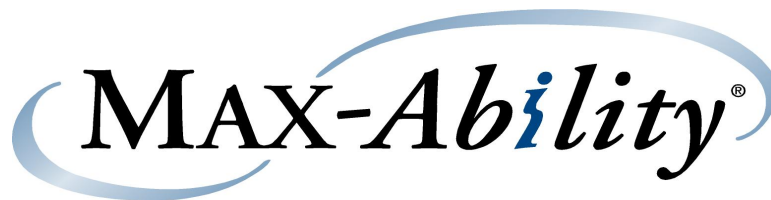- Make sure brakes are on.
- Press 'Up' for 2 seconds to activate.
- Position stretcher appropriately, using 'Up' & 'Down' pedals..
- Fold side bar down.*
- Move user to/from the Marina.
- Raise the bed base slightly. Close side bar.*
- Disengage brake to relocate shower bed.
- After 60 minutes inactivity, the Marina will go into sleep mode.

* Use gentle pressure to avoid bending side-release tabs.

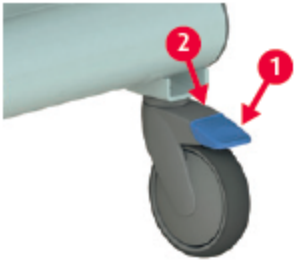

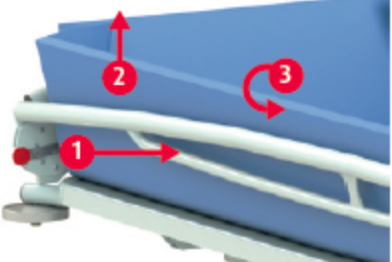

Battery-Powered Models

<h3>Wheel blocking</h3>  <p>1. Wheel blocking. 2. Unlocking.</p>	<h3>Electric manual control</h3> <p>Up  </p> <p>Down </p> <p>Activate  2 seconds</p>
<h3>Side-bars</h3>  <p>1. Unlock. 2. Tilt. 3. Turn. When closing, the bar locks automatically.</p>	<p>Make sure side bar does not pinch control panel cord.</p> <h3>Charging batteries (Electric Marina)</h3> <p>Connect battery to charger after use.</p> 
<h3>Emergency lowering (Electric Marina)</h3> <p>If the Marina breaks down, you can lower the stretcher using the emergency lowering button.</p> 	<h3>Cleaning and maintenance</h3> <ul style="list-style-type: none">• Clean after every use.• Use water & regular household cleaners.• Do not use bleach or abrasive cleaners.• Check wheels for hairs, debris, & soap residue.

Call us at 800 577-1555 for support or assistance.

Subject to alteration without further notice.

www.max-ability.com 800-577-1555 Fax: 707-575-3856 email: lk@max-ability.com

pub. May 2015