Oma™

Portable Movement Monitor
Moniteur mobile de mouvement
Monitor de Movimiento Portátil

SAFETY WARNINGS

-  **Read and Keep Instructions:** All the safety and operating instructions should be read before the movement monitor is operated.
-  **ATTENTION:** This product is a baby care aid, not a medical device. It is not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap including SIDS (Sudden Infant Death Syndrome).
-  **LEVANA® strongly recommends all new parents and caregivers become familiar with infant resuscitation and call 911 immediately if baby has stopped breathing.**
-  **RISK OF EXPLOSION IF BATTERY IS REPLACED BY AN INCORRECT TYPE!**
-  **Only use monitor when baby is in a motionless bed/cot. Do not use in car seat, stroller, swing or other mobile carrier as motion negates the functionality of the monitor.**
-  **This product does not replace responsible adult supervision. Check on your child at regular intervals and ensure that the battery is full.**
-  **Dispose of used batteries in accordance with your country's laws governing the disposal and recycling of batteries.**
-  **Use only batteries listed in the user manual.**

Water and Moisture: This product is designed for indoor use only. The unit should not be exposed to rain, moisture, dripping or splashing.

Non-Use Periods: Always remove the batteries during long periods of non-use.

Keep Away from Sources of Heat: This unit should not be exposed to direct heat sources such as a candle, fireplace, radiator, heat register, stove or other appliance that emits heat. The unit should not be left in direct sunlight.

Power Source: This unit should only be used with the CR14250, 3.0V battery included in this package.

Damage Requiring Service: This unit should be serviced by qualified personnel only when:

- a. Objects have fallen or liquid has been spilled into the product
- b. The units have been exposed to rain
- c. The units do not appear to operate normally or exhibits a distinct change in performance
- d. The unit has been dropped or the enclosure is damaged

LEVANA WARRANTY

Please visit our website at www.myLEVANA.com for information about your product's warranty

We take quality very seriously. This is why all of our products come with a one-year warranty from the original purchase date against defects in workmanship and materials. If you have warranty or support issues please contact us using any of the following methods:

Web: www.mylevana.com

Support: www.mylevana.com/support

LEVANA Canada

4080 Montrose Road
Niagara Falls, ON
Canada L2H 1J9

LEVANA USA

60 Industrial Parkway #Z64
Cheektowaga NY
USA 14227

Warranty Terms:

1. LEVANA® products are guaranteed for a period of one year from the date of purchase against defects in workmanship and materials. This warranty is limited to the repair, replacement or refund of the purchase price at LEVANA's option.
2. This warranty becomes void if the product shows evidence of having been misused, mishandled or tampered with contrary to the applicable instruction manual.
3. Routine cleaning, normal cosmetic and mechanical wear and tear are not covered under the terms of this warranty.
4. The warranty expressly provided for herein is the sole warranty provided in connection with the product itself and no other warranty, expressed or implied is provided. LEVANA assumes no responsibilities for any other claims not specifically mentioned in this warranty.
5. This warranty does not cover the shipping cost, insurance or any other incidental charges.
6. You **MUST** contact LEVANA before sending any product back for repair. You will be sent a Return Authorization form with return instructions. When returning the product for warranty service, please pack it carefully in the original box with all supplied accessories, and enclose your original receipt or copy, and a brief explanation of the problem (include RA #).
7. This warranty is valid only in Canada and the continental U.S.
8. This warranty cannot be re-issued.

DISCLAIMERS

Only use monitor when baby is in a motionless bed/cot. Do not use in car seat, stroller, swing or other mobile carrier as motion negates the functionality of the monitor.

This product is a baby care aid, not a medical device. It is not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap including SIDS (Sudden Infant Death Syndrome).

This product does not replace responsible adult supervision. Check on your child at regular intervals and ensure that the battery is full.

Customers use this monitor at their own risk and LEVANA®, its members, employees, agents, contractors, representatives, manufacturers, distributors and associates ("Snuza") shall not be held liable for any loss, damage or injury suffered by the customer or any other person arising from, or in connection with, the use or manufacture of the monitor, including loss, damage or injury suffered by the customer or any other person as a result of any defects in respect of the monitor, and all warranties, and any other liability from whatsoever cause arising, are hereby excluded and the customer hereby indemnifies LEVANA® in respect of any such liability.

TABLE OF CONTENTS

- SAFETY WARNINGS 2
- WARRANTY 3
- DISCLAIMERS 3
- WHAT'S INCLUDED 5
- IMPORTANT - READ FIRST! 5
- BUTTONS & FUNCTIONS..... 6
- USING OMA7
 - Startup 8
 - Monitoring 9
 - Oma and Clothing 10
- SETTINGS 11
 - Audible Alert 11
 - Alarm Activation Period 11
 - Movement-Rate Warning..... 12
- BATTERY 13
 - Battery Life 13
 - Replacing the Battery..... 13
 - Battery Disposal 13
- CARE & MAINTENANCE..... 14
 - Precautions 14
 - Cleaning and Storage..... 14
- TROUBLESHOOTING..... 14
- SPECIFICATIONS..... 16
- CUSTOMER SUPPORT 16
- FREQUENTLY ASKED QUESTIONS (FAQ) 17
- INSTRUCTIONS EN FRANÇAIS 18

WHAT'S INCLUDED

- Portable Movement Monitor
- Plastic Carrying Case
- Replaceable Battery
- Quick Start Guide
- 1 Year Warranty
- Lifetime Customer Support

Only use monitor when baby is in a motionless bed/cot. Do not use in car seat, stroller, swing or other mobile carrier as motion negates the functionality of the monitor.

This product is a baby care aid, not a medical device. It is not intended for the diagnosis, prevention or monitoring of a disease, injury or handicap including SIDS (Sudden Infant Death Syndrome).

This product does not replace responsible adult supervision. Check on your child at regular intervals and ensure that the battery is full.


Product may not be exactly as shown.

©2013 LEVANA®. All rights reserved. LEVANA®, the LEVANA® logo, Oma™ and other LEVANA® marks may be registered. All other trademarks are the property of their respective owners.



SNUZA is a registered trademark of Biosentronics

IMPORTANT - READ FIRST!

NOTE: Handle Oma with care to prevent damage to clip or sensor.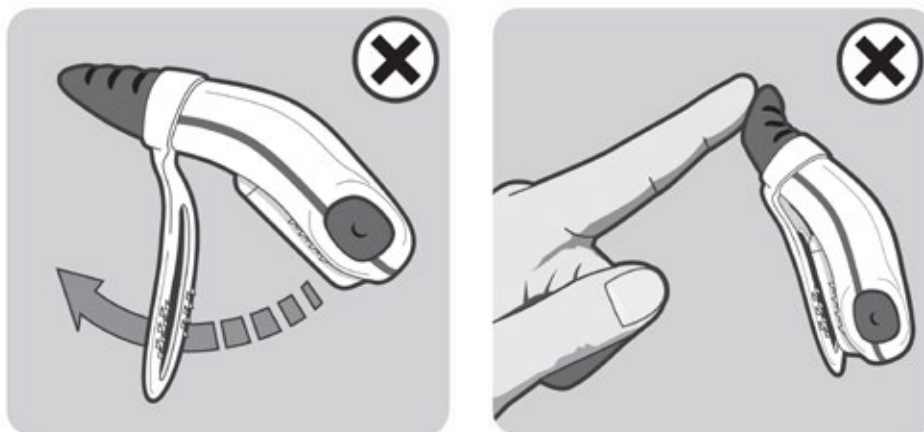

BUTTONS & FUNCTIONS

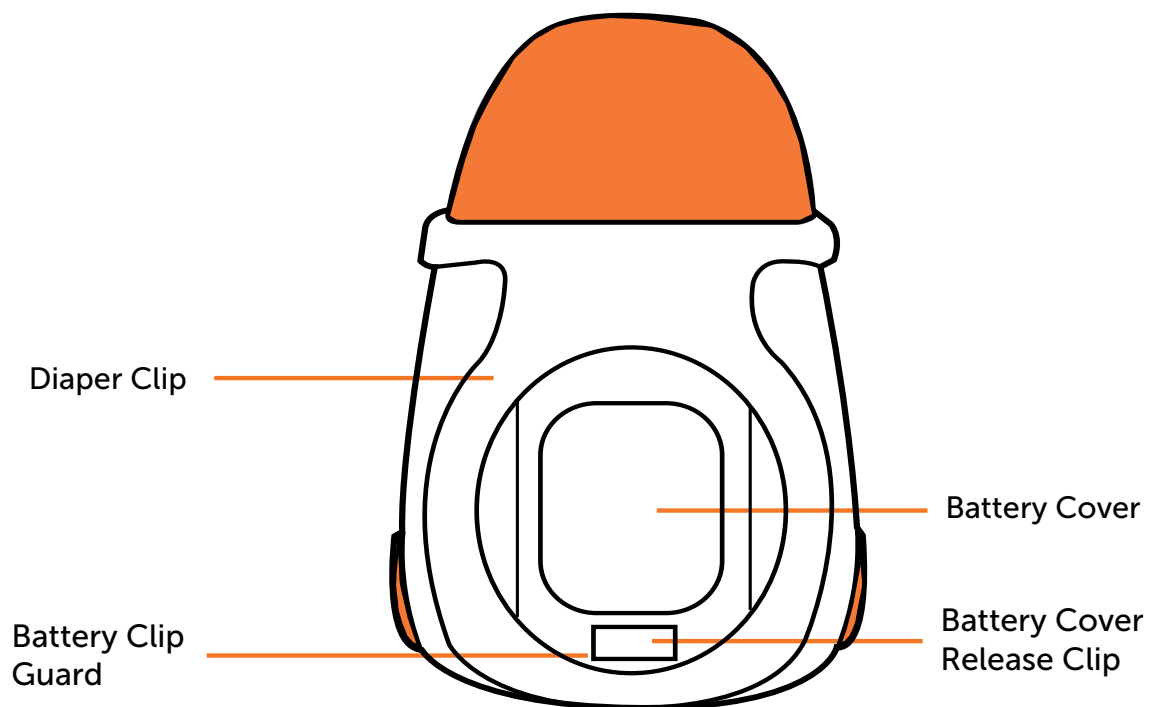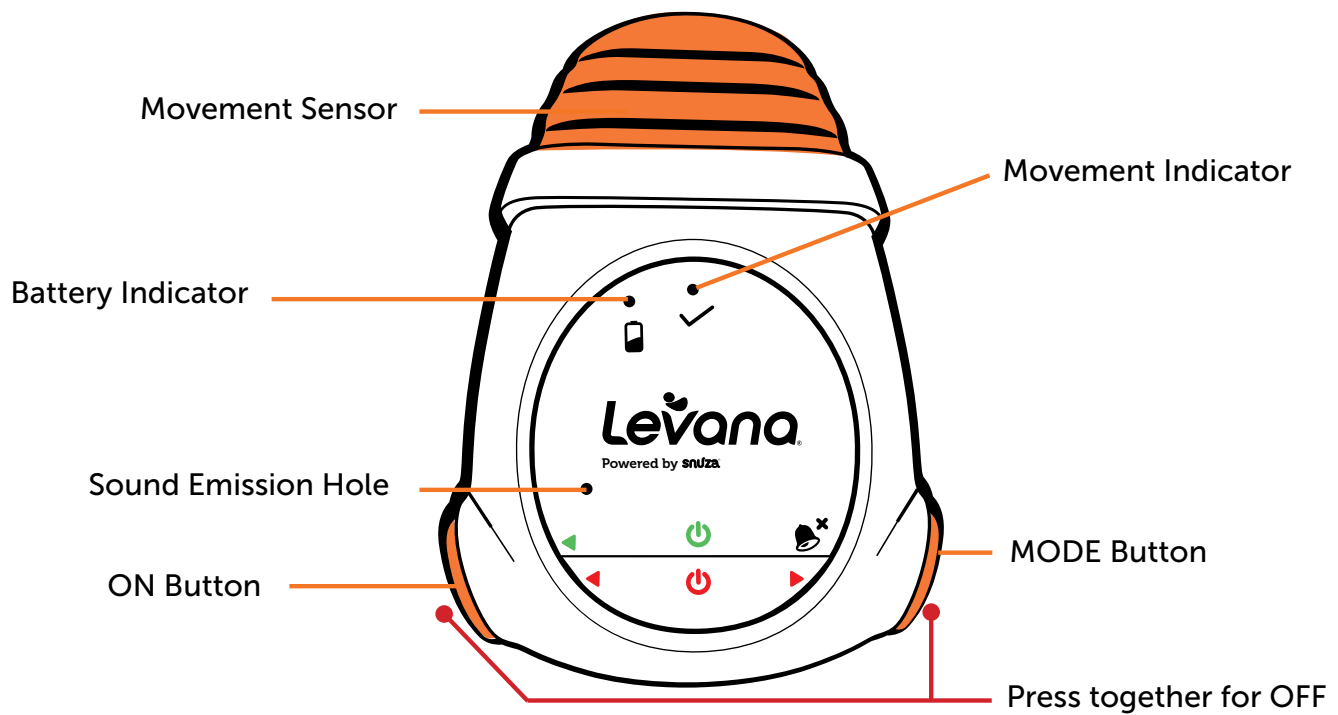
USING OMA

A properly fitting diaper is essential for the proper function of your Oma Portable Movement Monitor. The thin flap of material at the top of the diaper must be folded over the thicker part before the diaper is closed (this is the section to which Oma will be attached). The diaper should fit snugly on the baby with the top of the diaper just below the belly button. A very 'full' or wet diaper may also cause it to gape.

It is essential that the Oma Movement Sensor is in constant contact with the baby's tummy, around the belly button area, although closer to the sides may suit different sleeping positions.

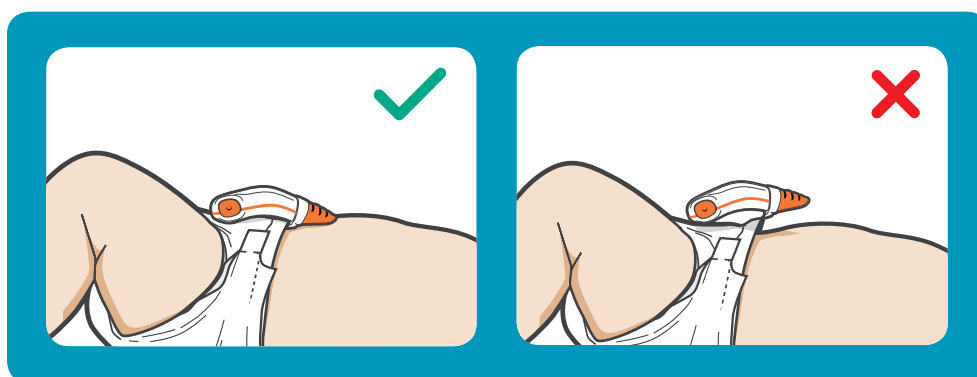Movement from external sources such as being pushed in a stroller or travelling in a vehicle or aircraft may be detected by the Oma sensor. **Oma should not be relied upon where external sources of movement are present.**

NOTE: The World Health Organisation (WHO), recommends that babies be placed to sleep on their back unless advised otherwise by a doctor.

USING OMA

STARTUP

Turn the unit ON, briefly press the ON button. A startup-test will run which will turn on (in sequence) all indicator lights and briefly sound the audible alarm. The Battery Indicator Light will flash briefly as follows:

	Battery OK	
	Battery nearing depletion	Replace battery
	Battery fully depleted	 Oma will not monitor; Replace battery

Warning: Ensure that all lights and alarm are working properly and that the battery level is high enough for proper operation (i.e. green or orange light). Otherwise, DO NOT use the monitor. If the battery is too low to operate, the Battery Indicator Light will flash red accompanied by a short beep every second. Please change the battery before further use (see page 13, "Replacing the Battery").

Unit On

Alert Off
(monitoring
will continue)

Unit Off
(monitoring
will stop)

To turn the unit OFF, press both the ON button and the MODE button at the same time.

USING OMA

MONITORING

When monitoring your child, the Movement Indicator Light provides information about your baby's movements, while the Alarm Indicator Light alerts you to potential issues.

The Movement Indicator Light flashes with each movement:

	Normal movements	
	Very small or weak movements (i.e. very shallow breathing)	
 (Dual-tone beep every second)	Movement rate below 8 movements per minute.	If movement rate increases to over 8 movements per minute, the Movement Indicator Light will reset to  and the warning will stop. To disable this function see page 12, " Movement-Rate Warning ".
 (Full alarm sounds)	No movement at all has been detected for the audible alarm sounds selected time period (15, 18 or 20 seconds).	You can select the alarm activation period (i.e. the period of no movement detection) to be 15, 18 or 20 seconds (see page 11, " Alarm Activation Period ")

NOTE: The alarm can be deactivated by briefly pressing the MODE or by turning Oma off completely (press ON and MODE buttons simultaneously). If the MODE button is used to deactivate the alarm, monitoring will continue.

USING OMA

OMA AND CLOTHING

- Oma can be clipped onto the folded waistband of the diaper or the waistband of elasticated pants, as long as the Movement Sensor touches the baby in the tummy area.

		There may also be a thin layer of clothing (i.e. a diaper shirt) between the Sensor and the skin.  Please ensure that the Sensor always maintains contact with the tummy area.
		The monitor can also be used if baby is dressed in light clothing (i.e. a sleeper) over the diaper.  Too much clothing may reduce or muffle the alarm. Please ensure the alarm can be heard if baby is covered in clothing and/or blankets.

NOTE: To extend the distance at which the alarm can be heard, Levana recommends using Oma together with an Audio Baby Monitor or a Video Baby Monitor. Please consult our website at mylevana.com to view our complete line of Levana Baby Monitors.

SETTINGS

Audible Alert - Emits an audible 'tick' on each movement.

Unit On

Press **MODE** button **ONCE** briefly


Press **MODE** button **ONCE** briefly to toggle volume settings:

Volume settings

- Press once-Low
- Press again-Medium
- Press again-High
- Press once more-OFF

Alarm Activation Period - Decide whether you prefer the alarm to activate after 15, 18 or 20 seconds of no movement.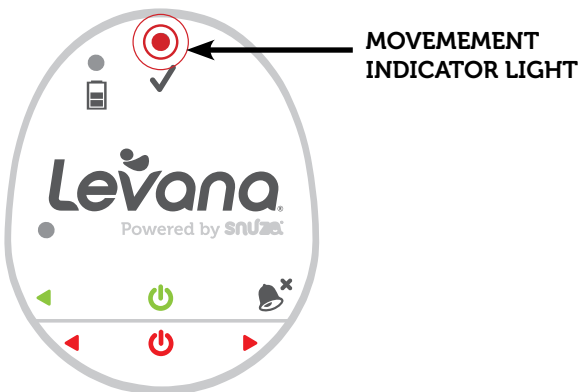

- Ensure that Oma is switched OFF.
- Press and hold the **MODE** button for 5 seconds until the unit beeps and both indicator lights flash briefly.
- The Movement Indicator Light will then start flashing with a pause between flashes.
- The following number of flashes relate to the alarm activation period:
 - 1 flash – 15 seconds
 - 2 flashes – 18 seconds
 - 3 flashes – 20 seconds

The factory pre-set value is for 20 seconds (3 flashes). Each press of the ON button will increase the number of flashes from 1 to 3.

- Press the **ON** button until the desired number of flashes (relating to the desired setting) is reached.
- Press the **MODE** button to store the setting. Oma will be in the power OFF mode after the setting has been saved.
- If the settings mode is entered and no button presses are detected within 30 seconds, Oma will switch to the monitoring mode (as if the unit had been turned on using the **ON** button).

SETTINGS

Movement-Rate Warning - Alerts you if the baby moves less than 8 times per minute.

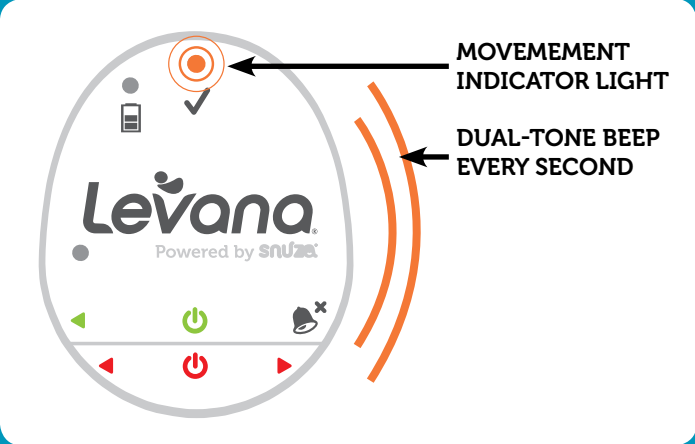
Diagram illustrating the Movement-Rate Warning feature. The monitor displays the Levana logo and "Powered by snuz". The Movement Indicator Light (orange) is shown flashing, and the Dual-Tone BEEP EVERY SECOND is indicated by orange curved lines.

- The Movement Indicator Light flashes orange and the Movement-Rate Warning sounds (dual-tone beep every second). This feature is automatically enabled each time Oma is switched ON.
- To disable this warning during monitoring, ensure that Oma is switched ON, and press and hold the ON button for more than 1 second until the unit beeps.
- If you want to enable the warning again, switch Oma off completely (press both ON and MODE buttons simultaneously), then switch it ON again. The Movement-rate Warning is enabled by default when Oma is switched on.

If the battery becomes depleted during monitoring, the Battery Indicator Light will flash as follows:

		Red flash on each movement	Battery nearing depletion; Replace battery
		Battery fully depleted–Low Battery Alarm beeps every 2 seconds	Oma will not monitor; Replace battery

NOTE: If the battery is nearing depletion or depleted, change the battery before further use of the monitor (see page 13, "Replacing the Battery").


BATTERY

Battery Life

- The life of the replaceable battery depends entirely on usage.
- Battery life will vary significantly depending on the average daily duration of use and/or alarm duration and/or number of alarms.
- Turning unit on and off unnecessarily shortens the battery life.
- If the number of alarms is high, please check that diaper is properly fitted.
- Please note that the product guarantee does not apply to the battery.

Replacing the Battery

- Use only a CR14250, 3.0V battery, which is available to purchase at www.mylevana.com.

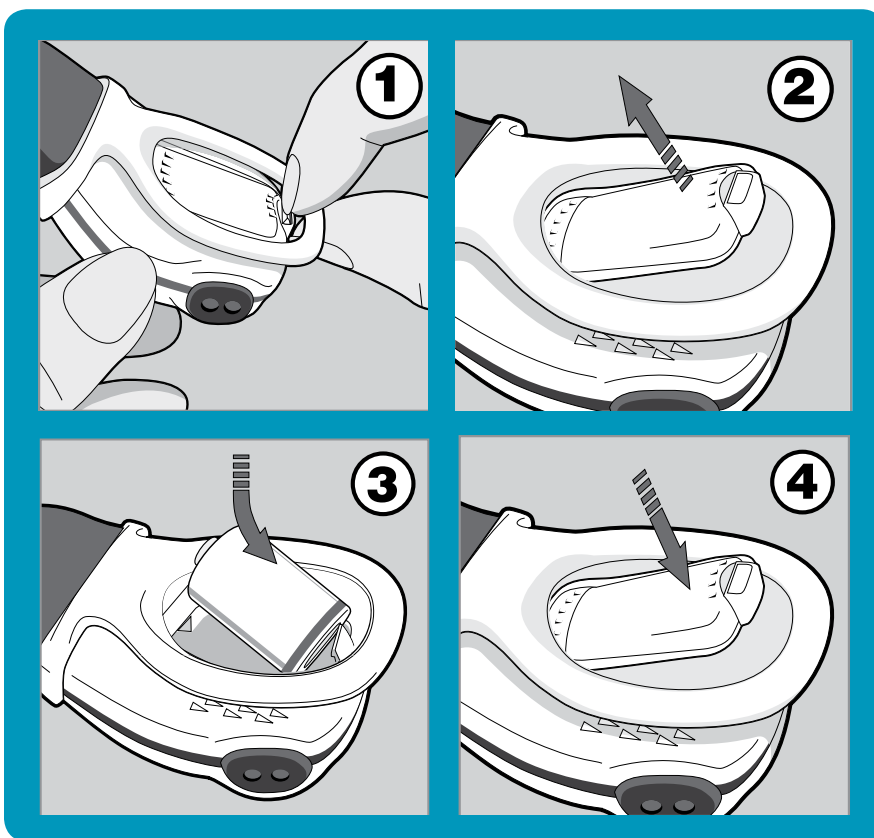1. Turn unit OFF by pressing the ON and MODE buttons at the same time.
2. Use your fingernail or a small, flat implement. With a pinching action, (Figure 1), gently pull back the Battery Cover Release Clip on the back of the monitor and lift the Battery Cover at the same time (Figure 2).
3. The Clip Guard prevents the clip from bending too far. Do not try to bend the clip beyond this point.
4. Remove the depleted battery.
5. Insert the new battery, negative end first (Figure 3), Ensure that the polarity is correct (align the positive end '+' of the battery with the '+' symbol in the battery housing). Replace the Battery Cover and ensure that it is correctly closed (Figure 4).

- Test the unit by switching ON and ensuring that the Battery Indicator Light flashes green.
- If the unit starts emitting a beep after the battery has been replaced, simply press the ON and MODE buttons simultaneously to turn it off. Power cycling may need to be attempted a few times.

Battery Disposal

The battery must be disposed of in accordance with your country's laws governing the disposal and recycling of batteries.

CARE & MAINTENANCE

PRECAUTIONS

- Do not insert anything into the Sound Emission Hole.
- Do not allow any liquid to enter the Sound Emission Hole.
- Avoid bending the sensor unnecessarily (Figure 1).
- Do not use undue force to bend the clip (Figure 2).
- Do not pull the Battery Cover Release Clip beyond the Guard Rail.

Cleaning and Storage

- Gently wipe with a damp cloth to remove any dirt. A mild soap may be used for stubborn dirt.
- Do not use any other cleaning agents as they may contain damaging chemicals.
- **The monitor is splash resistant, not waterproof.** Do not immerse in water. If it gets wet, dry it off immediately.
- When changing the battery, dirt that is trapped between the battery replacement hatch and the case may be cleaned with a dry cotton ball once the Battery Cover has been removed. Be sure that the rubber seal is free of dirt before replacing the Battery Cover, as this could compromise the seal.
- When not in use, keep your Oma in its protective case. Do not store in a damp area.

TROUBLESHOOTING

ISSUE	POSSIBLE REASON	POSSIBLE SOLUTION
I hear a ticking noise	• This feature causes the monitor to emit an audible tick on each movement.	• To turn OFF this feature, please see page 11, "Audible Alert".

TROUBLESHOOTING

ISSUE	POSSIBLE REASON	POSSIBLE SOLUTION
Alarm sounds for no reason	- Contact is lost between sensor and tummy. - Diaper is too large. Diapers that are too big loosen around the waistband, causing Oma to lose contact. - External movement is affecting sensor reading	- Check that diaper fits properly and is not too large; see page 7 to properly fit a diaper. - Avoid using Oma when child is in a car seat, stroller, swing or other mobile carrier as motion negates the functionality of the monitor.
	- Battery is nearing depletion - Battery is fully depleted	Battery needs to be replaced (see page 13, "Replacing the Battery")
I am having difficulty hearing the alarm	Too much clothing or too many blankets	- Make sure the alarm can be heard if baby is covered in clothing and/or blankets - To extend the distance at which the alarm can be heard, Levana recommends using Oma together with an Audio Baby Monitor or a Video Baby Monitor. Please consult our website at mylevana.com to view our complete line of Levana Baby Monitors.
The Battery Indicator Light is flashing red	- Battery is nearing depletion - Battery is fully depleted	Battery needs to be replaced (see page 13, "Replacing the Battery")
I just replaced the battery and I am still getting a low battery alert	Unit was not turned OFF <u>before</u> removing the depleted battery	- Turn unit OFF <u>before</u> removing the battery - Power cycle the unit a few times; this should correct issue
My baby is now mobile and the Movement Monitor is not staying in place	Movement Monitor needs to be secured in place	- We suggest a snug fitting sleeper and pant set. It will keep it in place when baby becomes mobile. - Insert a safety pin through the material (of the sleeper) and through the back of the clip - the clip is designed in such a way that this is possible. It won't come off until you remove the safety pin. Obviously this is only recommended <u>before</u> the diaper goes on so that there is no risk of hurting baby.

SPECIFICATIONS

SPECIFICATIONS	
Materials	Medical grade polycarbonate FDA approved thermoplastic-elastomer
Battery	CR14250, 3.0V
Noise level of alarm	85 dBm at 10cm
IP Rating	IP52
Unit Dimensions	2.75"(L) x 1.78"(W) x 0.98"(D) 6.98 cm (L) x 4.52 cm (W) x 2.48 cm (D)
Unit Weight	1.09 oz
Package Dimensions	6.38"(L) x 4.96"(W) x 1.57"(D) 16.20 cm (L) x 12.59 cm (W) x 3.98 cm (D)
Unit and Package Weight	5.3 oz

CUSTOMER SUPPORT

Contacting Customer Support

Customer Support is just a click away whenever you need it! Our customer support team can be reached by live web chat or on our website at www.mylevana.com/support. Our experts will be happy to assist you in English, French and Spanish.

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LEVANA Canada
4080 Montrose
Road
Niagara Falls, ON
Canada L2H 1J9

LEVANA USA
60 Industrial
Parkway, Suite #Z64
Cheektowaga, NY
USA 14227

QUESTION	ANSWER
Can I use the Movement Monitor with reusable/cloth diapers?	Yes. If the cloth diapers are too thick for attachment, the movement monitor should be securely attached to the diaper cover or waterproof pants - ensure a snug fit so that the movement monitor maintains body contact. You may need to position the movement monitor slightly to the side to ensure maximum contact.
Can the movement monitor be used with a baby that sleeps on his tummy?	The movement monitor relies on movement in the area where the sensor is located. Usually, when sleeping on its back (as recommended by the World Health Organization). The sensor contained in the tip of the movement monitor should be located as close to the tummy as possible as this is where most of the baby's breathing movements occur. Because of the restriction of the mattress, when baby sleeps on its tummy, tummy movements are reduced. When sleeping on the tummy, more breathing movements occur at the baby's sides than when sleeping on the back. You will need try and position the movement monitor away from the tummy (to avoid discomfort to baby), and place towards the sides where you can visibly see movement. <u>There is a greater possibility of false alarms occurring in this position</u> , however after a few trials you will find a position to attach the movement monitor that will provide successful monitoring of breathing movement. Please also be aware that if your baby sleeps on its tummy, alarm sounds may be muffled, therefore the use of an audio/video monitor may be required.
Is the movement monitor safe?	Absolutely! Baby's safety is paramount. Our movement monitors utilize an ultra-low voltage (3.0V) battery, emits no radiation and has no side effects. The Movement Sensor Tip is made from a medical grade polycarbonate FDA approved thermoplastic-elastomer which is completely synthetic and latex free thereby minimizing allergy risks.
Is the movement monitor rechargeable?	No, the battery is non-rechargeable however it can be replaced. Use only a CR14250, 3.0V battery, which is available to purchase at www.mylevana.com .
Until what age is the movement monitor recommended?	Levana recommends that Oma be used until the age of 1 year.

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Éviter les sources de chaleur: This unit should not be exposed to direct heat sources such as a candle, fireplace, radiator, heat register, stove or other appliance that emits heat. The unit should not be left in direct sunlight.

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