BioHug Vest™

An effective, portable, and non-restraining stress relief solution.

User Manual

Date: Aug 2014
Doc Number: DOC-UM30-00
Rev: 03

BioHug Technologies, Ltd., 1 Alexander Zeid St., P.O Box 372, Kiryat Tiv'on 3601201 Israel,

Tel: +972-4-6343460, info@biohug.com

<http://biohug.com/>

Table of Contents:

English	3
Hebrew	19
Arabic	35

BioHug Vest™

An effective, portable, and non-restraining stress relief solution.

User Manual

Date: Aug 2014
Doc Number: DOC-UM30-00
Rev: 03

BioHug Technologies, Ltd., 1 Alexander Zeid St., P.O Box 372, Kiryat Tiv'on 3601201 Israel,

Tel: +972-4-6343460, info@biohug.com

<http://biohug.com/>

Table of Contents

1. SAFETY PRECAUTIONS	4
2. QUICK START	5
3. INTRODUCTION	6
4. BIOHUG VEST PARTS	6
5. USING YOUR VEST	7
5.1. SIZING	7
<i>Using the sizing zippers</i>	7
<i>Size Chart</i>	8
<i>Front Zipper</i>	8
<i>Tightening straps</i>	8
5.2. OPERATING MODES	9
<i>Automatic</i>	9
<i>Manual</i>	9
<i>Power Off</i>	9
6. CHARGING THE BATTERY	10
7. MOVING THE REMOTE CONTROL UNIT TO THE POCKET	12
8. LAUNDERING	14
8.1. REMOVING THE BATTERY PACK	14
8.2. REMOVING COMPONENT POCKET	14
8.3. REMOVING BUBBLES	14
8.4. WASHING INSTRUCTION	15
8.5. REPLACING THE BUBBLES	15
8.6. REPLACING THE COMPONENT POCKET	15
9. TROUBLE SHOOTING	16
10. SPECIFICATIONS	17
11. CERTIFICATIONS	17

1. Safety Precautions

Warning: Description of hazard and possible resulting injuries.

To avoid the risk of electric shock, do not remove the remote or pneumatic box cover or back.

To avoid the risk of fire or electric shock, do not expose the BioHug Vest to liquids or moisture.

Caution: Operation is critical to quality.

Adjustments or procedures other than those specified and/or changes or modifications made to this BioHug Vest not approved by BioHug could void the warranty, render the device inoperable, and possibly injure the wearer.

Important Safety Instruction:

1. Read these instructions before operating the BioHug Vest.
2. Place the user manual in a place accessible to all users for future reference.
3. The BioHug Vest is not intended for use as sleepwear.
4. The BioHug Vest is not meant for use as a flotation device.
5. Unplug the Vest battery when unused for long periods of time.
6. Unplug the battery charger from the wall while not in use.
7. Do not block any of the ventilation openings.
8. Do not wear or store near any heat source.
9. Protect the battery charger and the battery power cord from being pinched.
10. Use only with battery pack and battery charger supplied by the manufacturer.
11. When batteries need to be replaced, please place old batteries in suitable battery recycling receptacle.
12. If battery pack falls on the floor, discontinue use and contact a BioHug representative.
13. To avoid injuries from tripping, and damage to the charger, please do not leave battery charger cable on the floor.
14. Before charging the battery, turn off the power using the switch, unplug the battery from the vest and take it out of the vest.
15. If the BioHug Vest is not working properly, follow the instructions/message in the Troubleshooting table. If you require assistance please contact a BioHug representative.
16. If the BioHug Vest smells or smoke, immediately remove the vest, turn it off and report it to a BioHug representative.
17. If the battery or charger smells or smokes while charging, immediately remove it from the power source and report it to a BioHug representative.

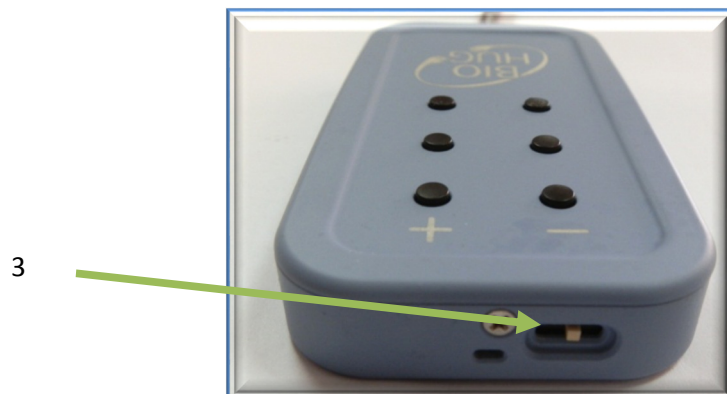
Exclusion criteria:

- A diagnosed disease, heart disease, congestive heart failure or respiratory conditions in particular
- Diagnosis of osteoporosis, or a brittle bone condition
- Any spinal or skeletal disease or disorder
- Obesity
- Pregnancy

2. Quick Start

The BioHug Vest is as easy to use as 1-2-3:

1. Adjust sizing (see page 7)
2. Connect the battery. (see page 9)
3. Slide the slide switch to automatic position. (see page 9)

3. Introduction

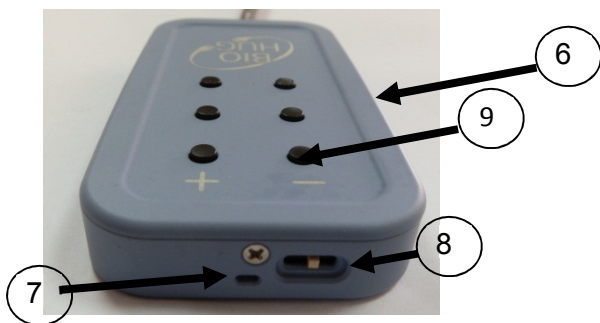
The BioHug Vest is an innovative device for applying deep pressure for effective, portable, and non-restraining stress relief.

Proper use of the BioHug Vest will help you get the most out of it. Please follow the instructions to ensure the best experience. Please take note of the safety notices.

4. BioHug Vest Parts

- 1. Component Pocket
- 2. Component Pocket zippers
- 3. Tightening strap
- 4. Sizing zippers
- 5. Front Zipper

Remote control front view

Remote control rear view

- 6. Remote control unit
- 7. Power/operation LED
- 8. Slide Switch
- 9. 6 push buttons
- 10. Instruction Panel

5. Using Your Vest

5.1. Sizing

Proper sizing of the vest is important to ensure the best experience. Ideally, the vest should be snug, but not uncomfortably tight.

Using the sizing zippers

The sizing zippers (4) will adjust the vest to small, medium and large sizes. The sizing zippers are located on the sides of the vest underneath the arm holes.

Size	Instruction
3-Large	Open all sizing zippers.
2-Medium	Fasten Zipper A to Zipper B on both sides of the vest.
1-Small	Fasten Zipper A to Zipper C on both sides of the vest

L /3

M/2

S/1

Size Chart

Vest Size	Zipper position	Size	*Waist circumference		**Vest length	
			cm	Inches	cm	Inches
M	1	S	100 -104	39.4 – 40.9	65	25.6
	2	M	104 -108	40.9 – 42.9	65	25.6
	3	L	108 -112	42.5 – 44.1	65	25.6
L	1	L	112 -116	44.1 – 45.7	73	28.7
	2	XL	116 -120	45.7 – 47.2	73	28.7
	3	XXL	120 -124	47.2 - 49	73	28.7

* / ** See diagram

Front Zipper

The front zipper (5) may be closed as far up as the upper chest. If this is uncomfortable it may be zipped up only about 1/3 of the way. In either case, the vest will operate properly.

Tightening straps

Once the vest has been sized properly using the sizing zippers, use the tightening straps (3) to insure the best fit for the lower back.

The tightening straps are found on the front of the vest

Tightening Strap

Tighten or loosen the straps to comfortable snugness.

5.2. Operating Modes

The BioHug Vest has two operating modes:

Automatic

In automatic mode, the vest will automatically press and release pressure on various parts of the wearers body without any need for intervention. The location and duration of the "hugs" are controlled automatically by the vest.

To use the vest in Automatic Mode, hold the Control Unit with the instruction panel (10) facing you, slide the three-position slide switch (8) to the right

Automatic 

After sliding the slide switch, the Power/operation LED (7) will flash green for about a minute, as the vest fills with air. It will then stay green.

Every now and then, when the air pressure drops as a result of inflation/release cycles the green LED will flash again until the air pressure builds up to the optimum level.

Manual

In manual mode, the wearer or an attendant chooses the location and duration of the pressure. Pressure can be applied to specific parts of the body at the touch of a button, using the buttons found on the control unit.

To use the vest in Manual Mode, hold the Control Unit with the instruction panel facing you; slide the three-position slide switch (8) to the left

Manual 

After sliding the slide switch, the Power/operation LED (7) will flash green for about a minute, as the vest fills with air. It will then stay green.

In manual mode use the following push buttons on the control unit to inflate and deflate specific areas:

- +
 -  Press/Release Shoulders
 -  Press/Release Mid-Back
 -  Press/Release Lower Back
- Press "+" buttons to apply pressure
Press "-" buttons to release pressure

Power Off

To turn the vest off, slide the three position slide switch (8) to the center position. The vest will automatically deflate.

Off 

6. Charging the Battery

Your BioHug Vest is equipped with a rechargeable battery. A fully charged battery should provide approximately 4 hours of vest operation.

You may charge the battery whenever you like to get the longest period of use. You should also charge the battery when:

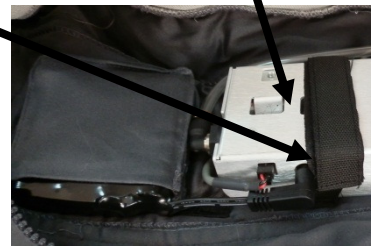
- The Power/operation LED (7) turns solid orange.
- The Power/operation LED (7) does not light up, and the unit does not operate
- The battery has not been used for a long time.

How to Charge the Battery:


1. Remove the battery pack from the components pocket (1)
 - a. Open the components pocket zipper (11)

- b. Disconnect the battery connector (12) from the pneumatic unit (13)

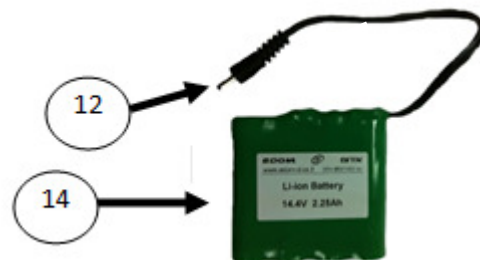
- c. Remove the battery pack (14) from its pocket

2. Connect the battery to the charger (15)

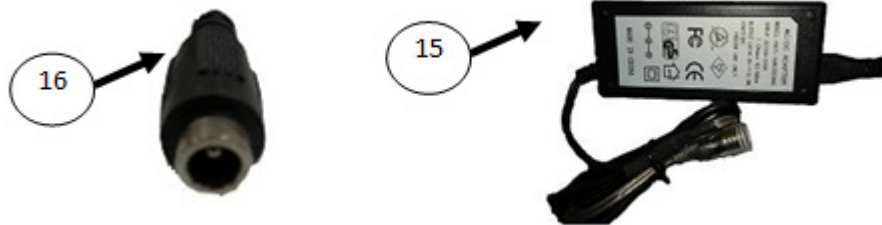- a. Plug the battery charger into a wall outlet
- b. Connect the battery connector (12) to the battery charger connector (16)

- c. Continue charging the battery until the indicator light on the charger turns green

3. Replace the battery in the components pocket

- a. Open the component pocket
- b. Replace the battery pack in its pocket
- c. Connect the battery connector to the Pneumatic unit (13). Make sure the connection is secure.
- d. Close the component pocket zipper

Warning!

Warning: The BioHug Vest should never be operated when connected directly to wall current as this may result in damage to the unit, or possible injury to the wearer.

7. Moving the Remote Control Unit to the pocket

The BioHug Vest is shipped with the Remote Control Unit – (6) located beneath the component pocket (1).

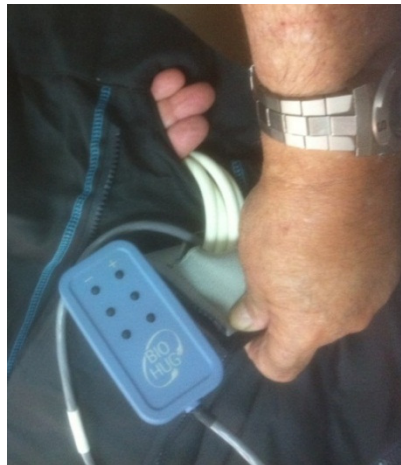
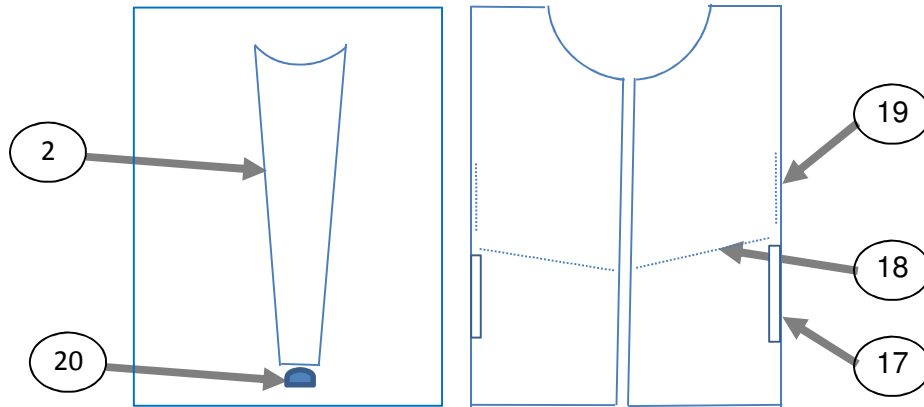
If you would like to place the Remote Control Unit in a pocket for manual mode use, follow these instructions.

- A. Unzip one of the Component Pocket Zippers (2) about half way. Grasp the Remote control unit and gently free it from under the Component Pocket.

- B. Locating the threading route – insert your hand into the pocket (17) (Left or right) and feel your way through the opening (18) at the top side of the pocket. Run your hand through it to locate the next opening (19) under the side zippers. Continue through that opening too until you see your hand in air pipes opening (20).

- C. Threading – pull the Remote control unit through the three openings to the pocket.
- D. Close the zipper (2) that was opened in step A

8. Laundering

Your BioHug Vest™ is washable. Please follow these instructions carefully. If you have any questions, please contact a BioHug representative for assistance.

Warning!

Warning: BioHug Vest™ components must be removed prior to washing in order to avoid damaging the mechanism.

Overview:

Laundering the BioHug Vest™ has the following basic steps:

1. Removing the battery pack from the vest
2. Removing the component pocket
3. Removing the bubbles
4. Washing and drip drying the vest
5. Replacing the bubbles
6. Replacing the component pocket.

- 8.1. Removing the battery pack
(see section 6.1)

- 8.2. Removing the component pocket

1. Lay the vest front side down on a table.
2. Open the component pocket
3. Unclip the "quick release" connector
4. Unzip the two zippers on the sides of the component pocket.
5. Gently guide the tubing through the hole in the pocket.
6. Close the component pocket zipper
7. Gently lift off the component pocket.


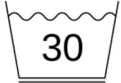
- 8.3. Removing the bubbles

1. Turn vest front side up on a table.
2. Unzip the front zipper (if closed)
3. Open the 2 Velcro strips found on the bottom left and right.
4. Reach through the opening in the lining, and gently pull the shoulder bubbles toward you.
5. Gently remove the bubble array from under the lining.

8.4. Washing instructions

 Warning!	<p>Warning</p> <ul style="list-style-type: none"> • Do not launder the component pocket. • Do not launder the bubbles
--	---

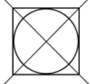	Machine Wash, Cold	Initial water temperature should not exceed 30° C or 65° F to 85° F.
	Machine Wash, Gentle or Delicate	Vest may be machine laundered only on the setting designed for gentle agitation for delicate items.
	Do Not Bleach	No bleach product may be used.
	Do Not Machine Dry	A machine dryer may not be used.
	Drip Dry	Hang wet vest on hanger or bar.

8.5. Replacing the bubbles

1. Lay vest, front side up, unzipped on a table
2. Place the bubbles over the vest, with the tubes and connector side down.
3. Slide the bottom part of the bubbles through the opening in the lining.
4. Push the fast connection connector through the hole in the back of the vest
5. Gently push the Velcro strips through the slots at the bottom of the lining, and press into place
6. Slip the shoulder strips of the bubbles under the upper lining of the vest, on the front of the vest. Pull them into place. They should sit between the lining and the outer part of the vest, going over the shoulders and down the front.
7. Smooth the bubbles with your hands, if necessary

8.6. Replacing the component pocket

1. Place the vest front side down on a table
2. Place the component pocket on top of the vest.
3. Unzip the component pocket
4. Reach your hand through the hole in the back of the component pocket, and push through the connector.
5. Connect the connector to its other half on the pneumatic unit (you should hear a click)
6. Zip the component pocket closed
7. Zip the component pocket into place using the two side zippers.

Please note: Your BioHug Vest may also be dry-cleaned. Component pocket and bubbles must be removed as above.

9. Trouble Shooting

Troubleshooting	Description	Solution
Power	Unit will not start	<ul style="list-style-type: none"> • Check that the power switch is turned to Automatic or Manual • Check battery connection • Charge the battery • Replace battery • Other: call BioHug representative
Power/operation LED	Flashing green for more than a minute	<ul style="list-style-type: none"> • Turn off the power switch and turn it on again • Other: call BioHug representative
	Red light	<ul style="list-style-type: none"> • Charge the battery or replace it
Bubbles pressure	Bubbles do not inflate at all	<ul style="list-style-type: none"> • Open the components pocket and ensure that the pump is working. If it is not: <ul style="list-style-type: none"> ○ Check electrical connection to pneumatic unit (13) ○ Replace pump • If the pump is working: <ul style="list-style-type: none"> ○ Check connection between pump and pneumatic unit (13) ○ Check the air-pipes connector • Other: call BioHug representative
	Switch in automatic mode, main bubble inflated, but no local pressure on shoulders, mid-back or lower back	<ul style="list-style-type: none"> • Check position of power switch. Is it on automatic? • Other: Call BioHug representative
	Switch in manual mode, main bubble inflated, but no local pressure on shoulders, mid-back or lower back	<ul style="list-style-type: none"> • Ensure that any of the inflate (+) buttons is pressed, on the remote control unit. • Other: Call BioHug representative
	Switch in automatic mode, but pressure does not change location	<ul style="list-style-type: none"> • Check position of power switch. Is it on automatic? • Other: call BioHug representative
Pump	Pump turns off and on	<ul style="list-style-type: none"> • This is normal. The BioHug Vest is designed to do this in order to increase the battery life
	Noisier than usual operation	<ul style="list-style-type: none"> • Check components sleeve zipper
Zippers	Cannot close zipper	<ul style="list-style-type: none"> • Check sizing • Try zipping up part-way

10. Specifications

Textile:

Outer Layer: Lycra 4% Cotton 96%

Lining: Lycra 4% Cotton 96%

Battery:

Li-ion 14.4V 2.25Ah

Up to 4 Hours of use when full charged

Charger:

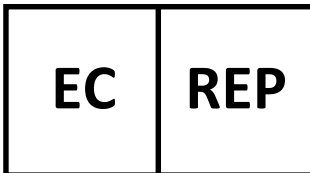
Input: 100-240V AC 50/60Hz

Output: 16.8 DC

11. Certifications

Authorized Representative In Europe:

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