Adapter for backrest actuator for Invacare® Bora/Spectra XTR with Modulite™ and electrical backrest recliner

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EN The information on the attached pages are meant for authorized Invacare® dealers only and are therefore not available in all languages.

At Invacare, our number one priority is to give you products that meet a wide variety of customer needs while maintaining their safety. The Bora/Spectra XTR continues to be highly popular because it is one of our most compact, lightweight, and transportable products available with a wide array of seating and positioning options. The new Modulite™ system has increased this flexibility with the addition of easy depth and center of gravity adjustment. With this, we have found that in a few of the more extreme combinations of seating system settings and customer weight distributions, the system can be less stable rearward increasing the likelihood that it will tip onto the anti-tip wheels. As a result, each time you receive a Bora with a recline, it will come pre-installed with a safety adapter to ensure each and every client is safe. In the majority of cases this can be removed and the full 120 degrees of recline can be utilized. The following details the procedure that can be used to verify that it is indeed safe to remove for your individual customer. These steps should only be performed by an authorized Invacare® dealer.

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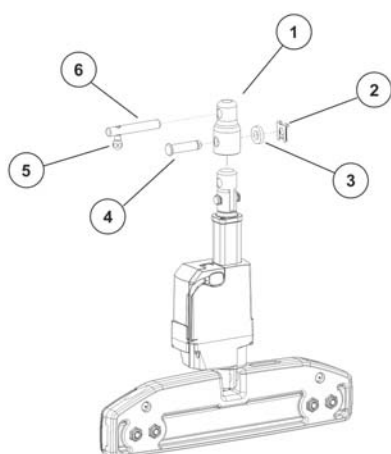
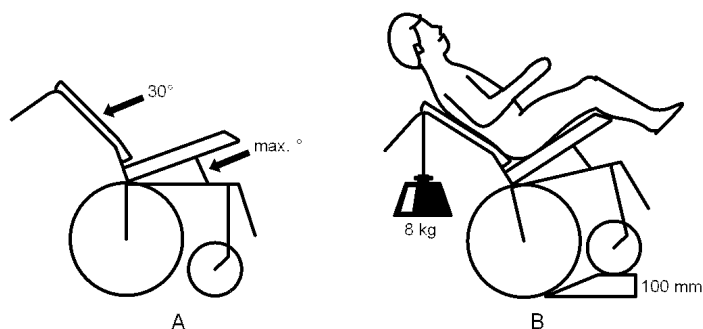
Tipping stability test

CAUTION! Risk of tipping!

Any change to the seat position can impair the stability of the electric wheelchair

- After setting the seat position, it is essential that you check that the wheelchair is stable (against tipping).

Any changes to the drive program can affect the driving characteristics and the tipping stability of the vehicle!

- Changes to the drive program may only be carried out by trained Invacare® specialist dealers!
- Invacare® supplies all mobility devices with a standard drive program ex-works. Invacare® can only give a warranty for safe vehicle driving behaviour - especially the tipping stability - for this standard drive program!

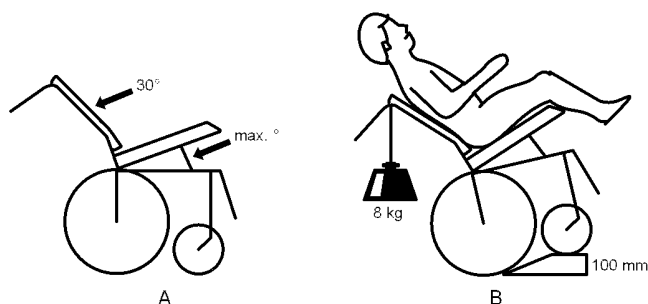
CAUTION! Risk of injury through tipping.

If the chair tips too far backwards when testing the tipping stability, there is a risk of injury for the user.

- Always additionally secure the wheelchair from behind against tipping.

1 Instruction:

- Adjust the following items to adjust the chair to the user's seating posture:
 - Lower leg length: Invacare offers a variety of legrests that can be individually adjusted. See user manual.
 - Seat depth: The seat depth is adjusted by moving the backrest. See user manual.
 - Center of gravity of the seat: See Chapter 2.
- Remove the adapter. See Chapter 3.
- Set the following angles on the wheelchair (Figure A):
 - Backrest 30°
 - Seat angle max. angle
- Allow the user to sit in the wheelchair.
- Drive the wheelchair by the wheelrests onto a step measuring 100 mm high. This can be done, e.g. using a ramp (Figure B).
- Ensure that the front wheels are in the direction of travel, facing backwards.
- Secure a reference weight of 8 kg (11.5 kg for Spectra XTR²HD) to the push bar (Figure B).
- Check that in this position the wheelchair is evenly balanced. This means that the front wheels may lose contact with the ground slightly but that the chair may not fall back onto the antitipper.
- If the wheelchair falls back on the antitipper, you must either:
 - move the seat's center of gravity forward until the point where the chair stops tipping or
 - re-fit the adapter

2 Adjusting the center of gravity of the seat

The center of gravity of the seat can be adjusted by mounting the seat frame further forward or further back on the seat support.

For more information about adjusting the seat's center of gravity, see the chapter "Adjusting the center of gravity of the seat" in the service manual.

The Modulite™ seat is available in two versions:

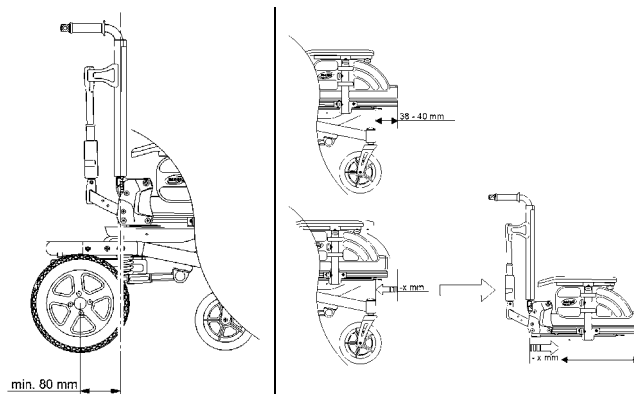
- Adjustable seats (plate and sling):
Adjust the center of gravity via the side profiles, as described in Chapter 2.1.
- Simple seat:
Adjust the center of gravity via the long holes of the seat adapter, as described in Chapter 2.2.

CAUTION!

Increased risk of tipping, if the center of gravity is moved too far to the rear.

The distance between the vertical backrest and rear axle must ALWAYS be at least 80 mm.

Note

The front edge of the seat profile should be 38 - 40 mm in front of the control heads of the castors.

If you reduce this dimension, make sure that

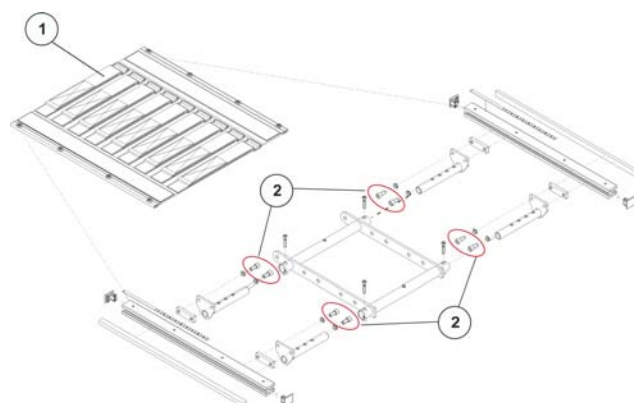
- you shorten the seat depth by the same amount and
- the legrest hangers cannot collide with the castors.

2.1 Adjustable seats

Requirements:

- 8 mm Allen key


- Remove the seat plate or sling seat (1), cf. Section 7.21.1 in the service instructions.
- Loosen the Allen screws (2) at the front and rear, left and right - DO NOT remove.
- Move seat.
- Retighten screws.
- Re-attach the seat plate or sling seat.

2.2 Simple seat


Requirements:

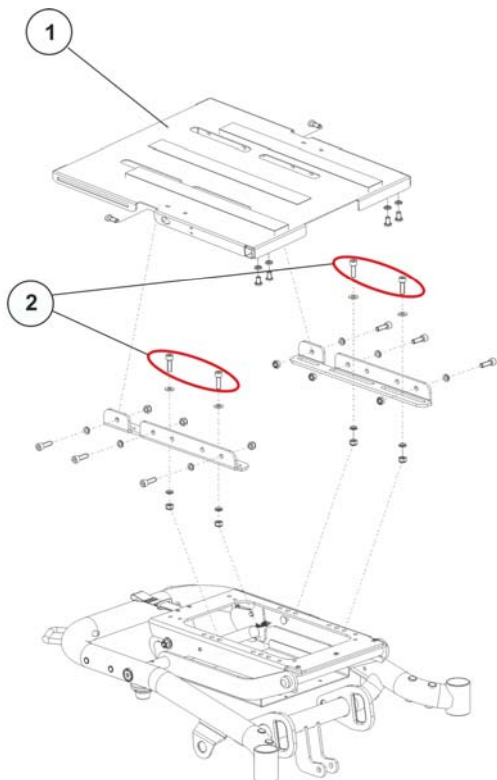- 8 mm Allen key

1. Remove the seat plate (1), cf. Section 7.21.1 in the service instructions.
2. Loosen the Allen screws (2) at the front and rear, left and right - DO NOT remove.
3. Move the seat frame.
4. Retighten screws.
5. Re-fit the seat plate.

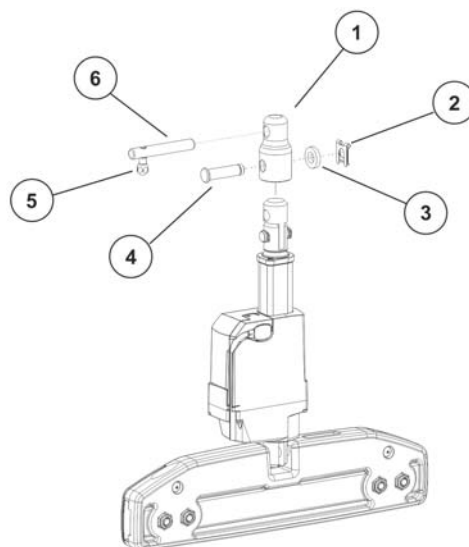3 Removing the adapter

1. Move bolts (6) to the side using the lever (5) and tip the actuator out of the top bracket.
2. Remove the SL clip (2), including washer (3) of the retaining bolt (4).
3. Pull out the retaining bolt (4).
4. Remove adapter (1).
5. Re-fit the actuator (without adapter) ensuring that the bolt (6) is inserted completely through the spindle of the actuator. Make sure that the actuator is securely fixed before usage.