Model: **pM-BP35 PLUS**
BPAP with Humidifier/ Nemlendiricili

Model: **pM-BP35**
BPAP

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EN Instruction Manual

www.plusmed-health.com

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This user manual covers the following models of the Plusmed BPAP series non-invasive ventilator:

PLUSMED pM-BP35 PLUS with Humidifier

PLUSMED pM-BP35 BPAP

The entire manual should be read before using the BPAP ventilator!

To have your device serviced, contact your provider.

Section 1: Introduction

1.1 Package contents

Your BPAP ventilator includes the following items. If any of these items are missing, contact your provider.

Fig.1 BPAP ventilator

Fig. 2 AC/DC Power Adapter

Fig. 3 AC Power Cord

Fig. 4 Sponge Filter

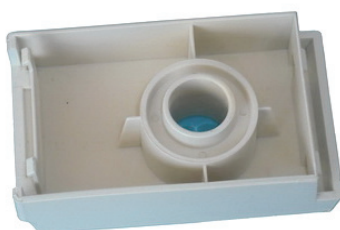
Fig. 5 Main Device Assembly Part

Fig. 6 SD Card

Fig. 7 Breathing Tubing

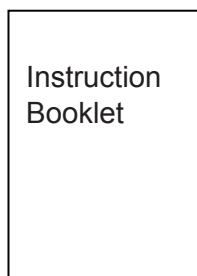
Fig. 8 Carry Bag

Note: You will also need a nasal mask/full face mask which is supplied separately.

1.2 Introduction

BPAP series non-invasive ventilator is a kind of Bi-level positive airway pressure device for the treatment of adult Obstructive Sleep Apnea Syndrome (OSAS) only.

The BPAP unit offers several options in how therapy is delivered, so treatment can be personalized to meet your needs. The unit delivers two different positive levels: IPAP (Inspiratory Positive Airway Pressure) and EPAP (Expiratory Positive Airway Pressure). Your home care provider will make the correct pressure settings.

When prescribed by your physician, the ramp function allows you to lower the pressure when you are trying to fall sleep. The air pressure will gradually increase until your prescription pressure is reached. You also have the option of not using the ramp feature at all.

1.3 BPAP Unit Feature

Name & Function

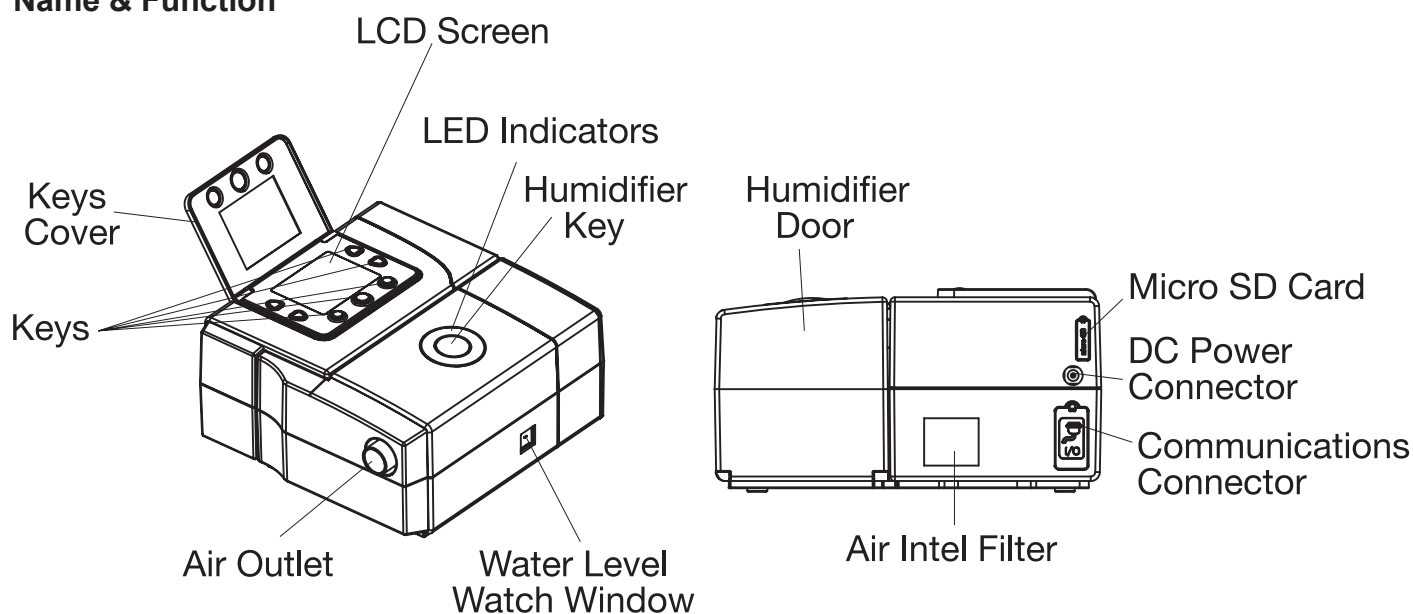
Fig. 9 Component Names

Unit Feature	Description
LCD Screen	The LCD screen displays the menus, treatment screens and alarm conditions.
Keys	Setting parameters and options
LED Indicators	Reflects the status of the humidifier heating level.
Humidifier Key	Use the key to set or modify the Humidifier heating level.
Air Outlet	Connect the flexible tubing here.
Water Level Watch Window	Watch the water level through the window
Humidifier door	Open the door to take out or put inside the humidifier tank.
Air Inlet Filter	The sponge filter prevents normal household dust.
DC Power Connector	Connect the 24V DC power cord here.
Micro SD Card	The micro SD card records device usage information (e.g., a history of pressure settings and times the unit was used for therapy).

Ventilation Modes

User-selectable Ventilation modes are:

- **Continuous Positive Airway Pressure (CPAP)**

In the CPAP mode, the device provides continuous pressure support ventilation at one pressure level.

- **Spontaneous (S)**

In the S mode, the device provides bi-level pressure support. The device triggers to inspiratory positive airway pressure (IPAP) in response to spontaneous inspiratory effort and cycles to Expiratory positive airway pressure (EPAP) during exhalation.

- **Timed (T)**

In the T mode, the device provides fixed time bi-level pressure support. The device can deliver IPAP and EPAP in response to setting parameters which breathe rate and inspiration time were set by users. The inspiration is time-triggered and time-cycled, the breath rate of patient does not affect the rate and pressure of the device.

- **Spontaneous/Timed (S/T)**

The S/T mode of the device is similar to the S mode, the difference is that it also provides a machine-triggered breath if the patient does not spontaneously breathe within a set time.

- **Pressure Control (PC)**

In the PC mode, the device triggers with S ventilation mode and the inspiratory time is controlled by setting parameters when patients breathe normally during the respiratory cycle.

Section 2: Warnings and Cautions

Carefully review and familiarize yourself with the following important notes and warnings, these instructions Must be followed to ensure correct and safe operation:

This device is not intended for life support.

Do not move, upside down, lean or shake the device if there is water in the humidifier, clear up the water first. Otherwise the water may flow into the machine and cause malfunction or damage.

2.1 Warnings

- The entire manual should be read and understand before using the BPAP unit.
- Advice contained in this manual should not supersede instructions given by the prescribing physician.
- The BPAP unit should only be used with masks recommended by Plumed, or by your physician or respiratory therapist. Do not attempt to wear your mask without the device turned on and operate properly. Doing so could result in CO₂ rebreathing. The vent holes associated with the mask should never be blocked. Otherwise it could result in suffocation.
- The BPAP unit may only be operated at temperatures between 41°F(5°C) and 95°F(35°C). Caution should be exercised if the room temperatures is warmer than 90°F(32°C).
- Do not use the device if the room temperature is above 95 °F (35 °C). If the device is used at room temperatures above 95 °F, the temperature of the airflow may exceed 105°F (41°C), which could cause irritation to your airway.
- This device is intended for adult use only.
- The BPAP unit is not suitable for use in the vicinity of flammable anesthetics.
- The BPAP unit should not be used with anesthetized patients, whose breathing depends on artificial ventilation.

- If oxygen is used with this device, the oxygen flow must be turned off when the device is not operating.
- Contact your doctor if symptoms of sleep apnea recur.
- Do not operate the device in direct sunlight or near a heating appliance because these conditions can increase the temperature of the air coming out of the device.
- When the BPAP is used with a humidifier, position the device so that the water level in the humidifier is lower than you.
- If you notice any unexplained changes in the performance of the device, if it is making unusual or harsh sounds, if it and/or the power supply has been dropped or mishandled, if the enclosure is broken, or if water has entered the unit, discontinue use and contact your home care provider.
- Repairs and adjustments must be performed by Plusmed - authorized service personnel only. Unauthorized service could cause injury, invalidate the warranty, or result in costly damage.
- Periodically inspect electrical cords, cables, and the power supply device for damage or signs of wear.
- To avoid electrical shock, unplug the device before cleaning it.

2.2 Cautions

The device may only be operated at temperatures between 11°F (5°C) and 90°F(32°C). A properly installed, undamaged reusable inlet filter is required for proper operation. Do not immerse the device or allow any liquid to enter the enclosure or the inlet filter. Condensation may damage the device. Always allow the device to reach room temperature before use. The humidifier key must remain on 0 if there is no water in the humidifier tank or the humidifier tank is not installed.

NOTE: Additional warnings, cautions, and notes are located throughout this manual.

2.3 Intended Use

The BPAP Bi-level systems delivers positive airway pressure therapy for the treatment of adult Obstructive Sleep Apnea Syndrome (OSAS) only.

2.4 Contraindications

The BPAP device should not be used if you have an insufficient respiratory drive to endure brief interruptions in non-invasive ventilation therapy. The BPAP device is not a life support ventilator and may stop operating with power failure or in the unlikely event of certain fault conditions.

If you have any of the following conditions, tell your physician before using the BPAP device:

- Acute sinusitis or otitis media.
- Epistaxis causing a risk of pulmonary aspiration.
- Conditions predisposing to a risk of aspiration of gastric contents.
- Impaired ability to clear secretions.
- Hypotension or significant intravascular volume depletion.
- Pneumothorax or pneumomediastinum.
- Recent cranial trauma or surgery.

2.5 Adverse Effects

You should report unusual chest pain, severe headache or increased breathlessness to your physician. An acute upper respiratory tract infection may require temporary discontinuation of treatment.

The following side effects may arise during the course of therapy with the BPAP device:

- Drying of the nose, mouth or throat

- Ear or sinus discomfort
- Eye irritation
- Mask-related skin irritations
- Chest discomfort

Section 3: Device Controls & Displays

3.1 Control Panel Keys

Şekil 10	Tuş	AÇIKLAMA
	▲ Up key	Allow you to increase the settings on the setup screens, also allow you to navigate to the previous Data menu.
	▼ Down key	Allow you to decrease the settings on the setup screens, also allow you to navigate to the next Data menu.
	Menu key	Enter the setting menu, also allow you to navigate to the next Setup menu.
	Modify & confirm key	Allows you to modify and confirm the Data menu.
	Return key	Quickly exit the Setup menu.
	On/Off key	Starts or stops treatment. In standby screen, press On/Off Button to put the device into the Operate state (and turn on the airflow). In monitor screen, press On/Off Button to put the device into the standby state (and turn off the airflow). Unplug the power if not use for a long time.
	Alarms Mute key	You can mute an alarm by pressing the Alarm mute key once. If the problem is still present, the alarm will sound again after two minutes.

3.2 Display Screen

There are two ways of display: Pressure bar graph/Data and Waveform/Data display

Fig. 11 S Mode Screen

Pressure bar graph/Data

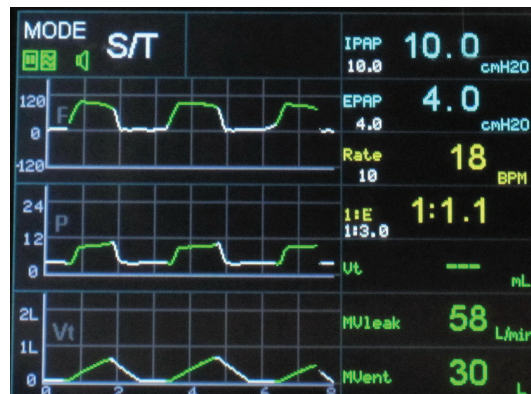
Fig. 12 S/T Mode Screen

Waveform/Data display

3.3 Menu Setting

Screen switch: Press the menu key to switch the device Screen. Follow monitor screen to Modes, Alarm screen and configuration menu. Shown in Fig. 13

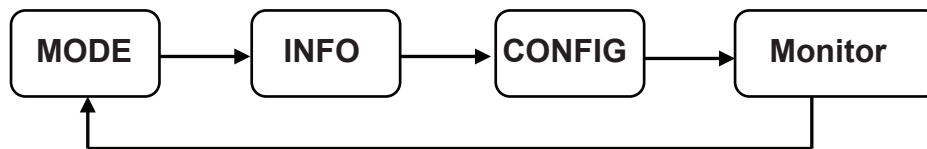
Fig 13: Screen Switch

Menu Setting for BPAP25

(Other Models have less options, refer to Section 8 Specification form)

Mode Setting

Choose mode and set parameter.

Parameter Setting Sequence

MODE		ALARM	CONFIG
Mode Select	S/T		
IPAP	10.0 cmH2O		
EPAP	4.0 cmH2O		
Rate	10 BPM		
Insp Time	1.5 Sec		

S/T Mode	T Mode	S Mode	CPAP Mode
IPAP	IPAP	IPAP	CPAP
EPAP	EPAP	EPAP	Ramp
Rate	Rate	Rise-time	Comfortable
Insp Time	Insp Time	I-trigger	
Rise Time	Exp Time	E-trigger	
I-trigger	Rise Time		
E-trigger			

Fig. 14 Mode Screen

Mode Parameter Modify

When option in blue condition **Mode Select S** press to enter modify status (the words turn white), Press ▲ or ▼ to select Mode, and then press to confirm. Press to return to monitor screen.
(Other parameter modification can also refer to this procedure)

Alarm Setting

Choice alarm functions and set parameter.

MODE		ALARM	CONFIG
Mask	OFF		
Apnea	OFF		
Mvent	OFF		
HRate	OFF		
LRate	OFF		

Alarm Setting Sequence	
Mask	(ON/OFF)
Apnea	(KAPALI, 10, 20, 30,40, 50, 60 Sec.)
Mvent	(OFF, 1 ~ 15 L)
HRate	(OFF, 1~99 BPM)
LRate	(OFF, 1~99 BPM)

Fig. 15 Alarm Screen

Configuration Setting

Choose configuration and set date

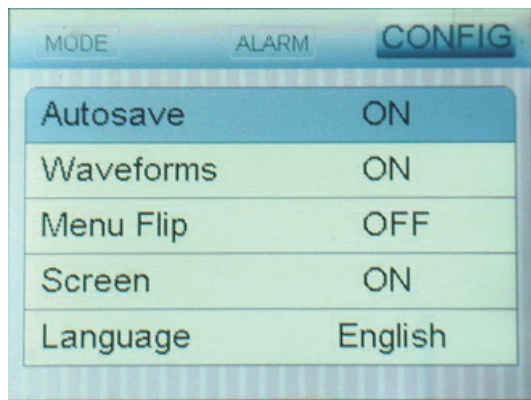
Fig. 16 Configuration Screen

Configuration Setting	
Autosave	(ON/OFF)
Waveforms	(ON/OFF)
Menu Flip	(ON/OFF)
Screen	(ON/OFF)
Language	(English/Chinese)
P-Units	(cmH2O/kPa)
Tube Length	(0, 1.8m, 2m)
Auto Start	(ON/OFF)
Data	(yy-mm-dd)
Time	(h-m)

- Autosave** Save the Data into Micro SD Card.
- Waveforms** Waveform display on the screen.
- Menu Flip** The screen will flip 180° to display.
- P-Unit** Pressure Unit, cmH2O/kPa.
- Tube Length** Selects the length of air tubing which connecting your mask and the device.

Navigate the Data menu: In modes screen, alarm screen, configuration screen, press (▲/▼) may to select the Parameter circularly.

3.4 Humidifier Setting:

Press the Humidifier key can adjust the water temperature level of the humidifier to the desired setting. The LED indicator on the humidifier will light on.

Humidifier Key: The humidifier key is used to control the temperature of the humidifier. The key has 6 settings, range from 0 to 5, with 0 indicating OFF.

LED Indicator: The LED indicator, locating on the humidifier key, reflects the status of the humidifier. It provides an indicator of the humidifier setting.

Fig. 17 Humidifier Key

CAUTION: The humidifier key must remain on 0 if there is no water in the humidifier tank or the humidifier tank is not installed.

NOTE: The LED indicator will not turn on if you did not start the blower on the therapy device or if the humidifier key is set to 0.

Section 4: Operation Instruction

4.1 Place the Device

Place the device on its base somewhere within easy reach of where you will use it. Make sure that the air inlet on the rear of the device is not blocked. If you block the air flow around the device may not work properly. Place the device in a hard, flat surface with clear, ventilated environment, away from heating equipment and cooling device.

WARNING: Position the device with humidifier so the water level is lower than you, see the humidifier instructions for complete setup information.

4.2 Setup the BPAP Unit

4.1.1 Assemble Main Device

The main therapy device can be used alone.

Install the air outlet port connection to the main therapy device, and then connecting the breathing circuit and power (refer to 4.3 and 4.4).

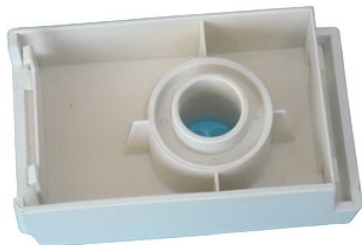
Fig. 18 air outlet port

Fig. 19 install air outlet port

4.1.2 Micro SD Card Installation

Micro SD card may save treatment and BPAP system's information. Install the SD card before using the BPAP device.

Open the micro SD cover in the nail, and put the micro SD card into the SD seat.

4.1.3 Connecting the Breathing Circuit

To use the unit, you will also need a Nasal Mask (with integrated exhalation port) or Full Face Mask (supplied separately).

You will need to assemble the following accessories to get the recommended circuit:

- Plusmed interface (e.g. nasal mask) with integrated exhalation port.
- Plusmed 1.8m (6 ft. flexible tubing.
- Plusmed headgear (for the patient interface).

(1) Connect the flexible tubing to the air outlet on the side of the device, as shown in Figure 20.

(2) Connect the mask's connector to the flexible tubing, as shown in figure 21.

(3) Attach the headgear to the mask. See the instructions that came with your mask for more information.

(4) Adjust your mask, mask cushion and headgear until you have a secure and comfortable fit.

WARNING: Only Plusmed air tubing and mask, or physician recommended accessories should be used with the BPAP unit. Do not attempt to wear your mask without the device turned on and operating properly. If you are using a full face mask, the mask must be equipped with a safety (entrainment) valve.

Fig. 20 tube connect

Fig. 21 tube and mask connect

4.1.4 Using AC Power

Complete the following steps to operate the device using AC power.

- (1) Connect the AC power cord into DC Power Adapter,
- (2) Insert the plug of AC power cord into the electrical outlet (not controlled by a wall switch), as shown in Fig 23,
- (3) Plug the Adapter connector into the power inlet on the side of the device, as shown in Fig 22.

Fig. 22 power cord connect

Fig. 23 power cord connect

CAUTION: To remove AC power, disconnect the power supply cord from the electrical outlet. Inspect the power cord often for any signs of damage. Replace a damaged power cord immediately. Only use the Plusmed DC power adapter available from your provider. Use of any other adapter may cause damage to the device.

4.1.5 Assembly the Humidifier

A humidifier may be required if you are experiencing dryness of the nose, throat or mouth.


Place the main device on the humidifier base. Position the main device so that the air inlet port on the humidifier connects to the air outlet port on the device. Use the latch on the humidifier base as a guide to properly connect the main device to the humidifier.

Fig. 24 connect the main device

Filling the water into Humidifier tank

- (1) Flip the humidifier door on the right side of the humidifier.
- (2) Remove the humidifier tank by sliding it out of the humidifier base.
- (3) Rinse the tank with water. Fill the tank to the fill line with room temperature distilled water (Maximum 225ml)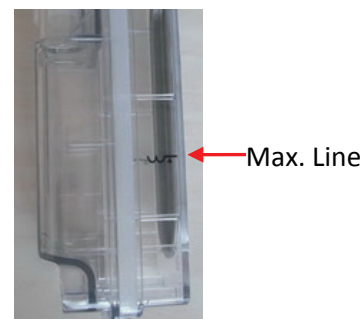

CAUTION: Use only distilled water in the tank.

Fill the humidifier with room temperature water only. Filling the humidifier with water that is too cold or too hot may cause errors in the humidifier, cause the LED indicator to blink.

WARNING: Do not move, upside down, lean or shake the device if there is water in the humidifier, clear up the water first. Otherwise the water may flow into the machine and cause malfunction or damage.

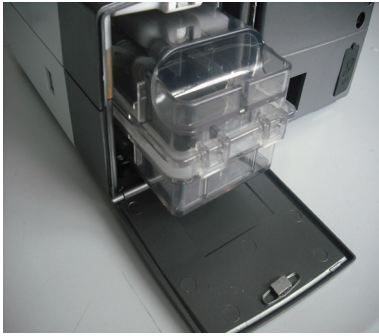
Fig. 25 Filling the water tank

4.3 Start Treatment

- (1) Plug in the device to an AC power source to power up the unit. The LCD screen lights up.
- (2) Set the parameter by physician.
- (3) Put on your mask assembly when the air starts to flow.
- (4) Lie down and arrange the air tubing so that it is free to move.
- (5) Relax. Take normal breaths through your nose.

NOTE: If you have trouble with your mask, see Section 7 troubleshooting, for some suggestions.

To stop treatment at any time, remove your mask and unplug the power plug.

CAUTION: Always empty the water tank before move the BPAP unit.

4.4 Traveling with the BPAP Unit

When you are traveling, the BPAP unit carrying case can be used for carry-on luggage only. The carrying case will not protect the BPAP, thus is not suitable for transportation.

For your own safe, take the user manual with the device.

CAUTION: Always empty the water tank before packing the humidifier.

Section 5: Cleaning and Maintenance

This section provides information on how to clean and maintain the BPAP unit. The cleaning and maintenance described in this section should be regularly carried out.

WARNING: To avoid electrical shock, always unplug the BPAP unit before cleaning and be sure that it is dry before reconnecting.

WARNING: DO NOT immerse the BPAP into any fluids or allow any liquid to enter the enclosure, inlet filter, or any openings


5.1 Cleaning the Cabinet

Disconnect the power, tubing, mask and humidifier tank. Wipe the exterior cabinet of the BPAP unit with a cloth slightly dampened with water and a mild detergent.

Allow the BPAP unit dry completely before plugging in the power cord.

5.2 Cleaning and Replacing the Inlet Filter

The gray sponge filter should be cleaned at least once every two weeks under normal usage and replaced with a new one every six months. Do not knead while washing.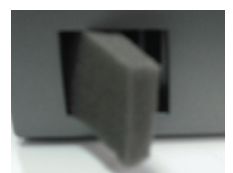

NOTE: Clean this filter at least once per week or more frequently if operated in a dusty environment. Also the replace time should be shorten accordingly.

5.3 Cleaning Tubing and Mask

Cleaning the tubing and mask is an important daily work.

(1) Disconnect the flexible tubing from the BPAP unit. Gently wash the flexible tubing daily in a solution of warm water and a mild detergent. Rinse thoroughly, air-dry.

(2) For details on cleaning your mask and accessories, refer to the instructions packaged with the accessories.

5.4 Cleaning Humidifier Tank

Disconnect the power, take out the humidifier tank.

Wash the humidifier tank in water using mild detergent. Rinse thoroughly, air- dry.

CAUTION: Operating the BPAP unit with a dirty filter may keep the device from working properly and damage the device.

CAUTION: Never install a wet filter into the BPAP unit. We recommend that you clean the filter in the morning and alternate using the spare sponge filters provided with the unit to ensure enough drying time for the cleaned filter.

NOTE: Do not use bleach, chlorine-, alcohol- or aromatic-based solutions (including all scented oils), moisturizing or antibacterial soaps to clean the tubing or the device. These solutions may cause hardening and reduce the life of the product.

The water temperature for wash should not over 50°C

Do not hang the tubing in direct sunlight as the tubing may harden over time and eventually crack


Section 6: Device Alarm

This section describes the device alarms and what you should do if an alarm occurs.

BPAP alarm conditions are signaled in two ways:

A sound——Continue beep sound

A display message—— the type of the alarm on screen

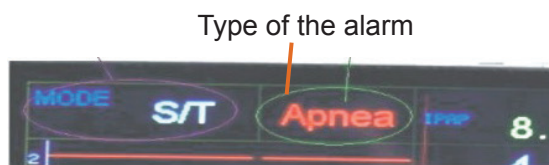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Fig. 27 Alarm mute key

Alarm mute: When the alarm appears, press the Alarm mute key to mute for two minutes.

When the device alarms, listen to the alert sound, look at the displayed message, and refer to the following table for possible solutions.

Alarm Summary Table

Alarm	Possible Cause	Patient Action
Mask alert	Mask removed or dislodged.	Press Alarm mute key on the device to silence the alert tone. The system has detected a large leak in the circuit or you have taken the mask off without turning the airflow off.

Apnea alert	Patient not breathing or unable to trigger ventilator. separated from the ventilator	Reevaluate the patient and check the patient circuit. Reevaluate the patient and alarm setting
Low Minute Ventilation alert	Patient disconnect or large leak. Decrease in patient rate or tidal volume. Improperly set alarm limit.	Check circuit and patient connection. Reevaluate the patient and alarm settings.
High Rate alert	Increase in patient breathing rate. Improperly set alarm limit.	Reevaluate the patient and alarm settings.
Low Rate alert	Decrease in patient breathing rate. Patient unable to trigger ventilator. Improperly set alarm limit.	Reevaluate the patient and alarm settings.

Section 7: Troubleshooting

The table below lists common problems you may have with the BPAP unit and possible solutions to those problems. If none of the corrective actions solve the problem, contact your home care provider.

Problem	Possible Cause	Solution(s)
The device does not operate	Power not connected or Standby status	Check the power connection, ensure the power cord is well connected, or press on/off key enter operate state.
The device deliver low pressure	Air filter or air outlet is blocked, or Ramp Time is in use	Clean or replace air filter, clean air outlet. If necessary, cancel Ramp time or reset time.
The blower works, but the pressure is obviously different from setting parameter.	Air tubing not connected properly. Mask or air tubing leak	Check the tubing connect, Check leak and adjust position of mask and tubing.
Insufficient air delivery	Air Inlet blocked, or Ramp time is in use.	Check and clean air inlet. Shorten ramp time if necessary.
The airflow from the BPAP seems warm.	Air filter dirty, Air inlet blocked, Near with wall, bedding, heating equipment etc.	Replace or clean the filter. Make sure the BPAP unit is away from bedding or curtains that could block the flow of air around the device. Make sure the device is away from heating equipment (e.g., forced air vents, radiators).
Excessive noise	Breathing circuit not connected properly; Mask or air tubing leak, Humidifier not installed properly.	Check the breathing circuit connect, Check leak and adjust position of mask and tubing. Check and reinstall the humidifier.
The noise level of the BPAP has changed to include unusual or harsh sounds during operation.	Component failure	Stop use. Contact your provider for service. Please have the serial number of BPAP unit ready when you call.
The BPAP unit has been dropped or mishandled; The enclosure of the BPAP unit is broken	Component may break	Stop use. Contact your provider for service. Please have the serial number of BPAP unit ready when you call.
Water has entered the BPAP unit	The device may break due to water in.	Stop use. Contact your provider for service. Please have the serial number of BPAP unit ready when you call.

The "Mask Alert" message appears on the display screen and the alert tone is sounding.	Mask removed or dislodged.	Press Alarm mute key on the device to silence the alert tone. The system has detected a large leak in the circuit or you have taken the mask off without turning the airflow off. Check the circuit connections for leaks.
The "blower fault" message appears on the display screen.	Blower fault or improperly connect	Stop use. Contact your provider for service. Please have the serial number of BPAP unit ready when you call.
The "Card error" message appears on the display screen.	The micro SD card inserted improperly	Remove the micro SD card and reinsert the card. If the message appears again, Contact your provider for service. Please have the serial number of BPAP unit ready when you call.
Sore or dry eyes	The mask is not properly fitted or positioned. Unsuitable type or size	Readjust the headgear. If the problem continues, contact your provider to be fitted for a different mask style or size.
Nasal, sinus, or ear pain.	inflammation	You may have a sinus infection or middle ear infection. Stop use the device, and contact your health care professional.
Redness on your face where the mask or accessory contacts your skin.	Improper mask fitting or improper mask cleaning Irritation or allergic reaction to the mask material.	Loosen the headgear. Check the mask's spacer to see that it is in good condition. Use a barrier between your skin and the mask, such as 3M's Microfoam tape. Be sure to cleaning the mask thoroughly according to mask cleaning instructions. If the problem continues, contact your provider to be fitted for a different mask style or size. Contact your health care professional if the problem persists.

Section 8: Models & Specifications

8.1 Specifications

Volume Tidal.....0 to 3000ml
Leak.....0 to 999ml
Minute Volume.....0 to 30L

8.2 Power Requirements

AC power:100 to 240VAC, 50/60Hz, 1.0A max.
DC power:.....24VDC, 2.5A max.
Type of protection against electric shock:.....Class II Equipment.
Degree of protection against electric shock:Type B Applied Part.
Degree of protection against ingress of water:Drip Proof, IPX1

8.3 Environmental Conditions

Atmospheric Pressure:.....83 to 102kPa
Operating Temperature:5°C to 35°C(41°F to 95°F)
Storage Temperature:-20 to 60°C(-4°F to 140°F)
Operating & Storage Humidity:15 to 95%RH

8.4 Standards Compliance

IEC 60601-1 Medical Electrical Equipment-General Requirements for Safety
IEC 60601-1-2 Electromagnetic Compatibility
EN ISO 17510-1 Sleep Apnea Breathing Therapy Devices

8.5 Symbols

	European CE Declaration of Conformity
	Our medical product is labeled with this symbol in accordance with European Directive 2002/96/EG (Waste Electrical and Electronic Equipment - WEEE) to indicate that, at the end of its life, it must be disposed separately from other household waste. Please contact your local authority or waste disposal service for the return and recycling of this product.
	Warning
	DC Power
	Type B Applied Part
	Observe the operating manual
IPX1	Drip Proof Equipment
	Class II (Double Insulated)

8.6 Models & Specifications

Parameters	
mode	CPAP, S, S/T, T, PC
IPAP range	4 - 35cmH ₂ O
EPAP range	4 - 20cmH ₂ O
CPAP range	4 - 20cmH ₂ O
I-trigger	Auto, 1 to 6
E-trigger	Auto, 1 to 6
IPAP Rise Time	100-600 msec
Breath rate (BPM)	Adjustable 0 - 40
Noise Level(dBA)	<30dB@ 10cmH ₂ O
CPAP Ramp (minutes)	0 - 60minutes. 1 minute increment
Display Data	IPAP, EPAP, Vt, Vleak, I:E Ratio, BR, MV
Alarms	Power failure, Mask Alarm, Apnea Alarm, High/Low Rate Alarm, low Minute Ventilation
Data storage	micro SD 2G
Screens	2.8" TFT
Displays	Pressure bar graph & Waveform/data display: Pressure curve, Flow rate curve, Volume curve
Humidifier	0 to 5
Humidifier Volume	225ml
Pressure Unit	cmH ₂ O/kPa
Expended	Yes
Tube Length Choice	1.8m, 2m
Dimensions (L*W*H)	21X21X11cm
Weight (kg)	2.1
Language setting	English & Chinese

Specifications subject to change without notice.

Section 9: Quality Warranty

Model	Warranty Period
pM-BP35 PLUS	2 year
pM-BP 35	2 year

Please contact your provider to order additional filters, and accessories.

Do not attempt to open the BPAP unit. There are no user serviceable parts inside. Repairs and internal servicing should only be performed by an authorized service agent.

During the warranty period, if the product fails under conditions of normal use, the damages to the equipments are not caused by man-made reasons, the provider or the appointed maintenance company is responsible to the maintenance or replacement free of charge.

The warranty does not include consumables like air tubing, inlet filter etc.

The warranty does not cover:

- Any breakdown or damages caused by improper operation, abuse of the product, accident, or alteration of the products.
- Any products damaged by reason of repairs made to any components without the specific consent of the authorized service agent.
- Any breakdown or damages caused by water being spilled on or into the BPAP device.
- This warranty does not include normal wear and tear or shipping charges. Plusmed and the dealer's sole obligation and your exclusive remedy under this warranty shall be limited to such repair or replacement. Plusmed shall not be liable for any consequential or incidental damages whatsoever.

After the guarantee period, or damages excluded in the warranty, the provider, or the appointed maintenance company will supply customers the paid service.

Trimpeks İth.İhr.Tur.ve Tic.A.Ş.

Eski Büyükdere Cad. Yunus Emre Sok.
Topçu İş Merkezi No:1/12,
34418, 4.Levent, İSTANBUL, TÜRKİYE
Tel +90 212 319 50 00
Fax +90 212 319 50 50

Trimpeks_IB_pM-BP35PLUS_ver1512

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Revizyon:0