



PRO COMFORT MEDIUM CUSHION USER MANUAL FFC PC MEDIUM

The packaging of this cushion is sealed prior to despatch and the cushion itself will be clean prior to closure. If the cushion has become exposed or damaged during transit, please contact us at the address below

Multi-layer CMHR profiled foam construction. Multi-stretch, wipe clean water proof, vapour permeable, PU coated knitted nylon cover with non-slip cushion base.



FOAM FOR CARE
Azalea Close, Clover Nook Industrial Park,
Alfreton, Derbyshire DE55 4QX
Tel 01773 838800 Fax 01773 838870

The manufacturing process operates under Class 1 of the Medical Devices Directive by fully complying with the essential requirements for CE marking this product.

Flammability

The Pro Comfort Medium Cushion meets the requirements of BS6807:2006 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

User Instructions

This Pro Comfort Medium Cushion has been designed for patients who require additional comfort♣, with a weight guide of up to 114kg / 18 Stone.

Ensure that the surface of the chair seat is clear from debris before placing the cushion as debris can damage the coating of the fabric.

Before Positioning the patient on the cushion ensure the cushion is paced the correct side upwards and the correct way round, see cushion base for further guidance.

Always ensure that, if the cover is removed, it is returned to the same inner. Using this cover on a different cushion core MAY render your warranty invalid.

The label on this cushion has a unique batch number which should be used in the event of a warranty claim. NEVER remove the label.

♣ *Indication of use based upon interface pressure readings– not a prescriptive criteria. Always defer to clinician’s judgement. Safe working load based on test methods devised at the Recticel International Development Centre, Wetteren, Belgium.*

Warranty

Covered against faulty manufacture and materials as replacements goods.

Normal wear and tear expected would not be included.

All warranties subject to terms and conditions of trading.

Please refer to our warranty schedule for terms.

In order to claim under warranty refer to the batch number printed on the product label.

Cleaning

For general cleaning and superficial dirt, use a disposable wipe with a solution of warm water and neutral detergent. Do not use abrasive or phenolic-based cleaners.

To disinfect in situ, use a 0.1% sodium hypochloride solution (1,000 ppm chlorine). If required, a 1% chlorine solution (10,000ppm) and cold water may be applied for not more than 2 minutes

Removable contaminated covers should be laundered at 65°C for not less than 10 minutes or at 95°C for not less than 3 minutes.

Thoroughly rinse and dry the cover before re-use or storage. Store in a cool dry area.

Spin and tumble dry at no more than 60°C

Do not mangle or iron.

Storage & Care

- Store the cushion flat – do not fold
- Do not place heavy objects on the cushion surface when not in use
- Do not allow sharp objects to penetrate the cover coating
- Keep the cushion clean and use protective cover when not in use
- Regularly inspect the exterior of the cushion
- Do not store in damp conditions or next to radiators or other heating devices.
- Do not store directly onto the floor