The packaging of this product is sealed prior to despatch and the product itself will be clean prior to closure. If the product has become exposed or damaged during transit, please contact us at the address below.



Unit 11 Lovett Road Hampton Lovett Industrial Estate Droitwich UK WR9 0QG

Tel +44 (0)1905 774695 Fax +44 (0)1905 796081 Email sales@nexusdms.co.uk Web: www.nexusdms.co.uk

July 2011

 ϵ



USER MANUAL

(please read carefully & retain for reference)

SAFETY FOAM BUMPERS

For side rails & head / foot boards

CME foam bed bumpers with non-removable PU coated nylon cover, sewn seams and velcro fastenings.

The manufacturing process operates under Class 1 of the Medical Devices Directive by fully complying with the essential requirements for CE marking this product.

User Instructions

The Safety Bumpers are wrapped around the bed side-rails or head and foot boards and fixed in position with the velcro located along the bottom edge of the product.

General Cleaning

- Wipe down with a suitable sponge using a solution of warm water (maximum of 40°C) and neutral detergent
- Check your chosen detergent on a small inconspicuous surface area prior to general cleaning
- Do not machine wash or autoclave
- Do not tumble dry
- Do not use abrasive or phenolic-based cleaners
- To disinfect in situ, use a 0.1% sodium hypochloride solution (1,000 ppm chlorine)
- Always wipe dry

Flammability

The Safety Bumpers meet the requirements of BS 7177: 2008 for medium hazard.

The interior materials meet the requirements of The Furniture & Furnishings (Fire) (Safety) Regulations 1988.

Warranty

Covered against faulty manufacture and materials by replacement.

Normal expected wear and tear in use is not covered.

All warranties are subject to the terms and conditions of trading.

Replacement value under warranty reduces proportionally to the period in use up to 12 months after purchase.

If making a claim under warranty please quote the production date stamped on the reverse of the fire retardancy label

Storage & Care

Store the product flat – do not fold

Do not place heavy or sharp objects on the surface of the bumpers when in use / in store.

Do not allow sharp objects to penetrate the cover coating (incl. hypodermic needles)

Keep the product clean and away from heat (above 40°C), high temperatures will delaminate the PU coating.

Keep the product clean

Regularly inspect the product for visible soiling or damage

Do not store in damp conditions