
Sing and Swing Olaf - #2129

USER'S GUIDE

For Technical Support:

Call our Technical Service Department
Monday through Friday, 9 a.m. to 5 p.m. (EST)

1-800-TEC-TOYS or 1-800-832-8697

customer_support@enablingdevices.com



50 Broadway

Hawthorne, NY 10532

Tel. 914.747.3070 / Fax 914.747.3480

Toll Free 800.832.8697

www.enablingdevices.com

Talks, sings & dances!

Sing and Swing Olaf speaks four fun phrases, sings “In Summer” and also moves his head back and forth in a delightful way. Size: 7"L x 6"W x 13"H. Requires 3 AA Batteries. Weight: 1 lb.

Operation:

1. Birthday Bear requires 3 AAA batteries (Not included) to operate. Use alkaline batteries only (e.g. Duracell or Energizer brand). Do not use rechargeable batteries or any other type of batteries because they supply lower voltage and the unit will not perform properly. Never mix old and new batteries together or different brands or types together.
2. The battery compartment is located under Olaf. Open the Velcro flap then, using a Phillips head screwdriver open and remove the batter cover. Next install 3 AA batteries paying close attention to proper battery polarity.
3. Replace the battery compartment cover and secure it with the screw. Set the ON/OFF switch to ON.
4. Connect Olaf to any external capability switch (not included) via the jack located at the end of his cord. Olaf is equipped with a 1/8" jack for use with external capability switches with a 1/8" plug. Make sure you have pushed the switch plug all the way in. There should be no gaps. Olaf can also be activated by squeezing his hand.

5. Actuate your external capability switch or squeeze Olaf’s hand and Sing and Swing Olaf will speak four fun phrases, sings “In Summer” and also moves his head back and forth in a delightful way. Press your switch or squeeze his hand while he is singing or talking with stop him.
6. Each switch activation or foot squeeze will start or stop Birthday Bear.

Troubleshooting:

Problem: Olaf does not function correctly.

Action #1: Replace the batteries paying close attention to proper battery polarity.

Action #2: Make sure that the connection between the external capability switch is pushed in all the way. There should be NO gaps.

Action #3: Try your switch with another switch operated toy/device in an effort to rule out your switch as the source of the problems.

Care of Unit:

If you will not be using Olaf for an extended amount of time, we recommend removing the batteries.

Do not submerge the unit in water, as it will damage the electrical components. Olaf is surface washable only.