

Breeze

by ACCESS™



Forward
Thinking
Mobility

www.Breeze-walker.com

USER MANUAL

- | | |
|----------------------------|--------------------------------|
| 1. Content | 6. Transportation and assembly |
| 2. Introduction | 7. Personal safety |
| 3. Parts description | 8. Operation |
| 4. Care and maintenance | 9. Specification |
| 5. Adjustments for comfort | 10. Accessory |

Congratulations on your new Breeze rollator. The Breeze rollator is an evolutionary design from Access Vital, of Hamar, Norway. The Breeze is lightweight and sleek, modern and elegant in design. The rollator is designed to aid the user in a number of everyday tasks such as walking and shopping.

We at Access believe that everyone should be able to enjoy the outdoors, and get around in public or in their home with the freedom of enhanced mobility. Our products have exceptional function while being modern looking and not 'stigmatising' as a mobility aid.

The Breeze is suitable for everyday indoor and outdoor use. It features easy adjustability, three different seat heights, and great portability.

All Breeze rollators are manufactured to the highest quality standards and are CE marked as Medical Devices. The Breeze rollator is a robust and visually appealing product with Scandinavian design, whilst being a cost-effective and reliable companion to help with special mobility needs.

3.

PARTS DESCRIPTION



4.

CARE AND MAINTENANCE

- The rollator should only be used as an aid for mobility.
- Ensure the rollator is kept clean. The rollator can be wiped down with a cloth or sponge using mild detergent, or can be cleaned with a high pressure washer or garden hose.
- Check the moving parts (for ex. wheels and height adjusters) regularly
- Ensure that the handle operated cable brakes have been adjusted properly (see OPERATION section).
- All the fasteners (screw, nuts, and bolts) should be checked regularly to ensure they are securely tightened.
- Only use an authorized dealer for any maintenance or repairs to your Breeze rollator.

5.

ADJUSTMENTS FOR COMFORT



Handle Height Adjustment

To adjust the handle height of the rollator, pull the Handle Height Adjuster out whilst repositioning the handle. Once the correct height has been selected, release the Adjuster and push the handle down to insure it is locked into position.

Repeat for the other handle.



Seat Height Adjustment

To adjust the height of the seat, push both Seat Lock Tabs inward, and push the seat forward out of the slots. Move the seat to the desired height, and pull the seat until the Seat Lock Tabs lock the seat in place.

6.

TRANSPORTATION AND ASSEMBLY



- The Breeze rollator comes fully assembled and ready to use. There are no tools required.
- To fold up the Breeze for transportation or storage, pull up the Folding Handle while lifting the Breeze by the seat handle. The Breeze will lock in the folded position resembling the photo to the left.
- To unfold the Breeze for use, pull the Folding Handle up until the back legs rotate. Push down on the handles to lock the Breeze open for use.
- Use caution when lifting the rollator into or out of the boot of a car.

- For your health and comfort, be sure to use good posture and hold the rollator so that you can operate the brakes with ease.
- Please store the rollator in a safe environment.
- Fold the rollator out completely in the locked position before use.
- Possible risk of pinch - do not put fingers or hands into the hinge of the Breeze rollator when folding or opening for use.
- Ensure that the wheels are stable and on the ground before using.
- Ensure the brakes are adjusted properly and tested before using.
- Ensure that the seat is locked in position before sitting.
- Always sit in the center of the seat, with the seat adjusted so that the user has their feet on the floor.
- Always have the parking brakes locked in position before sitting.
- Never sit on the rollator when it is folded for storage.
- Never stand on the rollator, or use it as a step.
- Never transport a person on the seat of the rollator.
- Use caution when lifting and carrying the rollator.




Basket

The convenient shopping basket hangs on the front of the seat of the Breeze rollator. The basket can hold up to 10kg max.



Obstacles

The rollator is equipped with large front castors and a stepper pad to negotiate large obstacles (such as a kerb). Press down on the stepper pad with your foot, and pull back on the handles at the same time to lift the front wheels to negotiate the obstacle.

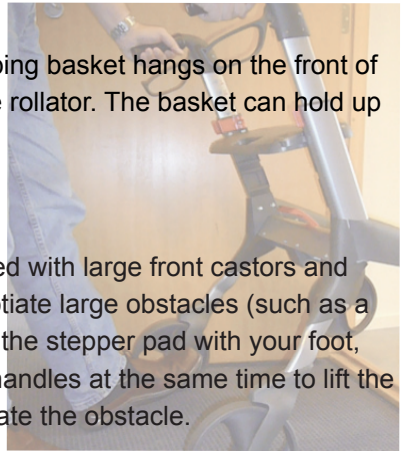


Brakes.

The brakes are a specially designed and essential function of your Breeze rollator. There are no exposed brake cables, and the patented system is both safer and requires less maintenance than other solutions.

To use the brakes, simply squeeze the brake lever up towards the handle. Only a light pull is required, and the brakes can easily be modulated with varying resistance. To lock the brakes (and to activate the parking brakes), simply push the brakelever down until it locks.


To unlock the brakes, pull up on the brake lever, and it will snap back to the neutral position.

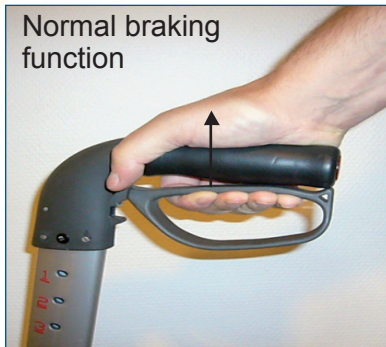

Brake maintenance.

Adjustment of the brakes should be checked before using your Breeze rollator. Periodic checks should be made on the brake functions to keep them in optimal working condition due to normal wear on brake and tire parts during usage.

There is a convenient adjustment screw (hex head) located inside the rear wheel fork (see picture to right) just over the back wheel. Adjust the screw so that the screw is about 3mm from the tire. Turning the screw in (to the right) increases the distance, and turning the screw out (to the left) decreases the distance to the tire.

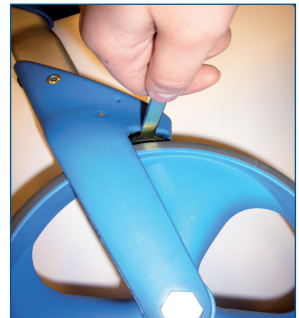

9.

SPECIFICATIONS

Type	Breeze
Max. user weight	125kg (275lbs)
Handle height	77-95cm (30-37in.)
Seat height	48-62cm (18-24in.)
Size (WxL)	60 x 72cm
Size folded	60 x 27 x 96cm
Weight	7 kg (15 lbs)
Material	Aluminum and fiber re-inforced plastic
Cleaning	Detergent or highpressure wash

10.

ACCESSORY

Your Breeze rollator can be equipped with optional accessories to suit your special needs. Contact your dealer for price and availability on the following:



- Backrest - padded backrest for comfort



- Cane/Crutch holder - for carrying a cane



- Serving tray - dishwasher safe, holds food and drink (3 kg limit)



- Deluxe Case Logic bag - Elegant and ergonomic for carrying your personal items