TV-Direct™ Listener with High Definition Sound Technology

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Model TV-100 (2.3/2.8 Mhz Stereo)

Model TV-200 (2.3/2.8 Mhz Stereo and 95Khz Mono)

Thank you for purchasing this TV-Direct™ Listener. Our proprietary High Definition Sound (HDS) technology converts any TV audio output into an infrared (IR) light beam for wireless transmission to a receiver. This unique technology dramatically broadens the acoustic sweet spot of your living room. This means you hear crystal clear dialogues and perfect stereo sounds anywhere in the room.

Please read this operating guide carefully before use and keep it for future reference. If you have any questions, please feel free to contact our Customer Care Department.

IMPORTANT SAFETY INSTRUCTIONS

1. Please read all these instructions carefully before using this TV listener.
2. Always keep the system dry and do not expose it to direct sunlight.
3. Furniture polish, over time, may degrade the rubber feet of the transmitter which may stain your furniture. To avoid such potential stain, you may want to place the transmitter on a pad or mat.
4. This listener allows you to amplify sounds to a loud volume, but listening to earphones at a high volume over an extended period of time may damage your hearing, especially in children. Thus, we suggest that you set the volume to a level that is comfortable to your hearing.
5. Some models may use rechargeable batteries for operation. Please observe the safety and handling instructions for the rechargeable batteries. Always recycle defective batteries through your local waste recycler.
6. This listener is intended for home and personal use only.

BATTERY SAFETY AND DISPOSAL

TV-Direct™ is supplied with two rechargeable NiMH batteries. Do not replace with regular disposal batteries. Recharging disposal batteries may cause them to explode, release dangerous chemicals, and cause serious injury to the user and others.

Please do not dispose of used batteries in the household trash. Please take the batteries to a retail or community collection point for recycling.

RECYCLING INSTRUCTIONS


Help protect the environment!
Please take the time to dispose any used equipment properly.

Product Recycling for Customers in the European Union:


Please do not dispose of your old equipment in the household trash. Please take the equipment to an nearby recycle center; or, return the product to the factory for proper disposal.

PACKAGE CONTENT CHECK

Your package includes the all the items as shown here. If there is anything missing, please contact our Customer Care Department immediately.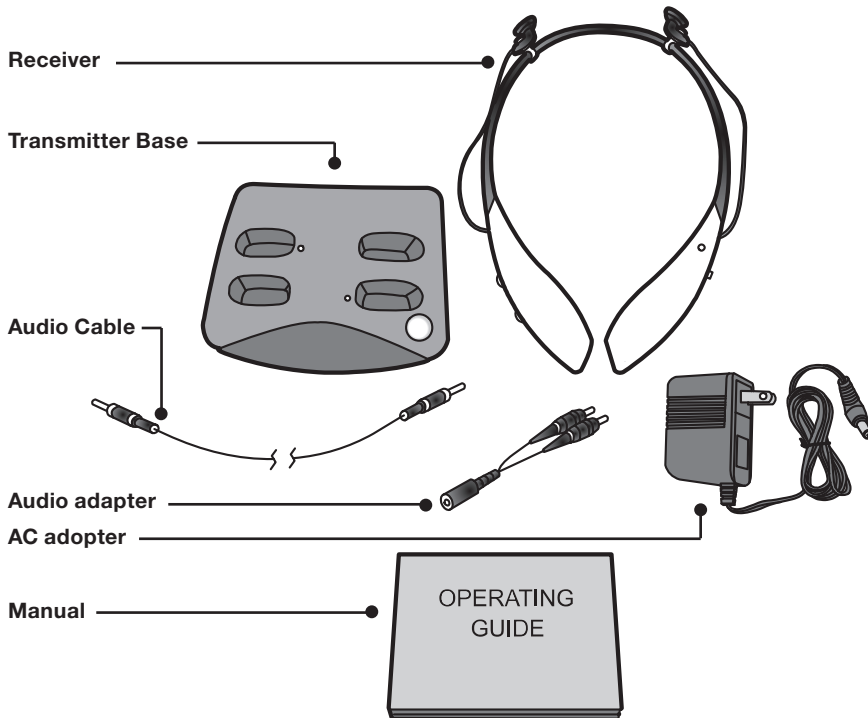


FEATURE IDENTIFICATION

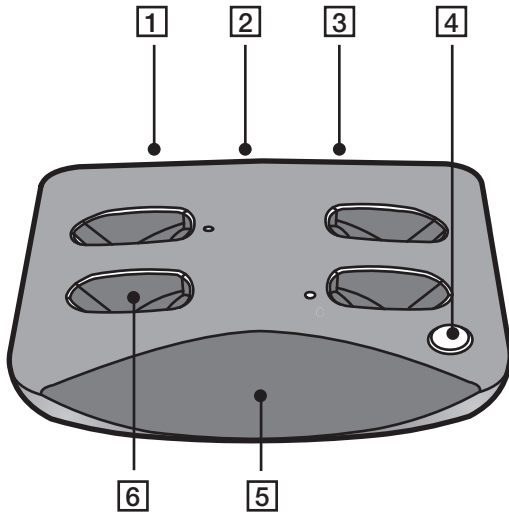TRANSMITTER BASE

1. Stereo / Mono / Microphone selector switch.
2. Audio-in jack for all audio inputs.
3. AC adapter input jack.
4. Transmitter On / Off switch.
5. IR transmitter window.
6. Receiver charging port with LED indicator


FEATURE IDENTIFICATION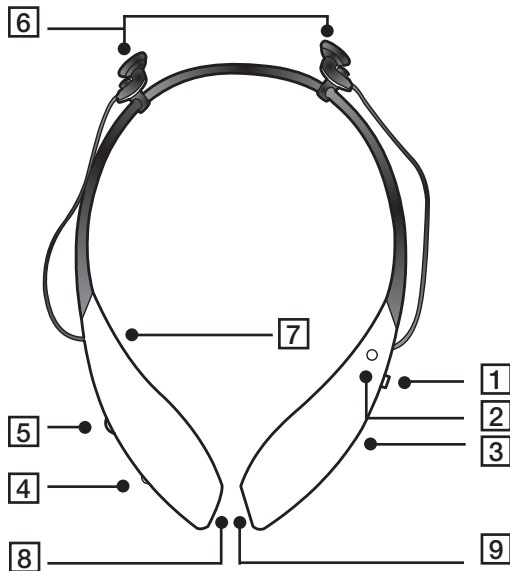

RECEIVER

1. Receiver On/Off pushbutton
2. Receiver On indicator.
3. Battery compartment (rear).
4. Tone control to enhance dialog clarity and intelligibility (Low-Mid-High).
5. Volume control.
6. Left and right earphone.
7. Left and right earphone balance adjustment (rear).
8. 3.5 mm direct audio input jack: For direct connection to other audio source such as MP3 or airplane entertainment system.
9. Battery charging contacts

CONNECTING THE TRANSMITTER TO YOUR TV, CABLE BOX, OR OTHER SOUND SOURCE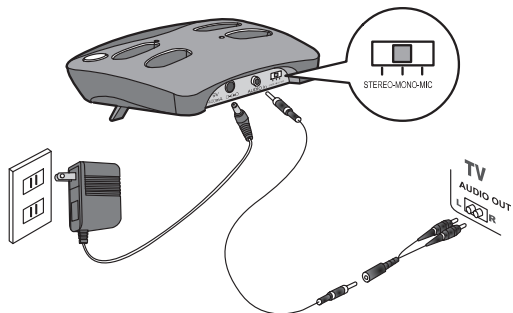

Option 1 (Recommended)

1. Set the Selector switch on the transmitter base to Stereo mode.
2. Use the provided audio cable (and audio adapter) to connect “Audio-in” on the transmitter base to the audio jacks of your TV (or cable box) as shown.
3. Set the TV volume to a comfortable level. On many TV sets, you may “Mute” the TV speaker output and still be able to hear via TV-Direct.
4. Press the transmitter On / Off switch to turn on the power. Verify the IR diodes are glowing faintly behind the IR transmitter window.


Option 2

Same procedures as Option 1 except that you connect the transmitter through the headphone port (usually located at the front of your TV). If you connect it to the headphone port, adjust the volume of the headphone output to a medium level. If this connection mutes the TV speaker sound, you can deactivate the muting function by using the TV menu setting

Option 3

If your TV does not have a headphone or audio output jacks, you can place an optional microphone module directly in front of your TV loudspeaker to pick up the sound for transmission. Connect the other end of the microphone module to “Audio-in” on the rear of the transmitter base. Set the Selector switch to the MIC position. (This optional microphone module is available from a Serene Dealer).

PLACEMENT OF THE TRANSMITTER BASE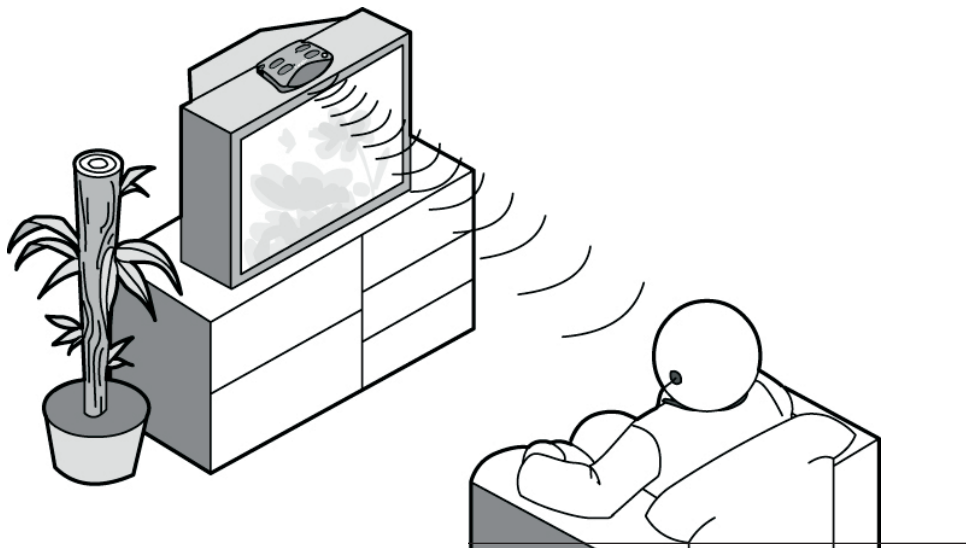


Place the transmitter on top of your TV or on your TV stand so that its front (line of sight) is pointed directly at the viewer. Make sure there is nothing in front of the transmitter blocking the path. If the mounting surface is too low, flip the mounting feet underneath the transmitter for the desired alignment. The powerful transmitter can cover any room size up to 40 ft x 40 ft.

Turn off the transmitter base if the TV is not being used.


Note: Transmitter automatically shuts itself off if no audio input is detected for more than 60 seconds.

USING THE TV-DIRECT RECEIVER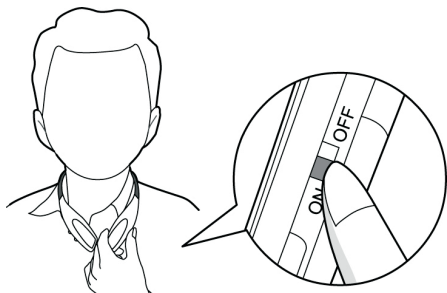

1. Set the volume to a low level (to protect your hearing from sudden loud sound) before putting on the ear buds.
2. Place the receiver gently on your shoulder and press the On / Off button to turn on the receiver. Its ON/OFF indicator should be on.
3. Adjust the sound level to a comfortable level.
4. Set the Tone control to a position that enables you to listen with the best clarity and intelligibility.
5. If you feel the loudness is not the same in both ear buds, adjust the Balance control until they sound about the same.

6. After use, press the On / Off button again to turn off the receiver and return it to the base for charging. The charger LED blinks while the battery is being charged. A fully charged battery provides about 24 hours of listening enjoyment.
7. The transmitter can charge two receivers at the same time. There is no limit on the number of receivers used with a transmitter.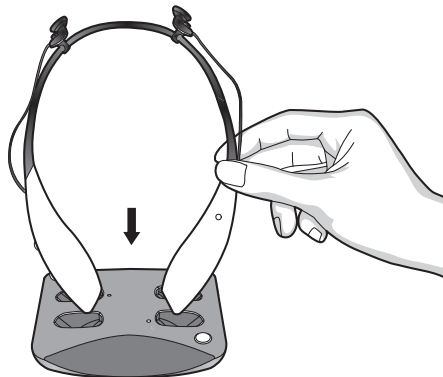


Note: To conserve battery, the receiver automatically shuts itself off if no IR signal is detected for 45 seconds or longer

USING THE RECEIVER FOR DIRECT AUDIO INPUT

1. You may connect a TV-Direct receiver directly to a MP3 music player.
2. Turn off the power on the receiver unit. Plug the audio cable directly into the input jack of the receiver as shown.

(Note: This application does not use any battery power. Audio input signals travel directly from the MP3 or other audio source to the ear buds)

3. Connect the other end of the audio cable to any audio source output (MP3, airplane entertainment system).
4. Adjust volume control on your MP3 or the airplane entertainment system. (Note: The Volume and the Tone controls have no effect on this input jack)

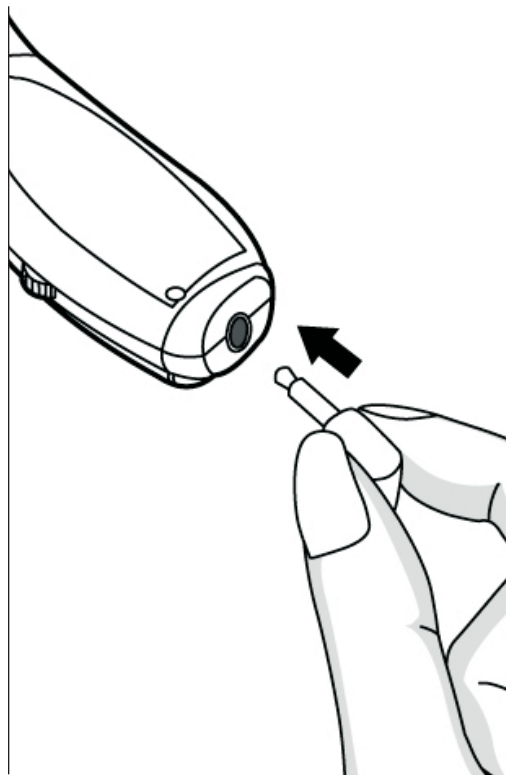SPECIFICATIONS

Selector switch	Stereo: For stereo input
	Mono: For mono input
	Mic: For microphone input
Modulation	FM stereo
Carrier frequencies	2.3/2.8 MHz
Range	Up to 40'
Frequency response	50Hz to 16 KHz
THD	Less than 3%
Transmitter auto shut-off level	Audio input below 10 mv for approx. 1-5 minutes
Receiver auto shut-off time	No IR signal is detected for approx. 1-5 minutes
Power supply (base)	12VDC / 400mA
Receiver battery life	Approx 24 hours
Charging indicator:	Blinking red LED
Weight Transmitter	Approx 8 oz (190 g)
Receiver	Approx 3 oz (80 g)

TV-DIRECT

Optional Accessories

The following accessories enhance your TV listening experiences. Please inquire availability from our dealers.

1. **TV 6005:** TV Microphone Module
2. **TV-6002:** Spare earbuds (1-pair)
3. **TV-6003:** Spare earbud cover or cushion (each package contains 2 pieces of each- Small, Medium, and Large).
4. **DT-2013:** Neckloop with 2.5 mm plug.
This is for coupling TV-Direct's output to a hearing-aids equipped with a T-coil.
5. **TC-1001:** Telecoil - Silhouette style with a 2.5 mm plug
6. **TV-6004:** NiMH AAA batteries (2 pieces)

CARE AND MAINTENANCE

1. Keep all liquids from entering the transmitter or receiver.
2. Before cleaning the transmitter, unplug the AC adapter.
3. Use a slightly damp cloth to clean the outside of the TV listener; do not use any cleansing agents or solvents, or take apart the TV listener.

TROUBLE SHOOTING

If you encounter any difficulties, please check the following before calling our Customer Care Department for assistance.

Cannot hear anything

- Make sure the ear buds are plugged in properly.
- Verify the transmitter On /Off switch is ON.
- Check and make sure all jacks were connected correctly and firmly, including the AC adapter at the wall outlet.
- Check and make sure the transmitter is properly connected to your TV or other sound source.
- Check the volume setting of the receiver.
- Check and make sure the volume of the sound source is sufficient.
- Check and make sure there is nothing

blocking the line of sight between the transmitter and the receiver.

There is only sound in one ear

- Make sure both ear buds are plugged in properly.
- Check if the Balance adjust is set correctly

The sound is distorted

- The receiver battery is too low. Put the receiver back on the transmitter base for recharging.

WARRANTY SERVICE INFORMATION

Your TV-Direct™ comes with one-year limited warranty. Batteries and ear buds are warranted for 30 days only. We warrant during this warranty period, from the date of purchase, to the original consumer, the TV-Direct to be free from defects in materials and workmanship under intended home use. In the event that TV-Direct fails to function properly within one year of original purchase due to defects in materials or workmanship, return the unit (freight prepaid) with proof of purchase (sales receipt or packing slip, no exceptions) to Serene Innovations. Serene Innovations will either repair or replace the unit (with a refurbished unit or unit of equal condition) and return it to you (using UPS/USPS ground shipping) at no cost to you if the unit is returned within 30 days of purchase. If the unit is returned after 30 days of purchase but within the warranty period, there will be a warranty handling charge of \$9.95 for each return.

This warranty does not apply to any product that has been accidentally damaged, abused, mis-

used or negligent, used on electrical frequency or voltage other than marked on product and/or described in manual. Defects or errors caused by unauthorized alterations, repairs, and/or tampering are also not covered by this warranty. This Warranty gives you specific legal rights, and you may also have other legal rights that vary from states to states.

If a defect covered by this warranty should occur, promptly contact a Customer Care Representative by phone or log onto www.sereneinnovations.com to obtain a Return Authorization Number (RAN) and shipping instructions before shipping the product to us. Any shipment without a RAN will not be accepted and will be returned to you at your expense. Any authorized returned product must be accompanied with proof of purchase and a brief explanation of the problem.

For out of warranty repaired and service, please contact our Customer Care Department for directions.

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Serene Innovations, Inc.