

Multipolar Rf+Electroporation Beauty Device



User Manual

WHAT IS RADIO FREQUENCY BEAUTY?

Technically speaking, radio frequency is the number of oscillations (or waves) per second of the electric and magnetic fields within the radio waves portion of the electromagnetic spectrum. It is lowest of the electromagnetic radiation frequencies within the spectrum, which itself spans from radio waves, microwaves, infrared, optical (visible light), ultraviolet, x-rays, to gamma ray. By feeding an alternating current or voltage through an antenna or electrode, electromagnetic waves can be generated that radiate through space at the speed of light; called radio waves. Radio frequency, or radio waves, have wavelengths ranging from less than a centimeter to as long as 100 kilometers

EP(ELETROPORATION)

- 1 It is electroporation principal used equipment which injects skin-effective drugs directly to skin without needles.
- 2 Electroporation is a way(method) of transmitting substances which hold certain amount of molecules(below 40,000 Dalton)through cell membrane.
- 3 Electronic pulse manipulates(disturbs) phospholipids bilayer and it makes temporary pores at skin layer .The,the drugs penetrate into internal c ell through the pores .The principal of electroportaion is that pore is closed when electronic pulse stop.
- 4 Electroporation does not change biological structures and functions of cells and is used as a non-chemical way of transmitting DNA and various outside substances into the living cells.

SYNERGY EFFECT

- 1.Wrinkle care
- 2.Whitening
- 3.Lifting
- 4.permeation
- 5.Scalp care
- 6.Shrink pores
- 7.Lymphatic Circulation
- 8.Elastic skin

SPECIFICATIONS & PACKING

Weight: 250g(Body only)

Size:115x205x140mm

Power:3.5W

Input Voltage:DC12V/2A

How to use

I



1 Power—Power button will be red once power input, the button will be on green when working when press power button

2 Mode---Multipolar RF function(P1) and electroporation function (P2),please chose P1 and P2 by press mode button when the machine is on

3 SART/STOP--- Please press START/STOP button after choose mode,this device will work,5 minutes is default cycle working time,the machine will stop after 5 minutes,please press START/STOP button again,the machine will go on working 5 minutes

4 DOWN/UP –Intensity button,please adjust DOWN/UP to adjust intensity

II

Installation for main machine

1. Check all accessories if they are connected right or not, clean or disinfect the accessories.
2. Plug in the power and switch on the power on of the control panel.
- 3 Chose P1 or P2 mode,press Button
- 4 Press Down/UP button to adjust intensity


III

Installation for electroporation head

Fill serum(water-solubility is better) in the bottle,then install the bottle with the electroporation handle

Notice:

The beautician adjust the Intensity from low to high according to the customer's feeling. After adjustment,Please follow the gesture below:

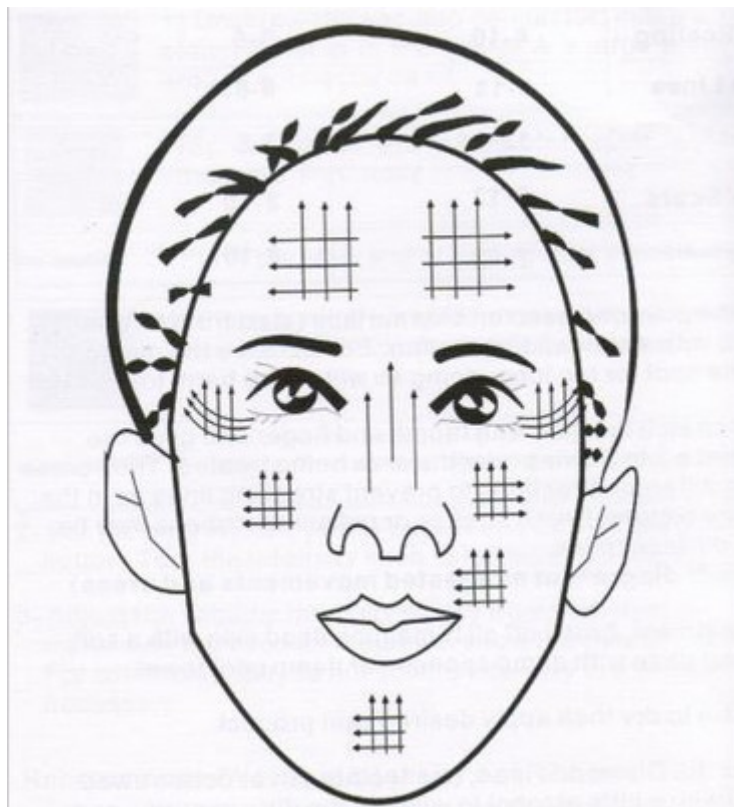

SUGGESTION on PERIOD of TREATMENTS (Only for reference)

Course	Period of Treatments (P D)	PD/Week	Quotation/PD (RMB)
Eyes Revitalizing	12times/PD	2-3times	2200
Wrinkle removal	12times/PD	2-3times	2500
Detoxification	10times/PD	3times	1800

Pores shrinking	12times/PD	1-2times	1500
Skin rejuvenation	12times/PD	2-3times	2200
Neck care	10times/PD	2-3times	1800
Skin whitening	12times/PD	2-3times	3980

NOTES

Conventional operation notes

Before using the machine, please be sure to check whether the accessories are connected well or not.

1. First plug in the power, then switch on the power of the control panel, to avoid the phenomenon of reflux current.

2. Cables of accessories must be connected well with the sockets of the mainframe. Be sure to avoid inferior connection and lead to the useless of accessories

3. Be careful to move the machine.

Place it in the stable, dry place, when pushing it, please be sure to hold the handle, put the accessories onto the accessories holder, to avoid accessories falling to the ground and useless.

4. After finish, be sure to clean and disinfect the accessories. But don't use the alcohol or evaporating disinfectant to clean the mainframe. It will induce the mainframe broken. Make sure to clean the residual products, to avoid products back to the mainframe, and damage the machine.

5. When filling the serum into the serum bottle, please be careful. Before please use the warm water to clean the rudimental products first, then place it in dry place. Be sure not place it in the oven to be disposed in high temperature.

6. Before operation, please the customer and beautician take off all metal decorations.

7. Please keep treatment away of the skin with wounds. Do not operate it on Patients with facial paralysis.

8. Do not operate to whom with serious vasodilator.

9. Do not operate to whom with heart disease, high blood pressure, diabetic.