IS Medical Grade™

Isolation Transformers

⚠ CAUTION!

DO not connect to an ungrounded outlet. DO not use with extension cords or adapters that eliminate the connection to ground. The isolation transformer is designed for indoor use only. Install the isolation transformer away from heat emitting appliances such as radiators or heat registers. Do not install where excessive moisture or other conductive contaminants are present. Do not use with life support appliances if a malfunction or failure could cause failure or significantly alter the performance of the life-support device. Do not use in the presence of a flammable anaesthetic mixture with air, oxygen or nitrous oxide.

Installation

1. Connect Your Isolation Transformer to Your AC Power Line

Plug the power cord into a standard, 3-wire grounded, 120V AC, 60 Hz utility outlet. Note: the isolation transformer only meets UL60601-1 when plugged into a hospital-grade outlet.

2. Connect Equipment to Your Isolation Transformer

Plug the AC power cords of your equipment into your isolation transformer. Do not use extension cords or adapters that eliminate the connection to ground. Connect equipment that requires a external connection to ground to the isolation transformer's potential equalization conductor, labeled "∇" on the unit.

3. Turn Your Isolation Transformer On

Flip the latched switch to provide clean AC power to the isolation transformer's receptacles. "I" is the ON position, "O" the OFF.

Do not overload your isolation transformer. Excess load on the isolation transformer will cause one or both of its circuit breakers to trip; if it does, the outlets will cease to provide power. Reduce the load on the isolation transformer down to your unit's Output Power rating by unplugging excess equipment, then depress the circuit breaker button(s) to reset the breaker(s) and restore power.

Maintenance

Your isolation transformer requires no routine maintenance. There are no user-serviceable parts inside your isolation transformer; do not open it for any reason.

IMPORTANT SAFETY INSTRUCTIONS. SAVE THESE INSTRUCTIONS.

Specifications

	Output Power (Watts)	Outlets
IS250HG	250 W	2 NEMA 5-15RHG
IS500HG	500 W	4 NEMA 5-15RHG
IS1000HG	1000 W	4 NEMA 5-15RHG
IS1800HG	1800 W	6 NEMA 5-15/20RHG

Surge and Noise Suppression

Your isolation transformer is a surge suppressor that protects against surges on its input line and on its outlets. It also suppresses disruptive EMI/RFI line noise on the utility power line.

Medical/Hospital Grade Protection (UL60601-1 Listed)

Your isolation transformer is UL60601-1 listed, classified as Class I, Type B, Continuous Operation, Ordinary Equipment. The isolation transformer is Faraday-shielded and reduces the cumulative leakage currents of all connected equipment below 100 microamps. As a further safety measure, your isolation transformer is equipped with a hospital grade plug and receptacles to reduce the risk of accidental equipment disconnection.

2-Year Limited Warranty

Seller warrants this product, if used in accordance with all applicable instructions, to be free from original defects in material and workmanship for a period of 2 years from the date of initial purchase. If the product should prove defective in material or workmanship within that period, Seller will repair or replace the product, in its sole discretion. To obtain service under this warranty you must obtain a Returned Material Authorization (RMA) number from Tripp Lite or an authorized Tripp Lite service center. Products must be returned to Tripp Lite or an authorized Tripp Lite service center with transportation charges prepaid and must be accompanied by a brief description of the problem encountered and proof of date and place of purchase.

1111 W. 35th Street
Chicago, IL 60609
Customer Support:
(773) 869-1234
www.tripplite.com

This warranty applies only to the original purchaser who must have properly registered the product within 10 days of purchase.

THIS WARRANTY DOES NOT APPLY TO NORMAL WEAR OR TO DAMAGE RESULTING FROM ACCIDENT, MISUSE, ABUSE OR NEGLIGENCE. SELLER MAKES NO EXPRESS WARRANTIES OTHER THAN THE WARRANTY EXPRESSLY SET FORTH HEREIN, EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW. ALL IMPLIED WARRANTIES, INCLUDING ALL WARRANTIES OF MERCHANTABILITY OR FITNESS, ARE LIMITED IN DURATION TO THE WARRANTY PERIOD SET FORTH ABOVE; AND THIS WARRANTY EXPRESSLY EXCLUDES ALL INCIDENTAL AND CONSEQUENTIAL DAMAGES.

WARNING: The individual user should take care to determine prior to use whether this device is suitable, adequate or safe for the use intended. Since individual applications are subject to great variation, the manufacturer makes no representation or warranty as to the suitability or fitness of these devices for any specific application.

Do not use Tripp Lite Isolation Transformers for life support devices in which a malfunction or failure of a Tripp Lite UPS System could cause failure or significantly alter the performance of a life-support device.

The policy of Tripp Lite is one of continuous improvement. Specifications are subject to change without notice.

Regulatory Compliance Identification Numbers

For the purpose of regulatory compliance certifications and identification, your Tripp Lite product has been assigned a unique series number. The series number can be found on the product nameplate label, along with all required approval markings and information. When requesting compliance information for this product, always refer to the series number. The series number should not be confused with the marking name or model number of the product.

This product designed and engineered in the USA.

Warranty Registration

Visit www.tripplite.com/warranty today to register the warranty for your new Tripp Lite product. You'll be automatically entered into a drawing for a chance to win a FREE Tripp Lite product!*

* No purchase necessary. Void where prohibited. Some restrictions apply. See website for details.

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