

Alligator SC Ramps User's manual



MOBILEX A/S
Grønlandsvej 5
DK-8660 Skanderborg
Denmark
www.mobilex.dk

1. General Information

Dear customer,

This user manual contains a description of the medical device and important guidances to ensure a correct and safe usage of the product. It is important prior to use to read this manual carefully. It is especially important to read the safety requirements and follow these. We continuously develop our products and we reserve the right without further notice to change the specifications and functions of products.

Intended use of the device

The Mobilex Alligator Ramps are medical devices which is intended to be used only to assist patients in wheelchairs or scooters. They enable the passage of elevated points (f.e. Stairs) with or without the assistance of a helper. Mobilex Ramps thereby enables patients, sitting in a wheelchair because of invalidity, to enter or leave buildings or cars, buses etc. The ramps can be used as well inside as outside. Any other use is prohibited.

By questions please take contact to your dealer or directly to Mobilex A/S.

Mobilex A/S

Grønlandsvej 5
DK-8660 Skanderborg
Denmark

Telephone: +45 87 93 22 20
Telefax: +45 87 93 17 77
E-Mail: info@mobilex.dk

2. Safety requirements

- Only use the product in flawless condition.
- If defects or errors are detected you must immediately contact the dealer.
- Be aware of the information on the product label.
- Make sure that this user manual is read by all persons using the ramp. Damages, caused by the fact that the user manual has not been followed, will not be accepted.
- Ramps must not be loaded with more than the “max load” described in Technical Data and on the label.
- The ramps can with assistant be used at maximum inclination of:
 - SC-045 10 cm**
 - SC-090 20 cm**
 - SC-120 25 cm**
 - SC-150 30 cm**
- The ramps can without assistant be used at maximum inclination of:
 - SC-045 5 cm**
 - SC-090 10 cm**
 - SC-120 12 cm**
 - SC-150 15 cm**
- Only use the ramp for the purpose it is intended for.
- Avoid making constructive changes on the ramp, unless you have the producers written accept.
- The ramp must only be used on a stable surface.
- Be careful during folding or unfolding of the ramp. There is a risk of finger stuck or cut between parts of the ramp.

3. Description

Product description

Mobilex SC-Ramps are foldable platforms made of aluminium. The surface is specially “rib” shaped for anti-slipping purposes.

SC-Ramps are supplied in cartons. The contents are:

- ✓ 1 Ramp
- ✓ 1 User manual



5. Technical Data

Alligator SC Ramps	SC-045	SC-090	SC-120	SC-150
Length	46 cm	92 cm	122 cm	152 cm
Width (unfolded)	71 cm	71 cm	71 cm	71 cm
Width of the tread surface	70 cm	70 cm	70 cm	70 cm
Height of side protections	3,5 cm	3,5 cm	3,5 cm	3,5 cm
Max. elevation / Height difference	12° / 10 cm	12° / 20 cm	12° / 25 cm	12° / 30 cm
Total weight	3,54 kg	6,68 kg	9,14 kg	11,43 kg
Max. load	300 kg	300 kg	300 kg	300 kg
Folded ramp measures (L x W x H) in cm	46 x 36 x 10	92 x 36 x 10	122 x 36 x 10	152 x 36 x 10

Product label

The product label is located on one of the ramp sides

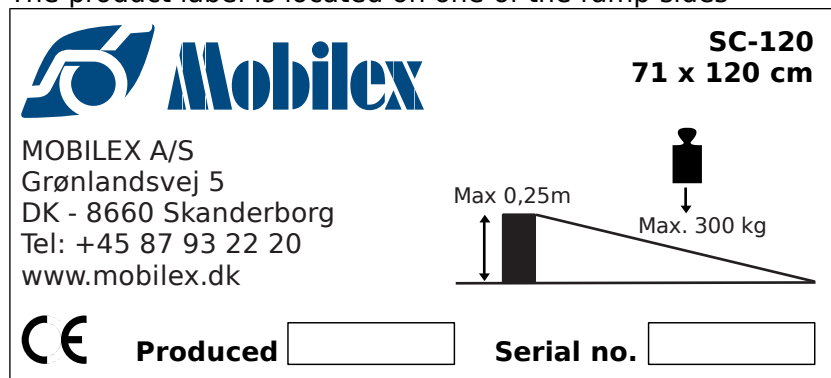


Image of the product label (not original)

6. Maintenance

Ramps are basically maintenance free. Although it is recommended to check always before use if the anti-slip surface isn't dirty or torn and if hinge connection between two ramp halves is rigid and strong. In case of any doubts concerning technical condition of your ramp stop using it and contact your dealer.

Despite the solid construction and the use of resistant materials the product is subject to wear. It is therefore recommended to let check the product at regular intervals by a professional

7. Most common problems and solutions

Symptoms	Possible cause	What to do?
Ramp is not stable	The surface under the ramp is not flat	Check the position of the ramp. Put it a flat, smooth surface
Ramp is not stable	Ramp is mounted upside down	Check if the upper end of ramp (the end with a small hole in the middle of every tread) is at the top and if <i>not so</i> , turn the ramp upside down

8. Disposal of the product

The product can not be disposed of with household waste but must be brought to the local recycling centre.

9. Guarantee terms

Mobilex A/S offers 2 years of guarantee by errors in production. The guarantee is not valid by not-intended use of the product. Parts subject to normal wear are not covered by the guarantee, unless wear is caused by a manufacturing fault. In the event of damage, please notify your dealer. The guarantee does not cover transport cost. The guarantee does not cover injury or other damages eventually related to a malfunction of this product.

10. Declaration of Conformity (CE)

The company Mobilex A/S, Grønlandsvej 5, 8660 Skanderborg, Denmark declares that the product


Alligator Ramp - SC series

is designed in accordance with the technical documentation and made of which conforms to the following basic requirements:

- **EU Council Directive 93/42/EEC for medical equipment
Annex I of the June 14, 1993**

Lars Nygaard
Managing Director
Mobilex A/S

Skanderborg, the 29th November 2009

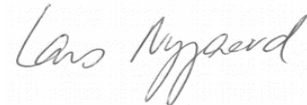

.....
signature