SOEHNLE

PROFESSIONAL

Bedienungsanleitung	2
Operating Instructions	13
Mode d'emploi	23
Instrucciones de manejo	33

7700, 7701, 7702

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Thank you for choosing this Soehnle product. All the features of this product were designed to the state of the art and are optimised for simple and straightforward use.


If you have any queries or problems which are not dealt with in these operating instructions, please contact your dealer or visit our website at www.soehnle-professional.com.

Type	Execution	Applicator	Weighing range	e=	Min	Parts catalog
7700	Stand	Platform	1 = 150 kg	50 g	1 kg	470.014.019
			2 = 250 kg	100 g	2 kg	
7701	Case	Platform	1 = 150 kg	50 g	1 kg	470.014.020
			2 = 250 kg	100 g	2 kg	
7702	Chair	Seat	1 = 150 kg	50 g	1 kg	470.014.021
			2 = 200 kg	100 g	2 kg	

CE-Marking

The product is provided with the CE mark according to the following directives

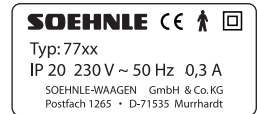- 89/336/EWG Electromagnetic compatibility.
- 73/23/EWG "Low-voltage Directive"
- 90/384/EWG Non-automatic scales
- 93/42/EWG Medical product (allocation to class I with measuring function)

Classification

- Class I medical product class I with measuring function
-  Electrical protection class II (totally insulated, no protective conductor connection)
-  Application part (see above) of type B
- Water protection according to IEC529: IP20

Safety Instructions

The appliance has no power switch. When the power plug is connected to the power supply and the scales, the appliance is supplied with power. When the appliance is switched off, the "OFF" message appears in the display.

Appliances which have become moist due to a change in ambient temperature may only be switched on after they are completely dry. Power supplies which have become moist may not be switched on at all. Replace them with a genuine power supply. Only use power supplies which comply with European Standard EN 60601-1.

Intended use

These Soehnle scales are intended for weighing patients. With stand-mounted and case scales, weighing may only take place if the patient is standing. With chair scales, weighing may only take place if the patient is sitting.

These operating instructions are part of the appliance. They must always be kept close to the appliance. Compliance with these operating instructions is mandatory in order to ensure the correct function and operation of the appliance. If the appliance fails to function correctly, the appliance may be damaged. The appliance must then be sent to an authorised service centre. Only genuine spare parts may be used if an authorised service station carries out repairs. Genuine spare parts are described in the service documentation together with order numbers (see table for a list of spare part numbers). Soehnle will only assume responsibility for the safety of the appliance if these instructions are observed and the appliance is operated in compliance with the operating instructions. The appliance is a medical device and may only be used by persons who are capable of handling the appliance correctly as a result of their training or their knowledge. Before using the appliance, the user must ensure that the appliance is functional and in proper working order.

The operator must know how to operate the appliance. The appliance is not intended for use in medical rooms subject to explosion hazards.

Explosion-hazard areas may occur if flammable anaesthetics, skin cleansing agents and skin disinfectants are used.

Use of additional appliances

Only additional appliances (printer, computer) may be connected to the serial data port if they conform to European Standard EN 60601-1 or a suitable disconnecting device is connected in series.

Applicator

Only the applicator listed in the table may come into contact with the patient.

Maintenance and service

The measuring inspection test was performed during the initial calibration process in production. Further regular measuring inspection tests (recalibration) must be carried out by the calibration authorities responsible in compliance with national regulations. Repairs may only be carried out by a centre authorised by Soehnle using genuine spare parts.

Cleaning

Before cleaning, disconnect the appliance from the mains by removing the power plug from the wall socket. Only clean the appliance using a moist cloth. On no account may water be allowed to enter into the appliance. Disinfectant may only be used on the membrane keyboard of the display and on the applicator.

The following disinfectants are allowed: Methylated spirit, Isopropanol, 2% Kohrsolin; 1% aqueous Sokrena solution; 5% Sagrotan; 5% Gigasept. It is prohibited to spray the appliance and power plug.

Before putting the appliance into use, please read with care the information given in the Operating Instructions. They contain important instructions for installation, proper use and maintenance of the appliance.

The manufacturer shall not be liable for damages arising out of failure to heed the following instructions:

When using electrical components under increased safety requirements, always comply with the appropriate regulations. Never perform work on the appliance while the power is switched on.

Improper installation will render the warranty null and void. The mains power supply must be compatible with the specifications printed on the appliance power supply. The appliance is dust-proof and splash-proof in accordance with protection class IP 65. Please make sure that all seals are undamaged. When the appliance is running off the battery, close the power socket using the supplied rubber cap.

The device meets the requirements for electromagnetic compatibility. Do not exceed the maximum values specified in the applicable standards.

Hinweis

Interference suppression of the unit corresponds to the valid EU directive 89/336/EWG.

The display value may be influenced by extreme electrostatic and electro-magnetic interference, e.g. operation of a radio unit or a mobile phone in close vicinity to the unit. When the electrostatic interference is no longer present, the unit can again be used for its intended purpose. You may need to switch it on again. If there is permanent electrostatic interference, we recommend earthing the platform.

The unit is a measuring instrument. Air currents, vibrations, rapid changes in temperature and direct sunlight may have an effect on the accuracy of the scale.

A short warm-up period of a couple of minutes after switch-on stabilises the measured values. Place the weighing product carefully on the scale. Do not keep any loads permanently on the platform. Avoid any knocks or overloading above the stated maximum load. These could damage the scale.

Supplement: Remove the batteries if the scales are not in use for a long period of time.

Correct battery disposal

Batteries must not be disposed of as household waste. The law requires that you, as consumer, return the waste batteries either to public collection points in your town or to any outlet selling batteries of the same kind.

Note: Batteries containing pollutant substances are marked as follows:

Pb = Battery contains lead

Cd = Battery contains cadmium

Hg = Battery contains mercury

Warranty - Liability

If a fault or defect is present on receipt of the unit which is within SOEHNLE's scope of responsibility, SOEHNLE shall have the right to either repair the fault or supply a replacement unit. Replaced parts shall be the property of SOEHNLE. Should the fault repairs or replacement delivery not be successful, the statutory provisions shall be valid. The period of warranty shall be 2 years, beginning on the day of purchase. Please retain your receipt as a proof of purchase. Should your scale require servicing, please contact your dealer or Soehnle Customer Service.

No responsibility shall be accepted for damage caused through any of the following reasons: Unsuitable or improper storage or use, incorrect installation or commissioning by the owner or third parties, natural wear, changes or modifications, incorrect or negligent handling, overuse, chemical, electrochemical or electrical interference or humidity, unless this is attributable to negligence on the part of SOEHNLE.


With scales subject to calibration, the fault limits shall refer to the official design and calibration regulations valid on the day the order is placed for the scale construction which is applicable according to the purchaser's specifications or to the recognisable purpose of use. For shipments which are destined to SOEHNLE's knowledge for shipment overseas, the design and calibration regulations of the destination country shall be valid providing SOEHNLE is informed of these by the ordering party.

If operating, climatic or any other influences lead to a major change in conditions or material quality, the warranty for perfect unit functioning shall be rendered null and void. If SOEHNLE provides an individual warranty, this means that the unit supplied will be free of faults for the length of the warranty period. Always keep the original packaging in case you have to return the scale.

Disposal

If the scales or parts of the scales are no longer used, your local waste disposal authorities will be pleased to inform you about the measures you should take to dispose of the scales in an environmentally responsible manner.

Description of the type label

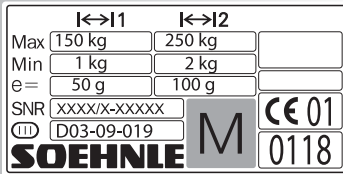- I<->I1** Weighing ranges
- Max** Maximum permitted weighing range
- Min** Minimum permitted weighing range
- e=** Calibration value (graduation)
- SNR** Serial number of scale (scale type, last digits of year of manufacture, number)
- CE 01** EU conformity symbol with year of application, also start of calibration validity
- (II)** Precision class
- D03-09-019** Number of EU admission
- 0118** e.g. 0118, Code of the "office quoted" (place of initial calibration)
- M** Symbol for EU calibration
- 001** Calibration count

Calibration counter

The calibration count indicates how often the scale was calibrated. The stored count must match the calibration count printed on the adhesive label affixed to the calibration plug.

Calling up the calibration count

Press key **⊖** and hold and then press key **F**. Hold both keys for 3 seconds.

Press key **⊖** twice.


Exiting the calibration count display

Press key **⊖** once.

Press key **→0←** once.

Press key **⊖** and hold and then press key **→0←**.

Installation and commissioning

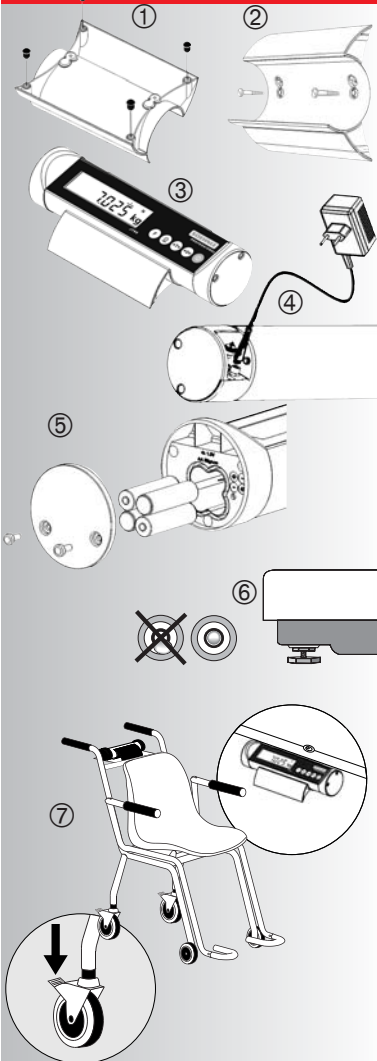The display unit is already mounted on the scale. If necessary, use the supplied bracket to place it on a table or fix it to the wall.

1. For table use, insert the rubber feet.
2. For wall use, fix the bracket to the wall approx. 110-150cm above the ground.
3. Insert the display unit into the bracket.
4. Connect the power supply ...
5. ... and/or insert (rechargeable) batteries (4 x 1.5V AA). Then slacken the two screws on the left side of the housing.

If the appliance is operated with the charge adapter, only use rechargeable batteries. Normal batteries will destroy the appliance.


6. Set up the scales at the point of application. Align 7700 and 7701 using a spirit level and ensure that the appliance is horizontal.
7. Align the 7702 using the spirit level and the adjustable feet until it is horizontal. During the weighing process, secure the appliance using the thumbwheels on the adjustable feet. The scale is ready for use.

The operating keys

- On-Off
- Zero position
- Tare
- Print
- Function key (see Page 19)

Display symbols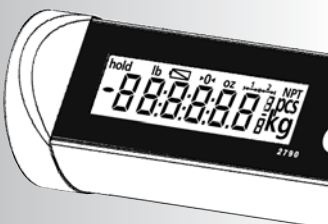

- hold** Display is fixed to improve legibility.
- Replace the batteries or charge the rechargeable batteries.
- Zero message
- N** Net weight
- pcs** Piece mode
- Active weighing area

Switching on

Only when the scale is not loaded.

After the test routine, the display is set to zero. The scale is ready for use.

Step onto the scales or sit on the scales. The body weight appears in the display.

Setting to zero

Necessary when the display does not show zero, when unloaded. Possible zero range -1 % ... +3 % of the weighing range.

Tare weighing

Place a towel or similar on the scales. Press the tare key. The scale is ready for net weighing.

Deleting tare weight

The display shows the gross weight or 0.000 when unloaded

Switching off

This is immediately possible when the display is "0". In other cases, depress the button for 3 seconds. When the appliance is running off the mains power supply, the message "Off" appears in the display.

Printer/IT

Sends the dataset to the printer or IT equipment.

Note

These functions are only executed when the scale is at a standstill and up to 5 seconds after the key is pressed.

Extended functions

H: 170

BI: 26.2

82.8 kg

The **F** key can be programmed with the following functions: Hold, unit switch-over, BMI and special Dialysis function. The standard function programmed is BMI. You can make changes in User mode (p. 20/21).

- Press the function key **F**.
- Display, e.g. 170 = body height in cm
- Body height > 170cm: Press **→0←** key until body height is correct.
- Body height < 170cm: Press **→T←** key until body height is correct.
- Confirm by pressing **⊙**.
- Display switches to weighing mode 0.00 kg.
- Weighing.
- After the scales have stopped moving, the display shows the BMI and weight alternately.

BMI classification (as per DGE - German Society for Nutrition, Nutrition Report 1992)

Classification		
Underweight	< 20	< 19
Normal weight	20 - 25	19 - 24
Overweight	25 - 30	24 - 30
Adiposity	30 - 40	30 - 40
Severe adiposity	> 40	> 40

Special dialysis function (intermediate tare)

If this function is set, the operator may give the patient a blanket or a book or remove them without affecting the weight reading.

Press the **F** key. "Hold" appears in the display. Give or take back book and press **F** again. Previous weight reading will be displayed without Hold but with "N". The reading is now a net weight. The tare is stored in the tare memory.

hold 82.8 kg

82.8 kg

Hold function

When this mode is active, press the **F** key. "Hold" flashes until the value is determined and frozen.

These 6 modes are settable in User mode:

Mode	Hold	Remove by pressing
Manual	At a standstill outside the zero range	⊙ key
Automatic		Remove load from scale
Slave pointer, manual	For weight increase and repeated standstill	⊙ key
Slave pointer, automatic		Remove load from scale
Maximum value, manual	of the maximum weight reached	⊙ key
Maximum value, automatic	(without being at a standstill)	Remove load from scale

hold 0.0 kg

hold 82.8 kg

kg/lb switch (with non-calibratable units)

Your service partner can activate 2 different functions:
Assignment of the **F** key to switch units.
(On restarting, the most recently chosen unit is displayed).

General restriction to kg or lb (**F** key can be assigned to Count or Hold).

User Mode

The User mode is an operating mode to set user operating parameters.

Switch from Weigh mode to User mode: press the **⊖** key and the **⊙** key for 5 seconds.

To call up the User mode position, press the **⊖** key.

Increment and decrement the User mode position
Increment by pressing the **→T←** key. Decrement by pressing the **F** key.

Switch to the Edit function by pressing the **⊖** key.

Select the decade by pressing the **→0←** key. The active decade flashes.

Set the value of the User mode position: move forward by pressing the **→T←** key. Back by pressing the **F** key.

Save the set value by pressing the **⊖** key.


Switch to the starting level by pressing the **→0←** key.

Exit User mode and save the data by pressing the **→0←** and the **⊖** key simultaneously.

CAL POS	R/W	CONTENTS	VALUE RANGE	COMMENT	DEFAULT
00	R	Version number	0.00..9.99	Current version	1.00
01	W	Backlighting In rechargeable mode, on for 30 sec. (always on with power supply)	0 1	Off On by pressing a key or making a weight change	0
02	W	Auto Off delay	0 1..60	No automatic switch-off Switch off delay in minutes	3
03	W	Empty value	0..99.9	Empty value in % of max. load	1.0
04	W	Function key	0 1 2 3 4	Hold shift Count mode Kg/lb switch Special dialysis function BMI operation (medicine)	1
05	W	Hold mode Function only > Empty Value	0 1 2 3 4 5 6	Inactive Hold on : Remove by: Standstill Pressing On key Standstill Removing load Max. value Pressing On key Max. value Removing load Slave counter Pressing On key Slave counter Removing load	0
06	W	Filter setting	0 1 2 3 4	Very rapid filter Rapid filter Normal filter Slow filter Very slow filter	2
07	W	Printer	0 1 2 3 4	No printer Strip printer Label printer TSP800 IT Customer-specific print image *1)	1
08	W	External alibi memory	0 1	No external alibi memory External alibi memory available	0
09	W	Line break symbol	0 1 2 3	CR+LF CR LF Semicolon	0
10	W	Decimal separator	0 1	Comma Full stop	0
11	W	IT data set with STX-ETX frame	0 1	Inactive STX-ETX active	0
12	W	Default function	0 1 2	Inactive Default values, user data Default values, print image	0

		Messages	Elimination
①		The scale sets the value zero automatically when it is switched on.	① Either take the load off the platform or press the $\rightarrow 0 \leftarrow$ key to activate the calibrated zero point.
②		The zero range at power-on is -5%+15% of the weighing range. If the scale is outside the set tolerance range, the display shown adjacent appears.	② There is a mechanical or electrical fault.
		Parts weight too small	The scale is not suitable for these parts.
		Zeroing not possible	The zero limit has been exceeded or undershot.
		Taring is not possible	Cannot tare the zero display or negative values.
		Print not permitted on over- or underload	Place an object on the scale / remove an object from the scale.
		Switch to lbs deactivated	Activation is described on Pages 20/21.
			In case of other error messages, please contact your SOEHNLE service

Technical data

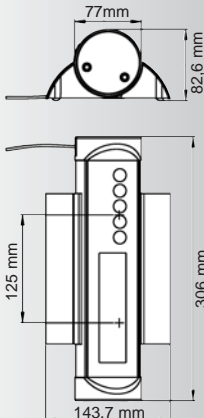- Housing made of anodised aluminium
- Operated by means of a tactile keypad.
- Desktop power supply: input 230VAC / 50Hz / 60mA, output 9VDC / 0.3A
- Charge adapter (optional): input 100-240VAC / 50-60Hz / 0.3 A; output charge current for 4 cells 0.5A, operating voltage 9VDC / 0.3 A
- Operating temperature -10° +40° C
- Storage temperature -20° +60° C
- Zero range -1 %... +3 % of the weighing range (max.)
- Zero range at power-on -5%...+15% of the weighing range.
- RS 232 (V24) interface for printer or IT connection. For a description of the interface, please contact your Soehhle service partner.

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Soehnle-Waagen GmbH & Co. KG